

Well No. HLA-B allele^{1,2,3} 1 2 3 4 5 6 7 8 9 10 11 12 13 14 15 16 17 18 19 20 21 22 23 24 25 26 27 28 29 30 31 32

Negative Control

*08:255	2			5																21															
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32			
*13:14, 13:83, 40:02:01:01-40:02:09, 40:02:11-40:02:17, 40:02:19-40:02:30, 40:04:01-40:04:02, 40:06:01:01-40:06:09, 40:06:11-40:06:14, 40:06:16-40:06:18, 40:06:20-40:06:22, 40:08, 40:11:01:01-40:11:02, 40:13, 40:19, 40:27:01-40:27:03, 40:29, 40:35:01-40:35:02, 40:37, 40:39-40:40, 40:56, 40:64:01:01-40:64:02, 40:68, 40:70:01-40:71, 40:74-40:75, 40:78, 40:82-40:83, 40:85-40:86, 40:91, 40:93-40:94, 40:96-40:97, 40:99, 40:103-40:105, 40:107, 40:109, 40:111, 40:115, 40:119, 40:122, 40:127, 40:133Q, 40:142N-40:145, 40:157, 40:162, 40:165, 40:167, 40:169, 40:173, 40:176-40:177, 40:181, 40:189-40:190, 40:200-40:203, 40:205-40:206, 40:209-40:211, 40:214, 40:220, 40:224-40:226, 40:229, 40:232, 40:243-40:244, 40:246, 40:254-40:255, 40:266, 40:268-40:271, 40:274-40:276, 40:283, 40:287, 40:289-40:297, 40:302-40:304, 40:306, 40:311, 40:313-40:314, 40:317, 40:320, 40:322, 40:331, 40:334-40:335, 40:337N, 40:340, 40:343, 40:345N, 40:347-40:350, 40:354, 40:356, 40:359, 40:361N-40:363, 40:366, 40:369, 40:371, 40:373-40:376, 40:378, 40:384, 40:387-40:390, 40:399N, 40:405, 40:408-40:413, 40:415, 40:423, 40:425, 40:427, 40:432, 40:434-40:435, 40:437-40:438N, 40:440-40:442, 40:444-40:445, 40:451, 48:01:10															15				19																
*13:31, 13:131, 15:30:01:01-15:30:01:03, 15:58, 15:73:01:01-15:73:01:02, 15:150, 15:303, 15:324, 15:453, 15:486, 15:541, 35:44, 35:413, 46:61, 46:74, 54:33, 55:21														14					19		w														
*13:46, 15:179:01-15:179:02, 15:218Q, 15:518, 27:47, 27:127, 35:38, 35:115, 35:169, 37:60, 44:02:01:01-44:02:65, 44:05:01:01-44:05:04, 44:08-44:09, 44:11-44:12, 44:15:01:01-44:17, 44:19N-44:24, 44:27:01:01-44:27:03, 44:33-44:34:02, 44:41:01-44:42, 44:44, 44:48-44:49, 44:51-44:52N, 44:55, 44:58N-44:59:02, 44:63, 44:66-44:68, 44:70-44:73, 44:75, 44:78, 44:80, 44:83-44:84:02, 44:86-44:91, 44:93, 44:97, 44:99-44:102, 44:104, 44:106, 44:112-44:113, 44:116, 44:118-44:119, 44:123, 44:126:01-44:127, 44:132-44:134, 44:136-44:140, 44:142, 44:145, 44:148-44:149N, 44:151-44:152, 44:158, 44:162, 44:168-44:173, 44:176-44:177, 44:179, 44:185, 44:187, 44:190-44:191, 44:195N-44:196, 44:200-44:201, 44:206, 44:208, 44:211, 44:214, 44:216-44:220, 44:225-44:226, 44:229-44:230, 44:235-44:236, 44:238, 44:240-44:244, 44:249, 44:253-44:255, 44:257, 44:260-44:265, 44:267N, 44:269-44:270:02, 44:273-44:274, 44:279, 44:287-44:289, 44:291-44:293, 44:296-44:297, 44:301, 44:303N, 44:306N, 44:308, 44:311-44:316, 44:320-44:322, 44:326-44:327, 44:329-44:331, 44:334N-44:335, 44:337-44:338, 44:341N-44:342, 44:344, 44:437, 44:439, 44:441-44:442, 44:445-44:446, 44:448N-44:449N, 44:451, 44:453, 44:455, 44:457, 44:461-44:465, 44:467-44:468, 44:470, 44:473, 44:477-44:480Q, 44:482, 44:484, 45:01:01:01-45:01:09, 45:03-45:05, 45:07-45:13, 45:15-45:25, 57:24, 57:119, 83:01																				21															
*13:86, 35:02:01:01-35:02:16, 35:81, 35:83, 35:95, 35:129N, 35:154, 35:157, 35:172, 35:182-35:184, 35:211, 35:220, 35:258, 35:270, 35:273, 35:285, 35:309, 35:311, 35:323, 35:335, 35:339, 35:357, 35:361, 35:367, 35:372, 35:374, 35:377-35:379, 35:387, 35:391, 35:394, 35:396, 35:403, 35:410, 35:419, 35:443, 40:01:28, 41:49, 53:36																			w																
*14:01:05	1		3											14																					
*14:02:01:01-14:02:02, 14:02:04-14:02:05, 14:02:07, 14:02:09-14:02:19, 14:02:21-14:03, 14:05-14:06:02, 14:09, 14:15-14:17, 14:20, 14:22-14:23, 14:25, 14:27, 14:29-14:31, 14:33-14:35, 14:37-14:39, 14:44-14:45, 14:50-14:52, 14:58, 14:61-14:62, 14:64, 14:67-14:70Q, 14:74-14:76N, 14:79N, 14:84-14:85N			3				8							14																			31		
*14:02:08, 14:02:20, 14:60, 14:63, 14:72N			3				8							14																					
*14:04			3				w							14																				31	
*14:11			3				8		10					14																				31	
*14:13			3				8							14					19			23												31	
*14:14	1	2	3											14	15																				
*14:18			2	3			8							14																					
*14:28			3						10					14																				31	
*14:36, 14:57							8							14																				31	
*14:41N, 14:43, 14:56, 14:59			3				8							14																				31	
*14:48							8		10					14																					31
*14:53, 39:107	1		3											14					19			23												31	
*14:54	1													14						20														31	
*14:73		2				6	8							14																					
*15:03:01:01-15:03:04, 15:03:06-15:03:11, 15:47:01-15:47:02, 15:54, 15:62, 15:98, 15:103, 15:127, 15:131-15:132, 15:156, 15:158, 15:173, 15:210, 15:220:01:01-15:220:02, 15:242:01-15:242:02, 15:251:01-15:251:03, 15:253, 15:266, 15:274, 15:281-15:282, 15:369, 15:379, 15:389, 15:429, 15:444, 15:448, 15:475, 15:502, 15:517, 15:527, 15:545, 15:560			2			6	7							14																					
*15:03:05, 15:64:01-15:64:02		2				6								14																					
*15:09:01-15:09:02, 15:229, 15:252, 15:329, 15:435		2	3											14					19			23													
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32			
*15:10:01:01-15:10:05, 15:18:01:01-15:18:07, 15:23, 15:37, 15:72, 15:80, 15:90, 15:93, 15:99, 15:108, 15:119, 15:133-15:134, 15:153, 15:161, 15:197:01-15:198, 15:221, 15:226N, 15:238, 15:263, 15:290, 15:292-15:293, 15:306-15:307, 15:311, 15:313-15:314, 15:323, 15:335, 15:337-15:338, 15:351, 15:354, 15:380N, 15:382, 15:388, 15:414, 15:426-15:427, 15:433-15:434, 15:436, 15:439, 15:445, 15:447, 15:451, 15:455, 15:487N, 15:498, 15:503, 15:506, 15:526, 15:530, 15:548		2	3											14																					

Negative Control

