













Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45			
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45			
*13:02:17, 13:02:23, 13:08, 13:34-13:35, 13:72, 13:89, 13:121, 13:126-13:127, 13:159, 40:06:10, 40:06:19, 40:44, 40:131, 40:159, 40:307, 40:341																													29																43			
*13:02:34, 13:16, 13:19, 13:54, 13:75, 13:85, 13:115, 13:154, 15:42, 15:83:01:01-15:83:01:02, 27:130, 40:95, 40:103, 40:148, 40:161, 40:244, 40:374, 40:376, 40:492, 41:01:01:01-41:01:08, 41:05-41:07, 41:09, 41:12, 41:14, 41:16, 41:20, 41:25-41:26, 41:28-41:30, 41:32-41:35, 41:37, 41:44, 41:50, 41:53-41:54, 41:57, 41:59-41:61, 41:64, 41:66-41:67, 41:70, 41:73, 41:75-41:79, 46:18, 51:16, 51:21, 51:34, 51:36, 51:82, 51:143, 51:153:01-51:153:02, 51:295, 51:300, 51:376, 52:08, 52:37, 52:67, 54:12, 54:24, 55:01:17, 55:34, 55:47, 55:90, 55:103, 55:118N, 55:121, 55:129, 78:07, C*14:92																													29	31																		
*13:03, 13:120, 13:146, 15:552, 35:06, 35:59:01-35:59:02, 35:432, 40:71, 44:10, 44:339, 51:15, 56:33, 56:44, 56:67, 56:89, 59:04																												28		31																43		
*13:15																												27	29	31															43			
*13:18											11																		29	31															43			
*13:31, 13:41, 13:128, 13:131, 13:145											11																		29	31																		
*13:46, 27:14, 27:81, 41:21, 45:08, 45:10, 49:07, 49:76, 50:31, 50:45, 51:197, C*05:152, C*07:463, C*07:758, C*12:183																													29																			
*13:48, 15:460, 45:14, 46:11, 56:07-56:08, 56:14, 56:46, 56:65, 57:118, 58:73, 59:12																													28		31																	
*13:73											11																																					
*13:86, 35:02:01:01-35:02:05, 35:02:07-35:02:19, 35:81, 35:83, 35:95, 35:129N, 35:154, 35:157, 35:172, 35:182-35:184, 35:211, 35:220, 35:258, 35:270, 35:273, 35:285, 35:309, 35:311, 35:323:01-35:323:02, 35:335, 35:339, 35:357, 35:361, 35:367, 35:372, 35:374, 35:377-35:379, 35:387, 35:391, 35:396, 35:403, 35:410, 35:419, 35:443, 35:483, 35:492, 35:501, 35:505, 35:508N, 35:522, 35:550, 35:553, 35:562, 35:565, 40:01:28, 40:305, 53:19, 53:36																																																
*13:96													13																29	31																	43	
*13:140, 40:86							8																						29	31																	43	
*13:158																													29	w																	43	
*13:164, 15:58, 15:73:01:01-15:73:01:02, 35:413, 46:61											11																	28		31																		
*14:01:01:01-14:01:05, 14:01:07, 14:01:09-14:01:13, 14:07N, 14:12, 14:19, 14:26, 14:32, 14:40, 14:47, 14:65, 14:71, 14:77-14:78, 14:80-14:82, 14:86-14:87, 14:103-14:105, 14:108Q, 14:112-14:113N		2			5														19				23	24		26																						
*14:01:06, 14:02:01:01-14:02:02, 14:02:04, 14:02:07-14:02:08, 14:02:10-14:02:19, 14:02:21-14:03, 14:09, 14:11, 14:15-14:17, 14:20, 14:22-14:23, 14:25, 14:27, 14:29-14:31, 14:33-14:35, 14:38-14:39, 14:41N, 14:44-14:45, 14:50-14:52, 14:58, 14:61, 14:64, 14:67-14:70Q, 14:74-14:76N, 14:79N, 14:84-14:85N, 14:89-14:92, 14:94, 14:98, 14:100N-14:101N, 14:106-14:107, 14:109-14:110		2			5													19				23		26																								
*14:01:08, 14:54					5													19				23	24		26																							
*14:02:03, 14:02:06		2																19						26																								
*14:02:05		2																19				23		26																								
*14:02:09, 14:05-14:06:02, 14:62		2			5													19				23																										
*14:02:20, 14:60, 14:72N		2			5													19						26																								
*14:04		2			5													19				23		26																								
*14:13		2			5													19				23		26																								
*14:14		2	3		5													19					24		26																							
*14:18		2	3		5													19						26																								
*14:21													14					19						26																								
*14:24		2			5													19				23		26																								
*14:28		2			5		8							15				19				23		26																								
*14:36, 14:48, 14:99					5													19				23		26																								
*14:37		2			5													19				23		26					29																			
*14:42:01																		19						26																								
*14:42:02				4	5													19				23		26																								
*14:43, 14:59		2			5													19				23		26																								
*14:46		2			5		8											19				23	24		26																							
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45			
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23																									











