

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
HLA-B allele ^{1,2,3}																								
*41:01:01-41:01:04, 41:20, 41:25-41:26, 41:30, 41:34-41:35, 41:37, 41:44, 41:50	1					6		8		10	11										21	22		
*41:02:01-41:02:03, 41:02:05-41:02:07, 41:18, 41:31, 41:38-41:42		2						8		10		12										22		
*41:02:04		2						8		10		12												
*41:03:01			3					8		10		12										22		
*41:03:02								8		10		12										22		
*41:04, 41:36 ⁵		2		4				8		10		12										22		
*41:05	1				5	6		8		10	11										21	22		
*41:06						6	7	8			11										21	22		
*41:07	1			4		6		8		10	11										21	22		
*41:08								8	9			12										22		
*41:09						6	7	8			11		13								21	22		
*41:10, 41:45N ⁵		2	3					8		10		12										22		
*41:11		2						8		10		12							19			22		
*41:12	1			4		6		8		10	11										21			
*41:13		2						8		10		12		14								22		
*41:14	1					6		8		10	11				15						21	22		
*41:15		2						8				12				16						22		
*41:16	1					6		8		10	11			14							21	22		
*41:17	1					6		8		10											21	22		
*41:19		2						8		10		12						17				22		
*41:21	1					6		8		10	11											22		
*41:22	1		3			6		8		10	11										21	22		
*41:23		2			5					10		12												
*41:24		2						8		10		12						18				22		
*41:27		2			5			8		10		12										22		
*41:28	1					6		8		10	11							19			21	22		
*41:29						6		8			11								20		21	22		
*41:32						6		8			11										21	22		
*41:33	1					6				10	11										21	22		
*41:43		2						8		10		12										22	23	
*41:46-41:47, 44:166 ⁴		2								10		12										22		
*41:48								8		10												22		
*41:49		2						8		10		12		14			17					22		
*07:04, 07:25, 07:146, 42:05:01-42:05:02		2							9			12												
*07:19, 08:01:21, 08:01:25, 08:31, 08:43, 08:103, 08:113, 08:169		2										12												
*07:33:01, 07:53, 07:60, 07:139, 13:71, 15:421, 27:07:01-27:07:05, 27:34, 27:43, 35:31, 40:02:03, 40:08, 40:220, 40:322, 48:04:01-48:04:02, 48:13, 48:24, 51:81, 51:148, 55:04, 55:49, 55:51, 58:27, C*02:60, C*03:05, C*03:25, C*03:27, C*03:167, C*03:296, C*03:335, C*08:08:01-08:08:02															16									
*07:68:01-07:68:03, 07:237, 07:262, 48:12, 48:14, 48:29																		18						
*07:100, 14:05, 14:13, 14:53, 15:07:01-15:07:03, 15:45, 15:55, 15:68, 15:126, 15:197:02, 15:207, 15:324, 15:329, 15:331, 15:360, 15:383, 15:405, 27:32, 35:05:01:01-35:05:03, 35:51, 38:19, 39:03, 39:14, 39:24:01-39:24:02, 39:29, 39:37, 39:76, 39:120, 46:12, 51:64, 53:14, 58:18															16		18							
*07:143, 39:31			3		5																			
*08:01:01:01-08:01:20, 08:01:22-08:01:24, 08:01:26-08:01:31, 08:01:33-08:05, 08:07-08:08N, 08:10-08:11, 08:14-08:15, 08:17-08:19N, 08:21-08:24, 08:26:01-08:26:03, 08:28-08:30N, 08:32, 08:34-08:39, 08:41-08:42, 08:44-08:48, 08:50-08:54:02, 08:56:01-08:59:02, 08:61, 08:63-08:69, 08:72N-08:75, 08:77-08:78, 08:80-08:83, 08:85-08:88, 08:90-08:95, 08:97-08:102, 08:105-08:106, 08:108-08:112, 08:114-08:115, 08:117-08:128, 08:130-08:133, 08:135-08:142, 08:144-08:153, 08:155, 08:157-08:164, 08:166-08:168, 08:170, 08:172-08:174, 35:87, 42:01:01-42:01:03, 42:06-42:07, 42:11-42:12, 42:15-42:16, 42:20-42:24		2								10		12												
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
*08:01:32, 08:27, 08:33, 08:76, 08:104, 08:154		2								10														
*08:09, 08:84, 42:04	1					6				10	11													
*08:12:01-08:12:02, 08:16, 08:89										10		12												
*08:12:03, 08:49, 08:60, 08:129, 42:13										10														
*08:40, 08:171, 27:83																	17							
*08:62, 42:02:01:01-42:02:01:02, 42:09, 42:18		2	3		5					10		12												
*08:71, 42:10		2								10		12						18						
*08:96		2		4						10		12												
*08:107				4						10														

Negative Control

*35:317, 40:01:04-40:01:08, 40:01:10, 40:01:13-40:01:15, 40:01:18-40:01:24, 40:01:28-40:01:29, 40:01:38, 40:14:01-40:14:02, 40:15-40:16, 40:20, 40:25-40:26, 40:28, 40:30-40:31, 40:33-40:34, 40:36, 40:38, 40:42, 40:45-40:48, 40:59-40:60, 40:62, 40:65, 40:69, 40:74, 40:76, 40:80, 40:84, 40:92, 40:100-40:102, 40:108, 40:112, 40:114:02, 40:116-40:118N, 40:120-40:121, 40:124:02-40:126, 40:130:01, 40:132, 40:134-40:135, 40:156, 40:160:01-40:160:02, 40:163, 40:168, 40:183-40:184, 40:187-40:188, 40:192, 40:194-40:199, 40:217-40:218, 40:234-40:235, 40:240, 40:245, 40:273, 40:280-40:281, 40:288, 40:298, 40:308, 40:312, 40:315, 40:319, 40:326-40:327, 40:336, 40:339, 40:352								8														22	?	
*37:09		2	3					8				12				16								
*39:90, 55:67, C*07:463										11		13												
*40:01:42								8														22	23	
*40:02:01:01-40:02:02, 40:02:05-40:03, 40:05:01:01-40:05:01:02, 40:19, 40:29, 40:39, 40:56-40:57, 40:82, 40:85, 40:89, 40:91, 40:97, 40:122, 40:142N-40:144N, 40:174, 40:176, 40:205, 40:211, 40:214, 40:254-40:255, 40:266-40:267, 40:271, 40:284, 40:302-40:305, 40:309, 40:314, 40:331, 40:345N, 40:347								8								16						22		
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
*40:02:04, 40:94, 40:209								8								16								
*40:06:01:01-40:06:02, 40:06:04, 40:06:06-40:06:08, 40:53, 40:70:01-40:70:02, 40:86, 40:159, 40:161, 40:167, 40:230, 40:268-40:270, 40:307, 40:317, 49:01:01-49:02, 49:04:01-49:09, 49:11-49:13, 49:16-49:23, 49:27-49:30, 49:32-49:46, 50:01:01:01-50:02, 50:04-50:10, 50:14-50:16, 50:18-50:20, 50:31-50:42, 50:44, 50:46-50:48, 50:50							7	8				13										22		
*40:06:03							7	8				13										22	23	
*40:06:05, 40:06:09-40:06:14, 40:06:16, 40:44, 40:93, 40:95-40:96, 40:109, 40:127, 40:131, 40:148, 40:165, 40:177, 40:190, 40:244, 40:256N, 40:275, 40:294-40:295, 40:306, 40:311, 40:313, 40:330, 40:340-40:341, 40:348-40:350							7	8				13										22	?	
*40:06:15, 40:110, 40:318							7					13										22	?	
*40:09, 40:18, 40:24, 40:27:01, 40:37, 40:78, 40:90, 40:104-40:105, 40:107, 40:111, 40:115, 40:119, 40:133Q, 40:145, 40:157, 40:169, 40:173, 40:180-40:181, 40:189, 40:201-40:203, 40:206, 40:224, 40:246, 40:274, 40:276, 40:283, 40:287, 40:289-40:292, 40:296-40:297, 40:320, 40:334-40:335, 40:337N								8								16						22	?	
*40:13, 40:219																16						22	?	
*40:35:01-40:35:02, 40:71								8								16		18				22	?	
*40:40, 40:58, 40:164, 40:229																16						22		
*40:43					5			8														22		
*40:50					5			8								16						22		
*40:79								8							15							22		
*40:83							7	8					13										?	
*40:129, 40:191, 40:328																						22	?	
*40:136, 40:231, 40:279								8	9			12										22	?	
*40:149			3					8														22	?	
*40:166					5																		?	
*40:170, 40:185, 40:310								8															?	
*40:248				4				8								16							?	
*40:282, 40:351				4				8														22	?	
*40:293								8								16			19			22	?	
*40:325			4																			22	?	
*40:342, 44:47, 44:197, 49:10, 49:24-49:26, 49:31, 50:13, 50:43, 51:103, 51:203							7						13									22		
*42:17		2	3		5					10		12							18					
*44:02:33, 44:41:01, 44:230												12										22		
*44:15, 44:18, 45:01:01-45:04, 45:07, 45:09-45:13, 45:15-45:19						6	7	8					13									22		
*44:20, 44:100						6	7						13									22		
*44:34:02, 44:245, 53:30			4																			22		
*44:54, 44:135, 44:184																16		18				22		
*44:80, 44:219																			19			22		
*44:106, 44:158, 44:213		2										12				16		18				22		
*45:05						6		8					13									22		
*45:06					5	6	7						13									22		
*45:08					6	7	8			11			13									22		

Negative Control

