

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
HLA-B allele ^{1,2,3}																								
*41:01:01:01-41:01:07, 41:20, 41:25-41:26, 41:30, 41:34-41:35, 41:37, 41:44, 41:50, 41:53-41:54, 41:57, 41:59-41:61, 41:64, 41:67, 41:70, 41:73, 41:75	1					6		8		10	11										21	22		
*41:02:01:01-41:02:03, 41:02:05-41:02:11, 41:18, 41:31, 41:38-41:42, 41:51-41:52, 41:55, 41:58, 41:65, 41:68-41:69, 41:72, 41:74	2							8		10		12											22	
*41:02:04, 41:71	2							8		10		12												
*41:03:01			3					8		10		12									20		22	
*41:03:02								8		10		12									20		22	
*41:04, 41:36 ⁴		2		4				8		10		12											22	
*41:05	1				5	6		8		10		11										21	22	
*41:06					6	7	8					11										21	22	
*41:07	1			4	6			8		10		11										21	22	
*41:08, 40:136, 40:231, 40:279 ⁵								8	9			12									20		22	
*41:09					6	7	8				11		13									21	22	
*41:10, 41:45N ⁴		2	3					8		10		12										22		
*41:11		2						8		10		12							19			22		
*41:12	1			4	6			8		10		11										21		
*41:13		2						8		10		12		14									22	
*41:14	1				6			8		10		11			15							21	22	
*41:15		2						8				12				16							22	
*41:16	1				6			8		10		11		14								21	22	
*41:17, 41:56Q	1				6			8		10												21	22	
*41:19		2						8		10		12					17					22		
*41:21, 41:66	1				6			8		10		11										22		
*41:22	1	3			6			8		10		11										21	22	
*41:23, 42:27 ⁵		2			5					10		12												
*41:24		2						8		10		12						18				22		
*41:27		2			5			8		10		12										22		
*41:28	1				6			8		10		11							19			21	22	
*41:29					6			8				11		15								21	22	
*41:32					6			8				11										21	22	
*41:33	1				6					10		11										21	22	
*41:43		2						8		10		12										22	23	
*41:46-41:47, 44:166 ⁵		2								10		12										22		
*41:48								8		10												22		
*41:49		2						8		10		12		14			17					22		
*41:62, 40:02:20 ⁵		2						8								16						22		
*41:63								8		10		12										22		
*07:02:13, 07:02:35, 07:05:10, 07:22:02, 07:151:02, 07:209, 08:13, 08:20:01-08:20:02, 08:25, 08:55, 08:70, 08:143, 08:281, 42:19, C*07:294, C*07:526:01, C*07:530, C*07:681	2																							
*07:04:01-07:04:02, 07:25, 07:146, 07:349, 08:187, 42:05:01-42:05:02	2								9			12												
*07:19, 08:01:21, 08:01:25, 08:01:62, 08:31, 08:43, 08:103, 08:113, 08:169	2											12												
*07:33:01, 07:53, 07:60, 07:139:01, 07:442, 13:71, 15:421, 15:524, 27:07:01-27:07:06, 27:34, 27:43, 27:219, 27:233, 27:251, 35:31, 35:114, 40:02:03, 40:08:01:01-40:08:01:02, 40:27:02, 40:200, 40:220, 40:226, 40:232, 40:322, 40:457, 40:481N, 48:04:01:01-48:04:02, 48:13, 48:24, 48:51, 51:81, 58:27, C*02:60, C*03:05:01:01-03:05:01:03, C*03:25, C*03:27, C*03:167, C*03:296:02, C*03:335, C*03:386, C*03:407, C*03:482, C*03:494, C*03:555, C*04:443, C*06:264, C*08:08:01-08:08:02																16								
*07:68:01, 07:68:03, 07:237, 07:262:02, 48:14, 48:29																		18						
*07:68:02, 07:262:01, 07:424, 35:66, 48:12		2																18						
*07:100, 15:07:01:01-15:07:02, 15:45, 15:55, 15:68, 15:126, 15:197:02, 15:207, 15:324, 15:329, 15:331, 15:360, 15:405, 15:431, 15:450, 27:32, 27:180, 46:12																16		18						
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
*07:143			3		5																			
*07:335, 15:426, 18:07:01-18:07:03:02, 18:33, 18:79, 18:136, 18:155, 18:182N, 18:218, 27:75, 35:162, 37:21, 37:26, 37:57, 37:60, 37:67, 37:79N, C*06:77			3																					
*07:388, 27:83																	17							

Negative Control

*08:01:01:01-08:01:20, 08:01:22-08:01:24, 08:01:26-08:01:31, 08:01:33-08:01:43, 08:01:45-08:01:61, 08:01:63-08:05, 08:07-08:08N, 08:10-08:11, 08:14-08:15, 08:17-08:19N, 08:21-08:24, 08:26:01-08:26:03, 08:28-08:30N, 08:32, 08:34-08:39, 08:41-08:42, 08:44-08:48, 08:50-08:54:02, 08:56:01-08:59:02, 08:61, 08:63-08:69, 08:72N-08:75, 08:77-08:78, 08:80-08:83, 08:85-08:88, 08:90-08:95, 08:96:02-08:102, 08:105-08:106, 08:108-08:112, 08:114-08:115, 08:117-08:128, 08:130-08:133, 08:135-08:142, 08:144-08:153, 08:155, 08:157-08:164, 08:166-08:168, 08:170, 08:172-08:174, 08:176, 08:178-08:180, 08:182-08:183, 08:185-08:186, 08:188-08:203, 08:207-08:225, 08:227-08:231, 08:233-08:241, 08:243-08:253, 08:256-08:273, 08:275-08:280, 08:282-08:290, 08:292, 35:87, 42:01:01:01-42:01:05, 42:06-42:07, 42:11-42:12, 42:15-42:16:02, 42:20-42:25, 42:28-42:32	2									10	12													
*08:01:32, 08:27, 08:33, 08:76, 08:104, 08:154, 08:177, 08:204, 08:206	2									10														
*08:01:44	2									10	12									23				
*08:09, 08:84, 42:04	1					6				10	11													
*08:12:01-08:12:02, 08:16										10	12								20					
*08:12:03										10									20					
*08:40, 08:171, 08:205	2														17									
*08:49, 08:60, 08:129, 08:181, 08:184, 08:232, 42:13										10														
*08:62, 08:226, 08:242, 42:02:01:01-42:02:02, 42:09, 42:18, 42:26	2	3			5					10	12													
*08:71, 42:10	2									10	12				18									
*08:89, 08:291										10	12													
*08:96:01	2		4							10	12													
*08:107			4							10														
*08:116, 08:134, 08:175, 42:08, 42:14	2										12			16										
*08:165	2									10	12			17										
*08:254	1				6					10	11									21				
*08:274	2										12				18									
*13:02:01:01-13:02:23, 13:02:25-13:04, 13:08-13:09, 13:14-13:16, 13:18-13:19, 13:27, 13:31-13:35, 13:37-13:38, 13:40-13:42, 13:44-13:45, 13:47-13:49N, 13:54, 13:56:01N-13:56:02N, 13:58-13:59, 13:62, 13:65-13:70, 13:72, 13:74-13:75, 13:81, 13:84-13:85, 13:88-13:89, 13:91, 13:93-13:94, 13:96-13:105, 13:111-13:121, 13:123Q-13:128, 13:130, 13:132-13:134, 13:136-13:137N, 13:139N-13:140, 13:143, 13:145, 13:147, 13:149-13:153, 13:155-13:160, 13:164-13:166, 13:169, 15:04:01:01-15:04:03, 15:16:01:01-15:16:05, 15:67, 15:95, 15:137, 15:155, 15:222, 15:254, 15:293, 15:303, 15:310, 15:362, 15:499, 15:517, 15:531, 15:550, 15:552, 15:602, 15:609, 15:613, 27:14, 27:81, 27:130, 35:37, 35:235, 35:265, 35:273, 35:534, 40:75, 40:103, 40:162, 40:388, 40:492, 46:11, 46:32:01-46:32:02, 51:01:01:01-51:01:62, 51:01:64-51:01:65, 51:01:67-51:01:69, 51:01:71-51:01:74, 51:01:76-51:03, 51:05:01:01-51:05:01:02, 51:07:01-51:07:02, 51:09:01-51:19, 51:22-51:24:05, 51:26-51:35, 51:37-51:41N, 51:43, 51:48-51:55, 51:57-51:58, 51:60-51:61:02, 51:63:01-51:63:02, 51:65-51:80, 51:82-51:95, 51:98N-51:100, 51:102, 51:104:01-51:115, 51:117-51:134, 51:136-51:138, 51:140, 51:142, 51:144-51:147, 51:150-51:152, 51:154-51:156, 51:158:01-51:193:02, 51:195-51:196, 51:198-51:202, 51:204:01-51:212, 51:214-51:219, 51:221-51:226, 51:228-51:239, 51:241-51:262, 51:264N-51:273N, 51:277-51:279, 51:282-51:292, 51:294, 51:296-51:298, 51:301-51:309, 51:311-51:318N, 51:320-51:339, 51:343-51:351, 51:355-51:358, 52:01:01:01-52:18, 52:20-52:36, 52:38-52:44, 52:46-52:50, 52:52-52:70, 52:72-52:83, 52:85-52:87, 52:89N-52:91, 52:93-52:105, 54:03, 54:26, 54:33, 54:40, 55:09, 55:22, 55:24, 56:01:01:01-56:01:07, 56:01:09-56:01:17, 56:05:01-56:08, 56:14-56:17, 56:19N-56:22, 56:24-56:30, 56:33-56:34, 56:36-56:50, 56:52-56:53, 56:55:01:02, 56:58-56:59, 56:61-56:62, 56:64-56:65, 56:68-56:86, 58:08:01-58:08:02, 58:122, 59:04, 59:09, 59:12, 73:01:01:01-73:03, 78:01:01:02-78:06, 78:08-78:10, C*03:464							7					13												
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
*13:02:24, 13:82, 51:01:63, 51:01:70, 51:240													13											
*13:23, 15:71, 15:175, 15:204, 15:225, 15:344, 15:381, 18:80, 18:98:01-18:98:02, 18:112, 18:120, 37:22:01-37:22:03, 37:41, 37:95							8																	
*13:30							7						13						19					
*13:46, 15:83:01:01-15:83:01:02, 51:36, 51:153, 55:56						6	7				11		13											

Negative Control

*13:53, 13:131, 15:42, 15:485, 35:60, 35:276, 46:18, 51:21, 51:101, 51:143, 51:157, 51:197, 51:280, 51:295, 51:300, 51:340, 52:37, 52:71, 52:84, 54:01:01:01-54:01:09, 54:04-54:05N, 54:07-54:08N, 54:10-54:25, 54:27-54:32, 54:34-54:37, 54:39, 54:41-54:44, 55:01:01:01-55:01:06, 55:01:08-55:03, 55:05, 55:07, 55:10-55:13, 55:15, 55:17-55:19, 55:21, 55:23, 55:25-55:48, 55:50, 55:52-55:55N, 55:57-55:66, 55:68-55:85, 55:87-55:109, 55:111-55:123, 56:23, 56:54, 59:01:01:01-59:03, 59:05-59:08, 59:10N-59:11, 78:07, C*05:152, C*12:183, C*14:92							7			11	13													
*13:55, 40:83, 40:362, 49:03, 49:69, 50:01:10, 52:51							7	8			13													
*13:108, 15:07:03, 15:383, 35:05:01:01-35:05:06, 35:22:01:01-35:22:01:02, 35:51:01-35:51:02, 35:58, 35:72, 35:89, 35:97, 35:199, 35:232, 35:368, 35:389, 35:416, 35:427N, 35:440, 35:459N, 35:472, 35:481, 35:498-35:499, 35:509, 35:519, 35:541, 51:64, 53:14, 53:67, 55:110, 58:18		2										16	18											
*13:154, 51:08:01:01-51:08:06, 51:20, 51:44N, 51:97, 51:141, 51:263, 51:274-51:275, 51:342, 52:19, 56:13							6	7			13													
*13:167, 40:06:15, 40:110, 40:318, 40:342, 40:449, 44:47, 44:197, 44:339, 44:459, 49:10, 49:24-49:26, 49:31, 50:13, 50:43, 51:103, 51:203, 51:276, 52:106							7				13									22				
*14:01:01:01-14:04, 14:06:01-14:07N, 14:10, 14:12, 14:31, 14:33-14:34, 14:41N, 14:42:02, 14:46, 14:49, 14:54, 14:59, 14:61-14:109, 35:01:25, 35:35, 35:471, 38:01:01:01-38:02:11, 38:05-38:06, 38:09, 38:11-38:12, 38:14, 38:17-38:18, 38:20:02, 38:24, 38:28, 38:35, 38:46-38:50, 38:55Q, 38:58, 38:60, 38:67-38:72, 38:74-38:97, 38:160-38:174, 39:01:01:01-39:01:01:25, 39:01:03:01-39:02:03, 39:04-39:05:05, 39:07-39:13:02, 39:15, 39:19:01, 39:20, 39:22-39:23, 39:25N-39:26, 39:28, 39:36, 39:38Q-39:39:01, 39:40:01N-39:42, 39:44-39:47, 39:49, 39:58-39:60, 39:63, 39:77, 39:84-39:88, 39:92, 39:97N-39:98, 39:103-39:104, 39:119, 39:121-39:123, 39:130-39:131, 39:134-39:139N, 39:142N-39:143, 39:145-39:151, 39:153, 39:155, 39:157N-39:177, 39:180-39:181, 39:183, 58:20, 67:01:01-67:04, 67:06-67:07																				23				
*14:05, 14:53, 39:03:01:01-39:03:01:03, 39:14:01:01-39:14:01:03, 39:24:01-39:24:03, 39:37:01:01-39:37:01:02, 39:144, 39:152, 39:182, 39:184-39:185		2										16	18								23			
*14:08:01-14:09, 14:11, 14:14-14:30, 14:32, 14:35-14:36, 14:38-14:40, 14:42:01, 14:43-14:45, 14:47-14:48, 14:50-14:52, 14:55, 14:57-14:58, 14:60, 38:03-38:04, 38:07-38:08, 38:10, 38:13, 38:15-38:16, 38:20:01, 38:21, 38:23, 38:25-38:27, 38:29, 38:31-38:34N, 38:36-38:45, 38:51-38:54, 38:56-38:57, 38:59, 38:61-38:66, 39:16-39:18, 39:19:02, 39:27, 39:30, 39:32, 39:35, 39:39:02, 39:48, 39:51-39:56, 39:61, 39:65-39:69, 39:71-39:75, 39:78-39:82, 39:89, 39:91, 39:93-39:96, 39:99-39:100, 39:105-39:108, 39:110, 39:112-39:115, 39:117-39:118, 39:124-39:125, 39:133N																					?			
*14:13, 38:19, 39:29, 39:76, 39:120		2										16	18								?			
*14:37, 39:57, 39:83:01, 39:109, 39:127							7			11	13										?			
*15:04:04, 39:33, 39:50							7				13										23			
*15:09:01, 15:30:01:01-15:30:01:03, 15:48, 15:150, 15:188, 15:243, 15:252, 15:361, 15:435, 15:541, 35:44, 35:251:01-35:251:02, 35:292, 35:367, 35:535, 40:01:39, 40:07, 40:12:01-40:12:02, 40:21, 40:61, 40:106, 40:252, 40:365, 40:428N, 40:462, 46:74, 48:03:01:01-48:03:02, 48:40, 51:06:01:01-51:06:04, 51:59, 51:194, 51:293, 51:341, 51:352, 55:08, 57:82, C*04:89, C*04:127, C*04:135, C*04:210, C*04:237, C*04:379, C*05:47, C*05:84, C*05:139, C*05:180N, C*05:182, C*05:193, C*05:236, C*05:240, C*05:256, C*07:775, C*08:208N																20								
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
*15:09:02, 38:73, 39:116N																				20			23	
*15:46, 15:106, 35:47, 35:154, 35:450, 35:530, 44:269, 53:15, 57:45, 58:111				4																				
*15:51, 15:179:01-15:179:02, 15:199, 15:218Q, 15:518, 15:578, C*07:04:03, C*07:199:02												12												

Negative Control

*40:01:01-40:01:17, 40:01:19-40:01:37, 40:01:40-40:01:41, 40:01:43-40:01:46, 40:01:48-40:01:50, 40:01:52-40:01:73, 40:01:75-40:01:76, 40:11:01:01-40:11:02, 40:14:01-40:14:03, 40:22N-40:23, 40:25-40:26, 40:46-40:47, 40:51, 40:54-40:55, 40:62-40:63, 40:65-40:67, 40:69, 40:72:01-40:74, 40:76, 40:81, 40:84, 40:87:01-40:87:02, 40:92, 40:100-40:102, 40:108, 40:112-40:114:02, 40:116-40:118N, 40:121, 40:123, 40:125:01-40:125:03, 40:128, 40:130:01-40:130:02, 40:132, 40:134-40:135, 40:138, 40:140-40:141, 40:146-40:147, 40:150-40:156, 40:163, 40:168, 40:171-40:172, 40:175, 40:178-40:179, 40:182-40:183, 40:186:01-40:188, 40:192-40:199, 40:204, 40:207-40:208, 40:212-40:213, 40:216N-40:218, 40:221, 40:223, 40:227-40:228, 40:234-40:239, 40:241-40:242, 40:245:01-40:245:02, 40:247, 40:249-40:251, 40:253, 40:257-40:265N, 40:272-40:273, 40:277-40:278, 40:281, 40:285-40:286N, 40:288, 40:299-40:301, 40:308, 40:312, 40:315, 40:319, 40:321, 40:323:01-40:324, 40:326-40:327, 40:329, 40:332-40:333, 40:336, 40:338N-40:339, 40:344, 40:346, 40:353:01:01-40:353:02, 40:355, 40:357-40:358, 40:360, 40:367, 40:370, 40:372N, 40:379-40:383, 40:385-40:386, 40:391-40:396, 40:398, 40:400-40:402, 40:404, 40:406-40:407, 40:416-40:419, 40:421Q, 40:424, 40:426N, 40:431, 40:433, 40:436, 40:439, 40:443, 40:446-40:448, 40:450, 40:452, 40:456, 40:461, 40:463, 40:466, 40:474-40:475:02, 40:477, 40:479-40:480, 40:483N, 40:485-40:486, 40:490-40:491, 40:495, 40:497, 40:499, 40:501, 40:506N-40:510N								8											20	22				
*40:01:42								8											20	22	23			
*40:01:51, 40:149			3					8											20	22				
*40:02:01:01-40:02:02, 40:02:05-40:02:19, 40:02:21-40:03:02, 40:05:01:01-40:05:02, 40:09, 40:18-40:19, 40:24, 40:27:01, 40:27:03, 40:29, 40:37, 40:39, 40:56-40:57, 40:78, 40:82, 40:85, 40:89-40:91, 40:97, 40:104-40:105, 40:107, 40:111, 40:115, 40:119, 40:122, 40:133Q, 40:142N-40:145, 40:157, 40:169, 40:173-40:174, 40:176, 40:180-40:181, 40:189, 40:201-40:203, 40:205-40:206, 40:211, 40:214, 40:224, 40:246, 40:254-40:255, 40:266-40:267, 40:271, 40:274, 40:276, 40:283-40:284, 40:287, 40:289-40:292, 40:296-40:297, 40:302-40:305, 40:309, 40:314, 40:320, 40:331, 40:334:01-40:335, 40:337N, 40:345N, 40:347, 40:356, 40:359, 40:369, 40:371, 40:375, 40:378, 40:384, 40:390, 40:405, 40:409-40:410, 40:412-40:415, 40:420, 40:432, 40:435, 40:437-40:438N, 40:440-40:442, 40:444, 40:455, 40:465, 40:467, 40:470, 40:489, 40:493, 40:496, 40:500, 40:504-40:505, 40:511N								8						16						22				
*40:02:04, 40:94, 40:209								8						16										
*40:06:01:01-40:06:02, 40:06:04:01-40:06:14, 40:06:16-40:06:29, 40:44, 40:53, 40:70:01-40:70:02, 40:86, 40:93, 40:95-40:96, 40:109, 40:127, 40:131, 40:148, 40:159, 40:161, 40:165, 40:167, 40:177, 40:190, 40:230, 40:244, 40:256N, 40:268-40:270, 40:275, 40:294-40:295, 40:306-40:307, 40:311, 40:313, 40:317, 40:330, 40:340-40:341, 40:348-40:350, 40:354, 40:361N, 40:363, 40:373-40:374, 40:376, 40:387, 40:389, 40:408, 40:411, 40:423, 40:425, 40:427, 40:434, 40:445, 40:451, 40:453-40:454, 40:459-40:460, 40:464, 40:468, 40:471, 40:473, 40:476, 40:478, 40:482, 40:484, 40:487N-40:488N, 40:494, 49:01:01:01-49:01:11, 49:01:13-49:01:21, 49:02, 49:04:01-49:09, 49:11-49:13, 49:16-49:23, 49:27-49:30, 49:32-49:46, 49:49-49:51, 49:53-49:60N, 49:62-49:66, 49:68, 49:70-49:75, 50:01:01:01-50:01:09, 50:01:11-50:02:01:02, 50:04:01-50:10, 50:14-50:16, 50:18-50:20, 50:31-50:42, 50:44, 50:46-50:48, 50:50, 50:53-50:76, 50:78-50:79							7	8					13							22				
*40:06:03							7	8					13								22	23		
*40:10:01:01-40:10:02			4																20					
*40:13, 40:40, 40:58, 40:164, 40:219, 40:229, 40:399N															16						22			
*40:35:01-40:35:02								8							16	18					22			
*40:35:03, 40:71			2					8							16	18					22			
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
*40:43					5			8													20	22		
*40:49, 40:191, 40:210, 40:328, 44:31								8													20	22		
*40:50:01:01-40:50:01:02					5			8								16						22		
*40:79								8							15						20	22		
*40:170, 40:185, 40:215, 40:310, 40:397, 40:430								8													20			
*40:248				4				8								16								
*40:282, 40:351				4				8													20	22		
*40:293								8								16			19			22		
*40:325				4																	20	22		
*40:364				4				7	8															
*40:366								7	8				13									21	22	
*40:503								8													19	20	22	

Negative Control

*42:17	2	3		5					10	12				18					22			Negative Control		
*44:02:33, 44:41:01, 44:230										12									22					
*44:05:05																			22	23				
*44:15:01:01-44:15:01:02, 44:18:01:01-44:18:01:02, 45:01:01:01-45:04, 45:07, 45:09-45:13, 45:15-45:25, 45:27-45:28N					6	7	8					13							22					
*44:20, 44:100					6	7						13							22					
*44:34:02, 44:245, 53:30			4																22					
*44:54, 44:135, 44:184	2													16	18				22					
*44:80, 44:219															19				22					
*44:106, 44:158, 44:213	2										12			16	18				22					
*44:342									10										22					
*45:05					6		8					13							22					
*45:06				5	6	7						13							22					
*45:08					6	7	8			11		13							22					
*45:14					6	7	8					13							21	22				
*49:01:12, 49:01:22, 50:11							8					13							22					
*49:14, 49:52						7	8												22					
*49:48			4			7	8					13							22					
*49:76						7	8			11									22					
*50:45, 50:49						7	8			11		13							22					
*50:52				5		7	8					13							22					
*50:77			3			7	8					13												
*51:116			3			7						13												
*51:148, 55:04, 55:49, 55:51, 55:86, C*03:296:01	2													16										
*51:354			3			6	7					13												
*54:02, 55:16				5		7				11		13												
*55:20	1		4		6					10	11													
*55:67, C*07:463, C*07:758											11		13											
*56:67			4			7						13												
C*07:714, C*07:724			3	w																				
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24