

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16
HLA-B allele^{1,2}																
*45:01:01-45:01:04, 45:15-45:21, 45:23	1	2	3													
*45:02	1	2			5											
*45:03	1	2		4												
*45:04		2	3			6										
*45:05	1	2	3				7									
*45:06	1	2	3					8								
*45:07	1	2	3	4												
*45:08		2	3			6			9							
*45:09	1	2								10						15
*45:10		2	3			6					11					
*45:11	1	2	3									12				
*45:12	1	2	3										13			
*45:13	1	2	3												14	
*45:14	1	2	3													15
*45:22, 44:15:01:01-44:15:01:02, 44:18 ³	1		3													
*07:20, 07:140, 07:153, 15:14, 15:91, 15:131, 15:161, 35:45, 35:71, 35:169, 44:10, 44:14, 44:31, 44:43:01-44:43:02, 44:45, 44:51, 44:62, 44:77, 44:82, 44:107, 44:123, 44:135, 44:184, 44:213, 46:17, 57:07, 57:93, 58:07																15
*07:78, 07:84, 13:02:01:01-13:02:24, 13:08-13:09, 13:14-13:16, 13:18-13:19, 13:27, 13:30-13:35, 13:37-13:38, 13:40-13:42, 13:44-13:45, 13:47, 13:49N, 13:54-13:56:02N, 13:58-13:59, 13:65-13:70, 13:72, 13:74-13:75, 13:81-13:82, 13:84-13:85, 13:88-13:89, 13:91, 13:93-13:94, 13:96-13:105, 13:111-13:119, 13:121, 13:123Q, 27:14, 27:81, 27:130, 39:50, 40:93, 40:96, 40:109-40:110, 40:340, 40:350, 49:07, 51:10, 51:16, 51:31, 51:34, 51:82, 51:93, 52:08, 52:67, 54:26, 55:09, 55:22, 55:24, 59:09, 73:01-73:02, 78:08						6					11					
*07:133, 40:166, 44:90		2						8								
*07:225, 13:86, 35:02:01:01-35:02:07, 35:02:09-35:02:11, 35:81, 35:83, 35:95, 35:129N, 35:146, 35:157, 35:162, 35:172, 35:182-35:184, 35:211, 35:220, 35:258, 35:270, 35:285, 35:311, 35:323, 35:335, 35:339, 35:357, 35:361, 35:366-35:367, 35:372, 35:374, 35:377-35:379, 35:387, 35:391, 35:394, 35:396, 53:19, 53:36, 58:65													13			
*08:09, 08:84, 13:46, 13:53, 14:37, 15:42, 15:83, 35:60, 35:276, 39:06:01-39:06:06, 39:34, 39:57, 39:62, 39:64, 39:83, 39:90, 39:109, 39:127, 39:129, 39:132, 39:140-39:141, 41:05, 41:26, 42:04, 46:18, 51:21, 51:36, 51:101, 51:143, 51:153, 51:157, 51:197, 52:37, 52:71, 54:01:01-54:02, 54:04-54:05N, 54:07-54:08N, 54:10-54:25, 54:27-54:32, 54:34-54:37, 55:01:01:01-55:01:06, 55:01:08-55:03, 55:05, 55:07, 55:10-55:13, 55:15-55:21, 55:23, 55:25-55:48, 55:50, 55:52-55:55N, 55:57-55:62, 55:64-55:85, 55:87-55:95, 56:23, 56:54, 59:01:01:01-59:03, 59:05-59:08, 59:10N, 78:07, C*05:152, C*07:463, C*12:183, C*14:92						6			9							
*08:28, 08:35, 08:37, 08:89, 08:107, 15:51, 15:179:01-15:179:02, 15:199, 15:218Q, 18:113, 35:115, 35:260, 37:35, 37:56, 42:09, 57:24, C*07:68, C*07:140, C*07:151, C*07:302										10						
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16
*08:155, 15:46, 15:53, 15:106, 15:143, 15:212, 18:48, 18:107, 35:19, 35:47, 35:63, 35:217, 35:274, 35:317, 39:69, 40:01:01-40:05:01:02, 40:07-40:11:02, 40:14:01-40:16, 40:18, 40:20:01:01-40:20:01:02, 40:22N-40:40, 40:42-40:43, 40:45, 40:48-40:52, 40:54-40:69, 40:71-40:74, 40:77-40:82, 40:84-40:85, 40:87:01-40:92, 40:94, 40:97-40:102, 40:104-40:108, 40:111-40:116, 40:118N-40:126, 40:128-40:130:02, 40:133Q-40:136, 40:138-40:147, 40:149-40:156, 40:158, 40:160:01-40:160:02, 40:163-40:164, 40:168-40:176, 40:179-40:184, 40:186:01-40:187, 40:189, 40:191-40:200, 40:202-40:207, 40:210-40:229, 40:231-40:243, 40:245-40:255, 40:257-40:264, 40:266-40:267, 40:271-40:274, 40:276-40:291N, 40:293, 40:296-40:297, 40:299-40:305, 40:309-40:310, 40:312, 40:314-40:316, 40:319-40:329, 40:331-40:339, 40:343-40:347, 40:351-40:353, 40:355-40:360, 40:367-40:372N, 40:375, 40:377-40:378, 40:380-40:386, 40:390-40:393, 40:395-40:399N, 41:02:01:01-41:04, 41:08, 41:10-41:11, 41:13, 41:15, 41:18-41:19, 41:23, 41:27, 41:31, 41:36, 41:38-41:43, 41:45N, 41:47-41:49, 41:52, 41:55, 44:09, 44:46, 44:75, 44:129, 44:131, 44:254, 44:277, 47:02, 48:41, 50:12, 50:17, 50:51		2														

Negative Control

