

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16
HLA-B allele^{1,2}																
*45:01:01:01-45:01:11, 45:15-45:21, 45:23, 45:25, 45:27	1	2	3													
*45:02	1	2			5											
*45:03	1	2		4												
*45:04		2	3			6										
*45:05	1	2	3				7									
*45:06	1	2	3					8								
*45:07	1	2	3	4												
*45:08		2	3			6			9							
*45:09	1	2								10						15
*45:10		2	3			6					11					
*45:11	1	2	3									12				
*45:12	1	2	3										13			
*45:13	1	2	3											14		
*45:14	1	2	3													15
*45:22, 44:15:01:01-44:15:01:02, 44:18 ³	1		3													
*45:24		2	3													
*07:20, 07:140, 07:153, 15:14, 15:91, 15:131, 15:161, 15:551, 18:169, 35:45, 35:71, 35:169, 35:452, 44:10, 44:14, 44:31, 44:43:01-44:43:02, 44:45, 44:51, 44:62, 44:77, 44:82, 44:107, 44:123, 44:135, 44:184, 44:213, 46:17, 57:07, 57:93, 58:07																15
*07:78, 07:84, 13:02:01:01-13:02:32, 13:08-13:09, 13:14-13:16, 13:18-13:19, 13:27, 13:30-13:35, 13:37-13:38, 13:40-13:42, 13:44-13:45, 13:47, 13:49N, 13:54-13:56:02N, 13:58-13:59, 13:65-13:70, 13:72, 13:74-13:75, 13:81-13:82, 13:84-13:85, 13:88-13:89, 13:91, 13:93-13:94, 13:96-13:105, 13:111-13:119, 13:121, 13:123Q-13:128, 13:130, 13:132-13:134, 13:136-13:137N, 13:139N-13:140, 13:143, 13:145, 13:147, 27:14, 27:81, 27:130, 39:50, 40:93, 40:96, 40:109-40:110, 40:190, 40:340, 40:350, 49:07, 51:10, 51:16, 51:31, 51:34, 51:82, 51:93, 52:08, 52:67, 54:26, 55:09, 55:22, 55:24, 59:09, 73:01:01:01-73:03, 78:08						6					11					
*07:133, 40:166, 44:90		2						8								
*07:225, 13:86, 35:02:01:01-35:02:07, 35:02:09-35:02:16, 35:81, 35:83, 35:95, 35:129N, 35:146, 35:157, 35:162, 35:172, 35:182-35:184, 35:211, 35:220, 35:258, 35:270, 35:285, 35:311, 35:323, 35:335, 35:339, 35:357, 35:361, 35:366-35:367, 35:372, 35:374, 35:377-35:379, 35:387, 35:391, 35:394, 35:396, 35:403, 35:407, 35:410, 35:419, 35:443, 35:468, 35:483, 35:487, 35:492, 53:19, 53:36, 58:65													13			
*08:09, 08:84, 08:254, 13:46, 13:53, 13:131, 14:37, 15:42, 15:83:01:01-15:83:01:02, 15:485, 35:60, 35:276, 39:06:01-39:06:06, 39:34, 39:57, 39:62, 39:64, 39:83:01-39:83:03, 39:90, 39:109, 39:127, 39:129, 39:132, 39:140-39:141, 39:154, 41:14, 41:60, 42:04, 46:18, 51:21, 51:36, 51:101, 51:143, 51:153, 51:157, 51:197, 51:280, 51:295, 51:300, 52:37, 52:71, 52:84, 54:01:01:01-54:02, 54:04-54:05N, 54:07-54:08N, 54:10-54:25, 54:27-54:32, 54:34-54:37, 54:39, 54:41-54:43, 55:01:01:01-55:01:06, 55:01:08-55:03, 55:05, 55:07, 55:10-55:13, 55:15-55:21, 55:23, 55:25-55:48, 55:50, 55:52-55:55N, 55:57-55:62, 55:64-55:85, 55:87-55:109, 56:23, 56:54, 59:01:01:01-59:03, 59:05-59:08, 59:10N-59:11, 78:07, C*05:152, C*07:463, C*07:758, C*12:183, C*14:92						6			9							
*08:28, 08:35, 08:37, 08:89, 08:107, 15:51, 15:179:01-15:179:02, 15:199, 15:218Q, 18:113, 35:115, 35:260, 37:35, 37:56, 42:09, 44:308, 57:24, C*07:68, C*07:140:01-07:140:02, C*07:151, C*07:302										10						

Negative Control

*08:155, 15:46, 15:53, 15:106, 15:143, 15:212, 15:525, 18:48, 18:107, 35:19, 35:47, 35:63, 35:217, 35:274, 35:317, 35:450, 39:69, 40:01:01-40:05:01:03, 40:07-40:11:02, 40:14:01-40:16:01:02, 40:18, 40:20:01:01-40:20:01:02, 40:22N-40:40, 40:42-40:43, 40:45, 40:48-40:52, 40:54-40:69, 40:71-40:74, 40:76-40:78, 40:80-40:82, 40:84-40:85, 40:87:01-40:92, 40:94, 40:97-40:102, 40:104-40:108, 40:111-40:116, 40:118N-40:126, 40:128-40:130:02, 40:133Q-40:136, 40:138-40:147, 40:149-40:156, 40:158, 40:160:01-40:160:02, 40:163-40:164, 40:168-40:176, 40:179-40:184, 40:186:01-40:187, 40:189, 40:191-40:200, 40:202-40:206, 40:210-40:229, 40:231-40:243, 40:245, 40:247-40:255, 40:257-40:264, 40:266-40:267, 40:271-40:274, 40:276-40:280, 40:282-40:288, 40:290-40:291N, 40:293, 40:296-40:297, 40:299-40:305, 40:309-40:310, 40:312, 40:314-40:316, 40:319-40:329, 40:331-40:339, 40:343-40:347, 40:351-40:353:02, 40:355-40:360, 40:367-40:372N, 40:375, 40:377-40:386, 40:390-40:393, 40:395-40:407, 40:409-40:410, 40:412-40:422, 40:424, 40:428N-40:429, 40:431-40:433, 40:435-40:444, 40:446-40:448, 40:450, 40:452, 40:455-40:458, 40:461-40:463, 40:465-40:467, 41:02:01:01-41:04, 41:08, 41:10-41:11, 41:13, 41:15, 41:18-41:19, 41:23, 41:27, 41:31, 41:36, 41:38-41:43, 41:45N, 41:47-41:49, 41:52, 41:55, 41:58, 41:62-41:63, 41:65, 44:09, 44:46:01-44:46:02, 44:75, 44:129, 44:131, 44:254, 44:277, 47:02, 48:41, 50:12, 50:17, 50:51																			
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16			
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16			
*13:03-13:04, 13:48, 13:62, 13:120, 15:04:01:01-15:04:04, 15:16:01:01-15:16:04, 15:67, 15:95, 15:137, 15:155, 15:222, 15:254, 15:293, 15:303, 15:310, 15:362, 15:408, 15:499, 15:517, 15:531, 15:550, 15:552, 27:153, 35:37, 35:235, 35:265, 39:33, 39:102, 44:459, 46:11, 46:32:01-46:32:02, 49:01:01:01-49:06, 49:08-49:14, 49:16-49:19N, 49:21-49:46, 49:48-49:60N, 49:62-49:69, 50:06, 51:01:01:01-51:01:81, 51:01:83-51:03, 51:05, 51:07:01-51:09:03, 51:11N-51:15, 51:17-51:20, 51:22, 51:24:01-51:24:05, 51:26-51:30, 51:32-51:33, 51:35, 51:37-51:41N, 51:43-51:44N, 51:48-51:55, 51:57-51:58, 51:60-51:61:02, 51:63:01-51:63:02, 51:65-51:69, 51:71-51:80, 51:83-51:92:02, 51:94-51:96, 51:98N-51:100, 51:102-51:107, 51:109-51:134, 51:136-51:138, 51:140-51:142, 51:144-51:147, 51:149N-51:152, 51:154-51:156, 51:158:01-51:174:02, 51:176-51:187, 51:189-51:193, 51:195-51:196, 51:198-51:211, 51:214-51:219, 51:221-51:226, 51:228-51:279, 51:282-51:291, 51:294, 51:296-51:299, 51:301-51:309, 51:311-51:318N, 51:320-51:324, 52:01:01:01-52:07, 52:09:01-52:14:02, 52:16-52:36, 52:38-52:47, 52:49N-52:54, 52:56-52:66, 52:68-52:70, 52:72-52:83, 52:85-52:87, 52:89N-52:100, 54:03, 54:33, 54:40, 56:01:01:01-56:01:11, 56:01:13-56:01:16, 56:05:01-56:08, 56:14-56:17, 56:19N-56:22, 56:24-56:30, 56:33-56:50, 56:52-56:53, 56:55:01:02, 56:58-56:59, 56:61-56:62, 56:64-56:65, 56:68-56:73, 56:76, 58:08:01-58:08:02, 58:122, 59:04, 59:12, 78:01:01:02-78:06, 78:09-78:10, C*03:464						6													
*15:518, 18:56, 44:17, 44:230, 53:22, 57:119										10						15			
*35:02:08														w					
*35:154		2												13					
*35:273						6								13					
*35:309														13	15				
*35:485, 37:25				4															
*38:158					5											15			
*40:06:01:01-40:06:12, 40:06:14-40:06:25, 40:44, 40:53, 40:70:01-40:70:02, 40:75, 40:83, 40:86, 40:95, 40:103, 40:127, 40:131, 40:148, 40:159, 40:161-40:162, 40:165, 40:167, 40:177, 40:230, 40:244, 40:256N, 40:268, 40:270, 40:275, 40:294-40:295, 40:298:01-40:298:02, 40:306-40:307, 40:311, 40:313, 40:317-40:318, 40:330, 40:341-40:342, 40:348-40:349, 40:354, 40:361N-40:364, 40:366, 40:373-40:374, 40:376, 40:387-40:389, 40:408, 40:411, 40:423, 40:425, 40:427, 40:434, 40:445, 40:449, 40:451, 40:453-40:454, 40:459-40:460, 40:464, 40:468, 50:31		2				6					11								
*40:06:13, 40:269		2									11								
*40:132, 40:178		2														15			
*41:01:01:01-41:01:07, 41:05-41:07, 41:09, 41:12, 41:16, 41:20-41:22, 41:25-41:26, 41:28-41:29, 41:32-41:35, 41:37, 41:44, 41:50, 41:53-41:54, 41:57, 41:59, 41:61, 41:64, 41:66, 50:45, 50:49		2				6			9										
*41:17, 41:56Q, 50:01:01:01-50:01:17, 50:04-50:05, 50:07-50:11, 50:13-50:16, 50:18-50:20, 50:32-50:35, 50:37-50:44, 50:46-50:48, 50:50, 50:52-50:55, 50:57-50:72N, 56:67		2				6													
*41:24, 41:51		2								10									
*41:30		2				6			9	10									
*44:20, 44:47, 44:100, 44:197, 49:20	1																		
*44:117, 44:251, C*04:345, C*05:102, C*05:184, C*08:93, C*15:25						5													
*44:140, 82:04			3																
*44:339, 51:23, 51:108, 52:15, 56:74	1															15			

Negative Control

*50:02:01:01-50:02:01:02, 50:56	1	2														
*50:36		2				6							13			
*51:97						6			10							
*55:56			3			6			9							
*55:63									9							
*56:13			3			6										
*82:01:01:01-82:03			3													15
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16