

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	
HLA-B allele^{1,2,3}																									
*46:01:01-46:01:02, 46:01:04-46:01:15, 46:01:17-46:01:18, 46:31, 46:35-46:37, 46:42, 46:44, 46:46, 46:48-46:50, 46:52-46:53, 46:60-46:61, 46:63, 15:318	1				5				9																
*46:01:03, 46:01:16	1				5				9						15										
*46:02	1	2			5				9																
*46:03	1		3		5																				
*46:04				4	5				9																
*46:05	1								9	10															
*46:06	1				5	6		8									17						23		
*46:07N	1				5		7		9																
*46:08, 15:289 ⁴	1				5			8																	
*46:09	1				5				9		11														
*46:10	1				5				9			12													
*46:11	1				5			8					13		15		17								
*46:12	1				5				9					14											
*46:13:01-46:13:02, 46:33	1				5			8							15		17								
*46:13:03, 46:43	1				5			8									17								
*46:14	1			4	5				9																
*46:15N	1				5				9	10															
*46:16	1	2			5		7		9																
*46:17	1				5				9								16								
*46:18	1		3		5			8					13		15		17								
*46:19	1				5			8				12													
*46:20, 46:41N ⁵	1				5	6			9																
*46:21:01	1		3		5			8									17								
*46:21:02	1		3		5			8							15		17								
*46:22	1				5				9				13		16										
*46:23, 46:57 ⁵	1				5				9								18								
*46:24	1				5				9									19							
*46:25	1				5			8									17							23	
*46:26	1				5				9													21			
*46:27	1				5				9														22		
*46:28, 46:34 ⁵	1				5				9											20					
*46:29	1		3		5				9																
*46:30	1				5				9															23	
*46:32	1				5				9				13												
*46:38	1				5				9	10								19							
*46:39	1				5			8	9																
*46:40				4	5				9			12													
*46:45, 46:54, 46:58-46:59, 46:62	1				5																				
*46:47					5				9																
*46:51Q	1				5				9													21	23		
*46:55N	1				5				9			12			15										
*46:56	1				5		7		9														22		
*07:09, 07:11, 07:17, 07:162, 07:237, 07:255, 18:35, 35:66, 40:38, 40:52, 40:59-40:60, 40:158, 40:222, 48:14, 48:23, C*07:38:01-07:38:02						6																			
*07:55, 08:70, 48:19, C*02:60														14											
*07:78, 13:02:01-13:02:14, 13:02:16-13:03, 13:08-13:09, 13:14-13:16, 13:18-13:19, 13:27, 13:30, 13:32-13:34, 13:37-13:38, 13:40, 13:42, 13:44-13:45, 13:48-13:49N, 13:53-13:56:02N, 13:58, 13:65-13:70, 13:74-13:75, 13:81-13:82, 14:37, 35:60, 44:15, 44:18, 45:01:01-45:01:02, 45:03-45:08, 45:10-45:15, 49:01:01-49:03, 49:06-49:14, 49:16-49:17, 49:19N-49:37, 50:01:01-50:02, 50:04-50:08, 50:10-50:11, 50:13, 50:15, 50:18-50:19, 50:31-50:32, 50:34-50:40, 51:15, 51:157, 52:25, 54:01:01-54:03, 54:05N, 54:07-54:08N, 54:10, 54:12-54:13, 54:16-54:30, 54:32, 54:34, 55:01:01-55:01:03, 55:01:05-55:01:06, 55:01:08-55:02:08, 55:05, 55:07, 55:09-55:12, 55:15-55:16, 55:18, 55:22, 55:24-55:26, 55:29-55:31, 55:33-55:38, 55:40-55:41, 55:43, 55:45-55:48, 55:50, 55:52, 55:54-55:57, 55:59-55:60, 55:62-55:67, 55:70-55:71, 55:73, 56:01:01-01-56:01:08, 56:07-56:08, 56:13-56:14, 56:16-56:17, 56:20:01-56:20:02, 56:23-56:30, 56:33-56:42, 56:44-56:47, 59:01:01-01-59:01:01:02, 59:04-59:09													13												
*07:100, 15:07:01-15:07:02, 15:45, 15:68, 15:126, 15:207, 15:324, 15:331									9					14											
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	
*07:204, 07:210, 14:02:13, 35:54, 51:140, C*05:01:28, C*07:02:30, C*08:16:02										10															
*08:15, 35:74, 35:186, 55:49					5																				
*13:31, 13:41, 15:04:01-15:04:02, 15:137, 15:303, 35:265, 51:61:01-51:61:02, 52:21, 54:33, 55:21, 56:43									9				13												

Negative Control

*55:03, 55:69					5								13												
*55:19						6							13												
*56:19N							7						13												
*73:01-73:02				w																					
C*01:73	1																								
C*02:28											11														
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	