

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
HLA-B allele^{1,2,3}																								
*46:01:01:01-46:01:02, 46:01:04-46:01:15, 46:01:17-46:01:23, 46:01:25, 46:01:27-46:01:32, 46:01:34-46:01:40, 46:35-46:37, 46:42, 46:44, 46:46, 46:48, 46:50, 46:52-46:53, 46:60-46:61, 46:63-46:71, 46:73-46:76, 46:78, 46:80-46:82, 46:84-46:85, 46:88-46:92, 46:94-46:98N	1				5				9															
*46:01:03, 46:01:16, 46:01:24	1				5				9						15									
*46:01:26, 46:31, 46:47, 46:49, 46:79N, 15:318, 15:579 ⁵					5				9															
*46:01:33, 46:72	1								9															
*46:02	1	2			5				9															
*46:03	1		3		5																			
*46:04				4	5				9															
*46:05	1								9	10														
*46:06	1				5	6		8									17						23	
*46:07N	1				5		7		9															
*46:08	1				5			8																
*46:09	1				5				9		11													
*46:10	1				5				9			12												
*46:11	1				5			8					13		15		17							
*46:12	1				5				9					14										
*46:13:01-46:13:02, 46:33	1				5			8							15		17							
*46:13:03, 46:43	1				5			8									17							
*46:14	1			4	5				9															
*46:15N	1				5				9	10														
*46:16		2			5		7		9															
*46:17	1				5				9								16							
*46:18	1		3		5			8					13		15		17							
*46:19, 46:93	1				5			8				12												
*46:20, 46:87	1				5	6			9															
*46:21:01	1		3		5			8									17							
*46:21:02	1		3		5			8							15		17							
*46:22					5				9				13		16									
*46:23, 46:57 ⁴	1				5				9								18							
*46:24	1				5				9									19						
*46:25	1				5			8									17						23	
*46:26	1				5				9													21		
*46:27	1				5				9														22	
*46:28, 46:34 ⁴	1				5				9															
*46:29	1		3		5				9													20		
*46:30	1				5				9															23
*46:32:01-46:32:02	1				5				9				13											
*46:38	1				5				9	10									19					
*46:39	1				5			8	9															
*46:40:01-46:40:02				4	5				9			12												
*46:41N, 46:86					5				9														22	
*46:45, 46:54, 46:58-46:59, 46:62	1				5																			
*46:51Q	1				5				9													21	23	
*46:55N	1				5				9			12			15									
*46:56	1				5		7		9														22	
*46:77, 15:401, 54:06 ⁵									9														22	
*46:83	1		3		5			8																
*07:09:01-07:09:02, 07:11, 07:17, 07:162, 07:237, 07:255, 07:287, 18:35, 18:110, 35:66, 35:392, 35:425, 40:38, 40:52, 40:59-40:60, 40:158, 40:222, 40:352, 40:422, 48:14, 48:23, 51:351, C*07:38:01-07:38:02						6																		
*07:13, 07:247, 07:270, 08:188, 27:198, 35:113, 39:121, 44:98, 52:31:01-52:31:02, 54:04, 54:09, 54:11, 54:14-54:15, 54:31, 54:38, 54:42, A*02:376, C*03:125, C*03:176																							22	
*07:55, 07:464, 08:70, 08:281, 15:524, 48:19, C*02:60, C*06:264														14										
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24

Negative Control

*15:57				w					9															
*15:78:04, 15:179:01, 15:578															15									
*15:84, 15:237																w								
*15:86, 15:188, 15:224			3						8						15	17								
*15:91									8						16	17								
*15:108, 15:136, 15:235, 15:249									8			12												
*15:117									9														23	
*15:125:01									9	10														
*15:131, 15:161									8						16									
*15:138						6			8														23	
*15:199			3																					
*15:241						6			9														23	
*15:289, 15:397, 15:526					5				8															
*15:292, 15:434									8														22	
*15:297						6			8															
*15:340									9													20		
*15:348			3						8								17							
*15:415, 15:507, 35:62, 53:70, 57:147									9	11														
*15:485			3						8					13	15	17								
*15:499									9	11				13										
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
*15:518, 18:56, 18:169, 35:45, 35:71, 44:17, 44:43:01-44:43:02, 44:144, 44:230, 44:256, 44:308, 45:09, 53:22, 57:93, 57:119, 58:07, C*02:82, C*06:165, C*07:979, C*14:61																16								
*15:552									8				13	15	17									
*15:592			3						8						16									
*18:06, 27:202, 40:465, C*03:278, C*07:714, C*07:724				4																				
*18:135:01:01-18:135:01:02, 51:334										10									19					
*18:142, 73:01:01:01-73:03				w																				
*35:74, 35:356, 40:73, C*16:86				4	5																			
*35:186				4	5							12												
*54:01:01:01-54:01:06, 54:01:08-54:03, 54:05N, 54:07-54:08N, 54:10, 54:12-54:13, 54:16-54:30, 54:32, 54:34-54:37, 54:39, 54:41, 54:43-54:46, 55:07, 55:124, 59:06													13									22		
*54:33, 54:40									9				13									22		
*55:03, 55:69					5								13											
*55:19						6							13											
*56:19N							7						13											
C*01:73	1																							
C*02:28, C*12:180											11													
C*02:167, C*04:459, C*05:205, C*06:284, C*07:828													12											
C*03:504, C*07:1011				4									12											
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24

Negative Control