

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16
HLA-B allele¹																
*48:01:01-48:01:03	1	2	3								11					
*48:02:01-48:02:03	1			4							11					
*48:03:01-48:03:02	1	2			5						11					
*48:04	1		3							10	11					
*48:05	1					6			9		11		13			16
*48:06, 81:02 ²		2	3				7									
*48:07	1	2	3					8			11					
*48:08	1								9		11					16
*48:09, 48:11 ⁴	1	2	3								11			14		
*48:10, 08:17, 08:54, 38:03, 39:02:01-39:02:02, 39:08, 39:13:01-39:13:02, 41:22, 42:11 ³	1										11		13			
*48:12	1										11		13		15	
*48:13	1									10	11		13			
*48:14	1										11		w	14	15	
*48:15, 48:28	1	2	3								11					16
*48:16		2	3					8			11					
*48:17	1	2			5			8			11					
*48:18	1	2	3								11				15	
*48:19, 48:27 ⁵	1	2	3						9		11					
*48:20	1	2	3			6					11					
*48:21	1	2				6					11					
*48:22	1	2	3								11	12				
*48:23	1	2			5						11	12				
*48:24	1		3						9	10	11					
*48:25	1			4			7				11					
*48:26	1	2			5	6					11					
*07:02:01-07:02:25, 07:02:27-07:03, 07:08, 07:10, 07:15-07:16, 07:20-07:24, 07:28-07:31, 07:35, 07:37, 07:39, 07:41-07:42, 07:44, 07:46-07:47, 07:49N, 07:55-07:59, 07:61-07:64, 07:67N, 07:70-07:76, 07:79, 07:82-07:83, 07:85-07:96:02, 07:98-07:99, 07:101-07:104, 07:106-07:111N, 07:113-07:114, 07:116-07:121, 07:124-07:130, 07:133-07:134, 07:136, 07:141-07:142, 07:144-07:145, 07:148-07:154, 07:156-07:161N, 07:166-07:173, 07:175								7		9						16
*07:02:26, 07:04, 07:26, 07:43, 07:45, 07:52, 07:66, 07:84, 07:131, 07:135N, 07:146-07:147								7								16
*07:05:01-07:06, 07:32, 07:69, 07:97, 07:105, 07:112, 07:138, 07:140, 07:176, 08:79								7		9			13			16
*07:07, 07:12, 07:14, 07:18:01-07:18:02, 07:77, 07:162-07:163, 15:138					5		7									16
*07:09, 07:11, 07:17							7		9					14		16
*07:13, 07:54, 07:65, 07:132									9							16
*07:19, 07:25, 07:48, 07:164, 08:01:12, 08:07, 08:12:02, 08:14, 08:64, 08:74, 08:83, 08:87, 08:89, 14:01:01-14:03, 14:06:01-14:09, 14:11-14:12, 14:14-14:33, 15:16:01-15:16:03, 15:18:01-15:18:05, 15:23, 15:29, 15:39:01-15:40, 15:67, 15:72, 15:74, 15:93, 15:95, 15:108, 15:115, 15:119, 15:134, 15:153, 15:161, 15:176, 15:195, 15:197-15:198, 15:226N, 15:238, 15:254-15:255, 15:263, 38:29, 39:07, 39:15, 39:17, 39:25N, 39:43-39:44, 39:55, 42:16, 46:08, 50:15, 51:37, 51:45, 56:32, 58:06, C*03:175, C*15:09, C*15:19, C*15:27, C*15:61							7									
*07:27, 07:50							7		9		11					16
*07:33, 07:60, 07:139							7		9	10						
*07:34, 07:80							7						13			16
*07:36, 07:38, 07:68:01-07:68:03, 07:81							7		9						15	16
*07:40, 08:20, 08:40, 08:70, 08:77							7		9				13			
*07:51, 07:122, 07:155, 07:165							7		9							
*07:53							7		9	10			13			
*07:78							7	8					13			16
*07:100							7		9	10					15	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16
*07:115							7	8	9							16
*07:123									9				13			16

*07:137					5	7						13			16	
*07:143						7		9			12				16	
*07:174		2	3			7		9							16	
*08:01:01-08:01:11, 08:01:13-08:02, 08:05, 08:08N-08:09, 08:11-08:12:01, 08:12:03-08:13, 08:15-08:16, 08:18-08:19N, 08:21-08:27, 08:29-08:34, 08:36, 08:39, 08:41-08:48, 08:50-08:51, 08:53:01-08:53:02, 08:55-08:63, 08:65-08:69, 08:72N-08:73, 08:75-08:76, 08:80-08:82N, 08:85-08:86N, 08:88, 08:90-08:100, 08:102-08:105, 38:02:01-38:02:04, 38:08, 38:15, 38:17-38:18, 38:23, 38:35, 39:01:01:01-39:01:01:03, 39:01:03-39:01:15, 39:04-39:06:02, 39:09-39:10:01, 39:11-39:12, 39:16, 39:18-39:20, 39:26-39:28, 39:30-39:36, 39:38Q, 39:40:01N-39:41, 39:45-39:48, 39:50-39:54, 39:57-39:75, 39:77, 39:79, 42:01:01-42:01:03, 42:04-42:07, 42:12-42:13, 42:15, 67:01:01-67:01:02, 67:03					7						13					
*08:03, 08:52, 08:71, 08:78, 38:01:01, 38:01:03-38:01:07, 38:05-38:07, 38:09-38:14, 38:16, 38:21, 38:24, 38:26-38:28, 38:30-38:34N, 38:36-38:37, 38:39-38:42, 42:10						7						13		15		
*08:04, 39:39:01-39:39:02						7			11			13				
*08:10, 13:03, 13:45, 13:48, 13:62, 15:09, 15:30, 15:42, 15:48, 15:58, 15:73, 15:83, 15:86, 15:150, 15:188, 15:224, 15:229, 15:243, 15:252, 27:20, 27:27, 35:09:01-35:09:03, 35:18, 35:44, 35:60, 35:87-35:88, 39:10:02, 39:78, 40:04, 40:06:01:01-40:06:06, 40:68, 40:70, 46:11, 46:18, 51:01:01-51:01:18, 51:01:20-51:01:30, 51:01:32-51:02:02, 51:02:04-51:03, 51:05-51:16, 51:19-51:24:04, 51:26, 51:28-51:36, 51:38-51:41N, 51:43-51:44N, 51:48-51:55, 51:57-51:63, 51:65-51:74, 51:76-51:80, 51:82-51:83, 51:86-51:89, 51:91, 51:95-51:115, 51:117-51:121, 51:123-51:129, 51:131-51:134, 51:136-51:138, 51:140-51:145, 51:147, 52:01:01:01-52:01:10, 52:01:12-52:11, 52:13-52:15, 52:17-52:25, 52:28-52:31, 54:01:01-54:05N, 54:07-54:08N, 54:10, 54:12-54:13, 54:17-54:21, 54:23-54:26, 54:28, 55:01:01-55:03, 55:05, 55:07-55:13, 55:15-55:22, 55:24-55:26, 55:28-55:40, 55:42-55:45, 55:47-55:48, 55:52-55:57, 56:01:01-56:02, 56:04-56:08, 56:10, 56:13-56:17, 56:19N-56:30, 56:33-56:38N, 58:08:01-58:08:02, 58:20, 59:01:01:01-59:05, 67:02, 78:01:01-78:07, 82:01-82:03, C*06:21, C*15:24												13				
*08:28, 08:37						7								14		
*08:35						7						13	14			
*08:38, 08:101, 38:04, 39:22, 41:01-41:03:02, 41:05-41:07, 41:09, 41:11-41:14, 41:16-41:17, 41:19-41:20, 41:23, 45:12, 45:14	1											13				
*08:49, 14:10, 15:80, 15:194, 15:223, 35:11:01-35:11:02, 35:21, 35:188, 53:06, 53:08:01-53:08:02, 53:28, 55:14, 57:22, 58:19				4		7										
*08:84, 39:56						7						w				
*13:01:01-13:01:06, 13:06-13:07N, 13:12-13:13, 13:17, 13:20, 13:25-13:26, 13:28-13:29, 13:36, 13:39, 13:43, 13:50-13:52, 13:57, 13:60-13:61, 13:63N, 15:02:01-15:02:06, 15:13:01-15:13:02, 15:20-15:21, 15:25:01-15:25:03, 15:36, 15:44, 15:77, 15:85, 15:88-15:89, 15:106, 15:112, 15:121, 15:139, 15:144, 15:154, 15:165, 15:170, 15:213-15:214, 15:240, 15:250, 27:19, 27:30, 35:01:01:01-35:01:36, 35:03:01-35:03:12, 35:07-35:08:06, 35:10, 35:13-35:15, 35:19-35:20:02, 35:23-35:29:02, 35:33-35:36, 35:38-35:42:02, 35:45-35:49, 35:52, 35:54-35:57, 35:61:01-35:63, 35:65Q, 35:69-35:71, 35:74, 35:76-35:78, 35:80, 35:82, 35:85, 35:90-35:94, 35:96, 35:98, 35:100-35:101:02, 35:103-35:112, 35:115-35:116, 35:120-35:124, 35:126, 35:128, 35:130N-35:134N, 35:136-35:145N, 35:147-35:148, 35:150, 35:152-35:153, 35:155-35:156, 35:158-35:161, 35:163, 35:165N-35:171, 35:173N, 35:175-35:181, 35:186-35:187, 35:189-35:198, 35:200, 35:202-35:204, 35:206-35:210, 35:212-35:219, 44:02:01:01-44:09, 44:11-44:13, 44:16-44:17, 44:19N, 44:21-44:30, 44:32-44:40, 44:42-44:46, 44:48-44:52N, 44:55-44:61N, 44:63-44:64:02, 44:66-44:76, 44:78-44:81, 44:83-44:98, 44:101-44:105, 44:107-44:109, 44:111-44:129, 44:131-44:134, 44:136-44:137, 44:139-44:149N, 44:151-44:155, 44:157, 44:159-44:165, 46:33, 51:42, 53:01:01-53:05, 53:07, 53:09-53:13, 53:15-53:18, 53:20-53:27, 53:29, 56:09, 56:11, 57:01:01-57:01:04, 57:01:06-57:01:14, 57:04-57:06, 57:08, 57:10, 57:14-57:16, 57:18-57:20, 57:23-57:27, 57:29-57:30, 57:32-57:38, 57:40-57:41, 57:43-57:45, 57:48-57:56, 57:58, 57:60, 58:01:01-58:01:02, 58:01:04-58:01:12, 58:04, 58:09-58:15, 58:17N, 58:21-58:24, 58:29, 58:31N-58:37, 58:39N, 83:01, C*03:102				4												
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16

*13:22:01-13:22:02, 35:02:01-35:02:05, 35:04:01-35:04:03, 35:06, 35:12:01-35:12:03, 35:59, 35:81, 35:83, 35:95, 35:129N, 35:146, 35:149, 35:154, 35:157, 35:172, 35:182-35:184, 35:201, 35:211, 35:220, 44:10, 44:14, 44:62, 44:77, 44:82, 51:04, 51:46, 51:56:01-51:56:02, 51:139, 53:19, 56:12, 57:02:01-57:03:02, 57:07, 57:09, 57:12, 57:17, 57:28N, 57:46, 57:57, 58:28				4										13				
*13:23, 15:204, 35:50, 35:84, 37:21-37:22, 37:26, 57:59, 58:05				4									12					
*13:40, 51:01:19, 51:75, A*26:01:19													w					
*13:55, 15:71, 15:116, 15:124, 15:175, 15:225, 18:07:01-18:07:02, 18:29, 18:33, 18:79-18:80, 27:75, 46:14, C*07:123, C*07:173													12					
*14:05, 39:29							7		10								15	
*14:13, 38:19, 39:03, 39:14, 39:24:01-39:24:02, 39:37, 39:76							7		10			13					15	
*15:03:01-15:03:03, 15:61, 15:64:01-15:64:02, 15:69, 15:98, 15:103, 15:127, 15:131-15:132, 15:158, 15:173, 15:210, 15:220, 15:242							7			11								
*15:07:01-15:07:02, 15:126, 15:207, 46:12									9	10						14	15	
*15:45									9	10								15
*15:47:01-15:47:02							w			11								
*15:49, 15:65, 15:221, 35:99, 46:03, 57:13							w											
*15:54, 15:91, 15:123, 15:151, 15:156, 15:235, 15:251, 15:253, 18:01:13, 18:25, 18:43										11								
*15:55, 35:05:01-35:05:03, 35:51, 35:72, 35:89, 35:97, 53:14, 58:18									10							14	15	
*15:62				4			7			11								
*15:68	1								9	10	11					14	15	
*15:230, 15:241, C*02:23					5													16
*18:01:01:01-18:01:12, 18:01:14-18:03, 18:05-18:06, 18:08-18:11, 18:13, 18:17N-18:20, 18:23N-18:24, 18:27-18:28, 18:31-18:32, 18:34, 18:36-18:42, 18:44:01-18:56, 18:58-18:60, 18:62-18:68, 18:70-18:78, 18:81-18:83, 37:01:08, 37:02											11	12						
*18:12											11	12						16
*18:14									10	11	12					14	15	
*18:15, 18:57							7			11	12							
*18:21, 18:30							w			11	12							
*18:22				4					10	11	12							
*18:26, 18:61, 37:12										11	12			13				
*18:35					5					11	12							
*18:69, 37:01:01-37:01:06, 37:01:09, 37:03N-37:06, 37:08, 37:10-37:11, 37:13-37:20, 37:23-37:25, 37:27-37:37				4							11	12						
*27:07:01-27:07:03, 27:11, 27:33, 27:43, 35:31, 40:02:01-40:02:09, 40:02:11-40:02:17, 40:05, 40:08, 40:13, 40:39-40:40, 40:94, 40:97, 40:122, 40:142N-40:144N, 40:214, 51:81, 51:148, 55:04, 55:49, 55:51, 58:27										10					13			
*27:24				3						10								
*27:32, 35:199, 44:54, 44:106, 44:135, 44:158										10								15
*27:34, 40:02:10, 40:89, C*03:05, C*03:25, C*03:27, C*03:143, C*03:167, C*08:08										10								
*27:36	2				5	6												
*35:11:03				4			w											
*35:16-35:17, 35:30, 35:113, 35:125, 35:164, 44:110				4						10								
*35:22, 51:64									10					13			15	
*35:58							7		10							14	15	
*35:66									9							14	15	
*35:114, 40:03									10							14		
*35:162, 57:39				4								12		13				
*37:01:07, 44:130, 44:156				4							11							
*37:07									9		11	12						16
*37:09									10	11	12							
*38:01:02, 38:22, 38:38							7											15
*38:20				4			7								13		15	
*38:25	1													13			15	
*39:23, 39:49	1										11							
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16		
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16		
*39:42				4			7								13			

*40:01:01-40:01:17, 40:01:19-40:01:29, 40:07, 40:60, 40:66, 40:73, 40:79, 40:124:01, 40:138-40:140, 40:146-40:147, 40:150, 40:152-40:153, 40:155N, 40:172, 40:186	1	2			5												
*40:01:18	1	2															
*40:09, 40:18, 40:24, 40:85, 40:91, 40:180	?								10								
*40:10:01-40:10:02, 40:25, 40:69, 40:106, 40:166, 44:31		2			5												
*40:11:01-40:11:02, 40:26, 40:53, 40:64, 40:75, 40:86, 40:93, 40:95-40:96, 40:99, 40:103, 40:110, 40:127, 40:129, 40:148, 40:161-40:162, 40:165, 40:167, 40:177, 40:184, 40:190, 40:225	?												13				
*40:12		2			5				11								
*40:14:01-40:14:03, 40:121, 40:136	?				5								13				
*40:15	?								9				13			16	
*40:16	1								9				13			16	
*40:19	?								10				13		?		
*40:21-40:22N, 40:33, 40:38, 40:42-40:43, 40:46-40:47, 40:49, 40:54-40:55, 40:61, 40:63, 40:67, 40:74, 40:76, 40:81, 40:84, 40:87:01-40:88, 40:100-40:102, 40:112-40:114, 40:116, 40:118N, 40:123, 40:124:02-40:126, 40:128, 40:132, 40:134-40:135, 40:137, 40:141, 40:151, 40:154, 40:156, 40:160:01-40:160:02, 40:168, 40:170-40:171, 40:175, 40:179, 40:182, 40:187-40:188, 40:191-40:197, 40:199, 40:204, 40:207-40:208, 40:210, 40:212-40:213, 40:215-40:218, 40:221-40:223	?	2			5												
*40:23	1	2			5												16
*40:27:01-40:27:02, 40:29, 40:37, 40:50, 40:56-40:58, 40:78, 40:82, 40:90, 40:104, 40:107, 40:111, 40:115, 40:119, 40:133Q, 40:145, 40:157, 40:164, 40:169, 40:173-40:174, 40:176, 40:181, 40:189, 40:200-40:203, 40:205-40:206, 40:211, 40:219-40:220, 40:224, 40:226	?								10				13				
*40:28	?			4									13				
*40:30, 40:34	?	2															
*40:31, 40:45, 40:80	?	2	3														
*40:32, 40:98	?								9								16
*40:35, 40:71	?								10				13		15		
*40:36, 40:48	?	2			5		8										
*40:44, 40:92, 40:120, 40:130-40:131, 40:159, 40:163, 40:178	?																
*40:51, 41:04, 41:10, 41:18, 41:21, 45:01-45:05, 45:07-45:11, 45:13, 49:02, 50:01:01-50:02, 50:04-50:14, 50:16, 50:18-50:19	1																
*40:52, 40:185	?	2			5							12					
*40:59	1	2			5							12					
*40:62, 40:65, 40:108	?				5												
*40:72:01-40:72:02	1				5								w				
*40:77	1				5		8						w				
*40:83	?											12	13				
*40:105	?								10				13	14			
*40:109	?												13		?		
*40:117	?	2			5										?		
*40:149	?	2			5					11							
*40:158	?				5								13				16
*40:183, 40:198	?	2			5												16
*40:209	?								10		12		13				
*41:08	1				5								13				
*41:15	1								10				13				
*41:24	1												13	14	15		
*41:25	1												w				
*42:02							7				12		13				
*42:08, 42:14							7		10				13				
*42:09							7				12			14			
*42:17							7				12		13		15		
*44:18, 49:01:01-49:01:04, 49:05-49:24	1														15		
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	
*44:150				4	5												16
*46:06		2			5							12					
*49:03	1											12				15	
*49:04	1						7									15	
*49:25	1										11					15	

