

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20
HLA-B allele^{1,2,3}																				
*48:01:01-48:01:06, 48:32-48:33, 48:35-48:37	1	2	3								11									
*48:02:01-48:02:03	1			4							11									
*48:03:01-48:03:02	1	2			5						11									
*48:04	1		3							10	11									
*48:05	1					6			9		11		13			16				
*48:06		2	3				7											18		
*48:07	1	2	3					8			11									
*48:08	1								9		11					16				
*48:09, 48:11 ⁵	1	2	3								11			14						
*48:10, 39:13:02, 42:11 ⁴	1										11		13							
*48:12	1										11		13		15					
*48:13	1									10	11		13							
*48:14	1										11		w	14	15					
*48:15	1	2	3								11					16				
*48:16		2	3					8			11									
*48:17	1	2			5			8			11									
*48:18, 48:29 ⁵	1	2	3								11				15					
*48:19, 48:27 ⁵	1	2	3						9		11									
*48:20	1	2	3			6					11									
*48:21	1	2				6					11									
*48:22, 48:30	1	2	3								11	12								
*48:23	1	2			5						11	12								
*48:24	1		3						9	10	11									
*48:25	1			4							11									19
*48:26	1	2			5	6					11									
*48:28	1	2	3													16				
*48:31	1	2	3																18	
*48:34	1	2	3								11								17	
*07:02:01-07:02:25, 07:02:27-07:02:38, 07:02:40-07:03, 07:08, 07:10, 07:15-07:16, 07:20-07:24, 07:28-07:31, 07:35, 07:37, 07:39, 07:41-07:42, 07:44, 07:46-07:47, 07:49N, 07:55-07:59, 07:61-07:64, 07:67N, 07:70-07:76, 07:79, 07:82-07:83, 07:85-07:96:02, 07:98-07:99, 07:101-07:104, 07:106-07:111N, 07:113-07:114, 07:116-07:121, 07:124-07:130, 07:133-07:134, 07:136:01-07:136:02, 07:141-07:142, 07:144-07:145, 07:148-07:154, 07:156-07:161N, 07:166-07:168, 07:170-07:173, 07:175, 07:177, 07:179, 07:184-07:192, 07:194-07:195, 07:199-07:201N, 07:203-07:205, 07:208, 07:211-07:212, 07:215-07:218, 07:221, 07:224, 07:229-07:234, 07:236, 07:238, 07:240-07:247, 07:251N-07:254								7		9						16				
*07:02:26, 07:02:39, 07:04, 07:26, 07:43, 07:45, 07:52, 07:66, 07:84, 07:131, 07:135N, 07:146-07:147, 07:178, 07:196, 07:198, 07:227, 07:239								7								16				
*07:05:01-07:06, 07:32, 07:69, 07:97, 07:105, 07:112, 07:138, 07:140, 07:176, 07:182N, 07:206, 07:210, 07:213, 07:249, 08:79								7		9			13			16				
*07:07, 07:12, 07:14, 07:18:01-07:18:02, 07:77, 07:163, 07:193					5			7								16				
*07:09, 07:11								7		9				14		16			w	
*07:13, 07:54, 07:65, 07:132, 07:181N										9						16				
*07:17								7		9				14		16				
*07:19, 07:25, 07:48, 07:164, 07:223, 07:226, 08:01:12, 08:01:23, 08:07, 08:12:02, 08:14, 08:64, 08:74, 08:83, 08:87, 08:89, 08:107, 08:111, 14:01:01-14:03, 14:06:01-14:09, 14:11-14:12, 14:14-14:46, 38:02:06, 38:29, 39:01:19, 39:07, 39:15, 39:17, 39:25N, 39:43-39:44, 39:55, 39:91, 39:96, 42:16, 42:20, 50:15								7												
*07:27, 07:50								7		9		11				16				
*07:33:01, 07:60, 07:139								7		9	10									
*07:33:02-07:33:03, 07:51, 07:122, 07:155, 07:165, 07:220								7		9										
*07:34, 07:80, 07:225								7						13		16				
*07:36, 07:38, 07:68:01-07:68:03, 07:81, 07:180								7		9						15	16			
*07:40, 08:20, 08:40, 08:70, 08:77								7		9				13						
*07:53								7		9	10			13						
*07:78								7	8					13		16				
*07:100								7		9	10					15				
*07:115								7	8	9						16				
*07:123										9				13		16				
*07:137					5			7						13		16				
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20
*07:143							7		9			12				16				
*07:162, 07:255					5		7									16				w
*07:169							7		9							16	17			
*07:174, 07:222		2	3				7		9							16				

Negative Control

*07:183							w	9									16			
*07:197					5	6	7										16			
*07:202		2	3				7	9												
*07:207							7	9				13					16		18	
*07:209, 08:01:01-08:01:11, 08:01:13-08:01:22, 08:01:24-08:02, 08:04-08:05, 08:08N-08:09, 08:11-08:12:01, 08:12:03-08:13, 08:15-08:16, 08:18-08:19N, 08:21-08:22, 08:24-08:27, 08:29-08:34, 08:36, 08:39, 08:41-08:48, 08:50-08:51, 08:53:01-08:53:02, 08:55-08:63, 08:65-08:69, 08:72N-08:73, 08:75-08:76, 08:80-08:82N, 08:85-08:86N, 08:88, 08:90-08:100, 08:102-08:106, 08:108-08:110, 08:112-08:115, 08:117-08:132, 08:135-08:146, 38:02:01-38:02:05, 38:02:07, 38:08, 38:15, 38:17-38:18, 38:23, 38:35, 38:43-38:48, 38:50, 39:01:01:01-39:01:01:03, 39:01:03-39:01:18, 39:01:20-39:01:22, 39:04-39:06:05, 39:09:01-39:10:01, 39:11-39:12, 39:16, 39:18-39:20, 39:26-39:28, 39:30-39:36, 39:38Q-39:41, 39:45-39:48, 39:50-39:54, 39:57-39:75, 39:77, 39:79-39:90, 39:92-39:95N, 39:97N, 39:99-39:100, 39:102, 42:01:01-42:01:03, 42:04-42:07, 42:12-42:13, 42:15, 42:19, 67:01:01-67:01:03, 67:03-67:05							7					13								
*07:214						5	7	8				13					16			
*07:219									9								15	16		
*07:228							7	9									16		18	
*07:235, 08:49, 14:10				4			7													
*07:237							7	9				14	15	16					w	
*08:03, 08:52, 08:71, 08:78, 38:01:01, 38:01:03-38:01:08, 38:05-38:07, 38:09-38:14, 38:16, 38:21, 38:24, 38:26-38:28, 38:30-38:34N, 38:36-38:37, 38:39-38:42, 38:51, 38:53-38:55Q, 42:10							7					13		15						
*08:10, 08:147, 15:09, 15:229, 15:243, 15:252, 15:303, 27:20, 27:27, 27:130, 39:10:02, 39:78, 40:04, 40:06:01:01-40:06:09, 40:06:11, 40:70:01-40:70:02, 40:244, 40:268-40:270, 44:166, 46:11, 46:18, 46:61, 51:01:01:01-51:01:18, 51:01:20-51:01:30, 51:01:32-51:01:49, 51:01:51-51:02:02, 51:02:04-51:03, 51:05-51:06:02, 51:08:01-51:16, 51:19-51:24:05, 51:26, 51:28-51:36, 51:38-51:41N, 51:43-51:44N, 51:48-51:55, 51:57-51:63, 51:65-51:74, 51:76-51:80, 51:82-51:83, 51:86-51:89, 51:91, 51:95-51:115, 51:117-51:121, 51:123-51:129, 51:131-51:134, 51:136-51:138, 51:140-51:145, 51:147, 51:149N-51:150, 51:152-51:162, 51:164-51:171, 51:173Q-51:176, 51:178N-51:184N, 51:186-51:189, 52:01:13, 52:06:01-52:06:03, 52:22-52:23, 54:01:01-54:05N, 54:07-54:08N, 54:10, 54:12-54:13, 54:17-54:21, 54:23-54:26, 54:28-54:34, 55:01:01-55:03, 55:05, 55:07-55:13, 55:15-55:22, 55:24-55:26, 55:28-55:40, 55:42-55:45, 55:47-55:48, 55:52-55:57, 55:59-55:67, 55:69-55:71, 55:73, 56:01:01:01-56:02, 56:04-56:08, 56:10, 56:13-56:17, 56:19N-56:30, 56:33-56:47, 58:08:01-58:08:02, 58:20, 59:01:01:01-59:09, 67:02, 78:01:01-78:01:02, 78:03, 78:07-78:08, 82:01-82:03, C*01:90, C*02:68, C*06:21, C*15:24, C*15:91												13								
*08:17, 08:38, 08:54, 08:101, 38:03-38:04, 39:02:01-39:02:02, 39:08, 39:13:01, 39:22, 39:101, 41:01:01-41:03:02, 41:05-41:07, 41:09, 41:11-41:14, 41:16-41:17, 41:19-41:20, 41:22-41:23, 41:26-41:36, 45:12, 45:14, 50:36	1											13								
*08:23, 08:133							7					13						18		
*08:28, 08:37							7					14								
*08:35							7					13	14							
*08:84, 38:49, 39:56, 39:98							7					w								
*08:116, 08:134, 42:08, 42:14							7		10			13								
*13:01:01-13:01:10, 13:06-13:07N, 13:12:01-13:13:02, 13:17, 13:20, 13:25-13:26:02, 13:28-13:29, 13:36, 13:39, 13:43, 13:50-13:52, 13:57, 13:60-13:61, 13:63N, 13:73, 13:76N-13:80, 13:83, 15:02:01-15:02:09, 15:13:01-15:13:02, 15:20, 15:25:01-15:25:03, 15:36, 15:77, 15:85, 15:88-15:89, 15:106, 15:112, 15:121, 15:144, 15:154, 15:165, 15:170, 15:213-15:214, 15:240, 15:250, 15:265, 15:271, 15:283, 15:289, 15:291, 15:301-15:302N, 15:308, 15:319, 15:325, 15:328, 15:330, 15:341, 15:345, 35:20:01-35:20:02, 35:28, 35:226, 37:55, 44:60, 44:83-44:84:01, 44:194, 44:205:01-44:205:02, 53:09, 53:17:01-53:17:02, 53:38				4						11										
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20

Negative Control

*15:65										11									w	
*15:68	1								9	10	11			14	15					
*15:124, 18:01:01:01-18:01:12, 18:01:14-18:03, 18:05-18:06, 18:08-18:11, 18:13, 18:17N-18:20, 18:23N-18:24, 18:27-18:28, 18:31-18:34, 18:36-18:42, 18:44:01-18:56, 18:58-18:60, 18:62-18:68, 18:70-18:71, 18:73-18:78, 18:80-18:96, 18:98-18:100, 18:103-18:104, 18:106, 18:108, 27:75, 46:14, 46:43, C*07:123, C*07:173												12								
*15:138					5						11					16			19	
*15:194, 15:223, 53:28			4								11								19	
*15:230, C*02:23, C*03:260					5											16				
*15:241					5						11					16				
*15:297			4		5						11					16				
*15:324									9	10	11		13	15						
*15:329, 44:184, 44:213, 51:64									10				13	15						
*18:07:01-18:07:02, 18:29, 18:79												12							18	
*18:12												12				16				
*18:14									10	12			14	15						
*18:15, 18:57, 18:101												12							19	
*18:21, 18:30												12							w	
*18:22			4						10	12										
*18:26, 18:61, 18:97, 51:116, 51:151												12	13							
*18:35					5							12								
*18:69, 18:105, 37:21, 37:26, 57:59, 57:73, 58:05, 58:41			4									12								
*27:07:01-27:07:04, 27:11, 27:33, 27:43, 40:02:01-40:02:03, 40:02:05-40:02:09, 40:02:11-40:02:17, 40:02:19-40:02:23, 40:05, 40:13, 40:39-40:40, 40:94, 40:97, 40:122, 40:142N-40:144N, 40:214, 40:229, 40:254-40:255, 40:266, 40:271, 40:302-40:304, 51:81, 51:148, 55:04, 55:49, 55:51, 58:27										10			13							
*27:24			3							10										
*27:32, 27:138, 44:54, 44:106, 44:135, 44:158										10				15						
*27:34, 40:02:10, 40:02:18, 40:89, C*03:05, C*03:25, C*03:27, C*03:143, C*03:167, C*08:08:01-08:08:02										10										
*27:36			2		5	6														
*27:125, 40:10:01, 40:12, 40:69, 44:31			2		5															
*35:01:01:01-35:01:45, 35:03:01-35:03:11, 35:03:13-35:03:17, 35:03:19-35:03:21, 35:07-35:08:08, 35:10, 35:13-35:15:02, 35:19, 35:23-35:25, 35:27, 35:33-35:36, 35:38-35:42:02, 35:45-35:49, 35:52, 35:54-35:57, 35:61:01-35:62, 35:65Q, 35:70-35:71, 35:74, 35:77-35:78, 35:80, 35:90-35:94, 35:96, 35:98, 35:100-35:101:02, 35:103-35:111, 35:115-35:116, 35:120-35:124, 35:126, 35:128, 35:130N-35:134N, 35:137-35:145N, 35:147-35:148, 35:150:01-35:150:02, 35:152, 35:155-35:156, 35:158-35:161, 35:163, 35:165N-35:171, 35:173N, 35:175-35:181, 35:187, 35:189-35:197, 35:200, 35:202, 35:204, 35:206-35:210, 35:213-35:219, 35:221-35:225, 35:227-35:228, 35:236-35:241, 35:244-35:248, 35:250, 35:252-35:253, 35:255-35:257, 35:259, 35:261-35:264, 35:267-35:269, 35:271-35:272, 35:274-35:275, 35:278-35:279, 35:281-35:282, 35:286, 53:05, 53:16, 53:33				4															18	
*35:02:01-35:02:07, 35:04:01-35:04:03, 35:06, 35:12:01-35:12:03, 35:59, 35:81, 35:83, 35:129N, 35:146, 35:149, 35:154, 35:157, 35:172, 35:182-35:184, 35:201, 35:211, 35:220, 35:233, 35:258, 35:266, 35:270, 35:285				4									13						18	
*35:05:01-35:05:03, 35:51, 35:89, 35:97										10				14	15				18	
*35:09:01-35:09:03, 35:18, 35:44, 35:60, 35:87-35:88, 35:251, 35:273, 35:276, 51:185, 78:02:01-78:02:02, 78:04													13						18	
*35:11:01-35:11:02, 35:21, 35:188, 35:277			4																18 19	
*35:11:03			4																18 w	
*35:16-35:17:02, 35:30, 35:113, 35:125, 35:164, 35:242			4							10									18	
*35:22										10			13	15					18	
*35:31, 40:08										10			13						18	
*35:50, 35:84, 35:231, 35:280			4									12							18	
*35:58										10				14	15				18 19	
*35:66									9					14	15				18	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20
*35:99																			18 w	
*35:114										10				14					18	
*35:162				4								12	13						18	
*35:199										10					15				18	
*35:232									9	10				14	15				18 w	
*37:07									9		11	12				16				
*37:09										10	11	12								
*37:12											11	12	13							
*38:01:02, 38:22, 38:38, 38:52, 49:26							7								15					
*38:20				4				7					13	15						

Negative Control

*38:25	1											13	15							
*38:56												13	15							
*39:23, 39:49, 40:51, 41:04, 41:10, 41:18, 41:21, 45:01:01-45:05, 45:07-45:11, 45:13, 45:15, 49:02, 50:01:01-50:02, 50:04-50:14, 50:16, 50:18-50:20, 50:31-50:35, 50:37-50:42	1																			
*39:42				4			7					13								
*40:01:01-40:01:17, 40:01:19-40:01:29, 40:01:31-40:01:41, 40:60, 40:66, 40:73, 40:79, 40:124:01, 40:138-40:140, 40:146-40:147, 40:150, 40:152-40:153, 40:155:01N-40:155:02N, 40:172, 40:186, 40:250-40:253, 40:258-40:265N, 40:272, 40:277-40:278, 40:285-40:286N, 40:301	1	2			5															
*40:01:18	1	2																		
*40:01:30	1				5															
*40:02:04									10	11		13								
*40:03, 40:267, 40:284									10			14								
*40:07	1	2			5													18		
*40:09, 40:18, 40:24, 40:85, 40:91, 40:180	?								10											
*40:10:02		2			5					11										
*40:11:01-40:11:02, 40:26, 40:53, 40:64, 40:75, 40:86, 40:93, 40:95-40:96, 40:99, 40:103, 40:110, 40:127, 40:129, 40:148, 40:161, 40:165, 40:167, 40:177, 40:184, 40:190, 40:225, 40:230, 40:243, 40:256N, 40:275, 40:294-40:295	?											13								
*40:14:01-40:14:03, 40:121, 40:136, 40:237	?				5							13								
*40:15	?								9			13		16						
*40:16	1								9			13		16						
*40:19, 40:292	?								10			13		?						
*40:21, 40:137	?	2			5					11										
*40:22N, 40:33, 40:38, 40:42-40:43, 40:46-40:47, 40:49, 40:54-40:55, 40:61, 40:63, 40:67, 40:74, 40:76, 40:81, 40:84, 40:87:01-40:88, 40:100-40:102, 40:112-40:114:02, 40:116, 40:118N, 40:123, 40:124:02-40:126, 40:128, 40:132, 40:134-40:135, 40:141, 40:149, 40:151, 40:154, 40:156, 40:160:01-40:160:02, 40:168, 40:170-40:171, 40:175, 40:179, 40:182, 40:187-40:188, 40:191-40:197, 40:199, 40:204, 40:207-40:208, 40:210, 40:212-40:213, 40:215-40:218, 40:221-40:223, 40:227-40:228, 40:231, 40:234-40:236, 40:239-40:242, 40:245, 40:247, 40:249, 40:257, 40:273, 40:279, 40:281-40:282, 40:288, 40:299	?	2			5															
*40:23	1	2			5													16		
*40:25, 40:166		2			5		?													
*40:27:01-40:27:02, 40:29, 40:37, 40:50, 40:56-40:58, 40:78, 40:82, 40:90, 40:104, 40:107, 40:111, 40:115, 40:119, 40:133Q, 40:145, 40:157, 40:164, 40:169, 40:173-40:174, 40:176, 40:181, 40:189, 40:200-40:203, 40:205-40:206, 40:211, 40:219, 40:224, 40:226, 40:246, 40:274, 40:276, 40:283, 40:287, 40:289, 40:291N, 40:293, 40:296-40:297	?								10			13								
*40:28	?			4								13								
*40:30, 40:34, 40:233, 40:298	?	2																		
*40:31, 40:45, 40:80	?	2	3																	
*40:32, 40:98	?								9									16		
*40:35:01-40:35:02, 40:71	?								10			13	15							
*40:36, 40:48, 40:280	?	2			5		8													
*40:44, 40:92, 40:120, 40:130:01-40:131, 40:159, 40:163, 40:178	?																			
*40:52	?	2			5						12									
*40:59	1	2			5						12									
*40:62, 40:65, 40:108, 40:238	?				5															
*40:68							?					13						18		
*40:72:01-40:72:02	1				5							w								
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20
*40:77	1				5			8					w							
*40:83	?										11	12	13							
*40:105	?									10			13	14						
*40:106		2			5		?												18	
*40:109	?												13		?					
*40:117	?	2			5										?					
*40:158	?				5								13			16				
*40:162	?												13						18	
*40:183, 40:198	?	2			5											16				
*40:185	?	2			5															
*40:209, 40:248	?									10	11	12	13							
*40:220	?									10			13						18	
*40:232								?		10			13						18	
*40:290	?									10			w							
*41:08	1				5								13							

Negative Control

*41:15	1								10			13								
*41:24	1											13	14	15						
*41:25	1											w								
*42:02, 42:18							7					12	13							
*42:09							7					12		14						
*42:17							7					12	13		15					
*44:10, 44:14, 44:62, 44:77, 44:82, 51:04, 51:46, 51:56:01-51:56:03, 51:139, 53:19, 53:36, 56:12, 57:02:01-57:03:02, 57:07, 57:09, 57:12, 57:17, 57:28N, 57:46, 57:57, 57:63, 57:66, 57:70, 58:28:01-58:28:02, 58:65				4									13							
*44:18, 49:01:01-49:01:08, 49:05-49:14, 49:16-49:25, 49:27-49:37	1														15					
*44:110, 44:211				4					10											
*44:150				4	5														16	
*46:06		2			5							12								
*46:12									9	10				14	15					
*49:03	1										11	12			15					
*49:04:01-49:04:02	1														15				19	
*50:17	1			4																
*51:01:19, 51:75, A*26:01:19													w							
*53:14, 58:18									10					14	15					
*57:39				4								12	13							
*57:42				4									w							
*73:01-73:02								w				12								
*81:01, 81:03-81:07		2	3																	
*81:02		2	3					7												
C*02:60									9	10				14						
C*04:77					5			8											16	
C*07:38:01-07:38:02					5															
C*07:294												12		14						
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20

Negative Control