

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19
<b>HLA-B allele<sup>1,2,3</sup></b>																			
*49:01:01-49:01:06, 49:25, 49:28-49:30	1				5	6													
*49:02		2	3		5	6													
*49:03, 14:37 <sup>4</sup>				4	5	6													
*49:04	1						7												
*49:05	1						7	8											
*49:06	1	2			5	6													
*49:07, 49:21 <sup>5</sup>	1				5	6			9										
*49:08, 49:16 <sup>5</sup>	1				5	6				10									
*49:09	1			4	5	6													
*49:10, 49:24, 49:31	1				5	6						12							
*49:11	1				5	6							13						
*49:12	1				5	6								14					
*49:13-49:14	1				5	6									15				
*49:15	1				5	6										16			
*49:17	1				5	6								14	15				
*49:18	1															16			
*49:19N	1				5	6		8						14					
*49:20	1				5	6	7												
*49:22	1		3		5	6													
*49:23	1				5	6												17	
*49:26		2			5	6													
*49:27	1				5	6													18
*07:78, 13:31, 13:41, 15:303, 54:16, 54:20, 55:09, 55:21, 55:46, 55:52, 55:56, 56:01:05, 56:13, 56:43					5			8											
*07:84, 08:09, 08:84, 13:35, 13:59, 13:62, 15:83, 27:14, 27:81, 40:06:01:01-40:06:10, 40:44, 40:53, 40:70:01-40:70:02, 40:75, 40:83, 40:93, 40:95, 40:103, 40:127, 40:131, 40:148, 40:161-40:162, 40:165, 40:177, 40:190, 40:230, 40:244, 40:256N, 40:268-40:270, 40:275, 41:01, 41:05-41:07, 41:09, 41:12, 41:14, 41:16-41:17, 41:20-41:22, 41:25-41:26, 41:28-41:29, 41:32, 42:04, 50:20, 51:01:01-51:01:18, 51:01:20-51:03, 51:05, 51:07:02, 51:09:01-51:12, 51:14, 51:16, 51:18-51:19, 51:21-51:22, 51:24:01-51:24:05, 51:26-51:35, 51:38-51:41N, 51:43, 51:48-51:55, 51:57-51:58, 51:60-51:61:02, 51:65-51:80, 51:82-51:91, 51:93, 51:95-51:96, 51:98N-51:105, 51:107, 51:109-51:111, 51:113-51:125, 51:127-51:129, 51:131-51:134, 51:136-51:138, 51:140, 51:142-51:147, 51:149N-51:152, 51:154-51:156, 51:158-51:161, 51:163-51:171, 51:173Q, 52:01:01:01-52:01:01:03, 52:01:03, 52:01:05-52:01:08, 52:01:10-52:01:20, 52:02:02, 52:04-52:08, 52:10:01-52:10:02, 52:11-52:13, 52:16-52:18, 52:20, 52:22-52:24, 52:26-52:31:01, 52:32, 52:34-52:38, 54:04, 54:11, 55:17, 55:20, 55:28, 56:05:01-56:06, 56:15, 56:21, 58:08:01-58:08:02, 59:02, 78:01:01-78:07								8											
*08:03, 08:52, 08:78, 37:34, 38:30, 44:06, 53:22, 57:09, 57:24												11							
*13:01:01-13:01:08, 13:06-13:07N, 13:11-13:12:02, 13:17, 13:22:01-13:22:02, 13:25, 13:28-13:29, 13:36, 13:39, 13:43, 13:50, 13:52, 13:57, 13:60-13:61, 13:63N-13:64, 13:76N-13:78, 15:86, 15:224, 40:48, 40:71, 56:02, 56:04, 56:10						6							13						
*13:02:01-13:02:05, 13:02:07-13:02:14, 13:02:16-13:03, 13:08-13:09, 13:14, 13:19, 13:27, 13:30, 13:32-13:34, 13:37-13:38, 13:40, 13:42, 13:44-13:45, 13:49N, 13:53-13:54, 13:56N, 13:58, 13:65-13:70, 13:75, 35:60, 46:11, 46:18, 51:15, 51:157, 54:01:01-54:03, 54:05N, 54:07-54:08N, 54:12-54:13, 54:17-54:19, 54:21-54:30, 54:32, 55:02:01-55:02:08, 55:07, 55:10, 55:12, 55:16, 55:18-55:19, 55:22, 55:26, 55:30, 55:34-55:35, 55:41, 55:43, 55:47-55:48, 55:50, 55:62-55:63, 56:01:01-56:01:04, 56:01:06-56:01:07, 56:07-56:08, 56:14, 56:16-56:17, 56:19N-56:20:02, 56:23-56:24, 56:26-56:29, 56:33, 56:35-56:36, 56:38N-56:42, 59:01:01:01-59:01:01:02, 59:04-59:08					5	6		8					13						
*13:02:06, 13:74					5	6							13						
*13:02:15, 13:47, 54:31, 55:39, 55:42, 55:61					6		8						13						
*13:04, 13:46, 15:04:01-15:04:02, 15:16:01-15:16:02, 15:67, 15:95, 15:137, 15:155, 15:222, 15:254, 15:293, 15:310, 39:06:01-39:06:04, 39:33-39:34, 39:50, 39:57, 39:62, 39:64, 39:83, 39:90, 40:86, 40:159, 41:30, 46:32, 51:13:01-51:13:02, 51:23, 51:37, 51:63, 51:92, 51:108, 51:172, 52:14-52:15, 52:39, 54:14, 55:13, 55:23, 55:32, 55:58, 56:22, 59:03, 73:01-73:02							7	8											
*13:15, 56:34				4	5	6		8					13						
<b>Well No.</b>	<b>1</b>	<b>2</b>	<b>3</b>	<b>4</b>	<b>5</b>	<b>6</b>	<b>7</b>	<b>8</b>	<b>9</b>	<b>10</b>	<b>11</b>	<b>12</b>	<b>13</b>	<b>14</b>	<b>15</b>	<b>16</b>	<b>17</b>	<b>18</b>	<b>19</b>
<b>Well No.</b>	<b>1</b>	<b>2</b>	<b>3</b>	<b>4</b>	<b>5</b>	<b>6</b>	<b>7</b>	<b>8</b>	<b>9</b>	<b>10</b>	<b>11</b>	<b>12</b>	<b>13</b>	<b>14</b>	<b>15</b>	<b>16</b>	<b>17</b>	<b>18</b>	<b>19</b>
*13:16, 13:48, 15:42, 55:01:01-55:01:03, 55:01:05-55:01:06, 55:01:08-55:01:12, 55:03, 55:05, 55:11, 55:15, 55:24-55:25, 55:29, 55:31, 55:33, 55:36, 55:38, 55:40, 55:45, 55:54-55:55N, 55:57, 55:59-55:60, 55:64, 56:25, 56:30, 56:37					5	6		8											
*13:18, 54:10, 55:37					5			8					w						
*13:20, 39:17, 50:12, 50:16-50:17, 55:01:07						6													
*13:23		2				6							13						
*13:55		2			5	6		8					13						
*13:72, 15:16:03, 35:37, 35:235, 45:02, 45:09, 50:09, 50:33							7												

Negative Control

*13:73													w								
*15:32:02, 18:04, 40:01:11				4																	
*15:33, 15:248, 44:03:16, 44:07, 44:49, 44:81, 44:111, 44:145, 57:54, C*03:87, C*05:27, C*05:39			3																		
*15:46, 15:53, 15:106, 35:47, 35:154, 40:10:01-40:10:02, 40:40, 40:49, 40:129, 40:164, 40:210, 40:219, 40:229, 44:02:13, 44:02:19, 44:05:03, 44:09, 44:11, 44:46, 44:50:02, 44:75, 44:90, 44:129, 44:152, 44:170, 47:01:01:01-47:03, 47:05-47:09													12								
*18:09, 27:01, 37:10, 39:31, 58:13			2																		
*35:01:10, 35:04:02, 40:28, 50:14, 51:56:01-51:56:02, 53:28, C*02:02:23																					16
*40:13, 40:19, 40:117, 57:51	1																				
*40:47, 40:157, 40:201, 44:55, 44:103, 44:188		2	3																		
*40:58				4									12								
*40:96		2	3						8												
*40:109, 51:112	1								8												
*40:110			3						8				12								
*44:02:01:01-44:02:09, 44:02:11-44:02:12, 44:02:14-44:02:18, 44:02:20-44:03:15, 44:03:17-44:05:02, 44:05:04, 44:12-44:14, 44:16-44:17, 44:19N, 44:21-44:24, 44:26-44:33, 44:34:02-44:39, 44:41:01-44:43:02, 44:45, 44:48, 44:51-44:54, 44:56N, 44:58N-44:59:02, 44:61N-44:74, 44:76-44:80, 44:82, 44:84:01-44:89, 44:91-44:94, 44:96-44:99, 44:101-44:102, 44:104-44:110, 44:112-44:128, 44:132-44:133, 44:135-44:144, 44:146-44:151, 44:153-44:155, 44:157-44:169, 44:171N-44:187, 44:189-44:193, 47:04			3										12								
*44:02:10, 44:34:01			3	4									12								
*44:10			3				6						12	13							
*44:15		2	3			5	7														
*44:18	1					5	7					11									
*44:20, 44:47, 44:100			3				7	8					12								
*44:25, 44:95	1											11	12								
*44:50:01, 53:30, 57:45, 57:69	1											12									
*45:01:01-45:01:02, 45:03, 45:05-45:07, 45:11-45:13						5	7														
*45:04, 45:08						5															
*45:10						5						9									
*45:14						5	7	8													
*50:01:01-50:01:04, 50:04-50:08, 50:10-50:11, 50:15, 50:18-50:19, 50:32, 50:34-50:38						5	6														
*50:02						5	6	7													
*50:13						5	6						12								
*50:31						5	6					9									
*51:07:01, 52:01:02, 52:01:04, 52:01:09, 52:02:01, 52:03, 52:09, 52:10:03, 52:21, 52:31:02				4								8									
*51:08:01-51:08:02, 51:20, 51:36, 51:44N, 51:141, 51:153, 52:19												8		11							
*51:62				4			6														
*51:97							7	8					11								
*51:106				4			6	8													
*51:126			3					8													
*51:130								8											14		
*52:25				4		5	6	8													
*52:33								8				10									
*55:01:04						5	6	8													16
*55:44, 55:53							6	8													
<b>Well No.</b>	<b>1</b>	<b>2</b>	<b>3</b>	<b>4</b>	<b>5</b>	<b>6</b>	<b>7</b>	<b>8</b>	<b>9</b>	<b>10</b>	<b>11</b>	<b>12</b>	<b>13</b>	<b>14</b>	<b>15</b>	<b>16</b>	<b>17</b>	<b>18</b>	<b>19</b>		

Negative Control