

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	
HLA-B allele ^{1,2,3}																								
*49:01:01:01-49:01:24, 49:25, 49:28:01-49:30, 49:32-49:36, 49:38-49:40, 49:42-49:46, 49:48-49:49, 49:51-49:60N, 49:62-49:69, 49:71-49:72, 49:74-49:78	1				5	6																		
*49:02, 49:70		2	3		5	6																		
*49:03				4	5	6										16								
*49:04:01-49:04:02	1						7													19				
*49:05:01-49:05:02	1						7	8																
*49:06	1	2			5	6																		
*49:07, 49:21 ⁴	1				5	6			9															
*49:08, 49:16 ⁴	1				5	6				10														
*49:09	1			4	5	6																		
*49:10, 49:24, 49:31	1				5	6						12												
*49:11	1				5	6							13											
*49:12	1				5	6								14										
*49:13-49:14 ⁴	1				5	6									15									
*49:17	1				5	6								14	15									
*49:18:01-49:18:02	1																				20			
*49:19N	1				5	6		8						14										
*49:20	1				5	6	7																	
*49:22	1		3		5	6																		
*49:23	1				5	6											17							
*49:26		2			5	6																22		
*49:27	1				5	6															18			
*49:37, 49:41	1				5																			
*49:47:01-49:47:02	1					6																		
*49:50	1						7																	
*49:61	1																					21		
*49:73, 59:12⁵					5	6																	22	
*07:16, 07:37:01-07:37:02, 07:50, 07:91, 07:207, 07:228:01-07:228:02, 07:306, 08:23, 08:54:01-08:54:02, 08:110, 14:14, 14:18, 14:73, 15:03:01:01-15:03:09, 15:03:11-15:03:13, 15:09:01-15:10:08, 15:18:01:01-15:18:10, 15:29:01:01-15:29:01:02, 15:37, 15:47:01-15:47:02, 15:49, 15:51-15:52, 15:54, 15:62, 15:64:01-15:64:02, 15:68-15:69, 15:72, 15:74, 15:80, 15:90-15:91, 15:93, 15:98-15:99, 15:103, 15:108, 15:114, 15:123-15:124, 15:127, 15:131-15:134, 15:151, 15:153, 15:156, 15:158, 15:161, 15:173, 15:176, 15:186, 15:197:01-15:198, 15:200, 15:210, 15:220:01:01-15:221, 15:226N, 15:229, 15:235, 15:238, 15:242:01-15:243, 15:252-15:253, 15:259, 15:263, 15:266, 15:274-15:275:02, 15:281-15:282, 15:290, 15:292, 15:306-15:307, 15:311-15:314, 15:323, 15:329, 15:335, 15:337-15:338, 15:351, 15:354, 15:369, 15:379-15:380N, 15:382, 15:388-15:389, 15:414, 15:426-15:427, 15:433, 15:435, 15:439, 15:444-15:445, 15:447-15:448, 15:451, 15:455, 15:475, 15:487N, 15:498, 15:502-15:504:02, 15:506, 15:509, 15:511, 15:526-15:527, 15:530, 15:539, 15:548, 15:560, 15:570-15:571, 15:573, 15:582, 15:588, 15:607, 15:610-15:611, 15:614:01:01-15:614:01:02, 15:616Q-15:617, 15:620, 15:628, 15:643, 15:650, 18:37:01-18:37:02, 27:18, 27:23, 27:44, 27:131, 27:257, 35:25-35:26, 35:49:01-35:49:02, 35:361, 39:05:01:01-39:05:05, 39:07-39:08, 39:11, 39:13:01-39:13:02, 39:20, 39:37:01:01-39:37:01:02, 39:49, 39:55-39:56, 39:113, 39:118, 39:147N, 39:149, 39:161N, 39:163, 39:165-39:168, 39:170, 39:172, 39:174, 40:12:01-40:12:02, 40:46, 40:93, 40:430, 42:11, 48:01:01:01-48:17, 48:19-48:40, 48:42-48:47, 48:49-48:55, 48:57, 81:08																								
*07:28, C*01:115, C*04:439, C*14:79																						w		
*07:38, 07:75:01:01-07:75:01:02, 07:180, 07:219, 08:295, 15:23, 27:119, 38:01:01:01-38:01:22, 38:02:07, 38:05, 38:09-38:12, 38:16, 38:19-38:22, 38:24-38:25, 38:27-38:28, 38:31-38:34N, 38:36-38:40, 38:42, 38:51-38:61, 38:63, 38:65-38:71, 38:73, 38:77-38:78, 38:80N-38:81, 38:83N, 38:86-38:88, 38:90-38:91, 38:93-38:96, 38:99-38:102, 38:161, 38:166, 38:168-38:171, 48:18, 51:267, 52:16, 58:44, 59:02, 59:11																							22	
*07:78, 13:31, 13:41, 13:128, 13:131, 13:145, 13:154, 13:164, 15:303, 45:04, 45:08, 45:24, 50:47, 50:76, 54:16, 54:20, 54:33, 54:40, 55:09, 55:21, 55:46, 55:52, 55:56, 56:01:05, 56:13, 56:43					5																			
*07:84, 07:429, 13:35, 40:44, 51:90, 52:12, 55:27																							19	
*08:03, 38:30												11											22	
*08:52, 08:78, 37:34, 44:06, 51:08:01:01-51:08:06, 51:20, 51:36, 51:44N, 51:141, 51:153:01-51:153:02, 51:263, 51:275, 51:342, 51:354, 51:362, 52:19, 53:22, 57:09, 57:24, 57:119												11												
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	
*08:254, 41:01:01:01-41:01:08, 41:05-41:07, 41:09, 41:12, 41:14, 41:16-41:17, 41:20, 41:22, 41:25-41:26, 41:28-41:29, 41:32, 41:34-41:35, 41:37, 41:53-41:54, 41:56Q-41:57, 41:59, 41:61, 41:64, 41:67, 41:70, 41:73, 41:75-41:79, 50:14, 50:20, 51:372																					20			

Negative Control

w

