

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22
HLA-B allele^{1,2,3}																						
*50:01:01-50:01:07, 50:32, 50:34-50:36, 50:38	1			4	5	6	7															
*50:02		2		4	5	6	7															
*50:04	1		3	4	5	6	7															
*50:05	1			4	5	6	7	8														
*50:06	1			4	5	6	7		9													
*50:07	1			4	5	6	7			10												
*50:08	1			4	5	6	7				11											
*50:09	1				5							12										
*50:10	1	10		4	5	6	7						13									
*50:11	1			4	5	6	7							14								
*50:12	1				5	6	7		9													
*50:13, 50:43	1			4	5	6	7								15							
*50:14	1				5											16						
*50:15	1			4	5	6	7									16						
*50:16	1				5	6	7			11												
*50:17	1				5	6	7	8														
*50:18	1			4	5	6	7					12										
*50:19	1			4	5	6	7					12			15							
*50:20	1				5												17					
*50:31	1			4	5	6	7											18				
*50:33, 41:21 ⁴	1				5														19			
*50:37	1			4	5	6	7													20		
*50:39	1			4	5																	21
*50:40	1			4	5	6	7										17					
*50:41	1				5	6	7															21
*50:42	1				5	6	7															
*07:78, 55:09	1			4			w											18				
*07:84, 13:35, 13:72	1																	18	19			
*07:133, 08:155, 15:143, 15:212, 18:48, 35:19, 35:274, 39:69, 40:01:01-40:01:05, 40:01:07-40:03, 40:05, 40:07, 40:09, 40:11:01- 40:11:02, 40:14:01-40:16, 40:18, 40:20, 40:22N-40:24, 40:27:01- 40:27:02, 40:29-40:39, 40:42-40:43, 40:45, 40:50-40:52, 40:54-40:57, 40:60-40:63, 40:65-40:67, 40:69, 40:72:01-40:74, 40:77-40:82, 40:84- 40:85, 40:87:01-40:92, 40:94, 40:97-40:102, 40:104-40:105, 40:107- 40:108, 40:111-40:116, 40:118N-40:119, 40:121-40:126, 40:128, 40:130:01-40:130:02, 40:132-40:136, 40:138-40:147, 40:149-40:156, 40:158, 40:160:01, 40:163, 40:168-40:169, 40:171-40:176, 40:178- 40:184, 40:186-40:187, 40:189, 40:191-40:200, 40:202-40:207, 40:211- 40:218, 40:220-40:228, 40:231, 40:233-40:243, 40:245-40:255, 40:257- 40:264, 40:266-40:267, 40:271-40:274, 40:276-40:291N, 40:293, 40:296-40:297, 40:299-40:305, 40:309-40:310, 40:312, 40:314-40:316, 41:02:01-41:04, 41:08, 41:10-41:11, 41:13, 41:15, 41:18-41:19, 41:23- 41:24, 41:27, 41:31, 41:36, 41:38-41:40, 44:131				5																		
*07:177																w						
*07:182N, 15:18:03, 15:73, A*02:339, A*02:510, C*03:19, C*03:102, C*15:26			3																			
*08:09, 08:84, 13:46, 15:83, 35:273, 35:276, 39:06:01-39:06:05, 39:33- 39:34, 39:57, 39:62, 39:64, 39:83, 39:102, 41:05, 41:26, 42:04, 51:01:01:01-51:03, 51:05, 51:07:01-51:09:03, 51:11N-51:14, 51:17- 51:22, 51:24:01-51:24:05, 51:26-51:30, 51:32-51:33, 51:35-51:36, 51:38-51:41N, 51:43-51:44N, 51:48-51:55, 51:57-51:58, 51:60- 51:61:02, 51:63, 51:65-51:69, 51:71-51:80, 51:83-51:84, 51:86-51:89, 51:91, 51:94-51:102, 51:104-51:105, 51:109-51:118N, 51:120-51:134, 51:136-51:138, 51:140-51:147, 51:149N-51:156, 51:158:01, 51:159- 51:171, 51:173Q-51:174, 51:176-51:187, 51:189-51:191, 51:193, 51:195, 52:01:01:01-52:07, 52:09-52:11, 52:13-52:14, 52:16-52:24, 52:27-52:47, 52:49N-52:54, 52:56, 54:04, 54:11, 55:13, 55:17, 55:20, 55:28, 55:32, 55:58, 56:05:01-56:06, 56:15, 56:21-56:22, 58:08:01- 58:08:02, 59:02-59:03, 78:01:01-78:07, 78:09	1																					
*08:102, 08:122, 14:48, 15:345, 18:80, 18:102, 27:04:03, 27:23, 27:92, 35:01:10, 35:04:02, 35:90, 35:150:01-35:150:02, 35:184, 35:201, 38:16, 38:46, 51:56:01-51:56:03, C*02:02:23, C*12:03:31																16						
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22
*13:01:01-13:01:11, 13:06, 13:12:01-13:12:02, 13:17, 13:22:01-13:23, 13:25, 13:28-13:29, 13:39, 13:43, 13:50, 13:52, 13:57, 13:60-13:61, 13:63N, 13:77-13:78, 13:80, 13:83						6		8														
*13:02:01-13:02:14, 13:02:16-13:02:18, 13:08-13:09, 13:14-13:15, 13:19, 13:27, 13:30, 13:32-13:34, 13:37-13:38, 13:40, 13:42, 13:44- 13:45, 13:49N, 13:54-13:56:02N, 13:58, 13:65-13:70, 13:74-13:75, 13:81-13:82, 13:84-13:85, 54:26, 55:22, 59:09	1			4		6													18			
*13:02:15, 13:47	1					6													18			
*13:03, 13:53, 35:60, 46:11, 46:18, 49:01:02, 49:11, 51:15, 51:157, 54:01:01-54:03, 54:05N, 54:07-54:08N, 54:12-54:13, 54:17-54:19, 54:21-54:25, 54:27, 54:29-54:30, 54:32, 54:34, 55:02:01:01-55:02:09, 55:07, 55:10, 55:12, 55:16, 55:18-55:19, 55:26, 55:30, 55:34-55:35, 55:41, 55:43, 55:47-55:48, 55:50, 55:57, 55:62, 55:65, 55:67, 55:69- 55:72, 56:01:01:01-56:01:04, 56:01:06-56:01:08, 56:07-56:08, 56:14, 56:16-56:17, 56:19N-56:20:02, 56:23-56:24, 56:26-56:30, 56:33-56:35, 56:38N-56:42, 56:44-56:49, 59:01:01:01-59:01:01:02, 59:04-59:08	1			4		6																

Negative Control

*45:09		2		5													19					
*45:10	1		4	5													18					
*47:03				w								15										
*49:07	1		4	6	7												18					
*49:10, 49:24, 49:31	1		4	6	7							15										
*49:20		2	4	6	7																	
*49:26	1		4	6	7												16					
*49:37	1		4		7																	
*51:23, 51:108, 52:15		2																				
*51:62			3		6	w																
*51:85, 51:192	1																17					
*51:106, 55:44, 55:53	1				6	w																
*52:26	1																17	19				
*54:20, 55:52	1		4			w																
*54:28	1		4	6					11													
*54:31, 55:39, 55:42, 55:61	1			6																		
*55:01:04	1		4	6	w												16					
*55:01:07				6	w																	
*55:63			4	6																		
*56:36	1		4	6													w					
*73:01-73:02	1																w	18				
C*01:90						w																
C*15:91					6	7																
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22