

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
HLA-B allele ^{1,2,3}																								
*50:01:01:01-50:01:01:20, 50:01:22-50:01:23, 50:22, 50:32, 50:34-50:36, 50:38, 50:44, 50:48, 50:50, 50:52-50:55, 50:57-50:65, 50:67-50:69, 50:71-50:74, 50:77-50:79	1			4	5	6	7																	
*50:01:21	1				5	6	7									16								
*50:02:01:01-50:02:01:02, 50:21, 50:56		2		4	5	6	7																	
*50:04:01-50:04:02	1		3	4	5	6	7																	
*50:05	1			4	5	6	7	8																
*50:06	1			4	5	6	7		9															
*50:07	1			4	5	6	7			10														
*50:08	1			4	5	6	7				11													
*50:09	1				5							12												
*50:10	1			4	5	6	7						13											
*50:11	1			4	5	6	7							14										
*50:12	1				5	6	7	9																
*50:13, 50:43	1			4	5	6	7								15									
*50:14	1				5											16								
*50:15	1			4	5	6	7									16								
*50:16	1				5	6	7			11														
*50:17	1				5	6	7	8																
*50:18	1			4	5	6	7					12												
*50:19	1			4	5	6	7				12			15										
*50:20	1				5												17							
*50:31	1			4	5	6	7											18						
*50:33	1				5														19					
*50:37	1			4	5	6	7													20				
*50:39	1			4	5																21			
*50:40	1			4	5	6	7										17							
*50:41	1				5	6	7														21			
*50:42	1				5	6	7																	
*50:45	1			4	5	6	7															22		
*50:46	1			4	5	6	7	9																
*50:47, 50:66, 50:70	1				4	5	6																	
*50:49	1			4	5	6																22		
*50:51					5	6	7																23	
*50:75, 49:01:01:01-49:01:01:18, 49:01:03-49:03, 49:06, 49:08-49:09, 49:12-49:14, 49:16-49:17, 49:19N, 49:21-49:23, 49:25, 49:27-49:30, 49:32-49:36, 49:38-49:40, 49:43-49:44, 49:46, 49:48-49:49, 49:51-49:60N, 49:62, 49:64-49:66, 49:68-49:75, 49:78, 59:12 ⁴	1			4		6	7																	
*50:76	1			4	5	w							14											
*07:78, 13:128, 55:09	1			4			w											18						
*07:84, 07:429, 13:35, 13:72, 13:127	1																	18	19					
*07:133, 08:155, 15:143, 15:212, 15:525, 18:48, 35:19, 35:274, 35:317, 39:69, 40:01:01-40:01:05, 40:01:07-40:03:02, 40:05:01:01-40:05:02, 40:07, 40:09, 40:11:01:01-40:11:02, 40:14:01-40:16:01:02, 40:18, 40:20:01:01-40:20:01:02, 40:22N-40:24, 40:27:01-40:27:03, 40:29-40:39, 40:42-40:43, 40:45, 40:50:01:01-40:52, 40:54-40:57, 40:60-40:63, 40:65-40:67, 40:69, 40:72:01-40:74, 40:77-40:82, 40:84-40:85, 40:87:01-40:92, 40:94, 40:97-40:102, 40:104-40:105, 40:107-40:108, 40:111-40:116, 40:118N-40:119, 40:121-40:126, 40:128, 40:130:01-40:130:02, 40:132-40:136, 40:138-40:147, 40:149-40:156, 40:158, 40:163, 40:168-40:169, 40:171-40:176, 40:178-40:184, 40:186:01-40:187, 40:189, 40:191-40:200, 40:202-40:207, 40:211-40:218, 40:220-40:228, 40:231, 40:233-40:243, 40:245:01-40:255, 40:257-40:264, 40:266-40:267, 40:271-40:274, 40:276-40:291N, 40:293, 40:296-40:297, 40:299-40:305, 40:309-40:310, 40:312, 40:314-40:316, 40:319-40:324, 40:326-40:329, 40:331-40:339, 40:344-40:347, 40:351-40:353:02, 40:355-40:360, 40:367-40:372N, 40:375, 40:378-40:386, 40:390-40:393, 40:395-40:398, 40:401-40:407, 40:409-40:410, 40:412-40:422, 40:424, 40:426N, 40:428N-40:429, 40:431-40:433, 40:435, 40:437-40:444, 40:446-40:448, 40:450, 40:452, 40:455-40:457, 40:461, 40:463, 40:465-40:467, 40:469-40:470, 40:472, 40:474-40:475:02, 40:477, 40:479-40:481N, 40:483N, 40:485-40:486, 40:489-40:491, 40:493, 40:495-40:499, 40:501-40:518, 40:521, 40:523-40:525, 41:02:01:01-41:02:08, 41:02:10-41:04, 41:08, 41:10-41:11, 41:13, 41:15, 41:18-41:19, 41:23-41:24, 41:27, 41:31, 41:36, 41:38-41:40, 41:42-41:43, 41:45N, 41:49, 41:51-41:52, 41:55, 41:58, 41:62-41:63, 41:65, 41:68-41:69, 41:71-41:72, 41:74, 48:41					5																			
*07:177, 27:157																w								
*07:182N, 07:424, 15:18:03, 56:63, A*02:339, A*02:510, C*03:19, C*03:102, C*03:307, C*03:318N, C*15:26				3																				
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24

Negative Control

*13:59, 13:140, 13:160, 27:130, 35:534, 39:50, 40:93, 40:96, 40:109, 40:340, 40:350, 51:10, 51:16, 51:31, 51:34, 51:82, 51:93, 52:08, 52:67, 78:08	1																		18							
*13:62, 27:153, 49:18:01, 51:103, 51:119, 51:158:02, 51:372, 52:83, 55:28:02	1																		16							
*13:129, 44:340						6	8											15								
*14:19, 14:27, 15:488-15:489, 35:261, 35:264, 35:547, 37:39, 37:48, 37:97, 53:07, C*03:110, C*03:250, C*03:267, C*03:419, C*03:559, C*04:08, C*04:34, C*04:147, C*04:212, C*05:27, C*05:39, C*05:151, C*05:184, C*08:41, C*08:115, C*08:138, C*08:182, C*17:07, C*18:08												12														
*14:37, 49:76	1			4			6	7																		22
*15:46, 15:53, 35:47, 35:154, 35:450, 40:10:01:01-40:10:02, 40:40, 40:49, 40:58, 40:164, 40:210, 40:219, 40:229, 40:325, 40:399N, 40:462, 41:47, 44:90, 47:02							5												15							
*15:73:01:01-15:73:01:02, 15:224, 18:191, 39:17, 56:04:01:01-56:04:04, 56:57			3				6																			
*15:106, 40:129, 44:09, 44:46:01-44:46:02, 44:75, 44:129, 44:254, 44:277, 44:438N							5												15							
*15:137, 49:50, 51:107, 51:172	1											12														
*18:107							5												15	16						
*18:150, 37:50									w																	
*27:14, 27:81	1																			17	18					
*35:63, 40:04:01:01-40:04:02, 40:59, 40:64:01:01-40:64:02, 40:120, 40:160:01-40:160:02, 40:343, 40:458, 40:500, 41:48, 44:131							5																			
*35:84, 35:441, 53:28																				8						16
*35:217, 40:28, 40:68																				8						16
*39:90, 51:55	1																	14								
*40:01:06, 40:08:01:01-40:08:01:02, 40:25-40:26, 40:106, 40:232, 41:02:09							5																			16
*40:06:01:01-40:06:12, 40:06:14-40:06:29, 40:53, 40:70:01-40:70:02, 40:75, 40:86, 40:103, 40:127, 40:131, 40:148, 40:161-40:162, 40:165, 40:167, 40:177, 40:190, 40:230, 40:244, 40:256N, 40:268, 40:270, 40:275, 40:294-40:295, 40:298:01-40:298:02, 40:306-40:307, 40:311, 40:317-40:318, 40:330, 40:341, 40:348-40:349, 40:354, 40:361N-40:364, 40:366, 40:374, 40:376, 40:387-40:389, 40:408, 40:411, 40:423, 40:425, 40:427, 40:434, 40:445, 40:451, 40:453-40:454, 40:459-40:460, 40:464, 40:468, 40:471, 40:473, 40:476, 40:478, 40:482, 40:484, 40:487N-40:488N, 40:492, 40:494, 40:519-40:520, 40:526	1						5																			18
*40:06:13, 40:269							5																			18
*40:06:30, 40:95, 40:313	1						5																			16
*40:44, 40:159	1						5																			18
*40:48, 40:71			3				5	6																		19
*40:83, 40:373	1						5				10															18
*40:110	1																		15							18
*40:166							5														w					
*40:170, 41:41							5				10															
*40:208										9																
*40:342, 40:449	1						5												15							18
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24		
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24		
*40:377					5	6																				
*40:436					5				9																	
*40:522			3		5																					
*41:01:01:01-41:01:08, 41:06-41:07, 41:09, 41:12, 41:14, 41:16, 41:20, 41:22, 41:25, 41:28-41:30, 41:32, 41:34-41:35, 41:37, 41:44, 41:50, 41:53-41:54, 41:57, 41:59, 41:61, 41:64, 41:67, 41:70, 41:75-41:79	1				5																				22	
*41:17, 41:56Q, 41:66	1				5																					
*41:21	1				5																				22	
*41:33	1				5														15						22	
*41:73	1				5					10															22	
*44:02:01:01-44:02:21, 44:02:23, 44:02:25-44:02:31, 44:02:33, 44:02:35-44:02:64, 44:02:66-44:03:04, 44:03:06-44:03:14, 44:03:17-44:03:29, 44:03:31-44:03:35, 44:03:37-44:03:39, 44:03:41-44:03:44, 44:03:46-44:05:06, 44:11, 44:14, 44:16-44:17, 44:19N, 44:21-44:27:02, 44:27:04-44:30, 44:32-44:39, 44:42-44:43:02, 44:45, 44:48, 44:50:01-44:53:02, 44:58N-44:59:02, 44:61N, 44:63-44:64:02, 44:66, 44:68, 44:71-44:74, 44:76-44:80, 44:84:01-44:89, 44:91-44:95, 44:98, 44:101-44:102, 44:104-44:105, 44:107-44:109, 44:112, 44:114-44:128:02, 44:132-44:133, 44:136-44:137, 44:139, 44:141-44:144, 44:146-44:148, 44:150-44:152, 44:154-44:155, 44:157, 44:159-44:165, 44:167-44:170, 44:172-44:183, 44:185, 44:187, 44:189-44:193, 44:195N-44:196, 44:198N-44:199, 44:201-44:208, 44:214-44:224, 44:227-44:229, 44:231, 44:233-44:235, 44:237N-44:245:02, 44:247-44:253, 44:255-44:258, 44:260-44:262, 44:265-44:276, 44:278-44:290, 44:292, 44:294-44:295:02, 44:297-44:316, 44:318-44:331, 44:333N-44:338, 44:341N-44:354N, 44:357-44:363, 44:437, 44:439, 44:441-44:443, 44:446-44:458, 44:460-44:524, 44:526-44:527, 44:529, 44:531-44:537, 44:539-44:542, 44:544N-44:546																										
*44:02:22, 44:02:24, 44:02:34, 44:02:65, 44:03:05, 44:03:15, 44:03:36, 44:03:45, 44:13, 44:31, 44:41:01-44:41:02, 44:54, 44:56N, 44:65, 44:67, 44:69:01-44:70, 44:82, 44:96-44:97, 44:99, 44:106, 44:110, 44:113, 44:135, 44:138Q, 44:149N, 44:153, 44:158, 44:166, 44:171N, 44:184, 44:211, 44:213, 44:230, 44:236, 44:259, 44:264, 44:291, 44:317, 44:332, 44:440, 44:444, 44:538, 44:543, 47:01:01:02-47:01:06, 47:04-47:12, 53:30, 57:45, 57:69, 58:111, C*16:85																									15	
*44:03:40, 44:62							8												15	16						
*44:07, 44:194:01-44:194:02							8			10																
*44:10					6	w													15							

Negative Control

Native Control

*44:12, 44:232, 44:263														15	16										
*44:15:01:01-44:15:01:02, 44:18:01:01-44:18:01:02, 44:355, 45:01:09, 45:16	2		4																						
*44:20, 44:47, 44:100, 44:197	2													15		17		19							
*44:140, 44:226		3					8							15											
*44:186:01-44:186:02, 44:296							8			12				15											
*44:209							w							15											
*44:339	2		4			6								15											
*44:356						5								15	w										
*44:459	1													15	16										
*45:01:01:01-45:01:08, 45:01:10-45:01:12, 45:03, 45:05-45:07, 45:11-45:15, 45:17-45:21, 45:23, 45:25, 45:27-45:28N, 45:30	2		4			5																			
*45:02	2					5					12														
*45:04	1		4			5																			
*45:08	1		4			5																	22		
*45:09		2				5														19					
*45:10	1		4			5													18						
*45:22		2	4											16											
*45:24			4			5																			
*45:29	2					5																			
*47:03						w								15											
*49:07	1		4			6	7												18						
*49:10, 49:24, 49:31	1		4			6	7							15											
*49:20		2	4			6	7																		
*49:26, 49:63, 49:67	1		4			6	7							16											
*49:37	1		4			7																			
*49:41, 54:20, 55:52	1		4			w																			
*49:42, 49:77	1		4			6	7		10																
*49:47:01-49:47:02, 56:66, C*15:91, C*15:128						6	7																		
*51:23, 51:108, 52:15		2																							
*51:62			3			6	w																		
*51:85, 51:192, 51:214	1																		17						
*51:106:01, 51:223, 52:25:02, 55:44, 55:53, 55:94	1					6	w																		
*51:261, 51:274, 52:26	1																		17		19				
*53:64																						20			
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	
*54:10, 54:16, 55:37, 55:46, 55:56, 56:01:05, 56:13	1			4																					
*54:28, 56:73	1			4							11														
*54:31, 55:39, 55:42, 55:61	1					6																			
*54:38						6																		23	
*55:01:04	1			4		6	w									16									
*55:01:07						6	w																		
*55:02:11	1					6										16									
*55:63, 55:120, 56:01:12, 56:77N				4		6																			
*55:122		2		4		6	w																		
*56:36	1			4		6										w									
*56:67	1			4		5	6								15										
*56:74		2		4		6																			
*73:01:01:01-73:03	1															w		18							
C*01:90							w																		23
C*01:136, C*01:145:01N, C*06:168, C*06:224, C*12:03:66, C*12:15, C*15:37, C*15:58																									23
C*07:463	1																				19				22
C*08:207						w																			
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	