

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21
HLA-B allele^{1,2,3}																					
*52:01:01:01-52:01:29, 52:22-52:24, 52:25:02, 52:33-52:35, 52:37-52:42, 52:44-52:48, 52:50-52:52, 52:54-52:62, 52:64-52:66, 52:68-52:70	1		3									12	13								
*52:02:01-52:02:02	1			4								12	13								
*52:03			3		5	6						12	13								
*52:04, 52:09 ⁵	1	2	3									12	13								
*52:05	1		3		5	6						12	13								
*52:06:01	1		3						9		11		13								
*52:06:02-52:06:03	1		3								11		13								
*52:07	1		3									12		14							
*52:08	1		3							10		12	13								
*52:10:01-52:10:03, 52:71			3			6						12	13								
*52:11			3		5							12	13								
*52:12, 52:26, 52:63	1		3									12	13		15						
*52:13	1		3				7					12	13								
*52:14, 52:49N ⁵	1		3							9		12	13								
*52:15	1		3			6						12	13								
*52:16	1											12	13								20
*52:17	1		3								11	12	13								
*52:18	1									10			13								
*52:19			3									12	13								
*52:20, 52:25:01 ⁵	1		3					8				12	13								
*52:21:01-52:21:02	1		3		5							12	13								
*52:27	1		3									12	13								20
*52:28-52:29, 53:17:01-53:17:02 ⁴			3									12	13								
*52:30	1		3									12	13								18
*52:31:01-52:31:02, 53:28 ⁴	1		3									12	13			16					
*52:32	1											12	13								
*52:36	1		3	4								12	13								
*52:43	1		3									12	13								17
*52:53	1		3									12	13								19
*52:67			3		5						10	12	13								
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21
*07:02:01:01-07:02:10, 07:02:12-07:05:02, 07:05:04-07:26, 07:28-07:37, 07:39-07:49N, 07:51-07:57, 07:59-07:77, 07:79-07:83, 07:85-07:217, 07:219-07:298, 08:01:01:01-08:01:36, 08:05, 08:07-08:08N, 08:10-08:16, 08:18-08:51, 08:53:01-08:53:02, 08:55-08:77, 08:79-08:83, 08:85-08:109, 08:111-08:116, 08:118-08:125, 08:127-08:184, 13:07N, 13:36, 13:39, 14:41N, 14:43, 14:46, 14:56, 15:01:01:01-15:01:01:12, 15:01:03-15:01:04, 15:01:06-15:02:11, 15:05:01-15:08:02, 15:10:01-15:12, 15:14-15:15, 15:18:01:01-15:19, 15:21, 15:23, 15:25:01-15:35, 15:39:01-15:40:02, 15:42, 15:44-15:46, 15:48-15:53, 15:55-15:58, 15:60, 15:63, 15:65-15:66, 15:70-15:82, 15:84-15:86, 15:88-15:90, 15:92-15:94N, 15:96-15:97, 15:99, 15:101, 15:104-15:122, 15:124-15:126, 15:128-15:129, 15:133-15:136, 15:138-15:150, 15:152-15:154, 15:159-15:161, 15:163-15:167, 15:169-15:172, 15:174-15:176, 15:178-15:195, 15:197:01-15:201, 15:203-15:207, 15:209N, 15:211-15:215, 15:217-15:219, 15:221, 15:223-15:227, 15:229, 15:231-15:234, 15:236-15:238, 15:240-15:241, 15:244-15:250, 15:255, 15:257-15:265, 15:267, 15:269-15:272N, 15:275:01-15:276, 15:279-15:280, 15:283, 15:285-15:292, 15:294N-15:304N, 15:306-15:309, 15:311-15:322, 15:324-15:334, 15:336-15:338, 15:340-15:355, 15:357-15:360, 15:363:01-15:363:02, 15:365-15:367, 15:370-15:375N, 15:377Q-15:378, 15:380N-15:388, 15:390-15:395, 15:398-15:402, 15:404-																					

Negative Control

*13:02:01:01-13:02:21, 13:08, 13:14-13:16, 13:18, 13:27, 13:30-13:34, 13:37-13:38, 13:40-13:42, 13:44-13:45, 13:47, 13:49N, 13:54-13:56:02N, 13:59, 13:65-13:66, 13:68-13:70, 13:74-13:75, 13:81-13:82, 13:84-13:85, 13:88-13:89, 13:91, 13:93-13:94, 13:96-13:105, 40:96, 40:109-40:110, 40:340, 49:07										10	12	14										
*13:04, 44:47, 44:100, 44:197, 49:04:01-49:05												12	14	15								
*13:35, 13:72										10	12	14	15									
*13:46							7		9		12	14	15									
*13:58			3					8		10	12	14										
*13:62, 49:09	1										12	14										
*15:04:01:01-15:04:04, 15:155, 15:293, 15:310, 41:21, 45:09, 46:32, 50:33, 54:14-54:15, 55:23, 55:27, 55:68, 56:50													14	15								
*15:13:01-15:13:02, 15:17:01:01-15:17:06, 15:162, 15:168, 15:177, 15:196, 15:208, 15:216, 15:230, 15:268, 15:273, 15:356, 15:396, 15:403, 15:411, 15:418, 15:423-15:424, 57:01:01:01-57:01:10, 57:01:12-57:11, 57:13-57:15, 57:17-57:44, 57:46-57:50, 57:52-57:59, 57:61-57:68, 57:70-57:90			4										14									
*15:16:01:01-15:16:03, 15:67, 15:95, 15:222, 15:254, 15:362, 15:408			4										14	15								
*15:20, 15:228, 15:305, 35:01:01:01-35:01:07, 35:01:09, 35:01:11-35:01:14, 35:01:16-35:01:27, 35:01:29-35:01:46, 35:02:01:01-35:04:01, 35:04:03-35:09:03, 35:11:01-35:12:04, 35:14:01-35:15:02, 35:17:01-35:20:02, 35:22-35:27, 35:29:01, 35:30, 35:32:01-35:34, 35:36, 35:38-35:42:02, 35:44-35:45, 35:47-35:56, 35:58-35:59:02, 35:61:01-35:68:02, 35:70-35:72, 35:74-35:78, 35:81-35:86, 35:88-35:95, 35:97-35:184, 35:186-35:187, 35:189-35:225, 35:227-35:234, 35:236-35:250, 35:252-35:264, 35:266-35:269, 35:271-35:275, 35:277-35:278, 35:280-35:290, 35:292-35:317, 35:319-35:328, 35:330-35:349, 53:40, 58:01:01:01-58:01:21, 58:02:01-58:02:02, 58:04-58:07, 58:09-58:10N, 58:12-58:19, 58:21-58:29, 58:31N-58:35, 58:37-58:72N, 58:74-58:91, 67:05, 81:03												13										
*15:24:01-15:24:02, 15:157				4								12	14									
*15:87		2		4								12	14									
*15:137, 39:06:01-39:06:05, 39:33-39:34, 39:57, 39:62, 39:64, 39:83, 39:90, 39:102, 39:109, 39:127, 39:129, 41:50, 45:02, 50:09, 55:13, 55:58, 56:22									9				14									
*15:202, 15:239, 15:376, 37:57, 40:185, 40:209, 40:308			3										14									
*15:361				4	5								14									
*18:09								8					14								20	
*18:12:01-18:12:02, 37:01:09, 37:05, 37:11, 37:37, 37:58			3										14								20	
*18:15, 18:19, 18:30, 18:57	1												14								20	
*35:01:08, 35:01:15, 35:29:02, 35:276									9				13									
*35:01:10, 35:04:02					5	6							13		16							
*35:10, 35:13, 35:16, 35:28, 35:69, 35:80, 35:226, 35:270, 35:318			3										13									
*35:21, 35:96, 35:188, 51:01:17, 51:07:01-51:07:02, 51:22, 51:118N, 51:214, 51:222, 56:05:01-56:05:02, 56:21, 58:08:01-58:08:02, 78:01:01:01, 78:01:02-78:03, 78:07, 78:09	1												13									
*35:37, 35:265													13	15								
*35:60, 56:45								8					14									
*35:235									9				13	15								
*35:251, 78:04					5	6							13									
*35:291, 53:09, 58:73								8					13									
*35:329, 51:02:04, 51:53, 51:81, 51:115, 51:120, 51:148, 51:175, 51:181, 51:199, 53:01:01-53:04, 53:07-53:08:02, 53:10, 53:14-53:15, 53:18-53:27, 53:29-53:30, 53:32, 53:34-53:35, 53:37, 53:39, 53:41-53:48N											11	13										
*37:01:01:01-37:01:05, 37:01:07-37:01:08, 37:01:10-37:04:02, 37:07, 37:09, 37:12-37:20, 37:23-37:25, 37:27, 37:29-37:36, 37:38-37:56, 37:59, 37:62-37:66, 37:68			3									12	14								20	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	
*37:01:06, 37:06:01-37:06:02, 37:28, 37:61, 38:03, 44:130, 44:156, 44:221, 44:224, 44:258, 48:18, 49:25												12	14								20	
*37:10			3					8				12	14								20	
*37:21			3								11		14									
*37:22, 37:26			3									12	14									
*39:50, 40:86, 40:330									9	10			14									
*40:01:06					5	6							14									
*40:06:08											10											
*40:28	1												14	16								
*40:95					5					10			14									
*40:350				3						10			14									
*44:03:40					5	6						12	14	16								
*44:20							7					12	14	15								
*44:62					5	w						12	14	16								
*44:83, 44:134, 49:03			3					8				12	14									
*48:02:01													13								20	
*48:02:02, 56:36													13	14								
*48:02:03													13	14							20	
*49:18:01					5							12	14									

Negative Control

*51:01:01:01-51:01:16, 51:01:18-51:01:32, 51:01:34-51:01:55, 51:01:57, 51:03, 51:06:01-51:06:03, 51:11N-51:12, 51:14, 51:17-51:18, 51:21, 51:24:01-51:24:05, 51:26-51:30, 51:32-51:33, 51:35, 51:38-51:39, 51:41N, 51:43, 51:45, 51:48-51:52, 51:55, 51:57-51:58, 51:60, 51:63:01-51:64, 51:66-51:68, 51:70-51:72, 51:74-51:77, 51:79-51:80, 51:83-51:89, 51:91, 51:94-51:96, 51:98N-51:103, 51:105-51:107, 51:109-51:114, 51:116-51:117, 51:119, 51:121, 51:123-51:138, 51:140, 51:142, 51:145, 51:147, 51:149N-51:152, 51:154-51:156, 51:158:01-51:163, 51:165-51:171, 51:173Q-51:174:02, 51:176-51:179, 51:182-51:184N, 51:186-51:193, 51:195-51:196, 51:198, 51:200-51:213, 51:215, 51:218-51:219, 51:221, 51:223-51:224, 51:226, 53:06	1									11	13										
*51:01:33, 51:13:01-51:13:02, 51:172	1							9	11	13											
*51:01:56, 51:164	1								11	13											
*51:02:01:01-51:02:03, 51:02:05, 51:59, 51:143-51:144, 51:146, 51:180					5				11	13											
*51:04, 51:46, 51:56:01-51:56:03	1								11	13		16									
*51:05, 51:23					5	w			11	13											
*51:08:01-51:08:02, 51:20, 51:44N, 51:97, 51:141, 51:153							7		11	13											
*51:09:01-51:09:03, 51:19, 51:73, 51:122, 51:194					6				11	13											
*51:10									10	11	13										
*51:15, 53:11-53:13, 53:31, 53:36							8		11	13											
*51:16, 51:82	1								10	11	13										
*51:31					6				10	11	13										
*51:34					5				10	11	13										
*51:36					5	7			11	13											
*51:37, 51:90, 51:216	1								11	13	15										
*51:40, 51:108					5	6			11	13											
*51:54					5	w	8		11	13											
*51:61:01-51:61:02	1				5				11	13											
*51:62, 51:78:01-51:78:02, 51:225	1						8		11	13											
*51:65	1								11	13									17		
*51:69	1								11	13									18		
*51:92:01-51:92:02	1							9	11	13	15										
*51:93					5	6			10	11	13										
*51:104	1		4						11	13											
*51:139					6				11	13		16									
*51:157							8		11		14										
*51:185	1	2							11	13											
*51:197									11	13	15										
*51:217					w				11	13											
*51:220, A*24:328									11												
*53:05:01-53:05:02, 53:16, 53:33	2								11	13											
*53:38		3					8			12	13										
*57:01:11, 57:91			4																		
*57:60	2	4									14										
*58:36		4									13										
*59:03								9	11		14										
*59:09									10	11	14										
*73:01-73:02								9	10												
*78:01:01:02, C*01:30, C*08:51, C*08:114, C*12:87, C*14:76, C*15:39, C*15:130	1																				
*78:05-78:06	1	3									13										
*78:08					6				10		13	14									
*83:01											?	?									
C*02:02:23, C*12:03:31					5																
C*05:152								9													
C*07:463, C*12:183													15								
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21

Negative Control