

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
<b>HLA-B allele<sup>1,2,3</sup></b>																								
*53:01:01-53:01:10, 53:25, 53:27, 53:32, 53:34-53:35, 58:01:14 <sup>4</sup>	1								9	10	11					16								
*53:02	1	2				6			9	10	11					16								
*53:03, 35:203 <sup>4</sup>			3						9	10	11					16								
*53:04	1			4					9							16								
*53:05	1		3			6			9	10	11					16								
*53:06	1	2				6		8	9		11					16								
*53:07	1			4			7		9							16								
*53:08:01-53:08:02	1							8			11					16								
*53:09					5				9	10	11				15	16								
*53:10	1						7		9	10	11			14		16								
*53:11					5				9	w	11									20				
*53:12					5	6			9	10	11					16								
*53:13					5				9	10	11					16								
*53:14	1								9	10	11			14		16								
*53:15	1		3						9	10	11					16								
*53:16	1		3			6			9	w	11								20					
*53:17:01-53:17:02	1								9	10	11				15	16								
*53:18	1			4					9	10	11					16								
*53:19	1								9							16							22	
*53:20	1								9	10	11					16	17							
*53:21	1								9	10	11					16		18						
*53:22	1				5				9		11													
*53:23	1								9	10	11					16			19					
*53:24	1								9	w	11								20					
*53:26	1						7		9	10	11					16								
*53:28	1	2				6		8			11				15	16								
*53:29	1								9	10	11					16								23
*53:30, 35:01:10, 35:46-35:47, 35:207, 35:217 <sup>4</sup>									9	10	11					16						21		
*53:31				4	5				9							16								
*53:33	1		3	4		6			9							16								
*07:02:01-07:08, 07:10, 07:12-07:16, 07:18:01-07:19, 07:21-07:23, 07:27-07:33:03, 07:35-07:42, 07:44-07:47, 07:49N-07:50, 07:52-07:59, 07:61-07:64, 07:66-07:77, 07:79, 07:81-07:114, 07:116-07:130, 07:132-07:133, 07:135N-07:146, 07:148-07:154, 07:156-07:163, 07:165-07:201N, 07:203-07:208, 07:210-07:219, 08:20, 08:53:01-08:53:02, 08:79, 14:03, 15:30, 15:33, 15:45, 15:63, 15:77-15:78:03, 15:116, 15:141, 15:150, 15:168, 15:233, 15:248, 15:269, 15:287, 27:25, 27:108, 27:115, 35:44, 35:118, 35:213, 39:18, 39:36, 40:15-40:16, 40:23, 40:32, 40:98, 40:161, 40:198, 44:146, 44:150, 46:01:06, 46:45, 46:54, 48:05, 48:08, 57:09, 57:13								8																
*07:09, 07:11, 07:17, 07:115								8						14										
*07:20, 07:24, 07:60, 15:99, 44:76, 57:63								8												20				
*07:26, 07:34, 07:43, 07:80, 07:131, 07:147, 13:20, 13:62, 14:01:01-14:02:09, 14:02:11-14:02:12, 14:04, 14:06:01-14:09, 14:11-14:18, 14:20-14:26, 14:28-14:36, 14:38-14:40, 15:09-15:10:03, 15:17:01:01-15:17:02, 15:37, 15:72, 15:90, 15:177, 15:196, 15:208, 15:216, 15:230, 15:243, 15:252, 15:273, 15:313, 27:04:01-27:04:04, 27:06, 27:10, 27:15, 27:18, 27:20-27:21, 27:24, 27:40, 27:54, 27:63, 27:66N, 27:68-27:69, 27:79, 27:86, 27:91-27:92, 27:100, 27:103, 27:105-27:106, 27:109, 27:112-27:114, 27:120, 38:10, 38:32, 39:35, 40:05, 40:26, 40:51, 40:95, 40:148, 40:174, 40:183, 42:19, 44:79:01-44:79:02, 48:15, 49:18, 50:14, 50:20, 51:01:40, 51:104, 51:112, 51:147, 52:02:01-52:02:02, 52:16, 52:32, 55:01:04, 55:01:07, 55:17, 55:28, 55:44, 55:53, 55:58, 56:05:01-56:05:02, 56:21, 57:57, 78:02:01-78:02:02								8							16									
<b>Well No.</b>	<b>1</b>	<b>2</b>	<b>3</b>	<b>4</b>	<b>5</b>	<b>6</b>	<b>7</b>	<b>8</b>	<b>9</b>	<b>10</b>	<b>11</b>	<b>12</b>	<b>13</b>	<b>14</b>	<b>15</b>	<b>16</b>	<b>17</b>	<b>18</b>	<b>19</b>	<b>20</b>	<b>21</b>	<b>22</b>	<b>23</b>	<b>24</b>
<b>Well No.</b>	<b>1</b>	<b>2</b>	<b>3</b>	<b>4</b>	<b>5</b>	<b>6</b>	<b>7</b>	<b>8</b>	<b>9</b>	<b>10</b>	<b>11</b>	<b>12</b>	<b>13</b>	<b>14</b>	<b>15</b>	<b>16</b>	<b>17</b>	<b>18</b>	<b>19</b>	<b>20</b>	<b>21</b>	<b>22</b>	<b>23</b>	<b>24</b>

Negative Control







*18:10-18:11, 18:35									10	11			15	16	18										
*18:13					6				w	11			15		18										
*18:14					6				10	11		14	15	16	18										
*18:15, 18:57					6		8			11			15	16	18										
*18:18					6	7			10	11			15	16	18										
*18:19, 18:30					6		8			11			15		18										
*18:20					6				10	w			15	16	18										
*18:21							8						15		18										
*18:22, 18:69		2			6				10	11			15	16	18										
*18:26, 18:41, 18:61, 40:209													15	16	18										
*18:36					6				9	10	11		15	16	18										
*18:37, 18:48, 18:66, 39:43, 56:31					6					10	11			16											
*18:54		3			6				10	11			15	16	18										
*18:56					5				9	11			15		18										
*18:58									9	10	11		15	16	18										
*18:67		1	3		6				10	11			15	16	18										
*18:101							8			11			15	16	18										
*27:52			3										16		18										
*35:02:01-35:02:06, 35:83, 35:129N, 35:182-35:184, 35:211, 35:220									9					16					22						
*35:03:01-35:03:05, 35:03:07-35:03:14, 35:06, 35:13, 35:36, 35:55-35:56, 35:59, 35:70, 35:74-35:75, 35:85, 35:98, 35:106, 35:127-35:128, 35:150-35:153, 35:155, 35:160, 35:179, 35:181, 35:193, 35:195, 35:198, 35:204-35:205, 35:221, 35:223, 35:235-35:236, 35:242-35:243, 35:246, 39:32, 39:74, 40:71, 44:10, 44:178, 56:02, 56:04, 56:11, 56:22				4					9					16											
*35:04:02, 35:251, 78:04									9					16					21						
*35:05:03, 35:51									9	10	11		14	16											
*35:18, 35:164, 44:28:01-44:28:02, 57:02:01-57:02:02, 57:04-57:05, 57:12, 57:28N, 57:42									9										20						
*35:20:01-35:20:02, 35:28, 35:226									9	10	11			15	16										
*35:21, 35:188		2			6		8			11				16											
*35:24:01-35:24:02		2			6				9	10	11			16											
*35:29:01-35:29:02, 35:69, 35:76									9	10	11			16					19						
*35:35										w	11			16											
*35:38, 35:163, 35:169, 38:30, 44:140, 45:02, 82:01-82:03, C*04:34				4					9																
*35:50, 35:197									9	10	11			16		18									
*35:58							8			11			14	16											
*35:62, 57:27									9	w	11														
*35:66										w	11			14											
*35:72									9	10	11			14	15	16									
*35:81		2							9					16					22						
*35:84, 35:231			4						9					16		18									
*35:95									9					15	16				22						
*35:108:01							7		9	10	11			16											
*35:109		2	4						9					16											
*35:136			4						9					16					19						
*35:141, 46:25									9	11				16											
*35:146									9	10	11			16					22						
*35:154									9					16					21 22						
*35:156, 39:33			4						9										20						
*35:157		2												16											
*35:162									9					16		18			22						
*35:166									9	10	11														
*35:172									9										20 22						
*35:174									9	w									20						
*35:186									9	10	11			16		w									
*35:190		2			6					10	11			16											
*35:208									9	10	11			16					23						
*35:229									9	w	11								19 20						
*35:232							8			11			14						20						
*35:233		2							9					16											
*35:237			4											16											
<b>Well No.</b>	<b>1</b>	<b>2</b>	<b>3</b>	<b>4</b>	<b>5</b>	<b>6</b>	<b>7</b>	<b>8</b>	<b>9</b>	<b>10</b>	<b>11</b>	<b>12</b>	<b>13</b>	<b>14</b>	<b>15</b>	<b>16</b>	<b>17</b>	<b>18</b>	<b>19</b>	<b>20</b>	<b>21</b>	<b>22</b>	<b>23</b>	<b>24</b>	
<b>Well No.</b>	<b>1</b>	<b>2</b>	<b>3</b>	<b>4</b>	<b>5</b>	<b>6</b>	<b>7</b>	<b>8</b>	<b>9</b>	<b>10</b>	<b>11</b>	<b>12</b>	<b>13</b>	<b>14</b>	<b>15</b>	<b>16</b>	<b>17</b>	<b>18</b>	<b>19</b>	<b>20</b>	<b>21</b>	<b>22</b>	<b>23</b>	<b>24</b>	
*37:01:01-37:01:05, 37:01:07-37:01:09, 37:03N, 37:05, 37:08, 37:15-37:18, 37:20, 37:22-37:25, 37:27, 37:30N, 37:32-37:33N, 37:36, 37:38, 37:41, 37:43, 37:45-37:47, 37:49			3				7								15					18					
*37:01:06							7														18				
*37:02			3												15	16					18				

Negative Control

*37:04:01-37:04:02		2	3			7							15	18											
*37:07			3				8						15	18											
*37:09, 37:12, 37:19, 37:44			3										15	18											
*37:10					5	7							15	18											
*37:11, 37:14, 37:29						7							15	18											
*37:13			3			7	8						15	18											
*37:21, 37:26			3			7								18											
*37:31, 37:42N			3			7							15												
*37:34	1					7							15	18											
*37:35			3		5	7			11				15	18											
*37:37						7							15	18	19										
*37:39			3	4		7		9					15	18	18	20									
*37:40			3			7							15	16	18										
*37:48			3	4		7		9					15	18											
*38:06-38:07	1												16	18											
*39:09:01-39:09:02, 39:88						7							16												
*39:19:01-39:19:02													16	18	19										
*39:24:01-39:24:02, 39:37, 39:76												14	16												
*40:01:06, 40:07-40:08, 40:13, 40:68, 40:106, 40:162, 40:220, 40:232													16					21							
*40:03, 40:267, 40:284								10				14	16												
*40:20, 40:38, 40:52, 40:59-40:60, 40:105, 40:159, 46:06, 48:23								10					16												
*40:185													15	16											
*41:24					5				11			14													
*44:02:08, 44:101								w																	
*44:03:17								w					16												
*44:06, 51:08:01-51:08:02, 51:44N, 51:141	1							9																	
*44:15				4		6		9		12															
*44:18, 45:01:01-45:01:02, 45:04-45:07, 45:13-45:14, 56:13, 56:43				4				9		12															
*44:83, 44:134					5			9					15												
*44:96, 44:153								9					16					23							
*44:109						6		9					16												
*44:117				w				9																	
*44:176						6		9																	
*44:190, C*15:39		2																							
*45:03								9		12															
*45:11				4						12															
*45:12				4				9		12								22							
*46:09, 57:01:17								9		11															
*46:26							8	9	10	11			16												
*46:40							8			11					w										
*49:01:02				4			8			12			w												
*49:03, 52:25	1			4			8			12			15	16											
*50:36				4			8			12			16					22							
*51:01:01-51:01:39, 51:01:41-51:01:46, 51:01:48-51:03, 51:06:01-51:06:02, 51:11N-51:12, 51:14, 51:16-51:18, 51:21-51:24:05, 51:26-51:28, 51:30, 51:32-51:35, 51:38-51:39, 51:41N, 51:43, 51:48-51:52, 51:58-51:60, 51:64-51:71, 51:75-51:77, 51:79-51:80, 51:83-51:86, 51:88-51:90, 51:94-51:96, 51:98N-51:102, 51:105, 51:107, 51:109-51:111, 51:113-51:114, 51:117-51:119, 51:121, 51:123-51:138, 51:140, 51:142-51:146, 51:149N-51:152, 51:154-51:156, 51:158-51:171, 51:173Q, 52:01:13, 52:06:01-52:06:02, 52:22-52:23	1						8						16												
*51:01:47, 51:36, 51:55, 51:61:01-51:61:02, 51:82	1						8																		
*51:04, 51:46, 51:56:01-51:56:02	1	2					8						16												
*51:05	1							9								20	w								
*51:07:01-51:07:02, 52:01:01:01-52:01:12, 52:01:14-52:01:19, 52:04-52:05, 52:07-52:08, 52:11-52:13, 52:15, 52:17-52:18, 52:26-52:27, 52:30-52:31:02, 52:33-52:38	1						8						15	16											
*51:09:01-51:09:02, 51:40, 51:73, 51:108, 51:122	1							9					16					21							
<b>Well No.</b>	<b>1</b>	<b>2</b>	<b>3</b>	<b>4</b>	<b>5</b>	<b>6</b>	<b>7</b>	<b>8</b>	<b>9</b>	<b>10</b>	<b>11</b>	<b>12</b>	<b>13</b>	<b>14</b>	<b>15</b>	<b>16</b>	<b>17</b>	<b>18</b>	<b>19</b>	<b>20</b>	<b>21</b>	<b>22</b>	<b>23</b>	<b>24</b>	
<b>Well No.</b>	<b>1</b>	<b>2</b>	<b>3</b>	<b>4</b>	<b>5</b>	<b>6</b>	<b>7</b>	<b>8</b>	<b>9</b>	<b>10</b>	<b>11</b>	<b>12</b>	<b>13</b>	<b>14</b>	<b>15</b>	<b>16</b>	<b>17</b>	<b>18</b>	<b>19</b>	<b>20</b>	<b>21</b>	<b>22</b>	<b>23</b>	<b>24</b>	
*51:10, 51:81, 51:115	1														16										
*51:13:01-51:13:02, 51:62, 51:92, 51:106, 51:172	1			4				8							16										
*51:15	1			4					9			12			16										
*51:19, 51:42, 51:148	1								9						16										
*51:20	1								9									19							
*51:29	1							8											20						
*51:31, 51:93	1														16								21		

Negative Control

*51:37, 51:45, 51:63	1				6	8		11				16												
*51:53, 51:120	1											w												
*51:54					5			9							20	w								
*51:57, 51:103, 51:116	1						8					16		18										
*51:72, 51:87, 51:91	1						8					w												
*51:74	1						8					16							21					
*51:78:01-51:78:02					5		8					16												
*51:97	1				5	6		9	11															
*51:139	1	2						9				16							21					
*51:153	1																							
*51:157	1									12		16												
*52:01:20	1						8					15	w											
*52:03, 52:10:01-52:10:03	1							9				15	16						21					
*52:09	1	3					8					15	16											
*52:14	1		4				8					15	16											
*52:19	1							9				15												
*52:20					5		8					15	16											
*52:21	1						8					15												
*52:24	1						8					15							20					
*52:28-52:29	1											15	w											
*52:39	1				6	8		11				15	16											
*55:14							8		11			16								23				
*55:54									12			w												
*56:01:05								9		12		16												
*56:06, 78:01:01-78:01:02, 78:03, 78:07							8					16			19									
*57:14		2			6			9	10	11		16								23				
*57:30									11						20									
*58:01:01-58:01:02, 58:01:04-58:01:13, 58:01:15-58:02, 58:04, 58:07, 58:10N-58:13, 58:15-58:16:02, 58:18, 58:21, 58:23-58:26, 58:29, 58:31N-58:32, 58:39N-58:43, 58:45:01-58:45:02, 58:47-58:59	1							9	10	11	13	16												
*58:01:03	1							9	10		13	16												
*58:05	1								11	13	16													
*58:06, 58:19	1						8		11	13	16													
*58:08:01-58:08:02, 58:46	1						8			13	16													
*58:09	1	2			6			9	10	11	13	16												
*58:14	1							9	10	11	13	16								23				
*58:17N								9	10	11	13	16												
*58:20, 58:27	1									13	16													
*58:22	1							9	w	11	13				20									
*58:28	1							9			13	16												
*58:33	1		4					9	10	11	13	16												
*58:34	1		4					9			13	16												
*58:35, 58:37-58:38	1								10	11	13	16												
*78:05-78:06							8					15	16											
A*02:188, A*24:174									w															
C*02:56, C*06:20, C*07:81										w														
C*03:19, C*03:26, C*03:102				w																				
C*03:28, 03:49, C*14:20																			20					
C*04:08, C*04:147, C*05:39				4															20					
C*05:27, C*08:41, C*15:15, C*17:07				4																				
C*07:168						7				w														
C*07:342, C*14:53									10															
<b>Well No.</b>	<b>1</b>	<b>2</b>	<b>3</b>	<b>4</b>	<b>5</b>	<b>6</b>	<b>7</b>	<b>8</b>	<b>9</b>	<b>10</b>	<b>11</b>	<b>12</b>	<b>13</b>	<b>14</b>	<b>15</b>	<b>16</b>	<b>17</b>	<b>18</b>	<b>19</b>	<b>20</b>	<b>21</b>	<b>22</b>	<b>23</b>	<b>24</b>

Negative Control