

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
<b>HLA-B allele<sup>1,2,3</sup></b>																								
*53:01:01-53:01:11, 53:25, 53:27, 53:32, 53:34-53:35, 53:41, 53:43	1								9	10	11					16								
*53:02	1	2				6			9	10	11					16								
*53:03			3			6			9	10	11					16								
*53:04	1			4					9							16								
*53:05	1		3			6			9	10	11					16								
*53:06	1	2				6		8			11					16								
*53:07	1			4			7		9							16								
*53:08:01-53:08:02	1							8			11					16								
*53:09, 53:38					5				9	10	11				15	16								
*53:10	1						7		9	10	11			14		16								
*53:11					5				9	w	11									20				
*53:12					5	6			9	10	11					16								
*53:13					5				9	10	11					16								
*53:14	1								9	10	11			14		16								
*53:15	1		3						9	10	11					16								
*53:16	1		3			6			9	w	11									20				
*53:17:01-53:17:02, 53:40	1								9	10	11				15	16								
*53:18	1			4					9	10	11					16								
*53:19	1								9							16						22		
*53:20	1								9	10	11					16	17							
*53:21	1								9	10	11					16		18						
*53:22	1				5				9		11													
*53:23	1								9	10	11					16				19				
*53:24	1								9	w	11									20				
*53:26	1						7		9	10	11					16								
*53:28	1	2				6		8			11				15	16								
*53:29	1								9	10	11					16								23
*53:30	1								9	10	11					16	17					21		
*53:31				4	5				9							16								
*53:33	1		3	4		6			9							16								
*53:36					5				9							16							22	
*53:37	1								9	10	11					16							22	
*53:39									9	10	11					16	17							
*53:42, 51:19, 51:42, 51:148 <sup>4</sup>	1								9							16								
*07:02:01-07:08, 07:10, 07:12-07:16, 07:18:01-07:19, 07:21-07:23, 07:27-07:33:03, 07:35-07:42, 07:44N-07:47, 07:49N-07:50, 07:52-07:59, 07:61-07:64, 07:66-07:77, 07:79, 07:81-07:114, 07:116-07:130, 07:132-07:133, 07:135N-07:146, 07:148-07:154, 07:156-07:163, 07:165-07:201N, 07:203-07:208, 07:210-07:219, 07:221-07:222, 07:224-07:225, 07:227-07:234, 07:236, 07:238-07:245, 07:247-07:260, 07:262-07:270, 07:272N-07:275, 08:20, 08:53:01-08:53:02, 08:79, 08:156, 14:03, 14:47, 15:30, 15:33, 15:45, 15:63, 15:77-15:78:03, 15:116, 15:141, 15:150, 15:168, 15:233, 15:248, 15:269, 15:287, 15:324, 15:334, 15:342, 15:352, 15:375N, 27:25, 27:108, 27:115, 35:44, 35:118, 35:213, 39:18, 39:36, 40:15-40:16, 40:23, 40:32, 40:98, 40:161, 40:198, 44:146, 44:150, 46:01:06, 46:45, 46:54, 48:05, 48:08								8																
*07:09, 07:11, 07:17, 07:115, 07:246, 57:09, 57:13								8						14										
*07:20, 07:24, 07:60, 15:99, 44:76								8												20				
*07:26, 07:34, 07:43, 07:80, 07:131, 07:147, 13:20, 13:62, 14:01:01-14:02:09, 14:02:11-14:02:14, 14:04, 14:06:01-14:09, 14:11-14:18, 14:20-14:26, 14:28-14:36, 14:38-14:40, 14:42, 14:44-14:46, 14:48-14:54, 15:09-15:10:04, 15:17:01:01-15:17:04, 15:37, 15:72, 15:90, 15:177, 15:196, 15:208, 15:216, 15:230, 15:243, 15:252, 15:273, 15:313, 15:323, 15:329, 15:338-15:339, 15:356, 15:361, 27:04:01-27:04:05, 27:06, 27:10, 27:15, 27:18, 27:20-27:21, 27:24, 27:40, 27:54, 27:63, 27:66N, 27:68-27:69, 27:79, 27:86, 27:91-27:92, 27:100, 27:103, 27:105, 27:109, 27:112-27:114, 27:120, 27:138, 27:149, 27:153, 38:10, 38:32, 39:35, 39:107, 40:05, 40:26, 40:51, 40:95, 40:148, 40:174, 40:183, 40:309, 42:19, 44:79:01-44:79:02, 48:15, 50:14, 50:20, 51:01:40, 51:104, 51:147, 52:02:01-52:02:02, 52:16, 52:32, 55:01:04, 55:01:07, 55:17, 55:28, 55:44, 55:53, 55:58, 56:05:01-56:05:02, 56:21, 78:02:01-78:02:02, 78:09								8							16									
<b>Well No.</b>	<b>1</b>	<b>2</b>	<b>3</b>	<b>4</b>	<b>5</b>	<b>6</b>	<b>7</b>	<b>8</b>	<b>9</b>	<b>10</b>	<b>11</b>	<b>12</b>	<b>13</b>	<b>14</b>	<b>15</b>	<b>16</b>	<b>17</b>	<b>18</b>	<b>19</b>	<b>20</b>	<b>21</b>	<b>22</b>	<b>23</b>	<b>24</b>
<b>Well No.</b>	<b>1</b>	<b>2</b>	<b>3</b>	<b>4</b>	<b>5</b>	<b>6</b>	<b>7</b>	<b>8</b>	<b>9</b>	<b>10</b>	<b>11</b>	<b>12</b>	<b>13</b>	<b>14</b>	<b>15</b>	<b>16</b>	<b>17</b>	<b>18</b>	<b>19</b>	<b>20</b>	<b>21</b>	<b>22</b>	<b>23</b>	<b>24</b>

Negative Control









*35:136			4					9							16		19							
*35:141, 46:25								9	11						16									
*35:146								9	10	11					16						22			
*35:154								9							16					21	22			
*35:156, 39:33, 56:51			4					9									20							
*35:157		2													16									
*35:162								9							16		18				22			
*35:166								9	10	11														
*35:172								9												20	22			
*35:174								9	w											20				
*35:186								9	10	11					16		w							
*35:190		2			6				10	11					16									
*35:203			3					9	10	11					16									
*35:208								9	10	11					16							23		
*35:229								9	w	11							19	20						
*35:232							8			11		14						20						
*35:233, 44:203		2						9							16									
*35:237			4												16									
*35:257, 35:278			4					9							16						21			
*35:260				5				9																
*35:262									w	11														
*35:296			4					9						15	16									
*35:316								9							16		19							
*37:01:01-37:01:05, 37:01:07-37:01:11, 37:03N, 37:05, 37:08, 37:15-37:18, 37:20, 37:22-37:25, 37:27, 37:30N, 37:32-37:33N, 37:36, 37:38, 37:41, 37:43, 37:45-37:47, 37:49-37:54, 37:58-37:59			3			7								15		18								
*37:01:06						7											18							
*37:02		3												15	16		18							
*37:04:01-37:04:02		2	3			7								15		18								
*37:07			3				8							15		18								
*37:09, 37:12, 37:19:01-37:19:02, 37:44			3											15		18								
*37:10				5	7									15		18								
*37:11, 37:14, 37:29					7									15		18								
*37:13			3			7	8							15		18								
*37:21, 37:26, 37:57			3			7										18								
*37:31, 37:42N			3			7								15										
*37:34		1				7								15		18								
*37:35			3	5	7					11				15		18								
*37:37					7									15		18	19							
<b>Well No.</b>	<b>1</b>	<b>2</b>	<b>3</b>	<b>4</b>	<b>5</b>	<b>6</b>	<b>7</b>	<b>8</b>	<b>9</b>	<b>10</b>	<b>11</b>	<b>12</b>	<b>13</b>	<b>14</b>	<b>15</b>	<b>16</b>	<b>17</b>	<b>18</b>	<b>19</b>	<b>20</b>	<b>21</b>	<b>22</b>	<b>23</b>	<b>24</b>
<b>Well No.</b>	<b>1</b>	<b>2</b>	<b>3</b>	<b>4</b>	<b>5</b>	<b>6</b>	<b>7</b>	<b>8</b>	<b>9</b>	<b>10</b>	<b>11</b>	<b>12</b>	<b>13</b>	<b>14</b>	<b>15</b>	<b>16</b>	<b>17</b>	<b>18</b>	<b>19</b>	<b>20</b>	<b>21</b>	<b>22</b>	<b>23</b>	<b>24</b>
*37:39			3	4			7		9						15		18	20						
*37:40			3				7								15	16		18						
*37:48			3	4			7		9						15		18							
*37:55			3			6	7								15									
*37:56			3		5						11				15		18							
*38:06-38:07	1														16		18							
*38:19, 39:03, 39:24:01-39:24:02, 39:37, 39:76, 40:314, 48:07, 57:46														14		16								
*39:19:01-39:19:02															16		18	19						
*40:01:06, 40:07-40:08, 40:68, 40:106, 40:162, 40:220, 40:232															16							21		
*40:03, 40:105, 40:267, 40:284										10				14		16								
*40:13															16	17					21			
*40:19, 40:109, 40:117, 40:292															16	17								
*40:20, 40:38, 40:52, 40:59-40:60, 40:159, 46:06, 48:23										10					16									
*40:185, 40:308															15	16								
*41:24					5						11			14										
*41:46																	17					21		
*44:02:08, 44:02:39, 44:101, 44:220									w															
*44:03:17, 44:234									w						16									
*44:06, 51:08:01-51:08:02, 51:44N, 51:141	1								9															
*44:15				4		6			9		12													
*44:18				4					9		12						17							
*44:25, 44:95									9								17							
*44:50:01									9						16	17								
*44:83, 44:134					5				9					15										
*44:96, 44:153									9						16								23	
*44:109						6			9						16									
*44:117				w					9															
*44:176						6			9															
*44:190, C*15:39		2																						
*44:209, 44:248						7			9						16									
*45:01:01-45:01:02, 45:04-45:07, 45:13-45:19, 56:13, 56:43				4					9		12													
*45:03									9		12													

Negative Control

ive Control



