

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	
HLA-B Allele^{1,2,3}																									
*53:01:01-53:01:16, 53:25, 53:27, 53:32, 53:34-53:35, 53:41, 53:43-53:47, 53:49-53:50	1								9	10	11					16									
*53:02	1	2				6			9	10	11					16									
*53:03			3			6			9	10	11					16									
*53:04	1			4					9							16									
*53:05:01	1		3			6			9	10	11					16									
*53:05:02	1		3			6			9	10	11					w									
*53:06	1	2				6		8			11					16									
*53:07	1			4			7		9							16									
*53:08:01-53:08:02	1							8			11					16									
*53:09, 53:38					5				9	10	11				15	16									
*53:10	1						7		9	10	11			14		16									
*53:11					5				9	w	11									20					
*53:12					5	6			9	10	11					16									
*53:13					5				9	10	11					16									
*53:14	1								9	10	11			14		16									
*53:15	1		3						9	10	11					16									
*53:16	1	3				6			9	w	11								20						
*53:17:01-53:17:02, 53:40	1								9	10	11				15	16									
*53:18	1			4					9	10	11					16									
*53:19	1								9							16						22			
*53:20	1								9	10	11					16	17								
*53:21	1								9	10	11					16		18							
*53:22	1				5				9		11					16									
*53:23	1								9	10	11					16			19						
*53:24	1								9	w	11					16				20					
*53:26	1						7		9	10	11					16									
*53:28	1	2				6		8			11				15	16									
*53:29	1								9	10	11					16							23		
*53:30									9	10	11					16	17					21			
*53:31				4		5			9							16									
*53:33	1		3	4		6			9							16									
*53:36					5				9							16							22		
*53:37	1								9	10	11					16							22		
*53:39									9	10	11					16	17								
*53:42	1		3						9							16									
*53:48N	1								9	10	11					16		18	19						
*07:02:01:01-07:08:02, 07:10, 07:13-07:16, 07:19, 07:21-07:23, 07:27-07:33:03, 07:35-07:42, 07:44N-07:47, 07:49N-07:50, 07:52-07:59, 07:61-07:64, 07:66-07:77, 07:79, 07:81-07:114, 07:116-07:130, 07:132-07:133, 07:135N-07:146, 07:148-07:154, 07:156-07:162, 07:165-07:198, 07:200-07:201N, 07:203-07:208, 07:210-07:219, 07:221-07:222, 07:224-07:225, 07:228:01-07:234, 07:236, 07:238-07:245, 07:247, 07:249-07:260, 07:262-07:286, 07:288-07:300, 07:302-07:316N, 08:20:01-08:20:02, 08:53:01-08:53:02, 08:79, 08:156, 14:03, 14:47, 15:30:01:01-15:30:01:02, 15:33, 15:45, 15:63, 15:77-15:78:04, 15:116, 15:141, 15:150, 15:168, 15:233, 15:248, 15:269, 15:287, 15:324, 15:334, 15:342, 15:352, 15:375N, 15:400N, 15:416, 15:432, 27:25, 27:108, 27:115, 35:44, 35:118, 35:213, 39:18, 39:36, 40:15-40:16, 40:23, 40:32, 40:98, 40:161, 40:198, 44:146, 44:150, 46:01:06, 46:45, 46:54, 46:70, 46:74, 48:05, 48:08								8																	
*07:09, 07:11, 07:17, 07:115, 07:246, 07:287, 57:09, 57:13								8						14											
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	
*07:12, 07:18:01-07:18:02, 07:163, 07:199, 07:227, 07:248			3					8																	
*07:20, 07:24, 07:60, 15:99, 44:76								8												20					
*07:26, 07:34, 07:43, 07:80, 07:131, 07:147, 13:20, 13:62, 14:01:01:01-14:02:09, 14:02:11-14:02:16, 14:04, 14:06:01-14:09, 14:11-14:18, 14:20-14:26, 14:28-14:36, 14:38-14:40, 14:42, 14:44-14:46, 14:48-14:55, 14:57-14:58, 14:60-14:61, 15:02:12, 15:09:01-15:10:04, 15:17:01:01-15:17:04, 15:17:06-15:17:07, 15:37, 15:72, 15:90, 15:177, 15:196, 15:208, 15:216, 15:230, 15:243, 15:252, 15:273, 15:313, 15:323, 15:329, 15:338-15:339, 15:356, 15:361, 15:403, 15:420, 15:423-15:424, 15:427, 15:435, 15:439, 15:445-15:446, 27:04:01-27:04:06, 27:06, 27:10, 27:15, 27:18, 27:20-27:21, 27:24, 27:40, 27:54, 27:63, 27:66N, 27:68-27:69, 27:79, 27:86, 27:91-27:92, 27:100, 27:103, 27:105, 27:109, 27:112-27:114, 27:120, 27:138, 27:149, 27:153, 27:165, 27:168, 27:173, 38:10, 38:32, 39:35, 39:107, 40:05:01:01-40:05:01:02, 40:26, 40:51, 40:95, 40:148, 40:174, 40:183, 40:309, 40:355, 42:19, 44:79:01-44:79:02, 48:15, 50:14, 50:20, 51:01:40, 51:104, 51:147, 52:02:01-52:02:02, 52:16, 52:32, 55:01:04, 55:01:07, 55:17, 55:28, 55:44, 55:53, 55:58, 56:05:01-56:05:02, 56:21, 78:02:01-78:02:02, 78:09								8							16										

Negative Control

*57:30										11		14				20								
*57:45, 57:51, 57:69								9	10	11		14	16	17										
*57:60					6			9	10	11		14	16											
*57:63							8					14					20							
*57:76			4					9				14	16											
*58:01:01:01-58:01:02, 58:01:04-58:01:20, 58:01:22-58:02:01, 58:07, 58:10N-58:13, 58:15-58:16:02, 58:18, 58:21, 58:23-58:26, 58:29, 58:31N-58:32, 58:39N-58:43, 58:45:01-58:45:02, 58:47-58:60, 58:62-58:63, 58:66, 58:68-58:72N, 58:75, 58:77, 58:79-58:83, 58:85-58:90, 58:92-58:93N	1							9	10	11		13	14	16										
*58:01:03, 58:61	1							9	10			13	14	16										
*58:01:21	1							9	10	11		w	w	16										
*58:02:02	1							9	w	11		13	14	16										
*58:04, 58:67, 58:74	1							9	10	11		13	14	16										
*58:05	1									11		13	14	16										
*58:06, 58:19, 58:91	1						8			11		13	14	16										
*58:08:01-58:08:02, 58:46	1						8					13	14	16										
*58:09	1	2			6			9	10	11		13	14	16										
*58:14	1							9	10	11		13	14	16					23					
*58:17N, 58:64								9	10	11		13	14	16										
*58:20, 58:27	1											13	14	16										
*58:22	1							9	w	11		13	14				20							
*58:28:01-58:28:02	1							9				13	14	16										
*58:33	1		4					9	10	11		13	14	16										
*58:34, 58:73	1		4					9				13	14	16										
*58:35, 58:37-58:38, 58:84	1									10	11	13	14	16										
*58:65	1							9				13	14	16					22					
*58:76	1	2			6				10	11		13	14	16										
*58:78	1						8			11		13	14											
*78:08														16		19		21						
A*02:188, A*24:174, C*14:71, C*16:43									w															
A*25:37															17									
C*02:03, C*06:153, C*07:95, C*07:342, C*07:599, C*12:175, C*14:53									10															
C*02:56, C*06:20, C*07:81, C*07:450, C*14:82, C*16:98, C*16:102, C*16:110										w														
C*03:19, C*03:26, C*03:102			w																					
C*03:36								9	10															
C*03:250, C*03:267, C*03:307, C*05:27, C*08:41, C*08:138, C*15:15, C*17:07			4																					
C*04:08, C*04:147, C*05:39, C*08:115, C*18:08			4														20							
C*07:168						7				w														
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24

Negative Con