

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
<b>HLA-B allele<sup>1,2,3</sup></b>																								
*53:01:01:01-53:01:25, 53:25, 53:27, 53:32, 53:34-53:35, 53:41, 53:43:01-53:47, 53:49-53:54, 53:57-53:58, 58:100 <sup>4</sup>	1								9	10	11					16								
*53:02	1	2				6			9	10	11					16								
*53:03			3			6			9	10	11					16								
*53:04	1			4					9							16								
*53:05:01	1		3			6			9	10	11					16								
*53:05:02	1		3			6			9	10	11					w								
*53:06	1	2				6		8			11					16								
*53:07	1			4			7		9							16								
*53:08:01-53:08:02	1							8			11					16								
<b>*53:09, 53:38</b>					5				9	10	11				15	16								
*53:10	1						7		9	10	11			14		16								
*53:11					5				9	w	11										20			
*53:12					5	6			9	10	11					16								
*53:13					5				9	10	11					16								
*53:14	1								9	10	11			14		16								
*53:15	1		3						9	10	11					16								
*53:16	1		3			6			9	w	11									20				
*53:17:01-53:17:02, 53:40	1								9	10	11				15	16								
*53:18	1			4					9	10	11					16								
*53:19	1								9							16						22		
*53:20	1								9	10	11					16	17							
*53:21	1								9	10	11					16		18						
*53:22	1				5						11					16								
*53:23	1								9	10	11					16				19				
*53:24	1								9	w	11									20				
*53:26	1						7		9	10	11					16								
*53:28	1	2				6		8			11				15	16								
*53:29	1								9	10	11					16							23	
*53:30									9	10	11					16	17				21			
*53:31				4	5				9							16								
*53:33	1		3	4		6			9							16								
*53:36					5				9							16						22		
*53:37	1								9	10	11					16						22		
*53:39									9	10	11					16	17							
*53:42	1		3						9							16								
*53:48N	1								9	10	11					16		18	19					
*53:55			3			6			9	w	11									20				
*53:56	1								10	11						16								
*07:02:01:01-07:08:02, 07:10, 07:13-07:16, 07:19, 07:21-07:23, 07:27-07:33:03, 07:35-07:42, 07:44N-07:47, 07:49N-07:50, 07:52-07:59, 07:61-07:64, 07:66-07:77, 07:79, 07:81-07:114, 07:116-07:130, 07:132-07:133, 07:135N-07:146, 07:148-07:154, 07:156-07:162, 07:165-07:198, 07:200-07:201N, 07:203-07:208, 07:210-07:219, 07:221-07:222, 07:224-07:225, 07:228-01-07:234, 07:236, 07:238-07:245, 07:247, 07:249-07:260, 07:262-07:286, 07:288-07:300, 07:302-07:326, 07:328-07:331, 07:333-07:345, 07:347-07:349, 07:351N-07:357, 07:360Q-07:362, 07:364-07:385, 07:387-07:399, 08:20:01-08:20:02, 08:53:01-08:53:02, 08:79, 08:156, 14:03, 14:47, 14:82, 15:30:01:01-15:30:01:03, 15:33, 15:45, 15:63, 15:77-15:78:04, 15:116, 15:141, 15:150, 15:168, 15:233, 15:248, 15:269, 15:287, 15:324, 15:334, 15:342, 15:352, 15:375N, 15:400N, 15:416, 15:432, 15:541, 27:25, 27:108, 27:115, 35:44, 35:118, 35:213, 39:18, 39:36, 40:15-40:16:01:02, 40:23, 40:32, 40:98, 40:161, 40:198, 44:146, 44:150, 44:452, 46:01:06, 46:45, 46:54, 46:70, 46:74, 46:81, 48:05, 48:08								8																
*07:09:01-07:09:02, 07:11, 07:17, 07:115, 07:246, 07:287, 07:327, 15:524, 57:09, 57:13								8					14											
*07:12, 07:18:01-07:18:02, 07:163, 07:199, 07:227, 07:248, 07:350			3					8																
*07:20, 07:24, 07:60, 07:358, 15:99, 44:76								8												20				
*07:26, 07:34, 07:43, 07:80, 07:131, 07:147, 13:20, 13:62, 14:01:01:01-14:02:09, 14:02:11-14:02:25, 14:04, 14:06:01-14:09, 14:11-14:18, 14:20-14:26, 14:28-14:36, 14:38-14:40, 14:42, 14:44-14:46, 14:48-14:55, 14:57-14:58, 14:60-14:81, 14:83-14:86, 15:02:12, 15:09:01-15:10:05, 15:17:01:01-15:17:04, 15:17:06-15:17:07, 15:37, 15:72, 15:90, 15:177, 15:196, 15:208, 15:216, 15:230, 15:243, 15:252, 15:273, 15:313, 15:323, 15:329, 15:338-15:339, 15:356, 15:361, 15:403, 15:420, 15:423-15:424, 15:427, 15:435, 15:439, 15:445-15:446, 15:455, 15:500, 15:503, 15:506, 15:516, 15:523, 15:530, 15:532, 15:546Q, 15:553, 15:555, 27:04:01-27:04:06, 27:06:01:01-27:06:01:02, 27:10, 27:15, 27:18, 27:20-27:21, 27:24, 27:40, 27:54, 27:63, 27:66N, 27:68-27:69, 27:79, 27:86, 27:91-27:92, 27:100, 27:103, 27:105, 27:109, 27:112-27:114, 27:120, 27:138, 27:149, 27:153, 27:165, 27:168, 27:173, 27:183, 27:192, 27:208, 27:210-27:211, 27:214, 27:222, 35:384, 38:10, 38:32, 39:35, 39:107, 40:05:01:01-40:05:01:03, 40:26, 40:51, 40:95, 40:148, 40:174, 40:183, 40:309, 40:355, 42:19, 44:79:01-44:79:02, 44:459, 48:15, 50:14, 50:20, 51:104, 51:147, 51:267, 51:284, 52:02:01-52:02:02, 52:16, 52:32, 55:01:04, 55:01:07, 55:17, 55:28, 55:44, 55:53, 55:58, 55:94, 56:05:01-56:05:02, 56:21, 56:66, 78:02:01-78:02:02, 78:09								8							16									
<b>Well No.</b>	<b>1</b>	<b>2</b>	<b>3</b>	<b>4</b>	<b>5</b>	<b>6</b>	<b>7</b>	<b>8</b>	<b>9</b>	<b>10</b>	<b>11</b>	<b>12</b>	<b>13</b>	<b>14</b>	<b>15</b>	<b>16</b>	<b>17</b>	<b>18</b>	<b>19</b>	<b>20</b>	<b>21</b>	<b>22</b>	<b>23</b>	<b>24</b>

Negative Control







*15:383, 35:05:01:01-35:05:03, 35:51, 35:89, 35:97, 35:368, 35:389, 35:416, 35:427N, 35:459N, 57:01:01:01-57:01:02, 57:01:04, 57:01:06-57:01:11, 57:01:13-57:01:16, 57:01:18, 57:01:20-57:01:22, 57:01:24-57:01:31, 57:01:34-57:01:41, 57:06, 57:08, 57:11, 57:15-57:16, 57:18, 57:21, 57:23, 57:29, 57:33-57:35, 57:37-57:38, 57:40, 57:44, 57:48-57:50, 57:52-57:56, 57:58-57:59, 57:61, 57:65, 57:67-01-57:68, 57:71, 57:73-57:75, 57:77-57:79N, 57:81, 57:83, 57:85-57:88, 57:90-57:92, 57:95, 57:97-57:100, 57:102-57:106, 57:108, 57:110-57:114, 57:116-57:117, 57:121-57:123, 57:125-57:127, 58:36, 58:44									9	10	11		14	16						
*15:415, 15:499, 15:507, 46:09									9		11									
*15:421, 40:03:01:01-40:03:02, 40:105, 40:267, 40:284										10				14	16					
*18:01:01:01-18:01:03, 18:01:05-18:02:02, 18:05:01:01-18:06, 18:08, 18:12:01-18:12:02, 18:17N, 18:23N-18:24, 18:27-18:28, 18:31-18:32, 18:34, 18:38-18:39, 18:42-18:47, 18:49-18:53, 18:55, 18:59-18:60, 18:62-18:65, 18:70-18:78, 18:80-18:100, 18:102-18:104, 18:106Q, 18:109, 18:111-18:112, 18:114-18:121, 18:123-18:124, 18:126-18:131:01:02, 18:133-18:135:01:02, 18:137, 18:139-18:141, 18:144-18:147, 18:151, 18:153-18:154N, 18:156:01:01-18:163, 18:167, 18:169-18:170, 18:172-18:177, 18:179-18:181, 18:183-18:188					6						10	11		15	16	18				
*18:01:04, 18:40, 18:138N					6						10			15	16	18				
*18:03:01:01-18:03:01:02, 18:142, 18:171					6						10	11		15	16	18	19			
*18:03:02					6						10	11		16		18	19			
*18:04:01-18:04:02, 18:25, 18:68, 18:143					6						10	11		15	16					
*18:07:01-18:07:02, 18:29, 18:33, 18:79, 18:155, 18:182N					6						10	11		16		18				
*18:09				5	6						10	11		15	16	18				
*18:10-18:11, 18:35, 18:110											10	11		15	16	18				
*18:13					6						w	11		15		18				
*18:14					6						10	11		14	15	16	18			
*18:15, 18:57:01-18:57:02					6		8					11		15	16	18				
*18:18:01:01-18:18:01:02					6	7					10	11		15	16	18				
*18:19, 18:30					6		8					11		15	16	18				
*18:20					6						10	w		15	16	18				
*18:21					6		8							15		18				
*18:22, 18:69, 18:105, 18:149-18:150		2			6						10	11		15	16	18				
*18:26, 18:41, 18:61, 18:122, 18:148, 18:165-18:166, 18:168, 18:189, 40:209, 40:350														15	16	18				
*18:36					6					9	10	11		15	16	18				
*18:54, 18:108			3		6						10	11		15	16	18				
*18:56				5								11		15		18				
*18:58									9	10	11			15	16	18				
*18:67	1	3			6					10	11			15	16	18				
*18:101							8					11		15	16	18				
*18:113				5	6							11		15		18				
*18:132			3		6					10	11			15	16	18	19			
*18:136	1				6					10	11			16		18				
*18:152					6		8					11		15	16	18	19			
*18:164							8					11		15		18				
*27:52			3											16		18				
*27:106, 47:10					6		8							16						
*35:01:10, 35:46-35:47, 35:207, 35:217									9	10	11			16					21	
*35:02:08									9					16					w	
*35:03:01:01-35:03:05, 35:03:07-35:03:29, 35:06, 35:13, 35:36, 35:55-35:56, 35:59:01-35:59:02, 35:70, 35:74-35:75, 35:85, 35:98, 35:106, 35:127-35:128, 35:150:01-35:153, 35:155, 35:160, 35:179, 35:181, 35:193, 35:195, 35:198, 35:204-35:205, 35:221, 35:223, 35:235-35:236, 35:242-35:243, 35:246, 35:256, 35:261, 35:274, 35:279, 35:281-35:282, 35:290-35:291, 35:298, 35:312, 35:320, 35:326, 35:344, 35:354, 35:358, 35:360, 35:364, 35:371, 35:375, 35:395, 35:397-35:399, 35:415, 35:420, 35:422, 35:424, 35:432, 35:434-35:436, 35:438, 35:442, 35:446, 35:451, 35:455, 35:457, 35:460, 35:469, 38:65, 39:32, 39:74, 40:71, 56:02:01:01-56:02:03, 56:04:01:01-56:04:04, 56:11, 56:22, 56:56-56:57				4					9					16						
*35:04:02, 35:251, 78:04, 78:10									9					16					21	
*35:18, 35:164, 41:62, 57:04:02									9										20	
*35:20:01-35:20:02, 35:28:01:01-35:28:01:03, 35:226									9	10	11			15	16					
*35:21, 35:188		2			6		8				11			16						
*35:24:01-35:24:02, 35:287		2			6				9	10	11			16						
*35:29:01-35:29:02, 35:69, 35:76, 35:294									9	10	11			16		19				
*35:34, 35:39, 35:199			3						9					16						
*35:38, 35:163, 35:362, 38:30, C*04:34, C*04:212, C*05:151				4					9											
*35:45											w	11						20		
*35:48, 35:108:01, 35:433							7		9	10	11			16						
*35:50, 35:197									9	10	11			16		18				
*35:62, 35:166									9	w	11									
*35:66											w	11								
*35:72									9	10	11		14	15	16					
*35:81		2							9					16					22	
*35:84, 35:231, 35:267, 35:423				4					9					16		18				
*35:95									9					15	16					22
*35:109		2	4						9					16						
*35:114, 57:01:05, 57:31, 57:47									9	10			14	16						
*35:136				4					9					16		19				

Negative Control

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
*35:141									9	11						16								
*35:146									9	10	11					16							22	
*35:154									9							16					21	22		
*35:156, 39:33, 56:51, 56:63				4					9											20				
*35:157, 44:203:01-44:203:02		2														16								
*35:162									9							16	18						22	
*35:169, 44:140, 44:226, 45:02, 82:01:01:01-82:03, C*03:26:02, C*03:250, C*03:267, C*03:307, C*05:27, C*05:184, C*08:41, C*08:138, C*08:182, C*15:15, C*17:07				4																				
*35:172									9												20	22		
*35:174									9	w										20				
*35:186									9	10	11					16			w					
*35:190		2				6			9	10	11					16								
*35:203			3						9	10	11					16								
*35:208:01									9	10	11					16							23	
*35:229									9	w	11								19	20				
*35:232								8			11			14						20				
*35:233, 44:290		2							9							16								
*35:237, 35:452, 44:10, 44:178, 44:251, 44:340				4												16								
*35:257, 35:278, 35:450				4					9							16						21		
*35:260					5				9															
*35:262, 35:333Q, 35:341, 35:461N										w	11													
*35:296				4					9						15	16								
*35:316									9							16			19					
*35:329				4		6			9							16								
*35:349				4			7		9					14		16								
*35:355			3	4					9							16								
*35:381N				4					9							16			18	19				
*35:394				4					9							16							22	
*35:407			3						9							16							22	
*35:439:01:01-35:439:01:02								8			11				15	16								
*35:440, 57:76, 57:118				4					9					14		16								
*37:01:01:01-37:01:05, 37:01:07-37:01:14, 37:01:16-37:01:22, 37:03N, 37:05, 37:08, 37:15-37:18, 37:20, 37:22-37:25, 37:27, 37:30N, 37:32-37:33N, 37:36, 37:38, 37:41, 37:43, 37:45-37:47, 37:49-37:54, 37:58-37:59, 37:62-37:66, 37:68-37:73, 37:75-37:78, 37:80, 37:84, 37:86N-37:91			3				7								15			18						
*37:01:06, 37:67, 37:82N							7											18						
*37:01:15, 37:31, 37:42N, 37:74, 37:85			3				7								15	16								
*37:02			3												15	16		18						
*37:04:01-37:04:02		2	3				7								15			18						
*37:07			3					8							15			18						
*37:09, 37:12, 37:19:01-37:19:02, 37:44, 37:81			3												15			18						
*37:10					5		7								15			18						
*37:11, 37:14, 37:29							7								15			18						
*37:13			3				7	8							15			18						
*37:21, 37:26, 37:57, 37:60, 37:79N			3				7											18						
*37:34	1						7								15			18						
*37:35			3		5		7				11				15			18						
*37:37							7								15			18	19					
*37:39			3	4			7		9						15			18		20				
*37:40, 37:83			3				7								15	16		18						
*37:48			3	4			7		9						15			18						
*37:55			3			6	7								15									
*37:56			3		5						11				15			18						
*38:06-38:07		1														16		18						
*38:19, 39:03:01:01-39:03:01:03, 39:24:01-39:24:02, 39:37, 39:76, 39:120, 39:144, 39:152, 40:314, 48:07, 57:07, 57:46														14		16								
*39:19:01-39:19:02																16		18	19					
*40:01:06, 40:07-40:08, 40:68, 40:106, 40:162, 40:220, 40:232, 44:03:40, 44:232, 48:01:08, 55:02:11																16						21		
*40:13																16	17					21		
*40:19, 40:109, 40:117, 40:292, 40:340, 44:50:01, 44:50:03																16	17							
*40:20:01:01-40:20:01:02, 40:38, 40:52, 40:59-40:60, 40:159, 40:352, 46:06, 48:23										10						16								
*40:120, 44:107			3													16								
*40:185, 40:308, 40:365															15	16								
*40:394						6									16		17							
*41:24, 57:119					5						11			14										
*41:46																		17				21		
*41:60, 44:25, 44:95, A*25:37																		17						
*44:06, 51:153, 51:287N	1																							
*44:15:01:01-44:15:01:02				4		6						12												
*44:18				4								12						17						
*44:83, 44:134, 44:445					5										15									
*44:96, 44:153																16							23	
*44:117, C*03:19, C*03:26:01, C*03:102				w																				
*44:190, C*03:384, C*15:39, C*15:130		2																						
*44:275		2							9															

Negative Control

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
*44:308				5							w													
*44:339, 56:59				4								12				16								
*45:01:01:01-45:01:08, 45:05-45:07, 45:11-45:21, 45:23, 45:25-45:26				4								12												
*45:04, 54:40, 56:13, 56:43				4					9			12												
*45:22				4								12			15									
*45:24				w								12												
*46:26								8	9	10	11					16								
*46:40:01-46:40:02								8			11							w						
*48:14, 57:01:03, 57:01:12, 57:25-57:26, 57:43, 57:64, 57:93										10	11			14	16									
*49:01:01:01-49:01:01:09, 49:01:03-49:01:07, 49:01:09-49:01:16, 49:06, 49:08-49:10, 49:12-49:14, 49:16-49:17, 49:19N-49:25, 49:27-49:34, 49:36-49:40, 49:42-49:44, 49:46, 49:48-49:49, 49:51-49:54, 49:56-49:60N, 49:62-49:66				4				8				12				16	17							
*49:01:02, 49:35				4				8				12				w	17							
*49:01:08, 49:07								8				12				16	17							
*49:03, 52:25:01	1			4				8				12			15	16								
*49:04:01-49:04:02								8			11					16	17							
*49:05:01-49:05:02								8			w					16	17							
*49:11				4					9			12				16	17							
*49:18:01-49:18:02, 49:61, 51:112								8								16	17							
*49:41				4				8				12				16	17			20				
*49:45				4								12				16	17							
*49:47, 49:50				4				8								16	17							
*49:55				4								12				w	17							
*50:36				4				8				12				16							22	
*50:66				4				8				12				w								
*51:01:01:01-51:01:46, 51:01:48-51:03, 51:06:01-51:06:04, 51:11N-51:12, 51:14, 51:16-51:18, 51:21-51:24:05, 51:26-51:28, 51:30, 51:32-51:35, 51:38-51:39, 51:41N, 51:43, 51:48-51:52, 51:58-51:60, 51:64-51:71, 51:75-51:77, 51:79-51:80, 51:83-51:86, 51:88-51:89, 51:94-51:96, 51:98N-51:102, 51:105, 51:107, 51:109-51:111, 51:113-51:114, 51:117-51:119, 51:121, 51:123-51:134, 51:136-51:138, 51:140, 51:142-51:146, 51:149N-51:152, 51:154-51:156, 51:158:01-51:171, 51:173Q-51:180, 51:182-51:184N, 51:186, 51:188-51:193, 51:195-51:196, 51:198-51:207, 51:209-51:210, 51:212-51:213, 51:215, 51:219, 51:221, 51:224, 51:226, 51:229-51:250, 51:252-51:254, 51:256N-51:258, 51:260-51:262, 51:264N-51:266, 51:268-51:273N, 51:277-51:278, 51:282-51:283, 51:285-51:286, 51:288-51:299, 51:301-51:309, 52:01:13, 52:06:01-52:06:03, 52:22-52:23, 52:70, 52:89N	1							8								16								
*51:01:47, 51:36, 51:55, 51:61:01-51:61:02, 51:82, 51:197	1							8																
*51:04:01-51:04:02, 51:46, 51:56:01-51:56:03, 51:227	1	2						8								16								
*51:05, 51:217	1								9										20	w				
*51:07:01-51:07:02, 51:214, 51:222, 52:01:01:01-52:01:12, 52:01:14-52:01:19, 52:01:21-52:01:22, 52:01:24-52:01:25, 52:01:27-52:01:28, 52:01:30-52:01:32, 52:01:34-52:01:48, 52:04-52:05, 52:07-52:08, 52:11, 52:13, 52:15, 52:17-52:18, 52:26-52:27, 52:30-52:31:02, 52:33-52:38, 52:40-52:43, 52:45-52:60, 52:62-52:68, 52:72-52:73, 52:76-52:87, 52:90-52:95	1							8							15	16								
*51:08:01:01-51:08:04, 51:44N, 51:141, 51:263, 51:274-51:275	1								9															
*51:09:01-51:09:03, 51:40, 51:73, 51:122, 51:194, 51:259	1								9							16				21				
*51:10, 51:42, 51:81, 51:115, 51:181	1															16								
*51:13:01-51:13:02, 51:62, 51:92:01-51:92:02, 51:106:01, 51:172, 51:223	1			4				8								16								
*51:15	1			4					9			12				16								
*51:19, 51:148, 51:276	1								9							16								
*51:20	1								9										19					
*51:29, 51:279	1							8												20				
*51:31, 51:93, 51:108, 51:280	1															16						21		
*51:37, 51:45, 51:63:01-51:63:02	1						6	8			11					16								
*51:53, 51:120, 51:228	1															w								
*51:54					5				9											20	w			
*51:57, 51:103, 51:116, 51:218	1							8								16			18					
*51:72, 51:87, 51:91, 51:211	1							8								w								
*51:74	1							8								16						21		
*51:78:01-51:78:02					5											16								
*51:90, 51:135	1		3					8								16								
*51:97	1				5	6			9		11													
*51:106:02, 51:225	1			4				8				12				16								
*51:139	1	2							9							16						21		
*51:157	1											12				16								
*51:185	1		3			6		8								16								
*51:187	1							8								16	17							
*51:208	1							8								16			19					
*51:216	1					6		8			w					16								
*51:220	1							8			11													
*51:251	1				6			8			w				15	16								
*51:255	1					7		8								16								
*51:281	1	2		4				8								16								
*51:300	1							8				12				16								
*51:310, 52:88	1	2						8								15	16							
*52:01:20, 52:01:23	1							8								15	w							
*52:01:26, 52:21:01-52:21:02, 52:44, 52:61, 52:69	1							8								15								

Negative Control

*52:01:29, 78:05-78:06								8							15	16								
*52:01:33	1														15									
*52:03, 52:10:01-52:10:03, 52:74	1								9						15	16					21			
<b>Well No.</b>	<b>1</b>	<b>2</b>	<b>3</b>	<b>4</b>	<b>5</b>	<b>6</b>	<b>7</b>	<b>8</b>	<b>9</b>	<b>10</b>	<b>11</b>	<b>12</b>	<b>13</b>	<b>14</b>	<b>15</b>	<b>16</b>	<b>17</b>	<b>18</b>	<b>19</b>	<b>20</b>	<b>21</b>	<b>22</b>	<b>23</b>	<b>24</b>
*52:09:01-52:09:02	1		3			6		8							15	16								
*52:12	1		3					8							15	16								
*52:14:01-52:14:02, 52:25:02, 52:75	1			4				8							15	16								
*52:19	1								9						15									
*52:20								8							15	16								
*52:24	1				5			8							15					20				
*52:28-52:29	1														15	w								
*52:39	1					6		8			11				15	16								
*52:71	1														15	16								
*55:01:20								8				12				w								
*55:14								8			11				16							23		
*55:54												12				w								
*55:90						6		8				12			16									
*56:01:05									9			12			16									
*56:06, 78:01:01-78:01:02, 78:03, 78:07								8							16				19					
*56:45				4					9			12			16				19					
*56:46				4		6			9			12			16									
*57:01:17									9		11			14										
*57:01:19									9	10	11			14	16								23	
*57:01:32									9	10	11			14	16							21		
*57:01:33, 57:03:01-57:03:05, 57:17, 57:32, 57:39, 57:66, 57:70, 57:80, 57:82, 57:84, 57:94, 57:96, 57:101, 57:124									9					14	16									
*57:02:01-57:02:02, 57:04:01, 57:04:03-57:05, 57:12, 57:28N, 57:42									9					14								20		
*57:14:01		2				6			9	10	11			14	16								23	
*57:14:02		2				6			9	10	11			14	16									
*57:19									9	w	11			14								20		
*57:24						5			9		11			14										
*57:27, 57:72									9	w	11			14										
*57:30											11			14								20		
*57:45, 57:51, 57:69, 58:111									9	10	11			14	16		17							
*57:60						6			9	10	11			14	16									
*57:63								8						14								20		
*57:120, 58:01:01-58:01:02, 58:01:04-58:01:20, 58:01:22-58:01:25, 58:01:27-58:02:01:03, 58:10N-58:13, 58:15-58:16:02, 58:18, 58:21, 58:23-58:26, 58:29, 58:31N-58:32, 58:39N-58:43, 58:45:01-58:45:02, 58:47-58:60, 58:62-58:63, 58:66, 58:68-58:72N, 58:75, 58:77, 58:79-58:83, 58:85-58:90, 58:92-58:99, 58:101, 58:103-58:110, 58:112-58:113, 58:115-58:120	1								9	10	11		13	14	16									
*57:129											11			14										
*58:01:03, 58:61	1								9	10			13	14	16									
*58:01:21	1								9	10	11		w	w	16									
*58:01:26, 58:07, 58:35, 58:37-58:38, 58:84	1									10	11		13	14	16									
*58:02:02	1								9	w	11		13	14	16									
*58:04, 58:67, 58:74	1								9	10	11		13		16									
*58:05	1										11		13	14	16									
*58:06, 58:19, 58:91	1							8			11		13	14	16									
*58:08:01-58:08:02, 58:46	1							8					13	14	16									
*58:09	1	2				6			9	10	11		13	14	16									
*58:14	1								9	10	11		13	14	16								23	
*58:17N, 58:64									9	10	11		13	14	16									
*58:20, 58:27	1												13	14	16									
*58:22	1								9	w	11		13	14								20		
*58:28:01-58:28:02, 58:114	1								9				13	14	16									
*58:33	1			4					9	10	11		13	14	16									
*58:34, 58:73	1			4					9				13	14	16									
*58:65	1								9				13	14	16								22	
*58:76	1	2				6				10	11		13	14	16									
*58:78	1							8			11		13	14										
*58:102	1								9	10	w		13	14	16									
*58:121	1							7	9	10	11		13	14	16									
*78:08														16							19	21		
A*02:188, A*24:174, C*14:71, C*16:43											w													
C*02:03, C*06:153, C*07:95, C*07:342, C*07:599, C*12:175, C*14:53											10													
C*02:56, C*06:20, C*07:81, C*07:450, C*14:82, C*16:98, C*16:102, C*16:110											w													
C*03:36									9	10														
C*04:08, C*04:147, C*05:39, C*08:115, C*18:08				4																		20		
C*07:168								7			w													
C*15:177N																					18	19		
C*15:195		2		4																				
<b>Well No.</b>	<b>1</b>	<b>2</b>	<b>3</b>	<b>4</b>	<b>5</b>	<b>6</b>	<b>7</b>	<b>8</b>	<b>9</b>	<b>10</b>	<b>11</b>	<b>12</b>	<b>13</b>	<b>14</b>	<b>15</b>	<b>16</b>	<b>17</b>	<b>18</b>	<b>19</b>	<b>20</b>	<b>21</b>	<b>22</b>	<b>23</b>	<b>24</b>

Negative Control