

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	
HLA-B Allele^{1,2,3}																									
*53:01:01:01-53:01:29, 53:25, 53:27, 53:32, 53:34-53:35, 53:41, 53:43:01-53:47, 53:49-53:54, 53:57-53:61, 53:63-53:64, 53:66, 58:100 ⁴	1								9	10	11					16									
*53:02	1	2				6			9	10	11					16									
*53:03			3			6			9	10	11					16									
*53:04	1			4					9							16									
*53:05:01	1		3			6			9	10	11					16									
*53:05:02	1		3			6			9	10	11					w									
*53:06	1	2				6		8			11					16									
*53:07	1			4			7		9							16									
*53:08:01-53:08:02	1							8			11					16									
*53:09, 53:38					5				9	10	11				15	16									
*53:10	1						7		9	10	11			14		16									
*53:11					5				9	w	11									20					
*53:12					5	6			9	10	11					16									
*53:13					5				9	10	11					16									
*53:14	1								9	10	11			14		16									
*53:15	1		3						9	10	11					16									
*53:16	1		3			6			9	w	11									20					
*53:17:01-53:17:02, 53:40	1								9	10	11				15	16									
*53:18	1			4					9	10	11					16									
*53:19	1								9							16						22			
*53:20	1								9	10	11					16	17								
*53:21	1								9	10	11					16	18								
*53:22	1				5						11					16				19					
*53:23	1								9	10	11					16									
*53:24	1								9	w	11					16				20					
*53:26, 53:65	1						7		9	10	11					16									
*53:28	1	2				6		8			11				15	16									
*53:29	1								9	10	11					16							23		
*53:30									9	10	11					16	17				21				
*53:31				4	5				9							16									
*53:33	1		3	4		6			9							16									
*53:36					5				9							16						22			
*53:37	1								9	10	11					16						22			
*53:39									9	10	11					16	17								
*53:42	1		3						9							16									
*53:48N	1								9	10	11					16	18	19							
*53:55			3			6			9	w	11									20					
*53:56, 53:68-53:69	1									10	11					16									
*53:62, 35:471, 51:10, 51:42, 51:81, 51:115, 51:181, 51:337, 51:359 ⁴	1															16									
*53:67					5	6			9	10	11			14		16									
*53:70	1								9		11														
*07:02:01:01-07:02:05, 07:02:07-07:08:02, 07:10, 07:13-07:16, 07:19, 07:21-07:23, 07:27-07:33:03, 07:35-07:42, 07:44N-07:47, 07:49N-07:50, 07:52-07:59, 07:61-07:64, 07:66-07:77, 07:79, 07:81-07:114, 07:116-07:130, 07:132-07:133, 07:135N-07:146, 07:148-07:154, 07:156-07:162, 07:165-07:198, 07:200-07:201N, 07:203-07:208, 07:210-07:219, 07:221-07:222, 07:224-07:225, 07:228:01-07:234, 07:236, 07:238-07:245, 07:247, 07:249-07:260, 07:262:01-07:286, 07:288-07:300, 07:302-07:326, 07:328-07:331, 07:333-07:345, 07:347-07:349, 07:351N-07:357, 07:360Q-07:362, 07:364-07:385, 07:387-07:399, 07:401-07:406, 07:409-07:410, 07:412-07:419, 07:421-07:449, 07:451-07:465, 08:20:01-08:20:02, 08:53:01-08:53:02, 08:79:01-08:79:02, 08:156, 08:281, 08:296, 14:03, 14:47, 14:82, 15:30:01:01-15:30:01:03, 15:33, 15:45, 15:63, 15:77-15:78:04, 15:116, 15:141, 15:150, 15:168, 15:233, 15:248, 15:269, 15:287, 15:324, 15:334, 15:342, 15:352, 15:375N, 15:400N, 15:416, 15:432, 15:541, 15:585, 15:595, 27:25, 27:108, 27:115, 27:245, 35:44, 35:118, 35:213, 39:18, 39:36, 40:15-40:16:01:02, 40:23, 40:32, 40:98, 40:161, 40:198, 40:466, 44:146, 44:150, 44:452, 46:01:06, 46:45, 46:54, 46:70, 46:74, 46:81, 48:05, 48:08								8																	
*07:09:01-07:09:02, 07:11, 07:17, 07:115, 07:246, 07:287, 07:327, 15:524, 57:09, 57:13								8					14												
*07:12, 07:18:01-07:18:02, 07:163, 07:199, 07:227, 07:248, 07:350			3					8																	
*07:20, 07:24, 07:60, 07:358, 15:99, 40:496, 44:76								8												20					
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	

Negative Control

