

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32		
HLA-B allele <sup>1,2,3</sup>																																		
*55:01:01-55:01:06, 55:01:08-55:01:14, 55:53, 55:59-55:60, 55:64, 55:73	1	2	3												15																			
*55:01:07	1	2	3																															
*55:02:01:01-55:02:08, 55:61, 55:63, 55:65, 55:67, 55:69-55:71	1	2											13		15																			
*55:03	1	2	3	4											15																			
*55:04	1				5																													
*55:05	1	2	3			6									15																			
*55:07, 54:01:02 <sup>4</sup>	1	2								10			13	14	15																			
*55:08, 55:51	1						7																											
*55:09	1	2	3								11											22												
*55:10	1	2										12	13		15							21												
*55:11	1	2	3						9						15																			
*55:12	1	2						8					13		15																			
*55:13	1	2													15									24								31		
*55:14	1								9																									
*55:15	1	2	3								11				15																			
*55:16	1	2											13		15								23											
*55:17	1														15		17															30		
*55:18	1	2											13		15		18									25								
*55:19	1	2											13		15		18																	
*55:20	1														15		17			20														
*55:21	1	2													15							22	23											
*55:22	1	2									11		13																					
*55:23, 55:68	1	2													15							22	24											
*55:24	1	2	3								11																							
*55:25	1	2	3						9						15		18																	
*55:26, 55:62 <sup>5</sup>	1	2											13		15										25									
*55:27	1	2													15		17					22	24											
*55:28	1	2													15		17																	
*55:29	1	2	3												15								23											
*55:30	1	2											13		15			19																
*55:31	1	2	3												15		18																	
*55:32	1	2													15																			
*55:33	1	2	3												15									24										
*55:34	1	2											13		15										25	26								
*55:35	1	2											13		15																	30		
*55:36	1	2	3												15			19																
*55:37	1	2											w		15							22												
*55:38	1		3												15																	30		
*55:39	1	2											13		15		17																	
*55:40	1	2	3												15										25									
*55:41	1	2											13		15																28			
*55:42	1	2											13		15											26								
*55:43	1	2											13		15																27		31	
*55:44	1	2	3												15																	28		
*55:45	1	2	3												15											26								
*55:46	1	2											13		15											26					27			
*55:47	1	2											13		15											26								
*55:48	1	2									11		13		15																			
*55:49	1			4	5																													
*55:50	1	2											13		15																	29		
*55:52	1	2	w												15							22												
*55:54	1	2	3										13		15																			
*55:55N	1	2	3												15																		31	
*55:56	1	2													15						20													
*55:57	1	2													15																	28		
*55:58	1	2													15									24							28		31	
*55:66	1	2	3					8							15																			
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32		
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32		

Negative Control



*13:04, 15:16:01-15:16:03, 15:67, 15:95, 15:222, 15:254, 35:37, 49:04:01-49:05, 50:33	2														16						22	24										
*13:06	2														13													29				
*13:12:01-13:12:02, 15:86, 44:10	2														13																	
*13:13:01-13:13:02, 13:26:01-13:26:02, 13:51, 14:22, 15:20, 15:34, 15:62, 15:85, 15:194, 35:01:01:01-35:01:42, 35:01:44-35:01:45, 35:03:01-35:03:21, 35:05:01-35:05:02, 35:06-35:08:08, 35:10-35:11:03, 35:13-35:15:02, 35:19-35:21, 35:23-35:24:02, 35:26-35:30, 35:33, 35:35-35:36, 35:38, 35:40N-35:42:02, 35:45-35:50, 35:52, 35:54-35:58, 35:62-35:63, 35:65Q-35:66, 35:68:01-35:72, 35:75, 35:77-35:78, 35:80, 35:82, 35:84-35:85, 35:89-35:94, 35:96-35:98, 35:100-35:101:02, 35:103-35:112, 35:115-35:116, 35:119-35:123, 35:126-35:128, 35:130N-35:134N, 35:136-35:141, 35:143-35:148, 35:150:01-35:153, 35:155-35:161, 35:163, 35:166-35:171, 35:173N, 35:175-35:178, 35:180-35:181, 35:187-35:195, 35:197-35:198, 35:200, 35:202-35:204, 35:206-35:207, 35:209-35:210, 35:212-35:214, 35:216N-35:219, 35:221-35:229, 35:231-35:232, 35:236-35:241, 35:243-35:250, 35:252-35:257, 35:259, 35:261-35:264, 35:268-35:269, 35:271-35:272, 35:274-35:275, 35:277-35:282, 35:286, 44:48, 46:53, 48:02:01-48:02:03, 48:25, 49:01:08, 50:12, 50:17, 53:01:01-53:06, 53:08:01-53:18, 53:20-53:35, 53:37-53:39, 58:01:01-58:01:02, 58:01:04-58:02, 58:04-58:07, 58:09, 58:11, 58:13, 58:15-58:19, 58:21-58:26, 58:29, 58:31N-58:45:02, 58:47-58:58, 58:60, 58:62-58:64, 58:66-58:69, C*02:56, C*04:180, C*06:20, C*07:81, C*07:168	2																															
*13:16, 13:20	2	3													11																	
*13:18	2														11	w					22											
*13:31, 13:41	2														11						22	23										
*13:35, 40:44																					17	22	24									
*13:46	2														15						20		24		28		31					
*13:48	2	3													16																	
*13:53	2														13	15										29						
*13:59, 27:14	2																				17											
*13:62, 51:01:01:01-51:01:14, 51:01:16-51:01:31, 51:01:33-51:01:38, 51:01:40-51:01:47, 51:01:49-51:03, 51:07:01-51:07:02, 51:09:01-51:09:02, 51:11N-51:12, 51:18-51:19, 51:22-51:24:05, 51:26-51:28, 51:30, 51:32-51:33, 51:38-51:41N, 51:43, 51:48-51:53, 51:55, 51:57-51:58, 51:60, 51:65-51:80, 51:83-51:84, 51:86-51:89, 51:91, 51:94-51:96, 51:98N, 51:100, 51:102-51:105, 51:107-51:115, 51:117-51:126, 51:128-51:130, 51:132-51:134, 51:136-51:138, 51:140, 51:142, 51:145-51:147, 51:149N-51:152, 51:154-51:156, 51:158:01, 51:159-51:171, 51:173Q-51:175, 51:177-51:178N, 51:180-51:181, 51:183-51:189, 52:01:01:01-52:01:20, 52:01:22-52:01:24, 52:01:27-52:07, 52:09-52:11, 52:13, 52:15, 52:17-52:18, 52:20, 52:22-52:23, 52:28-52:36, 52:38, 52:40-52:45, 58:08:01-58:08:02, 78:01:01-78:06						7									16	17																
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32
*13:65, 40:48													11	13																		
*13:72		2																				22	24									
*13:73		2											11	w																		

Negative Control

*14:37	2													15																							
*14:43, 38:38, C*05:110, C*12:120																																					29
*15:04:01-15:04:02, 35:265														16					22	23	24																
*15:42	2	3												15																							
*15:76, 15:101																																				30	
*15:137	2													16							22	23	24											28	31		
*15:155, 15:293, 15:310, 51:37														16							22	24															
*15:224, 40:71, C*15:57												13																									
*15:244, 15:315, 39:54, C*03:70, C*03:127, C*03:179, C*03:274																	19																				
*15:289, 40:73, 46:01:01-46:01:09, 46:02-46:04, 46:06-46:10, 46:12-46:17, 46:19-46:31, 46:34-46:43, 46:52				4																																	
*15:303	2													16							22	23															
*18:01:01:01-18:03, 18:05-18:13, 18:15, 18:18-18:21, 18:23N-18:28, 18:30-18:51, 18:53-18:73, 18:75-18:77, 18:79, 18:81-18:97, 18:99-18:101, 18:103-18:104, 18:106, 18:108, 27:75, 35:125, 37:01:01-37:08, 37:10-37:21, 37:23-37:40, 37:42N-37:55, 44:130, 44:156, 57:58																					21																
*18:14, 37:09				5																	21																
*18:22	2			w																	21																
*18:52, 18:105, 35:25, 35:124, 35:142, 35:267, 58:12	2																				21																
*18:74N, 18:78																					21														29		
*27:81, 27:130, 40:06:01:01-40:06:08, 40:06:10-40:06:11, 40:53, 40:70:01-40:70:02, 40:75, 40:83, 40:93, 40:95-40:96, 40:103, 40:109-40:110, 40:127, 40:131, 40:148, 40:162, 40:165, 40:167, 40:177, 40:190, 40:230, 40:244, 40:256N, 40:268-40:270, 40:275, 40:294-40:295, 40:298, 51:01:32, 51:01:39, 51:10, 51:16, 51:31, 51:34, 51:93, 51:144, 52:01:21, 52:08, 78:08															17																						
*35:02:01-35:02:07, 35:04:01-35:04:03, 35:34, 35:81, 35:83, 35:95, 35:129N, 35:154, 35:172, 35:182-35:184, 35:211, 35:220, 35:258, 35:270, 35:285, 40:28, 51:56:01-51:56:03, 53:19, 53:36, 57:05, 58:28:01-58:28:02, 58:65	2					7																															
*35:60, 51:157, 54:24	2												13	15																							
*35:74, 35:186, 46:33	2			4																																	
*35:76, 56:09, 56:11	1	2																																			
*35:162	2					7															21																
*35:235	2													16							22	24													31		
*35:273, 49:18, 50:14, 51:158:02	2					7								16	17																						
*35:276, 39:06:01, 39:83														15								24													31		
*39:06:02-39:06:05, 39:34, 39:57, 39:62, 39:64, 39:90														15								24													30	31	
*39:33, 51:92:01-51:92:02														16							22	24													31		
*39:50, 39:102, 40:86																						24														31	
*40:10:01-40:10:02																18								25													
*40:149, 81:05																					?																
*40:159																					22	24															
*40:161, 51:82															17						22																
*40:166	?																				22																
*41:01:01-41:01:04, 41:05-41:07, 41:09, 41:12, 41:14, 41:16, 41:20, 41:22, 41:25-41:26, 41:28-41:29, 41:32-41:35														15	17						20																
*41:17, 51:08:01-51:08:02, 51:20, 51:44N, 51:141, 52:19						7									16	17					20																
*41:21														15	17						20	22	24														
*41:30														15							20																
<b>Well No.</b>	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32					
<b>Well No.</b>	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32					
*42:02, 42:09, 42:17-42:18					5																			23													
*42:04												12			15		17				20														30		
*44:15, 44:18, 45:01:01-45:01:02, 45:03-45:05, 45:07, 45:11-45:15	2														16						20																

Negative Control

*44:20, 44:100															16	17		20	22	24													
*44:47, 44:197, 52:26															16	17			22	24													
*45:02	2														16			20		24							31						
*45:06	2														16			20		23													
*45:08	2												15					20															
*45:09	2														16			20		22	24												
*45:10	2								11									20															
*46:11	2	4										13		16																			
*46:18	2	4										13	15																				
*46:32			4												16				22	23	24												
*49:01:01-49:01:07, 49:02-49:03, 49:06, 49:08-49:10, 49:12-49:14, 49:16-49:17, 49:19N-49:37, 50:01:01, 50:01:03-50:02, 50:05-50:08, 50:10-50:11, 50:13, 50:15-50:16, 50:18-50:19, 50:32, 50:34-50:38, 50:40-50:41	2														16																		
*49:07, 50:31	2								11																								
*50:01:02, 50:04															16																		
*50:09	2														16					24						31							
*50:20, 51:01:15, 51:14, 51:17, 51:85, 51:182															16	17																	
*50:39	2														16										27								
*50:42	2														16	17																	
*51:01:48, 51:35, 51:99, 51:131						7									16																		
*51:05, 51:29, 51:54, 52:24						7									16	17			22														
*51:13:01-51:13:02, 52:14															16						24					31							
*51:21, 51:101, 51:143, 52:37, 78:07													15		17											30							
*51:61:01-51:61:02, 52:21						7									16	17			22	23													
*51:62, C*15:91			3																														
*51:63, 52:39															16						24												
*51:90, 52:12						7									16	17			22	24													
*51:97															16			20			24												
*51:106, 52:25			3												16																		
*51:116, 51:176, 51:179, 52:16, 52:27						7									16	17			21														
*51:127						7									16	17									27		31						
*51:172															16						24				28		31						
*54:01:01, 54:01:03-54:01:06, 54:05N, 54:07-54:08N, 54:12-54:13, 54:17-54:19, 54:22-54:23, 54:27-54:29, 54:31-54:32, 54:34	1	2											13	14	15																		
*54:02	1	2											13	14	15						23												
*54:03	1	2											13	14	16																		
*54:04	1	2												14	15	17																	
*54:06, 54:09	1													14																			
*54:10	1	2										w	14	15						22													
*54:11	1													14	15	17																	
*54:14	1	2													14	15				22	24												
*54:15	1														14	15	17			22	24												
*54:16	1	2											13	14	15									27									
*54:20	1	2	w												14	15				22													
*54:21	1	2											13	14	15					21													
*54:25	1	2											13	w	15																		
*54:26	1	2								11				13	14																		
*54:30		2											13	14	15			18															
*54:33	1	2												14	16					22	23												
*56:01:01:01-56:01:03, 56:01:06-56:01:08, 56:07-56:08, 56:24, 56:26, 56:28N-56:29, 56:34, 56:36, 56:38N, 56:40, 56:45-56:47	1	2												13		16																	
*56:01:04, 56:20:01-56:20:02, 56:27	1													13		16																	
*56:01:05, 56:30	1	2														16																	
*56:02, 56:10, 56:19N	1	2												13																			
*56:04	1													13																			
<b>Well No.</b>	<b>1</b>	<b>2</b>	<b>3</b>	<b>4</b>	<b>5</b>	<b>6</b>	<b>7</b>	<b>8</b>	<b>9</b>	<b>10</b>	<b>11</b>	<b>12</b>	<b>13</b>	<b>14</b>	<b>15</b>	<b>16</b>	<b>17</b>	<b>18</b>	<b>19</b>	<b>20</b>	<b>21</b>	<b>22</b>	<b>23</b>	<b>24</b>	<b>25</b>	<b>26</b>	<b>27</b>	<b>28</b>	<b>29</b>	<b>30</b>	<b>31</b>	<b>32</b>	
<b>Well No.</b>	<b>1</b>	<b>2</b>	<b>3</b>	<b>4</b>	<b>5</b>	<b>6</b>	<b>7</b>	<b>8</b>	<b>9</b>	<b>10</b>	<b>11</b>	<b>12</b>	<b>13</b>	<b>14</b>	<b>15</b>	<b>16</b>	<b>17</b>	<b>18</b>	<b>19</b>	<b>20</b>	<b>21</b>	<b>22</b>	<b>23</b>	<b>24</b>	<b>25</b>	<b>26</b>	<b>27</b>	<b>28</b>	<b>29</b>	<b>30</b>	<b>31</b>	<b>32</b>	
*56:05:01-56:06, 56:21	1						7									16	17																
*56:13	1	2														16				20													
*56:15	1	2					7								16	17																	
*56:16	1	2											13		16					21													
*56:22	1	2													16										24								31
*56:23, 59:01:01:01-59:01:01:02, 59:05, 59:07-59:08		2											13	15				18															

Negative Control

*56:25, 56:37	1	2	3												16																																													
*56:35	1	2												13	16																																													
*56:39	1	2												13	16																																30													
*56:41	1	2						8						13	16																																													
*56:42, 56:44, 59:04		2												13	16	18																																												
*56:43	1	2													16																																													
*59:02															15	17	18																																											
*59:03		2													15	18																																		31										
*59:06		2							10					13	15	18																																												
*59:09		2								11				13		18																																												
*73:01-73:02																																																												
*81:01	1													12																																														
*81:03-81:04N, 81:06-81:07	?													12																																														
*82:01-82:03	1	2																																																										
*83:01	1																																																											
C*01:90			w																																																									
C*02:06, C*15:42												11																																																
<b>Well No.</b>	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32																												

Negative Control