

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32		
HLA-B allele <sup>1,2,3</sup>																																		
*55:01:01:01-55:01:29, 55:53, 55:59-55:60, 55:64, 55:73-55:76, 55:79, 55:84-55:85, 55:87, 55:90-55:95, 55:97N-55:106, 55:108-55:109, 55:111, 55:113-55:116, 55:119, 55:122, 55:125N-55:127	1	2	3												15																			
*55:02:01:01-55:02:16, 55:61, 55:63, 55:65, 55:67, 55:69-55:71, 55:77, 55:80, 55:88-55:89N, 55:96, 55:107, 55:112, 55:117N, 55:120, 55:129-55:132N	1	2											13	15																				
*55:03	1	2	3	4											15																			
*55:04, 55:86, 55:110	1				5																													
*55:05	1	2	3			6									15																			
*55:07	1	2								10			13	14	15																			
*55:08, 55:51	1							7																										
*55:09	1	2	w								11											22												
*55:10	1	2										12	13	15							21													
*55:11	1	2	3						9						15																			
*55:12	1	2						8					13	15																	31			
*55:13	1	2													15									24										
*55:14	1								9																									
*55:15	1	2	3								11			15																				
*55:16	1	2											13	15									23											
*55:17	1									10				15			17																	
*55:18		2											13	15			18								25									
*55:19	1	2											13	15			18																	
*55:20	1													15			17		20															
*55:21	1	2												15										22	23									
*55:22, 56:60 <sup>4</sup>	1	2									11		13																					
*55:23, 55:68	1	2												15								22	24											
*55:24	1	2	3								11																							
*55:25	1	2	3						9					15			18																	
*55:26, 55:62 <sup>5</sup>	1	2											13	15											25									
*55:27	1	2												15			17					22	24											
*55:28:01-55:28:02, 55:81	1	2												15			17																	
*55:29	1	2	3											15									23											
*55:30	1	2											13	15					19															
*55:31	1	2	3											15			18																	
*55:32	1	2												15										24										
*55:33	1	2	3											15									24											
*55:34		2											13	15											25	26								
*55:35	1	2											13	15																30				
*55:36	1	2	3											15				19																
*55:37	1	2											w	15								22												
*55:38	1		3											15																				
*55:39	1	2											13	15			17																	
*55:40	1	2	3											15											25									
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32		
*55:41	1	2												13	15												26	28						
*55:42	1	2												13	15												26							
*55:43	1	2												13	15												27				31			
*55:44	1	2	3											15													27	28						
*55:45	1	2	3											15												26								
*55:46	1	2												15													27							
*55:47		2											13	15												26								
*55:48	1	2												15																				
*55:49	1			4	5								11	13	15																			
*55:50	1	2			5									13	15																30			
*55:52	1	2	w											15								22												
*55:54	1	2	3											13	15																			
*55:55N	1	2	3											15																		31		
*55:56	1	2												15						20														
*55:57	1	2												15																	28			
*55:58	1	2												15										24			28			31				
*55:66	1	2	3					8						15																				
*55:72	1										10			13	15																			
*55:78, 55:118N, 55:121, 15:42, 51:300 <sup>4</sup>																																		

	w								11	12									22																				
*07:78										12									22																				
*07:84, 07:429										12									22																				
*07:143																			22			24																	
*07:209, 07:271, 81:01:01:01-81:01:05, 81:10	1																		21																				
<b>Well No.</b>	<b>1</b>	<b>2</b>	<b>3</b>	<b>4</b>	<b>5</b>	<b>6</b>	<b>7</b>	<b>8</b>	<b>9</b>	<b>10</b>	<b>11</b>	<b>12</b>	<b>13</b>	<b>14</b>	<b>15</b>	<b>16</b>	<b>17</b>	<b>18</b>	<b>19</b>	<b>20</b>	<b>21</b>	<b>22</b>	<b>23</b>	<b>24</b>	<b>25</b>	<b>26</b>	<b>27</b>	<b>28</b>	<b>29</b>	<b>30</b>	<b>31</b>	<b>32</b>							
*07:225, 07:227									9			12																											
*07:234												12							19																				
*07:235		2										12																											
*07:247, 07:270												10	12		14																								
*07:409												12																								30			
*08:01:01:01-08:01:55, 08:01:57-08:05, 08:07-08:08N, 08:10, 08:13-08:14, 08:17-08:20:02, 08:22-08:24, 08:26:01-08:39, 08:41-08:48, 08:50-08:59:02, 08:61-08:78, 08:80-08:82N, 08:85-08:88, 08:90-08:100, 08:102-08:106, 08:108-08:124, 08:126-08:128, 08:130-08:155, 08:157-08:164, 08:166-08:170, 08:172-08:174, 08:176-08:180, 08:182-08:183, 08:185, 08:187, 08:189-01-08:204, 08:206-08:231, 08:233-08:253, 08:256-08:266, 08:268-08:290, 08:292-08:303, 14:05, 14:13, 14:53, 35:87, 38:19, 39:03:01:01-39:03:02, 39:14:01:01-39:14:01:03, 39:24:01-39:24:03, 39:29, 39:37:01:01-39:37:01:02, 39:76, 39:120, 39:144, 39:152, 39:182, 39:184-39:185, 40:39, 41:02:01:01-41:02:11, 41:04, 41:10-41:11, 41:13, 41:15, 41:18, 41:23-41:24, 41:27, 41:31, 41:36, 41:38-41:43, 41:45N-41:47, 41:49, 41:51-41:52, 41:58, 41:65, 41:68-41:69, 41:71-41:72, 41:74, 42:01:05, 42:07, 42:11, 42:26, 44:166, 48:37, C*04:443, C*06:264, C*07:294, C*07:526:01, C*07:530:01, C*07:681, C*08:08:01-08:08:02						5																																	
*08:09, 08:84, 15:83:01:01-15:83:01:02, 41:66, 51:36, 51:153:01-51:153:02												10			15	17		20																					
*08:15			4	5																																			
*08:21, 08:25, 08:175, 15:09:01-15:10:08, 15:30:01:01-15:30:01:03, 15:37, 15:45, 15:48, 15:63, 15:90, 15:99, 15:150, 15:243, 15:248, 15:287, 15:312-15:313, 15:324, 15:329, 15:338, 15:361, 15:427, 15:435, 15:439, 15:445, 15:455, 15:503, 15:530, 15:541, 15:563, 15:611, 15:620, 27:76, 35:09:01:01-35:09:04, 35:12:01:01-35:12:04, 35:18, 35:22:01:01-35:22:01:02, 35:39, 35:44, 35:149, 35:199, 35:201, 35:233, 35:251:01-35:251:02, 35:266, 35:321, 35:372, 35:425, 35:474, 35:535, 35:539, 40:05:01:01-40:05:02, 40:26, 40:63, 40:92, 40:174, 40:324, 40:343, 40:496, 40:523, 41:62, 44:62, 44:77, 44:82, 44:107, 44:123, 44:184, 44:213, 46:74, 46:81, 51:04:01-51:04:02, 51:06:01:01-51:06:04, 51:46, 51:59, 51:64, 51:135, 51:139, 51:148, 51:194, 51:213, 51:293, 51:310, 51:341, 51:353, 57:02:01:01-57:03:05, 57:07, 57:09, 57:12, 57:17, 57:28N, 57:39, 57:42, 57:46, 57:57, 57:63, 57:66, 57:70, 57:82, 57:84, 57:94, 57:96, 57:101, 57:124, 57:137, 57:155, C*03:81, C*03:152, C*03:317, C*03:382, C*03:388, C*03:526																																							
*08:83, 14:43, 14:50, 38:38, A*31:147, C*04:467, C*05:110, C*12:120						5																																30	
*08:101, 41:55						5																																29	
*08:188						5						10																											
*08:254, 41:01:01:01-41:01:08, 41:05-41:07, 41:09, 41:12, 41:14, 41:16, 41:20, 41:22, 41:25-41:26, 41:28-41:29, 41:32-41:35, 41:53-41:54, 41:57, 41:59-41:61, 41:64, 41:67, 41:70, 41:73, 41:75-41:79																																							
*08:267, 18:14, 37:09																																							
*13:01:01:01-13:02:30, 13:02:32-13:02:33, 13:02:35-13:02:37, 13:07N-13:09, 13:11, 13:14-13:15, 13:17, 13:19, 13:22:01-13:23, 13:25, 13:27-13:30, 13:32-13:34, 13:36-13:40, 13:42-13:45, 13:47, 13:49N-13:50, 13:52, 13:54-13:58, 13:60-13:61, 13:63N-13:64, 13:66-13:70, 13:74-13:78, 13:80-13:85, 13:87, 13:90-13:104, 13:106-13:107, 13:109, 13:111-13:119, 13:121-13:124, 13:126, 13:129-13:130, 13:133-13:139N, 13:141-13:144, 13:147-13:153, 13:156-13:158, 13:161N-13:163, 13:165, 13:167-13:173																																							
*13:02:31, 13:65, 13:166, 40:48, 40:377												11	13																										
*13:02:34, 13:89, 13:110, 13:159, 49:07, 50:31												11																											
*13:03, 13:120, 15:552, 44:339, 49:11, 49:55, 50:70, 51:15, 56:14, 56:17, 56:33, 56:68																																							
*13:04, 15:16:01:01-15:16:05, 15:67, 15:95, 15:222, 15:254, 15:362, 15:408, 15:550, 15:602, 15:613, 35:37, 49:04:01-49:05:02, 50:33, 58:122																																							
*13:06																																							
<b>Well No.</b>	<b>1</b>	<b>2</b>	<b>3</b>	<b>4</b>	<b>5</b>	<b>6</b>	<b>7</b>	<b>8</b>	<b>9</b>	<b>10</b>	<b>11</b>	<b>12</b>	<b>13</b>	<b>14</b>	<b>15</b>	<b>16</b>	<b>17</b>	<b>18</b>	<b>19</b>	<b>20</b>	<b>21</b>	<b>22</b>	<b>23</b>	<b>24</b>	<b>25</b>	<b>26</b>	<b>27</b>	<b>28</b>	<b>29</b>	<b>30</b>	<b>31</b>	<b>32</b>							
*13:12:01-13:12:02, 13:146, 15:86, 35:432, 35:446, 44:10, 44:340, 58:73	2												13																										

Negative Control







