

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
HLA-B allele ^{1,2,3}																								
*56:01:01-56:01:07, 56:34-56:35, 56:39, 56:41, 56:43	1	2	3								11													
*56:02	1	2	3	4																				
*56:03		2			5																			
*56:04		2	3	4										14										
*56:05:01		2				6	7										17							
*56:05:02		2					7										17							
*56:06		2					7										17					22		
*56:07	1	2	3						9		11													
*56:08, 56:13 ⁵	1	2	3								11			15										
*56:09, 35:78 ⁴	1	2								10														
*56:10	1	2		4								12												
*56:11	1	2								10			13											
*56:12		2								10														
*56:14, 44:15, 44:18, 45:01, 45:04-45:05, 45:07, 45:11-45:14 ⁵	1		3								11			15										
*56:15	1	2															17							
*56:16	1	2	3								11							18						
*56:17, 56:26 ⁵	1	2	3								11										21			
*56:18, 07:65 ⁵		2																				22		
*56:19N, 56:29 ⁵	1	2	3								11								19					
*56:20:01-56:20:02		2	3								11												23	
*56:21		2					7	8									17							
*56:22	1	2											13											
*56:23, 55:31 ⁴	1	2									11	12									21			
*56:24	1	2	3								11												23	
*56:25, 56:37	1	2	3								11	12												
*56:27		2	3								11			14										
*56:28N	1	2	3								11				15			18						
*56:30	1	2	3								11		13											
*56:31		2							9														22	
*56:32		2				6																	22	
*56:33	1		3				7				11													
*56:36	1	2	3								11				16								22	
*56:38N	1	2	3								11						17							
*56:40	1	2	3		5						11													
*56:42	1		3								11												21	
*07:02:19, 15:01:04, 40:94																							20	
*07:09, 07:11, 07:162, 15:65, 18:21, 35:99, 57:13						w																		
*07:16, 07:37, 07:50, 07:75, 07:91, 07:207, 08:23, 08:54, 08:110, 14:14, 14:18, 15:51-15:52, 15:54, 15:68, 15:91, 15:114, 15:123-15:124, 15:151, 15:156, 15:200, 15:235, 15:252, 15:259, 15:275, 27:18, 27:23, 27:44, 39:05:01-39:05:02, 39:08, 39:11, 39:13:01-39:13:02, 39:20, 39:37, 39:49, 39:55-39:56, 40:12, 40:46, 40:93, 42:11, 48:01:01-48:01:06, 48:03:01-48:17, 48:19-48:24, 48:26-48:34																							21	
*07:20, 07:24, 07:60, 07:100, 07:131, 08:21, 08:25, 15:30, 15:45, 15:48, 15:63, 15:150, 15:248, 15:287, 27:76, 35:09:01-35:09:03, 35:18, 35:22, 35:44, 35:199, 40:05, 40:26, 40:63, 40:92, 40:174, 41:17, 51:135, C*03:81, C*03:152																	17							
*07:78											11	12												23
*07:134, 08:37, 08:89, 08:107, 15:50, 15:199, 18:11, 42:09, 46:21:01-46:21:02, 46:29, C*06:89, C*07:123, C*07:173, C*07:294, C*12:113, C*16:20																								22
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
*07:149, 08:02, 15:38:01-15:38:02, 15:185, 18:01:18, 18:36, 18:50, 18:86, 27:01, 38:02:01-38:02:06, 38:03-38:04, 38:08, 38:15, 38:18, 38:23, 38:29, 38:35, 38:43-38:50									9															

Negative Control

*07:176, 39:40:01N-39:40:02N																			19					
*07:177																16						22		
*07:182N, 39:17				4												14								
*07:206, 15:23, 15:39:01-15:40, 15:61, 15:119, 15:138, 15:195, 15:255, 15:268, 15:288, 15:294N, 27:05:18, 39:63, 46:08, 58:46, C*03:175, C*03:199							6																	
*08:03, 08:52, 08:78, 27:102, 40:13, 40:19, 40:109, 40:117, 51:10, 51:81, 58:27, 59:02, A*24:146																								
*08:49, 14:10, 15:02:01-15:02:08, 15:13:01-15:13:02, 15:25:01-15:25:03, 15:36, 15:77, 15:88-15:89, 15:106, 15:112, 15:121, 15:139, 15:144, 15:154, 15:165, 15:170, 15:204, 15:213-15:214, 15:240, 15:250, 15:265, 15:271, 15:283, 15:289, 15:291, 15:297, 15:301-15:302N, 27:19, 27:30, 35:16-35:17:02, 35:61:01-35:61:02, 35:113, 35:125, 35:164-35:165N, 35:196, 35:208, 35:215, 37:01:01-37:01:03, 37:01:05-37:01:07, 37:01:09, 37:03N-37:06:02, 37:08, 37:10-37:11, 37:13-37:34, 37:36-37:43, 37:45-37:47, 38:20, 39:42, 44:03:01-44:05:04, 44:07, 44:13-44:14, 44:17, 44:25-44:26, 44:28:01-44:30, 44:32, 44:35-44:39, 44:42-44:43:02, 44:45-44:46, 44:50:01-44:51, 44:56N-44:58N, 44:60-44:61N, 44:64:01-44:64:02, 44:69-44:70, 44:75-44:76, 44:78-44:79:02, 44:81, 44:85, 44:88, 44:92, 44:94, 44:96, 44:98, 44:103, 44:105, 44:108N-44:111, 44:114-44:115, 44:117, 44:120, 44:122, 44:124-44:125, 44:128-44:129, 44:133-44:134, 44:136, 44:139, 44:141, 44:143-44:144, 44:146-44:147, 44:150, 44:153-44:155, 44:157, 44:159-44:161, 44:163-44:165, 44:167, 44:175, 44:178, 44:180-44:181, 44:183, 44:186, 51:42, 53:07, 57:01:01-57:01:04, 57:01:06-57:01:17, 57:04, 57:06, 57:08, 57:10, 57:15-57:16, 57:18-57:20, 57:23-57:25, 57:27, 57:29-57:30, 57:32-57:38, 57:40-57:41, 57:43-57:45, 57:48-57:56, 57:59-57:62, 57:64-57:65, 57:67-57:69, 58:10N																								
*08:87, 44:20, 44:41:01-44:41:02, 44:53, 44:99-44:100, 44:106, 44:138Q																16								
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24

Negative Control

*15:18:03			4		6								14								21			
*15:21, 15:44						7			10													21		
*15:47:01-15:47:02, 15:49, 15:221					w																	21		
*15:58, 35:205, 38:30, 39:32-39:33, 39:74, 51:13:01, 51:92, 52:14													13											
*15:62, 48:25	1				6				10													21		
*15:66, 15:189, 35:135, 38:28, 40:95, 51:01:15, 51:14, 51:63, 51:85, 51:97, 52:26, 78:07						7																		
*15:69, 15:93					6																	21	22	
*15:73			3	4							w		14											
*15:80					6				10													21		
*15:86	1			4									12											
*15:115, 18:57, 51:45					6				9															
*15:133, 15:229, 15:253													13									21		
*15:186, 39:07																						21	22	
*15:194, 35:11:01-35:11:02, 53:08:01-53:08:02, 58:19	1				6				10															
*15:223, 57:22					6				10															
*15:224				4								12		14										
*15:303	1		3								11	w												
*18:01:01-18:01:13, 18:01:15-18:01:17, 18:01:19-18:09, 18:12-18:14, 18:17N-18:19, 18:23N-18:25, 18:27-18:29, 18:31-18:34, 18:38-18:40, 18:42-18:49, 18:51, 18:53-18:55, 18:59-18:60, 18:62-18:68, 18:70-18:85, 18:87-18:96, 39:43									9													22		
*18:15					6				9													22		
*18:22, 35:190	1								9	10												22		
*18:30					w				9													22		
*18:37									9													21	22	
*18:52	1								9													22		
*18:69									9	10												22		
*27:77			2																			21		
*35:02:01-35:02:06, 35:04:01-35:04:03, 35:34, 35:81, 35:83, 35:95, 35:129N, 35:154, 35:162, 35:172, 35:182-35:184, 35:211, 35:220, 40:28	1								10							17								
*35:03:01-35:03:05, 35:03:07-35:03:12, 35:06, 35:13, 35:36, 35:38, 35:55-35:56, 35:70, 35:74, 35:84, 35:98, 35:106, 35:109, 35:128, 35:136, 35:150, 35:152-35:153, 35:155-35:156, 35:160, 35:163, 35:167, 35:169, 35:181, 35:193, 35:195, 35:198, 35:204, 35:221, 35:223, 35:231, 35:236-35:237, 35:243, 35:246, 53:04, 53:31, 58:34	1								10				13											
*35:11:03	1				w				10															
*35:12:01-35:12:03, 35:39, 35:149, 35:201, 35:233, 44:62, 44:77, 44:82, 44:107, 44:123, 57:12, 57:28N									10							17								
*35:21, 53:06, 53:28	1				6				9	10														
*35:24:01-35:24:02, 53:02, 58:09	1								9	10														
*35:25, 35:49, 48:02:01-48:02:03	1								10													21		
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
*35:26	1						7			10												21		
*35:35, 35:93, 58:05, 58:35	1									10													22	
*35:59, 35:179, 35:242, 44:174										10			13											
*35:75, 35:127, 35:151, 35:235	1												13											
*35:76	1	2								10													22	
*35:82	1						7			10														
*35:85	1						7			10			13											
*35:186	1									10					15									
*35:188	1					6				9	10												22	
*35:232	1					w																		
*37:01:04										10											20			
*37:35, 57:26										10													22	
*40:10:02					5																			
*40:48				4										14									23	
*40:71			3	4										14										

Negative Control

*40:149, 41:22																		18						
*40:166	?													16						22				
*44:02:01:01-44:02:29, 44:06, 44:08-44:09, 44:11-44:12, 44:16, 44:19N, 44:21-44:24, 44:27:01-44:27:02, 44:33-44:34:02, 44:49, 44:52N, 44:55, 44:59:01-44:59:02, 44:63, 44:66-44:68, 44:71-44:74, 44:80, 44:83-44:84:02, 44:86-44:87, 44:89, 44:91, 44:93, 44:95, 44:97, 44:101-44:102, 44:104, 44:112-44:113, 44:116, 44:118-44:119, 44:121, 44:126:01-44:127, 44:131-44:132, 44:137, 44:142, 44:145, 44:148-44:149N, 44:151-44:152, 44:162, 44:168-44:173, 44:176-44:177, 44:179, 44:185, 44:187							10							16										
*44:10, 50:17	1	3							10															
*44:40, 44:130, 44:156, 57:14									9	10														
*44:44									9	10								16						
*44:48	1									10								16						
*44:90		2								10								16		22				
*44:140			3							10				14	15									
*45:06	1	3								11				15	16					22				
*45:08, 45:10	1									11				15										
*46:03						w														22				
*48:18																				21				
*49:18	1						7	8										17						
*49:25	1	3									11							18						
*50:01:02			3								11													
*50:04			3								11			14										
*50:12, 50:16	1	3																						
*50:14	1						7											17						
*51:01:01-51:01:03, 51:01:05-51:01:07, 51:01:09-51:01:12, 51:01:14, 51:01:16-51:01:17, 51:01:19-51:01:31, 51:01:33-51:01:38, 51:01:41-51:01:42, 51:01:44-51:02:03, 51:02:05-51:03, 51:05, 51:07:01-51:09:02, 51:11N-51:12, 51:18-51:20, 51:22-51:23, 51:24:03-51:24:05, 51:26-51:30, 51:32-51:33, 51:35, 51:39-51:41N, 51:43-51:44N, 51:48-51:49, 51:51-51:53, 51:55, 51:57-51:58, 51:60-51:61:02, 51:65-51:77, 51:79-51:80, 51:83-51:84, 51:86-51:89, 51:91, 51:94-51:96, 51:98N-51:100, 51:102-51:105, 51:107-51:134, 51:136-51:138, 51:140-51:142, 51:145-51:147, 51:149N-51:152, 51:154-51:156, 51:158-51:166, 52:01:01:01-52:01:04, 52:01:06-52:01:14, 52:01:16-52:07, 52:09-52:11, 52:13, 52:15-52:19, 52:21-52:24, 52:27-52:36, 58:08:01-58:08:02							7	8										17						
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
*51:01:04, 51:01:13, 51:02:04, 51:06:01-51:06:02, 51:24:01-51:24:02, 51:38, 51:59, 51:64, 51:148, 52:01:05, 52:01:15								8									17							
*51:01:18							7	8									17		20					
*51:01:32, 51:01:39, 51:16-51:17, 51:21, 51:31, 51:34, 51:36, 51:82, 51:93, 51:101, 51:143-51:144, 51:153, 52:08							7	8																
*51:04, 51:46, 51:139, 57:02:01-57:03:02, 57:07, 57:09, 57:17, 57:39, 57:42, 57:46, 57:57, 57:63, 57:66								8		10							17							
*51:13:02							7						13											
*51:37						6	7		9															
*51:56:01-51:56:02, 53:19, 57:05, 58:28	1							8		10							17							
*51:62			3	4								12		14										
*51:106			3				7					12												
*52:25			3								11	12												
*54:01:01, 54:01:03-54:02, 54:05N, 54:08N, 54:10, 54:12-54:13, 54:17-54:20, 54:22-54:23, 54:25, 54:27-54:30, 54:32, 55:02:07	1	2									11	12								20				

Negative Control

*54:01:02, 55:35	1	2		5						11	12													
*54:03	1	2	3							11								20						
*54:04, 54:15	1	2																20						
*54:06, 54:11		2																20						
*54:07, 54:24	1									11	12							20						
*54:09		2																20				22		
*54:14	1	2																20				22		
*54:16, 54:26	1	2								11								20						
*54:21	1	2								11	12						18	20						
*54:31	1	2									12							20						
*55:01:01-55:01:03, 55:01:05, 55:01:08-55:02:03, 55:02:05-55:02:06, 55:02:08-55:03, 55:05, 55:07, 55:09, 55:11-55:12, 55:15-55:16, 55:19, 55:21, 55:24-55:26, 55:29-55:30, 55:33, 55:36-55:37, 55:40, 55:43, 55:45, 55:48, 55:50, 55:52, 55:54- 55:55N, 55:57, 55:59-55:60, 55:62-55:63	1	2								11	12													
*55:01:04	1	2				7					12													
*55:01:06, 55:02:04		2								11	12													
*55:01:07, 55:39, 55:42, 55:44, 55:53, 55:61	1	2									12													
*55:04, 55:17, 55:20, 55:49, 81:01		2																						
*55:08, 55:51		2															17							
*55:10	1	2								11	12						18							
*55:13, 55:27-55:28, 55:32, 55:58	1	2																						
*55:14		2				6				10													22	
*55:18	1										11	12										21		
*55:23	1	2																					22	
*55:38		2								11	12		14											
*55:56	1	2								11	12		15											
*57:58										10							18							
*58:01:08	1									10												20		
*81:03-81:04N, 81:06		?																						
*82:01-82:03	1	2	3	4										15			18							
*83:01		2								10						16								
A*02:339, C*03:19, C*15:26														w										
C*01:90				4							12												23	
C*02:06, C*02:47, C*07:289, C*12:15, C*15:37, C*15:58, C*15:72				4																				
C*03:102										10				w										
C*15:09, C*15:19, C*15:27, C*15:61					6																		22	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24

Negative Control