

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
HLA-B allele ^{1,2,3}																								
*56:01:01:01-56:01:08, 56:34-56:35, 56:39, 56:41, 56:43, 56:46-56:47	1	2	3								11													
*56:02	1	2	3	4																				
*56:03		2			5																			
*56:04		2	3	4										14										
*56:05:01		2				6	7										17							
*56:05:02		2					7										17							
*56:06		2					7										17					22		
*56:07	1	2	3						9		11													
*56:08	1	2	3								11				15				19					
*56:09, 35:78 ⁴	1	2								10														
*56:10	1	2		4								12												
*56:11	1	2								10			13											
*56:12		2								10														
*56:13	1	2	3								11				15									
*56:14	1		3								11				15				19					
*56:15	1	2															17							
*56:16	1	2	3								11						18							
*56:17, 56:26 ⁵	1	2	3								11										21			
*56:18, 07:65 ⁵		2																				22		
*56:19N, 56:29 ⁵	1	2	3								11								19					
*56:20:01-56:20:02		2	3								11													23
*56:21		2					7	8									17							
*56:22	1	2											13											
*56:23, 55:31 ⁴	1	2									11	12									21			
*56:24	1	2	3								11													23
*56:25, 56:37	1	2	3								11	12												
*56:27		2	3								11			14										
*56:28N	1	2	3								11				15		18							
*56:30	1	2	3								11		13											
*56:31		2							9														22	
*56:32		2																					22	
*56:33	1		3				7				11													
*56:36	1	2	3								11				16								22	
*56:38N	1	2	3								11						17							
*56:40	1	2	3		5						11													
*56:42, 56:44	1		3								11											21		
*56:45	1	2	3								11												22	
*56:48-56:49		2	3								11													
*07:02:19, 15:01:04, 40:94																					20			
*07:09, 07:11, 07:162, 07:237, 07:255, 15:65, 18:21, 35:99, 46:62, 57:13						w																		
*07:16, 07:37, 07:50, 07:75, 07:91, 07:207, 07:228, 08:23, 08:54, 08:110, 14:14, 14:18, 15:51-15:52, 15:54, 15:68, 15:91, 15:114, 15:123-15:124, 15:151, 15:156, 15:200, 15:235, 15:252, 15:259, 15:275:01, 15:323, 27:18, 27:23, 27:44, 27:131, 39:05:01-39:05:02, 39:08, 39:11, 39:13:01-39:13:02, 39:20, 39:37, 39:49, 39:55-39:56, 40:12, 40:46, 40:93, 42:11, 48:01:01-48:01:06, 48:03:01-48:17, 48:19-48:24, 48:26-48:37																						21		
*07:20, 07:24, 07:60, 07:100, 07:131, 07:220, 07:223, 08:21, 08:25, 15:30, 15:45, 15:48, 15:63, 15:150, 15:248, 15:287, 15:324, 27:76, 35:09:01-35:09:03, 35:18, 35:22, 35:44, 35:199, 35:251, 40:05, 40:26, 40:63, 40:92, 40:174, 41:17, 44:184, 44:213, 51:135, C*03:81, C*03:152																	17							
*07:78											11	12												23
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24

Negative Control

*07:134, 08:37, 08:89, 08:107, 15:50, 15:199, 18:11, 18:110, 38:52, 39:91, 42:09, 46:21:01-46:21:02, 46:29, C*06:89, C*07:123, C*07:173, C*07:294, C*12:113, C*16:20																					22			
*07:149, 08:02, 08:117, 15:38:01-15:38:02, 15:185, 18:01:18, 18:36, 18:50, 18:86, 18:106, 27:01, 27:142, 38:02:01-38:02:06, 38:03-38:04, 38:08, 38:15, 38:18, 38:23, 38:29, 38:35, 38:43-38:50								9																
*07:176, 39:40:01N-39:40:02N, C*07:46, C*07:356																	19							
*07:177													16								22			
*07:182N, 39:17		4											14											
*07:206, 15:23, 15:39:01-15:40, 15:61, 15:119, 15:138, 15:195, 15:255, 15:268, 15:288, 15:294N, 15:310, 18:101, 27:05:18, 39:63, 46:08, 58:46, C*03:175, C*03:199, C*03:245						6																		
*07:235, 13:01:01-13:01:10, 13:07N, 13:12:01-13:13:02, 13:17, 13:22:01-13:23, 13:25-13:26:02, 13:28-13:29, 13:36, 13:39, 13:43, 13:50-13:52, 13:57, 13:60-13:61, 13:63N, 13:73, 13:76N-13:78, 13:80, 13:83, 15:20, 15:85, 35:01:01:01-35:01:42, 35:01:44-35:01:45, 35:03:06, 35:07-35:08:08, 35:10, 35:14:01-35:15:02, 35:19-35:20:02, 35:23, 35:27-35:30, 35:33, 35:40N-35:42:02, 35:45-35:48, 35:50, 35:52, 35:54, 35:57, 35:62-35:63, 35:65Q, 35:69, 35:71, 35:77, 35:80, 35:90-35:92, 35:94, 35:96, 35:100-35:101:02, 35:103-35:105, 35:107-35:108:02, 35:110-35:112, 35:115-35:116, 35:120-35:124, 35:126, 35:130N-35:134N, 35:137-35:148, 35:157-35:159, 35:161, 35:166, 35:168, 35:170-35:171, 35:173N, 35:175-35:178, 35:180, 35:187, 35:189, 35:191-35:192, 35:194, 35:197, 35:200, 35:202-35:203, 35:206-35:207, 35:209-35:210, 35:212-35:214, 35:216N-35:219, 35:222, 35:224-35:229, 35:238-35:241, 35:244-35:245, 35:247-35:250, 35:252-35:255, 35:259, 35:263-35:264, 35:268-35:269, 35:271-35:272, 35:275, 35:280, 35:286, 35:288-35:289, 35:291, 35:293, 35:295, 46:33, 53:01:01-53:01:11, 53:03, 53:05, 53:09-53:13, 53:15-53:18, 53:20-53:27, 53:29-53:30, 53:32, 53:34-53:35, 53:37-53:39, 58:01:01:01-58:01:02, 58:01:04-58:01:07, 58:01:09-58:01:15, 58:01:17-58:01:19, 58:04, 58:11-58:15, 58:17N, 58:21-58:24, 58:29, 58:31N-58:33, 58:36-58:37, 58:39N-58:42, 58:44-58:45:02, 58:47-58:58, 58:62-58:64, 58:66-58:69, 58:72N	1							10																
*08:03, 08:52, 08:78, 27:102, 40:13, 40:19, 40:109, 40:117, 40:292, 51:10, 51:81, 58:27, 59:02, A*24:146								8																
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
*08:49, 13:79, 14:10, 15:02:01-15:02:09, 15:13:01-15:13:02, 15:25:01-15:25:03, 15:36, 15:77, 15:88-15:89, 15:106, 15:112, 15:121, 15:139, 15:144, 15:154, 15:165, 15:170, 15:204, 15:213-15:214, 15:240, 15:250, 15:265, 15:271, 15:283, 15:289, 15:291, 15:297, 15:301-15:302N, 15:308, 15:319, 15:325, 15:328, 15:330, 15:341, 15:345, 15:357-15:358, 27:19, 27:30, 35:01:43, 35:16-35:17:02, 35:61:01-35:61:02, 35:113, 35:125, 35:164-35:165N, 35:196, 35:208, 35:215, 37:01:01-37:01:03, 37:01:05-37:01:07, 37:01:09, 37:01:11, 37:03N-37:06:02, 37:08, 37:10-37:11, 37:13-37:34, 37:36-37:43, 37:45-37:47, 37:49-37:50, 37:52-37:55, 38:30, 38:43, 44:03:01-01, 44:05:01, 44:07																								

Negative Control

*15:03:01:01-15:03:05, 15:18:01-15:18:02, 15:18:04-15:18:06, 15:29, 15:64:01-15:64:02, 15:72, 15:74, 15:98, 15:103, 15:108, 15:127, 15:131-15:132, 15:134, 15:153, 15:158, 15:161, 15:173, 15:176, 15:197-15:198, 15:210, 15:220, 15:226N, 15:238, 15:242, 15:263, 15:266, 15:274, 15:281-15:282, 15:290, 15:292-15:293, 15:306-15:307, 15:311, 15:314, 15:337, 15:354					6																	21		
*15:09-15:10:04, 15:37, 15:90, 15:99, 15:243, 15:312-15:313, 15:329, 15:338														17									21	
*15:16:01-15:16:03, 15:67, 15:95, 15:254, 15:362, 35:58, 49:04:01-49:04:02, 58:06	1					6																		
*15:18:03			4			6								14									21	
*15:21, 15:44								7		10														
*15:47:01-15:47:02, 15:49, 15:221						w																	21	
*15:58, 35:205, 38:30, 39:32-39:33, 39:74, 46:61, 51:13:01, 51:92:01, 52:14													13											
*15:62, 48:25	1					6				10													21	
*15:66, 15:189, 35:135, 38:28, 40:95, 51:01:15, 51:14, 51:63, 51:85, 51:97, 51:172, 51:182, 52:26, 52:39, 78:07-78:08								7																
*15:69, 15:93						6																	21	22
*15:73			3	4								w		14										
*15:80						6				10													21	
*15:86	1			4										12										
*15:115, 18:57, 51:45						6				9														
*15:133, 15:229, 15:253														13									21	
*15:186, 39:07																							21	22
*15:194, 35:11:01-35:11:02, 53:08:01-53:08:02, 58:19	1					6					10													
*15:223, 57:22						6					10													
*15:224				4									12		14									
*15:303	1		3									11	w											
*15:335						6				9													21	
*15:361, 51:01:04, 51:01:13, 51:02:04, 51:06:01-51:06:02, 51:24:01-51:24:02, 51:38, 51:59, 51:64, 51:148, 52:01:05, 52:01:15									8							17								
*18:01:01:01-18:01:13, 18:01:15-18:01:17, 18:01:19, 18:01:21-18:09, 18:12:01-18:14, 18:17N-18:19, 18:23N-18:25, 18:27-18:29, 18:31-18:34, 18:38-18:40, 18:42-18:49, 18:51, 18:53-18:55, 18:59-18:60, 18:62-18:68, 18:70-18:85, 18:87-18:96, 18:98-18:100, 18:102-18:104, 18:107-18:109, 18:111-18:113, 39:43										9													22	
*18:01:20										9													20	22
*18:15						6				9													22	
*18:22, 18:105, 35:190	1									9	10												22	
*18:30						w				9													22	
*18:37										9													21	22
*18:52	1									9													22	
*18:69										9	10												22	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
*27:77, 27:129		2																					21	

Negative Control

*46:03, C*01:102					w														22					
*48:18								8											21					
*49:18, 51:158:02	1						7	8						17										
*49:25	1		3										11					18						
*50:01:02			3										11											
*50:04			3										11		14									
*50:12, 50:16, 50:41-50:42	1		3																					
*50:14	1							7										17						
*51:01:01:01-51:01:03, 51:01:05-51:01:07, 51:01:09-51:01:12, 51:01:14, 51:01:16-51:01:17, 51:01:19-51:01:31, 51:01:33-51:01:38, 51:01:41- 51:01:42, 51:01:44-51:02:03, 51:02:05-51:03, 51:05, 51:07:01-51:09:02, 51:11N-51:12, 51:18- 51:20, 51:22-51:23, 51:24:03-51:24:05, 51:26- 51:30, 51:32-51:33, 51:35, 51:39-51:41N, 51:43- 51:44N, 51:48-51:49, 51:51-51:53, 51:55, 51:57- 51:58, 51:60-51:61:02, 51:65-51:77, 51:79-51:80, 51:83-51:84, 51:86-51:89, 51:91, 51:94-51:96, 51:98N-51:100, 51:102-51:105, 51:107-51:134, 51:136-51:138, 51:140-51:142, 51:145-51:147, 51:149N-51:152, 51:154-51:156, 51:158:01, 51:159- 51:171, 51:173Q-51:181, 51:183-51:190, 52:01:01:01-52:01:04, 52:01:06-52:01:14, 52:01:16- 52:01:20, 52:01:22-52:01:24, 52:01:27-52:07, 52:09- 52:11, 52:13, 52:15-52:19, 52:21-52:24, 52:27- 52:36, 52:38, 52:40-52:45, 52:47-52:50, 58:08:01- 58:08:02							7	8									17							
*51:01:18								7	8									17	20					
*51:01:32, 51:01:39, 51:16-51:17, 51:21, 51:31, 51:34, 51:36, 51:82, 51:93, 51:101, 51:143-51:144, 51:153, 52:01:21, 52:08, 52:37								7	8															
*51:04, 51:46, 51:139, 57:02:01-57:03:02, 57:07, 57:09, 57:17, 57:39, 57:42, 57:46, 57:57, 57:63, 57:66, 57:70									8	10								17						
*51:13:02, 51:92:02								7						13										
*51:37							6	7	9															
*51:56:01-51:56:03, 53:19, 57:05, 58:28:01- 58:28:02, 58:65	1								8	10								17						
*51:62			3	4									12	14										
*51:106			3					7					12											
*52:25			3										11	12										
*54:01:01, 54:01:03-54:01:05, 54:01:07-54:02, 54:05N, 54:08N, 54:10, 54:12-54:13, 54:17-54:20, 54:22-54:23, 54:25, 54:27-54:30, 54:32, 54:34, 55:02:07	1	2											11	12					20					
*54:01:02, 55:35	1	2			5								11	12										
*54:01:06		2											11	12					20					
*54:03	1	2	3										11						20					
*54:04, 54:15	1	2																	20					
*54:06, 54:11		2																	20					
*54:07, 54:24	1												11	12					20					
*54:09		2																	20					
*54:14	1	2																	20					
*54:16, 54:26	1	2											11						20					
*54:21	1	2											11	12			18		20					
*54:31	1	2												12					20					
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
*54:33	1	2	3									11	w							20				
*55:01:01-55:01:03, 55:01:05, 55:01:08-55:02:03, 55:02:05-55:02:06, 55:02:08-55:03, 55:05, 55:07, 55:09, 55:11-55:12, 55:15-55:16, 55:19, 55:21, 55:24-55:26, 55:29-55:30, 55:33, 55:36-55:37, 55:40, 55:43, 55:45, 55:48, 55:50, 55:52, 55:54- 55:55N, 55:57, 55:59-55:60, 55:62-55:63, 55:65- 55:67, 55:69-55:71, 55:73-55:75	1	2										11	12											
*55:01:04	1	2						7					12											

Negative Control

*55:01:06, 55:02:04		2								11	12													
*55:01:07, 55:39, 55:42, 55:44, 55:53, 55:61	1	2									12													
*55:04, 55:17, 55:20, 55:49, 81:01		2																						
*55:08, 55:51		2															17							
*55:10	1	2									11	12						18						
*55:13, 55:27-55:28, 55:32, 55:58	1	2																						
*55:14		2				6				10													22	
*55:18	1										11	12										21		
*55:23	1	2																					22	
*55:38		2									11	12				14								
*55:56	1	2									11	12				15								
*55:68	1	2				6																	22	
*57:58										10										18				
*58:01:08	1									10												20		
*59:08	1						7		9		11	12												
*81:03-81:04N, 81:06-81:07		?																						
*82:01-82:03	1	2	3	4												15			18					
*83:01		2								10							16							
A*02:339, C*15:26														w										
A*02:510														14										
C*01:90			4									12											23	
C*02:06, C*02:47, C*07:289, C*12:15, C*15:37, C*15:58, C*15:72			4																					
C*03:19			3											w										
C*03:102			3							10				w										
C*08:113, C*15:09, C*15:19, C*15:27, C*15:61, C*15:107						6																	22	
C*15:15			3																					
C*15:91																								23
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24

Negative Control