

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
HLA-B allele ^{1,2,3}																								
*56:01:01:01-56:01:08, 56:34-56:35, 56:39, 56:41, 56:43, 56:46-56:47	1	2	3							11														
*56:02:01-56:02:02, 56:51	1	2	3	4																				
*56:03		2			5																			
*56:04		2	3	4										14										
*56:05:01		2				6	7										17							
*56:05:02		2					7										17							
*56:06		2					7										17					22		
*56:07	1	2	3						9	11														
*56:08	1	2	3							11					15				19					
*56:09, 35:78 ⁴	1	2								10														
*56:10	1	2		4								12												
*56:11	1	2								10			13											
*56:12		2								10														
*56:13	1	2	3								11				15									
*56:14	1		3								11				15				19					
*56:15	1	2															17							
*56:16	1	2	3								11						18							
*56:17, 56:26 ⁵	1	2	3								11										21			
*56:18, 07:65 ⁵		2																				22		
*56:19N, 56:29 ⁵	1	2	3								11								19					
*56:20:01-56:20:02		2	3								11													23
*56:21		2					7	8									17							
*56:22	1	2											13											
*56:23, 55:31, 55:78 ⁴	1	2									11	12									21			
*56:24	1	2	3								11													23
*56:25, 56:37	1	2	3								11	12												
*56:27		2	3								11			14										
*56:28N	1	2	3								11				15		18							
*56:30	1	2	3								11		13											
*56:31		2							9														22	
*56:32		2																					22	
*56:33	1		3								11													
*56:36	1	2	3								11				16								22	
*56:38N	1	2	3								11						17							
*56:40	1	2	3		5						11													
*56:42, 56:44	1		3								11											21		
*56:45	1	2	3								11												22	
*56:48-56:49		2	3								11													
*56:50, 55:13, 55:27-55:28, 55:32, 55:58 ⁴	1	2																						
*07:02:19, 15:01:04, 40:94																						20		
*07:09, 07:11, 07:162, 07:237, 07:255, 15:65, 18:21, 35:99, 46:62, 57:13						w																		
*07:16, 07:37, 07:50, 07:75, 07:91, 07:207, 07:228, 08:23, 08:54:01-08:54:02, 08:110, 14:14, 14:18, 15:51-15:52, 15:54, 15:68, 15:91, 15:114, 15:123-15:124, 15:151, 15:156, 15:200, 15:235, 15:252, 15:259, 15:275:01-15:275:02, 15:323, 27:18, 27:23, 27:44, 27:131, 39:05:01-39:05:02, 39:08, 39:11, 39:13:01-39:13:02, 39:20, 39:37, 39:49, 39:55-39:56, 39:113, 39:118, 40:12, 40:46, 40:93, 42:11, 48:01:01-48:01:06, 48:03:01-48:17, 48:19-48:24, 48:26-48:40																							21	
*07:20, 07:24, 07:60, 07:100, 07:131, 07:220, 07:223, 08:21, 08:25, 15:30, 15:45, 15:48, 15:63, 15:150, 15:248, 15:287, 15:324, 27:76, 35:09:01-35:09:03, 35:18, 35:22, 35:44, 35:199, 35:251, 35:321, 40:05, 40:26, 40:63, 40:92, 40:174, 40:324, 40:343, 41:17, 44:184, 44:213, 51:135, C*03:81, C*03:152, C*03:317																	17							
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
*07:78											11	12												23

Negative Control

*07:134, 08:37, 08:89, 08:107, 15:50, 15:199, 15:348, 18:11, 18:110, 38:52, 39:91, 42:09, 46:21:01-46:21:02, 46:29, C*06:89, C*07:123, C*07:173, C*07:294, C*12:113, C*16:20																							22	
*07:149, 08:02, 08:117, 15:38:01-15:38:02, 15:185, 18:01:18, 18:36, 18:50, 18:86, 18:106, 27:01, 27:142, 38:02:01-38:02:06, 38:03-38:04, 38:08, 38:15, 38:18, 38:23, 38:29, 38:35, 38:43-38:50, 38:62, 38:64								9																
*07:176, 39:40:01N-39:40:02N, C*03:342, C*07:46, C*07:356																19								
*07:177														16									22	
*07:182N, 39:17	4												14											
*07:206, 08:26:03, 15:23, 15:39:01-15:40:02, 15:61, 15:119, 15:138, 15:195, 15:255, 15:268, 15:288, 15:294N, 15:310, 15:376, 15:397, 18:101, 27:05:18, 39:63, 46:08, 58:46, C*03:175, C*03:199, C*03:245						6																		
*07:235, 13:01:01-13:01:12, 13:07N, 13:12:01- 13:13:02, 13:17, 13:22:01-13:23, 13:25-13:26:02, 13:28-13:29, 13:36, 13:39, 13:43, 13:50-13:52, 13:57, 13:60-13:61, 13:63N, 13:73, 13:76N-13:78, 13:80, 13:83, 13:87, 13:90, 13:92, 13:95, 15:20, 15:85, 15:393, 35:01:01:01-35:01:42, 35:01:44- 35:01:45, 35:03:06, 35:07-35:08:09, 35:10, 35:14:01-35:15:02, 35:19-35:20:02, 35:23, 35:27- 35:30, 35:33, 35:40N-35:42:02, 35:45-35:48, 35:50, 35:52, 35:54, 35:57, 35:62-35:63, 35:65Q, 35:69, 35:71, 35:77, 35:80, 35:90-35:92, 35:94, 35:96, 35:100-35:101:02, 35:103-35:105, 35:107- 35:108:02, 35:110-35:112, 35:115-35:116, 35:120- 35:124, 35:126, 35:130N-35:134N, 35:137-35:148, 35:157-35:159, 35:161, 35:166, 35:168, 35:170- 35:171, 35:173N, 35:175-35:178, 35:180, 35:187, 35:189, 35:191-35:192, 35:194, 35:197, 35:200, 35:202-35:203, 35:206-35:207, 35:209-35:210, 35:212-35:214, 35:216N-35:219, 35:222, 35:224- 35:229, 35:238-35:241, 35:244-35:245, 35:247- 35:250, 35:252-35:255, 35:259, 35:263-35:264, 35:268-35:269, 35:271-35:272, 35:275, 35:280, 35:283-35:284, 35:286, 35:288-35:289, 35:291, 35:293, 35:295, 35:297, 35:299-35:308, 35:310, 35:313-35:315, 35:318-35:319, 35:322, 35:324- 35:325, 35:327-35:328, 44:256, 46:33, 46:67, 53:01:01-53:01:11, 53:03, 53:05, 53:09-53:13, 53:15-53:18, 53:20-53:27, 53:29-53:30, 53:32, 53:34-53:35, 53:37-53:40, 53:43, 53:45, 58:01:01:01-58:01:02, 58:01:04-58:01:07, 58:01:09- 58:01:15, 58:01:17-58:01:20, 58:04, 58:11-58:15, 58:17N, 58:21-58:24, 58:29, 58:31N-58:33, 58:36- 58:37, 58:39N-58:42, 58:44-58:45:02, 58:47-58:58, 58:62-58:64, 58:66-58:72N, 58:74-58:75, 58:77- 58:83	1								10															
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
*08:03, 08:52, 08:78, 27:102, 40:13, 40:19, 40:109, 40:117, 40:292, 40:340, 41:46, 51:10, 51:81, 58:27, 59:02, A*24:146								8																
*08:49, 13:79, 14:10, 15:02:01-15:02:11, 15:13:01- 15:13:02, 15:25:01-15:25:03, 15:36, 15:77, 15:88- 15:89, 15:106, 15:112, 15:121, 15:139, 15:144, 15:154, 15:165, 15:170, 15:204, 15:213-15:214, 15:240, 15:250, 15:265, 15:271, 15:283, 15:289, 15:291, 15:297, 15:301-15:302N, 15:308, 15:319, 15:325, 15:328, 15:330, 15:341, 15:345, 15:357- 15:358, 15:366, 15:374, 15:378, 15:399, 27:19, 27:30 27:150 35:01:43 35:16:35-17:02 35:61:01-																								

Negative Control

*13:31	1										11	w													
*13:48	1		3								11	12													
*13:54, 55:22, 55:41, 55:46	1	2									11														
*13:62, 27:153, 51:01:08, 51:01:40, 51:01:43, 51:50, 51:54, 51:78:01-51:78:02, 51:90, 52:12, 52:20, 78:01:01-78:06, 78:09							7												17						
*13:65											11														
*13:86, 35:02:01-35:02:08, 35:04:01-35:04:03, 35:34, 35:81, 35:83, 35:95, 35:129N, 35:154, 35:162, 35:172, 35:182-35:184, 35:211, 35:220, 35:258, 35:270, 35:285, 35:309, 35:311, 35:316, 35:323, 40:28, 53:36, 53:42	1										10								17						
*14:54, 15:66, 15:189, 35:135, 38:28, 40:95, 51:01:15, 51:14, 51:63:01, 51:85, 51:97, 51:182, 51:192, 52:26, 52:39, 52:62-52:63, 78:07-78:08							7																		
*15:03:01:01-15:03:06, 15:18:01:01-15:18:02, 15:18:04-15:18:06, 15:29, 15:64:01-15:64:02, 15:72, 15:74, 15:98, 15:103, 15:108, 15:127, 15:131-15:132, 15:134, 15:153, 15:158, 15:161, 15:173, 15:176, 15:197:01-15:198, 15:210, 15:220, 15:226N, 15:238, 15:242:01, 15:263, 15:266, 15:274, 15:281-15:282, 15:290, 15:292-15:293, 15:306-15:307, 15:311, 15:314, 15:337, 15:351, 15:354, 15:369, 15:380N, 15:382, 15:388-15:389							6															21			
*15:09-15:10:04, 15:37, 15:90, 15:99, 15:243, 15:312-15:313, 15:329, 15:338																			17				21		
*15:16:01:01-15:16:03, 15:67, 15:95, 15:254, 15:362, 35:58, 49:04:01-49:04:02, 58:06	1						6																		
*15:18:03				4			6								14								21		
*15:21, 15:44, 15:394, 15:404							7			10															
*15:47:01-15:47:02, 15:49, 15:221, 15:379							w																21		
*15:58, 35:205, 38:30, 38:65, 39:32-39:33, 39:74, 46:61, 51:13:01, 51:92:01, 52:14, C*08:138														13											
*15:62, 48:25	1						6			10													21		
*15:69, 15:93							6															21	22		
*15:73			3	4								w		14											
*15:80							6			10													21		
*15:86	1			4							12														
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	
*15:115, 15:364, 18:57, 51:45						6			9																
*15:133, 15:229, 15:253													13									21			
*15:186, 39:07																						21	22		
*15:194, 35:11:01-35:11:02, 53:08:01-53:08:02, 58:19	1					6				10															
*15:223, 15:402, 57:22						6				10															
*15:224				4								12		14											
*15:242:02	1					6																21			
*15:303	1		3								11	w													
*15:335						6			9													21			
*15:361, 51:01:04, 51:01:13, 51:02:04, 51:06:01- 51:06:03, 51:24:01-51:24:02, 51:38, 51:59, 51:64, 51:148, 51:194, 51:199, 51:213, 52:01:05, 52:01:15, 57:82								8											17						
*15:368, 18:01:01:01-18:01:13, 18:01:15-18:01:17, 18:01:19, 18:01:21-18:09, 18:12:01-18:14, 18:17N- 18:19, 18:23N-18:25, 18:27-18:29, 18:31-18:34, 18:38-18:40, 18:42-18:49, 18:51, 18:53-18:55, 18:59-18:60, 18:62-18:68, 18:70-18:72:01, 18:73- 18:85, 18:87-18:96, 18:98-18:100, 18:102-18:104, 18:107-18:109, 18:111-18:121, 18:123, 18:126- 18:130, 39:43									9													22			
*18:01:20, 18:72:02									9													20	22		
*18:15						6			9														22		
*18:22, 18:105, 35:190, 58:76	1								9	10													22		

Negative Control

*18:30						w			9														22		
*18:37									9														21	22	
*18:52	1								9														22		
*18:69									9	10													22		
*18:125						6			9														21	22	
*27:77, 27:129	2																						21		
*27:127, 44:02:01:01-44:02:33, 44:02:35-44:02:40, 44:06, 44:08-44:09, 44:11-44:12, 44:16, 44:19N, 44:21-44:24, 44:27:01-44:27:02, 44:33-44:34:02, 44:49, 44:52N, 44:55, 44:59:01-44:59:02, 44:63, 44:66-44:68, 44:71-44:74, 44:80, 44:83-44:84:02, 44:86-44:87, 44:89, 44:91, 44:93, 44:95, 44:97, 44:101-44:102, 44:104, 44:112-44:113, 44:116, 44:118-44:119, 44:121, 44:126:01-44:127, 44:131-44:132, 44:137, 44:142, 44:145, 44:148-44:149N, 44:151-44:152, 44:162, 44:168-44:173, 44:176-44:177, 44:179, 44:185, 44:187, 44:190-44:191, 44:195N-44:196, 44:200-44:201, 44:206, 44:208, 44:211-44:212, 44:214, 44:216, 44:218-44:221, 44:225, 44:229, 44:235, 44:238, 44:240-44:244, 44:249, 44:253-44:254, 44:257									10					16											
*35:03:01:01-35:03:05, 35:03:07-35:03:21, 35:06, 35:13, 35:36, 35:38, 35:55-35:56, 35:70, 35:74, 35:84, 35:98, 35:106, 35:109, 35:128, 35:136, 35:150:01-35:150:02, 35:152-35:153, 35:155-35:156, 35:160, 35:163, 35:167, 35:169, 35:181, 35:193, 35:195, 35:198, 35:204, 35:221, 35:223, 35:231, 35:236-35:237, 35:243, 35:246, 35:256-35:257, 35:261, 35:267, 35:274, 35:278-35:279, 35:281-35:282, 35:290, 35:296, 35:298, 35:312, 35:320, 35:326, 35:329, 53:04, 53:31, 53:33, 58:34	1								10			13													
*35:11:03	1					w			10																
*35:12:01-35:12:03, 35:39, 35:149, 35:201, 35:233, 35:266, 44:62, 44:77, 44:82, 44:107, 44:123, 57:12, 57:28N									10								17								
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	
*35:21, 53:06, 53:28	1					6			9	10															
*35:24:01-35:24:02, 35:287, 53:02, 58:09	1								9	10															
*35:25, 35:49, 48:02:01-48:02:03	1								10													21			
*35:26	1						7		10													21			
*35:35, 35:93, 35:262, 58:05, 58:35	1								10														22		
*35:59:01-35:59:02, 35:179, 35:242, 37:48, 44:174, 44:251, 57:76									10				13												
*35:75, 35:127, 35:151, 35:235	1												13												
*35:76	1	2							10														22		
*35:82, 35:294	1						7		10																
*35:85	1						7		10				13												
*35:186	1								10						15										
*35:188	1					6			9	10													22		
*35:232	1					w																			
*35:273, 35:292	1																	17							
*35:277, C*15:126	1					6			10														22		
*37:01:04									10													20			
*37:35, 37:56, 57:26									10														22		
*40:10:02					5																				
*40:48				4										14										23	
*40:71			3	4										14											
*40:149, 41:22																			18						
*40:166		?														16							22		
*44:10, 50:17, 58:73	1		3						10																
*44:15, 44:18, 45:01:01-45:01:02, 45:04-45:05, 45:07, 45:11-45:19	1		3								11				15										
*44:40, 44:130, 44:156, 44:210, 57:14:01-57:14:02									9	10															

Negative Control

Negative Control

*44:44									9	10					16										
*44:48	1									10					16										
*44:90		2								10					16								22		
*44:140			3							10				14	15										
*44:226										10				14											
*45:06	1		3								11				15	16							22		
*45:08, 45:10	1										11				15										
*46:03, C*01:102						w																	22		
*48:18												8											21		
*49:18:01, 51:158:02	1						7	8																	
*49:18:02	1							8																	
*49:25	1		3								11											18			
*49:40			3								11													23	
*50:01:02			3								11														
*50:04			3								11			14											
*50:12, 50:16, 50:41-50:42	1		3																						
*50:14	1						7																	17	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	
*51:01:01:01-51:01:03, 51:01:05-51:01:07, 51:01:09-51:01:12, 51:01:14, 51:01:16-51:01:17, 51:01:19-51:01:31, 51:01:33-51:01:38, 51:01:41-51:01:42, 51:01:44-51:02:03, 51:02:05-51:03, 51:05, 51:07:01-51:09:03, 51:11N-51:12, 51:18-51:20, 51:22-51:23, 51:24:03-51:24:05, 51:26-51:30, 51:32-51:33, 51:35, 51:39-51:41N, 51:43-51:44N, 51:48-51:49, 51:51-51:53, 51:55, 51:57-51:58, 51:60-51:61:02, 51:65-51:77, 51:79-51:80, 51:83-51:84, 51:86-51:89, 51:91, 51:94-51:96, 51:98N-51:100, 51:102-51:105, 51:107-51:134, 51:136-51:138, 51:140-51:142, 51:145-51:147, 51:149N-51:152, 51:154-51:156, 51:158:01, 51:159-51:171, 51:173Q-51:181, 51:183-51:191, 51:193, 51:195-51:196, 51:198, 51:200-51:212, 52:01:01:01-52:01:04, 52:01:06-52:01:14, 52:01:16-52:01:20, 52:01:22-52:07, 52:09-52:11, 52:13, 52:15-52:19, 52:21-52:24, 52:27-52:36, 52:38, 52:40-52:61, 58:08:01-58:08:02								7	8																
*51:01:18								7	8									17						20	
*51:01:32, 51:01:39, 51:16-51:17, 51:21, 51:31, 51:34, 51:36, 51:82, 51:93, 51:101, 51:143-51:144, 51:153, 52:01:21, 52:08, 52:37								7	8																
*51:04, 51:46, 51:139, 57:02:01-57:03:02, 57:07, 57:09, 57:17, 57:39, 57:42, 57:46, 57:57, 57:63, 57:66, 57:70, 57:84									8	10								17							
*51:13:02, 51:92:02, 51:172								7						13											
*51:37						6	7		9																
*51:56:01-51:56:03, 53:19, 57:05, 58:28:01-58:28:02, 58:65	1								8	10								17							
*51:62			3	4									12	14											
*51:106, 52:25:02			3					7					12												
*52:25:01			3									11	12												
*54:01:01, 54:01:03-54:01:05, 54:01:07-54:02, 54:05N, 54:08N, 54:10, 54:12-54:13, 54:17-54:20, 54:22-54:23, 54:25, 54:27-54:30, 54:32, 54:34-54:36, 55:02:07	1	2										11	12									20			
*54:01:02, 55:35	1	2			5						11	12													
*54:01:06, 54:37		2									11	12											20		
*54:03	1	2	3								11												20		
*54:04, 54:15	1	2																					20		
*54:06, 54:11		2																						20	
*54:07, 54:24	1										11	12												20	
*54:09		2																					20	22	
*54:14	1	2																					20	22	
*54:16, 54:26	1	2									11													20	
*54:21	1	2									11	12						18						20	

Negative Control

*54:31	1	2									12							20						
*54:33	1	2	3								11	w						20						
*55:01:01-55:01:03, 55:01:05, 55:01:08-55:02:03, 55:02:05-55:02:06, 55:02:08-55:03, 55:05, 55:07, 55:09, 55:11-55:12, 55:15-55:16, 55:19, 55:21, 55:24-55:26, 55:29-55:30, 55:33, 55:36-55:37, 55:40, 55:43, 55:45, 55:48, 55:50, 55:52, 55:54-55:55N, 55:57, 55:59-55:60, 55:62-55:63, 55:65-55:67, 55:69-55:71, 55:73-55:77, 55:80	1	2									11	12												
*55:01:04	1	2				7						12												
*55:01:06, 55:02:04, 55:72		2										11	12											
*55:01:07, 55:39, 55:42, 55:44, 55:53, 55:61	1	2										12												
*55:04, 55:17, 55:20, 55:49, 81:01		2																						
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
*55:08, 55:51		2															17							
*55:10	1	2									11	12						18						
*55:14		2				6					10											22		
*55:18	1											11	12									21		
*55:23	1	2																					22	
*55:38		2										11	12		14									
*55:56	1	2										11	12			15								
*55:68	1	2				6																	22	
*57:58										10								18						
*57:80							8			10								17	18					
*58:01:08	1									10											20			
*59:08	1					7			9		11	12												
*67:06-67:07																			?					
*81:03-81:04N, 81:06-81:07		?																						
*81:08		?																				21		
*82:01-82:03	1	2	3	4											15			18						
*83:01		2								10						16								
A*02:339, C*15:26														w										
A*02:510, C*03:307, C*03:318N														14										
C*01:90				4								12											23	
C*02:06:01-02:06:02, C*02:47, C*05:132, C*06:168, C*07:289, C*12:15, C*15:37, C*15:58, C*15:72				4																				
C*03:19			w												w									
C*03:102			w							10					w									
C*07:536				4																			23	
C*08:113, C*15:09, C*15:19, C*15:27, C*15:61, C*15:107						6																	22	
C*15:91																							23	
C*15:128												12											23	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24

Negative Control