

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
HLA-B allele^{1,2,3}																								
*56:01:01:01-56:01:01:04, 56:01:01:06-56:01:12, 56:01:15-56:01:18, 56:34-56:35, 56:39, 56:41, 56:43, 56:46-56:47, 56:52-56:53, 56:58-56:59, 56:61-56:62, 56:64-56:65, 56:69-56:70, 56:72-56:87	1	2	3								11													
*56:01:01:05S, 56:45 ⁴	1	2	3								11												22	
*56:01:13, 13:54, 55:22, 55:41, 55:46 ⁵	1	2	3								11													
*56:01:14, 54:03, 54:40 ⁵	1	2	3								11									20				
*56:02:01:01-56:02:03, 56:51, 56:56	1	2	3	4																				
*56:03		2			5																			
*56:04:01:01-56:04:04, 56:57, 56:63		2	3	4									14											
*56:05:01						6	7										17							
*56:05:02		2					7										17							
*56:06		2					7										17						22	
*56:07	1	2	3						9		11													
*56:08	1	2	3		5						11				15									
*56:09, 56:60, 35:78 ⁵	1	2								10														
*56:10, 56:66	1	2		4								12												
*56:11	1	2								10			13											
*56:12		2								10														
*56:13	1	2	3								11				15									
*56:14	1		3		5						11				15									
*56:15	1	2															17							
*56:16, 56:55:01:02⁴	1	2	3								11						18							
*56:17, 56:26 ⁴	1	2	3								11											21		
*56:18, 07:65 ⁴		2																					22	
*56:19N, 56:29 ⁴	1	2	3								11								19					
*56:20:01-56:20:02		2	3								11													23
*56:21, 51:272 ⁵		2					7	8									17							
*56:22	1	2											13											
*56:23, 55:31, 55:78 ⁵	1	2									11	12										21		
*56:24	1	2	3								11													23
*56:25, 56:37	1	2	3								11	12												
*56:27		2	3								11			14										
*56:28N	1	2	3								11				15		18							
*56:30	1	2	3								11		13											
*56:31		2							9														22	
*56:32		2				6																	22	
*56:33	1		3				7				11													
*56:36	1	2	3								11				16								22	
*56:38N	1	2	3								11						17							
*56:40	1	2	3		5						11													
*56:42, 56:44, 50:75 ⁵	1		3								11												21	
*56:48-56:49, 56:71		2	3								11													
*56:50, 55:13, 55:27-55:28:01, 55:32, 55:58, 55:81 ⁵	1	2																						
*56:54, 55:18 ⁵	1										11	12											21	
*56:67-56:68, 13:03, 13:120, 13:164, 15:552, 44:339, 46:11, 49:01:01:01-49:01:07, 49:01:09-49:01:18, 49:01:20-49:03, 49:06, 49:08-49:14, 49:16-49:17, 49:19N-49:24, 49:26-49:39, 49:41-49:46, 49:48-49:49, 49:51-49:60N, 49:62-49:75, 49:77-49:78, 50:01:01:01-50:01:01:13, 50:01:03-50:01:13, 50:01:15-50:01:20, 50:01:22-50:02:01:02, 50:05-50:08, 50:10-50:11, 50:13, 50:15, 50:18-50:19, 50:21, 50:32, 50:34-50:38, 50:40, 50:43-50:44, 50:46-50:48, 50:50, 50:52-50:59, 50:62-50:74, 50:76, 50:78-50:79, 51:15, 59:04 ⁵	1		3								11													
*07:02:19, 08:01:67, 15:01:04, 15:01:43, 15:12:03, 40:94, 46:77																					20			
*07:09:01-07:09:02, 07:11, 07:162, 07:237, 07:255, 15:65, 15:519, 18:21, 35:99, 46:62, 57:13						w																		
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24

Negative Control

<p>*07:235, 13:01:01:01-13:01:19, 13:07N, 13:12:01-13:13:02, 13:17, 13:22:01-13:23, 13:25-13:26:02, 13:28-13:29, 13:36, 13:39, 13:43, 13:50-13:52, 13:57, 13:60-13:61, 13:63N, 13:73, 13:76N-13:78, 13:80, 13:83, 13:87, 13:90, 13:92, 13:95, 13:106-13:107, 13:109-13:110, 13:122, 13:129, 13:135, 13:138, 13:141-13:142, 13:144, 13:148, 13:161N-13:163, 13:168, 13:170-13:171, 13:173, 14:102, 15:20, 15:85, 15:393, 15:407, 15:587, 15:599, 35:01:01:01-35:01:42, 35:01:44-35:01:54, 35:01:56-35:01:61, 35:01:63-35:01:70, 35:03:06, 35:07-35:08:15, 35:10, 35:14:01-35:15:02, 35:19-35:20:02, 35:23, 35:27-35:30:01:02, 35:33, 35:40N-35:42:02, 35:45-35:48, 35:50, 35:52, 35:54, 35:57, 35:62-35:63, 35:65Q, 35:69, 35:71, 35:77, 35:80, 35:90-35:92, 35:94, 35:96, 35:100-35:101:02, 35:103-35:105, 35:107-35:108:02, 35:110-35:112:02, 35:115-35:116, 35:120-35:124, 35:126, 35:130N-35:134N, 35:137-35:148, 35:157-35:159, 35:161, 35:166, 35:168, 35:170-35:171, 35:173:01N-35:173:02N, 35:175-35:178, 35:180, 35:187, 35:189, 35:191-35:192, 35:194, 35:197, 35:200, 35:202-35:203, 35:206-35:207, 35:209-35:210, 35:212-35:214, 35:216N-35:219, 35:222, 35:224-35:229, 35:238-35:241, 35:244-35:245, 35:247:01-35:250, 35:252-35:255, 35:259, 35:263-35:264, 35:268-35:269, 35:271-35:272, 35:275, 35:280, 35:283-35:284, 35:286, 35:288-35:289, 35:291, 35:293, 35:295, 35:297, 35:299-35:308, 35:310, 35:313-35:315, 35:318-35:319, 35:322, 35:324-35:325, 35:327-35:328, 35:330, 35:332-35:334, 35:336, 35:338, 35:340-35:342, 35:345, 35:347-35:348, 35:350-35:353, 35:356, 35:359, 35:363, 35:365, 35:370, 35:373, 35:375-35:376, 35:380:01:01-35:380:01:02, 35:382-35:383, 35:385-35:386, 35:388, 35:390N, 35:392-35:393, 35:400-35:402, 35:404-35:406, 35:408, 35:411-35:412, 35:417-35:418, 35:421, 35:426, 35:428Q-35:431, 35:437, 35:441, 35:444-35:445, 35:447-35:449, 35:453N-35:454, 35:456N, 35:458, 35:460Q-35:462, 35:464-35:468, 35:470, 35:473, 35:479-35:480, 35:485, 35:488, 35:490-35:491, 35:493-35:495, 35:497, 35:500, 35:502-35:504, 35:506, 35:512-35:518, 35:520, 35:524, 35:526, 35:528-35:533, 35:536-35:538, 35:542N, 35:546, 35:548, 35:551N-35:552, 35:558, 35:560, 37:101, 44:256, 44:276, 44:481, 46:33, 46:67, 53:01:01:01-53:01:29, 53:03, 53:05:01-53:05:02, 53:09-53:13, 53:15-53:18, 53:20-53:27, 53:29-53:30, 53:32, 53:34-53:35, 53:37-53:40, 53:43:01-53:43:02, 53:45-53:46, 53:48N-53:61, 53:63-53:66, 53:68, 53:70, 58:01:01:01-58:01:02, 58:01:04-58:01:07, 58:01:09-58:01:15, 58:01:17-58:01:42, 58:04, 58:11-58:15, 58:17N, 58:21-58:24, 58:29, 58:31N-58:33, 58:36-58:37, 58:39N-58:42, 58:44-58:45:02, 58:47-58:58, 58:62-58:64, 58:66-58:72N, 58:74-58:75, 58:77-58:83, 58:85-58:87, 58:89-58:90, 58:92-58:103, 58:105-58:113, 58:115-58:120, 58:123-58:126, 58:128N-58:131, 58:133N-58:137</p>	1									10																Negative Control
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24		
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24		

*55:23	1	2																				22		
*55:28:02	1	2					7																	
*55:38		2								11	12			14										
*55:56	1	2								11	12			15										
*55:68	1	2					6															22		
*57:58										10											18			
*57:80								8		10							17	18						
*58:01:08	1									10												20		
*59:08	1						7		9		11	12												
*59:11	1						8																	
*67:06-67:07																					?			
*81:03-81:04N, 81:06-81:07		?																						
*81:08		?																				21		
*82:01:01:01-82:04	1	2	3	4											15						18			
A*02:339, A*02:510, C*15:26														w										
C*01:90				4								12										23		
C*01:136, C*01:145:01N, C*02:06:01-02:06:02, C*02:47, C*05:132, C*06:168, C*06:224, C*07:289, C*07:725, C*12:03:66, C*12:15, C*12:208, C*15:37, C*15:58, C*15:72				4																				
C*03:19			w											w										
C*03:102			w							10				w										
C*03:307, C*03:318N														14										
C*15:91																						23		
C*15:128												12										23		
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24

Negative Control