

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	
HLA-B allele ^{1,2,3}																																
*57:01:01-57:01:04, 57:01:06-57:01:22, 57:38, 57:48, 57:50, 57:54, 57:56, 57:59-57:60, 57:64-57:65, 57:67-57:68, 57:71-57:72, 57:75, 57:77	1	2	3										13																			
*57:01:05, 57:47, 57:74		2	3										13																			
*57:02:01-57:02:02, 57:42	1		4	5								12	13																			
*57:03:01-57:03:02, 57:46, 57:66, 57:70	1		3	4								12	13																			
*57:04	1				5	6							13						19													
*57:05	1		4										13																			
*57:06	1	2	3		6								13																			
*57:07	1		3	4				8				12	13																			
*57:08	1	2	3						10				13																			
*57:09	1		4						9		11	12	13																			
*57:10	1	2	3										13	14																		
*57:11													13		15																	
*57:12	1		4	5								12	13		16																	
*57:13		2		5					9				13						17	19												
*57:14:01-57:14:02	1	2	3								11		13																			
*57:15, 57:37, 57:55 ⁵	1	2	3				7						13																			
*57:16	1	2	3					8																								
*57:17	1		3	4								12	13	14																		
*57:18	1	2	3										13		16																	
*57:19	1	2		5									13																			
*57:20	1	2	3																	20												
*57:21	1	2	3										13							21												
*57:22	1	2	3					9					13					17														
*57:23	1	2	3										13						18													
*57:24	1	2									11		13																			
*57:25, 57:41 ⁵	1	2	3										13						19													
*57:26	1	2	3										13							20												
*57:27	1	2	3										13					17														
*57:28N	1			5									13																			
*57:29	1	2	3										13	15																		
*57:30	1	2		5	6								13																			
*57:31		2	3										13					17														
*57:32	1		3		6								13																			
*57:33	1	2	3										13	15						21												
*57:34	1	2	3										13								22											
*57:35	1	2	3										13									23										
*57:36	1	2	3																			23										
*57:39	1		3	4								12	13																	28		
*57:40	1	w	3										13								22		24									
*57:43	1	2	3								11		13						19													
*57:44	1	2	3										13												25							
*57:45, 57:62, 57:69, 35:208, 58:14 ⁴	1	2	3																													
*57:49	1	2	3					8					13																			
*57:51	1	2	3																											28		
*57:52	1	2											13																			
*57:53, 57:76	1		3										13																			
*57:57	1		3	4					9			12	13					17														
*57:58	1	2	3										13														26					
*57:61	1	2											13																	27		
*57:63	1		4	5					9			12	13					17														
*57:73	1	2	3										13																	28		
*57:78	1	2	3										13																		29	
*57:79N	1	2	3										13																		30	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	
*07:02:01-07:24, 07:26-07:47, 07:49N-07:50, 07:52-07:119, 07:121-07:136:02, 07:138-07:154, 07:156-07:163, 07:165-07:201N, 07:203-07:208, 07:210-07:218, 07:221-07:222, 07:224-07:225, 07:228-07:234, 07:236-07:249, 07:251N-07:256, 08:20, 08:53:01-08:53:02, 08:79, 13:16, 13:31, 13:48, 13:62, 14:02:09, 14:05-14:06:02, 14:08:01-14:08:02, 14:13, 14:37, 15:01:01-01-15:01:04, 15:01:06-15:01:39, 15:03:01-15:04:02, 15:06-15:12, 15:14-15:19, 15:23-15:24:02, 15:26N-15:30, 15:32:01-15:35, 15:37-15:40, 15:42-15:43, 15:45-15:47:02, 15:49-15:50, 15:53-15:54, 15:56-15:58, 15:60-15:61, 15:63-15:74, 15:76, 15:78:01-15:79N, 15:81-15:82, 15:87, 15:90, 15:92-15:99, 15:101-15:104, 15:108-15:110, 15:113, 15:115-15:120, 15:122, 15:125-15:129, 15:131-15:135, 15:137-15:138, 15:140-15:143, 15:146-15:150, 15:152-15:153, 15:156-15:161, 15:163-15:164, 15:166-15:169, 15:171-15:175, 15:177-15:178, 15:180-15:184, 15:187, 15:189-15:193, 15:195-15:199, 15:201-15:203, 15:205-15:212, 15:215-15:217, 15:219-15:221, 15:225-15:234, 15:236, 15:238-15:239, 15:241-15:249, 15:251-15:264, 15:266-15:270, 15:272N-15:274, 15:276-15:282, 15:284-15:288, 15:290, 15:292-15:296, 15:298-15:300, 15:303-15:307, 15:309-15:311, 15:313-15:318, 15:320-15:324, 15:326-15:327, 15:329, 15:331, 15:333-15:339, 15:342-15:344, 15:346-15:347, 15:353-15:356, 15:359-15:363, 18:15, 18:19, 18:21, 18:30, 18:57, 18:101, 27:04:01-27:04:04, 27:06, 27:10, 27:15, 27:19, 27:20, 27:21, 27:24, 27:26, 27:40, 27:54																																

Negative Control

