

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
HLA-B allele^{1,2,3}																								
*57:01:01:01-57:01:40, 57:67:02, 57:95, 57:97-57:98Q, 57:100, 57:102-57:108, 57:110-57:111, 57:113-57:117, 57:121-57:123, 57:125-57:126	1	2									11													
*57:02:01-57:02:02, 57:12, 57:42	1					6																		
*57:03:01:01-57:03:04, 57:17, 57:66, 57:70, 57:84, 57:101, 57:124	1	2				6																		
*57:04:01, 57:04:03	1		3														17							
*57:04:02			3														17							
*57:05	1																					21		
*57:06, 57:18, 57:27	1	2	3								11													
*57:07, 57:46	1	2			5	6																		
*57:08	1	2				6					11	12												
*57:09	1					6	7																	
*57:10, 57:73, 57:87	1	2						8			11													
*57:13	1						7			10	11													
*57:14:01-57:14:02, 57:25, 57:31, 57:50	1	2					7				11													
*57:15, 57:29, 57:91	1	2		4							11													
*57:16		2			5						11													
*57:19, 57:30, 57:52, 57:61	1										11													
*57:20, 57:62		2		4							11													
*57:21, 57:33, 57:74	1	2							9		11													
*57:22, 57:43	1	2								10	11													
*57:23, 57:26, 57:34, 57:64, 57:93	1	2			5						11													
*57:24	1						7				11													
*57:32	1	2	3																					
*57:35, 57:60	1	2									11	12												
*57:36		2									11											21		
*57:37, 57:79N, 57:81, 57:86	1	2									11							18						
*57:38	1	2									11											21		
*57:39	1	2			6		8																	
*57:40	1	2							9		w													
*57:41, 57:68	1	2									11						17							
*57:44, 57:75	1	2									11										20			
*57:45, 57:69		2									11		13											
*57:47, 57:54, 57:77	1	2									11			14										
*57:48, 57:55-57:56, 57:78	1	2									11				15									
*57:49, 57:58, 57:71, 57:88, 57:90	1	2									11		13											
*57:51		2						8			11		13											
*57:53	1	2					7																	
*57:57	1	2				6					10													
*57:59	1	2									11												23	
*57:63	1					6					10													
*57:65	1	2				6					11													
*57:67:01	1	2									11					16								
*57:72, 57:92	1	2									11									19				
*57:76	1	2																		19				
*57:80	1	2				6							13											
*57:85	1	2									11												22	
*57:94	1	2				6									15									
*57:96	1	2				6								14										
*57:119	1				5		7				11													
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
*07:219								8					13											
*07:227							7			10														
*08:181					5	6	7																	
*14:20, 15:461, 27:188, 35:127, 44:317, 44:450, 50:54, 51:186, 53:41, 55:100									9															
*14:61, 15:214, 18:81, 35:250, 39:145, 40:427, 44:311, 51:165				4																				

Negative Control

