

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
HLA-B allele ^{1,2,3}																								
*57:01:01:01-57:01:46, 57:67:02, 57:95, 57:97- 57:98Q, 57:100, 57:102- 57:108, 57:110-57:111, 57:113- 57:117, 57:121-57:123, 57:125- 57:127, 57:130N, 57:132- 57:133, 57:135-57:136, 57:138- 57:139N, 57:141-57:146, 57:148-57:154, 57:156-57:159, 57:162N	1	2									11													
*57:02:01:01-57:02:02, 57:12, 57:42	1					6																		
*57:03:01:01-57:03:05, 57:17, 57:66, 57:70, 57:84, 57:101, 57:124, 57:137, 57:155	1	2				6																		
*57:04:01, 57:04:03	1		3														17							
*57:04:02			3														17							
*57:05	1																				21			
*57:06, 57:18, 57:27	1	2	3								11													
*57:07, 57:46	1	2			5	6																		
*57:08	1	2				6					11	12												
*57:09	1					6	7																	
*57:10, 57:73, 57:87	1	2						8			11													
*57:11, 57:28N, 57:82-57:83, 57:99, 57:112, 57:118, 57:131, 57:134, 57:160, 15:462, 58:36	1																							
*57:13	1						7			10	11													
*57:14:01-57:14:02, 57:25, 57:31:01-57:31:02, 57:50	1	2					7				11													
*57:15, 57:29, 57:91	1	2		4							11													
*57:16		2			5						11													
*57:19, 57:30, 57:52, 57:61, 57:129, 57:140, 57:147	1										11													
*57:20, 57:62		2		4							11													
*57:21, 57:33, 57:74	1	2							9		11													
*57:22, 57:43	1	2								10	11													
*57:23, 57:26, 57:34, 57:64, 57:93	1	2			5						11													
*57:24	1						7				11													
*57:32	1	2	3																					
*57:35, 57:60	1	2									11	12												
*57:36		2									11											21		
*57:37, 57:79N, 57:81, 57:86	1	2									11						18							
*57:38	1	2									11											21		
*57:39	1	2				6		8																
*57:40	1	2							9		w													
*57:41, 57:68	1	2									11						17							
*57:44, 57:75	1	2									11										20			
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24

Negative Control

*57:45, 57:69		2									11	13												
*57:47, 57:54, 57:77	1	2									11	14												
*57:48, 57:55-57:56, 57:78	1	2									11	15												
*57:49, 57:58, 57:71, 57:88, 57:90	1	2									11	13												
*57:51		2					8				11	13												
*57:53	1	2					7																	
*57:57	1	2			6					10														
*57:59	1	2									11											23		
*57:63	1				6					10														
*57:65	1	2			6						11													
*57:67:01	1	2									11		16											
*57:72, 57:92	1	2									11						19							
*57:76	1	2															19							
*57:80	1	2			6							13												
*57:85	1	2									11											22		
*57:89, 57:109, 57:120, 57:128, 35:208:01-35:208:02, 35:317, 37:94		2									11													
*57:94	1	2			6							15												
*57:96	1	2			6							14												
*57:119	1				5		7				11													
*57:161	1									w	11													
*07:219							8					13												
*07:227							7			10														
*08:181					5	6	7																	
*14:20, 15:461, 27:188, 35:127, 44:317, 44:450, 50:54, 51:186, 53:41, 55:100, 56:80									9															
*14:61, 15:214, 15:626, 18:81, 35:250, 37:93, 39:145, 39:158, 40:427, 44:311, 51:165, 53:59, 55:108			4																					
*14:104, 15:340, 35:300, 40:218, 44:307, 44:517, 45:21, 46:28, 51:58, 52:82, 58:83, A*23:72, A*33:12												15												
*15:33, 15:248, 15:617, 44:36, 49:22, 51:126, C*03:87:01-03:87:02, C*03:232, C*03:414, C*05:27, C*05:39, C*05:184, C*08:115, C*08:182												14												
*15:87, 15:625											12													
*15:403					6						12													
*15:446, 58:41							8																	
*18:167, 35:374, 44:249, 52:81																					20			
*40:30, 40:34		2			6	7																23		
*40:327																					?			
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24

Negative Control

*44:13, 44:67, 51:170, 51:356, 53:25																	18								Negative Control
*44:153	2	3			5												17								
*51:331																							22		
*55:14	2				5				10	11															
*58:12, 58:111, A*02:42:01- 02:42:02, A*02:285, A*02:310, A*02:597, A*02:799													13												
*58:14	2									11							16								
*58:54					5																				
C*06:72					5	7			w	11							16	17							
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	