

Well No.	1	2	3	4	5	6	7	8	9	10
HLA-B allele^{1,2,3}										
*59:01:01:01-59:01:01:02, 59:09-59:10N	1	2								
*59:02, 59:11	1		3							
*59:03	1				6					
*59:04, 59:12	1	2				7				
*59:05	1	2			6					
*59:06	1	2		4						
*59:07	1	2							9	
*59:08, 13:125 ⁴		2						8		
*07:65, 07:209, 07:271, 27:157, 35:76, 35:561, 44:90, 44:356, 54:06, 54:09, 54:14, 54:31, 54:38, 55:01:04, 55:01:07, 55:02:11, 55:04, 55:08, 55:14, 55:23, 55:32, 55:39, 55:42, 55:44, 55:49, 55:51, 55:53, 55:61, 55:68, 55:86, 55:94, 55:110, 56:03, 56:09-56:12, 56:18, 56:31-56:32, 56:50, 56:60, 56:66, 81:01:01:01-81:01:05, 81:09-81:10, 83:01					5					
*07:78, 13:02:01:01-13:02:14, 13:02:16-13:02:36, 13:08-13:09, 13:14-13:16, 13:18-13:19, 13:27, 13:30-13:34, 13:37-13:38, 13:40-13:42, 13:44-13:45, 13:49N, 13:53-13:56:02N, 13:58, 13:65-13:70, 13:74-13:75, 13:81-13:82, 13:84-13:85, 13:88-13:89, 13:91, 13:93-13:94, 13:96-13:105, 13:111-13:119, 13:121, 13:123Q-13:124, 13:126, 13:128, 13:130-13:134, 13:136-13:137N, 13:139N, 13:143, 13:145, 13:147, 13:149-13:159, 13:165-13:167, 13:169, 13:172, 14:37, 15:42, 15:485, 35:60, 45:03, 45:08, 45:10, 46:18, 49:01:08, 49:07, 49:76, 50:31, 50:39, 50:45, 50:49, 51:157, 51:300, 54:07, 54:24, 54:30, 55:18, 55:34, 55:47, 55:64, 55:78, 55:118N, 55:121, 56:23, 56:54		2								
*07:84, 07:429, 08:09, 08:84, 08:254, 13:35, 13:59, 13:62, 15:83:01:01-15:83:01:02, 27:14, 27:81, 27:130, 27:153, 35:273, 35:534, 40:06:01:01-40:06:08, 40:06:10-40:06:17, 40:06:19-40:06:30, 40:44, 40:53, 40:70:01-40:70:02, 40:75, 40:83, 40:93, 40:95-40:96, 40:103, 40:109-40:110, 40:127, 40:131, 40:148, 40:161-40:162, 40:165, 40:167, 40:177, 40:190, 40:230, 40:244, 40:256N, 40:268-40:270, 40:275, 40:294-40:295, 40:298:01-40:298:02, 40:306-40:307, 40:311, 40:313, 40:317-40:318, 40:340-40:342, 40:348-40:350, 40:354, 40:361N-40:364, 40:366, 40:373-40:374, 40:376, 40:387-40:389, 40:408, 40:411, 40:423, 40:425, 40:427, 40:434, 40:445, 40:449, 40:451, 40:453-40:454, 40:459-40:460, 40:464, 40:468, 40:471, 40:473, 40:476, 40:478, 40:482, 40:484, 40:487N-40:488N, 40:492, 40:494, 40:519-40:520, 41:01:01:01-41:01:08, 41:05-41:07, 41:09, 41:12, 41:14, 41:16-41:17, 41:20-41:22, 41:25-41:26, 41:28-41:29, 41:32-41:35, 41:37, 41:53-41:54, 41:56Q-41:57, 41:59-41:61, 41:64, 41:66-41:67, 41:70, 41:73, 41:75-41:78, 42:04, 44:20, 44:47, 44:100, 44:197, 44:459, 45:29, 49:18:01-49:18:02, 50:14, 50:20, 51:01:01:01-51:01:47, 51:01:49-51:03, 51:05:01:01-51:05:01:02, 51:07:01-51:12, 51:14, 51:16-51:24:05, 51:26-51:34, 51:36, 51:38-51:41N, 51:43-51:44N, 51:48-51:55, 51:57-51:58, 51:60-51:61:02, 51:65-51:80, 51:82-51:91, 51:93-51:96, 51:98N, 51:100-51:105, 51:107-51:130, 51:132, 51:134, 51:136-51:138, 51:140-51:147, 51:149N-51:156, 51:158:01-51:171, 51:173Q-51:193:02, 51:195-51:212, 51:214-51:215, 51:217-51:219, 51:221-51:222, 51:224, 51:226, 51:228-51:250, 51:252-51:280, 51:282-51:292, 51:294-51:299, 51:301-51:309, 51:311-51:318N, 51:320-51:340, 51:342-51:351, 51:354-51:358, 51:360-51:374, 52:01:01:01-52:13, 52:15-52:24, 52:26-52:38, 52:40-52:74, 52:76-52:87, 52:89N-52:96N, 52:98-52:109, 58:08:01-58:08:02, 78:01:01:02-78:10, C*03:464, C*14:92			3							
*13:03, 13:48, 13:120, 13:164, 15:303, 15:552, 44:15:01:01-44:15:01:02, 44:18:01:01-44:18:01:02, 44:339, 44:355, 45:01:01:01-45:01:12, 45:04-45:07, 45:11-45:23, 45:25, 45:27-45:28N, 45:30, 46:11, 49:01:01:01-49:01:07, 49:01:09-49:03, 49:06, 49:08-49:14, 49:16-49:17, 49:19N-49:46, 49:48-49:49, 49:51-49:60N, 49:62-49:75, 49:77-49:78, 50:01:01:01-50:01:20, 50:01:22-50:02:01:02, 50:04:01-50:08, 50:10-50:11, 50:13, 50:15, 50:18-50:19, 50:21, 50:32, 50:34-50:38, 50:40, 50:43-50:44, 50:46-50:48, 50:50, 50:52-50:79, 51:15, 51:106:02, 51:225, 52:25:01, 56:14, 56:17, 56:33, 56:42, 56:44, 56:67-56:68		2					7			
*13:46, 13:140, 15:137, 35:235, 35:276, 39:06:01:01-39:06:07, 39:33-39:34, 39:50, 39:57, 39:62, 39:64, 39:83:01-39:83:03, 39:90, 39:102, 39:109, 39:127, 39:129, 39:132, 39:140-39:141, 39:154, 39:178, 39:186-39:187, 40:86, 40:330, 41:50, 45:02, 49:50, 50:09, 51:13:01-51:13:02, 51:92:01-51:92:02, 51:172, 52:14:01-52:14:02, 52:75, 73:01:01:01-73:03, C*05:152						6				
*13:146, 15:73:01:01-15:73:01:02, 15:486, 35:432, 35:446, 35:545, 40:71, 44:10, 44:140, 44:340, 49:47:01-49:47:02, 50:01:21, 50:12, 50:16-50:17, 50:41-50:42, 50:51, 51:62, 51:106:01, 51:223, 52:25:02, 58:73							7			
*27:198				4						
*40:166, 81:03-81:04N, 81:06-81:08					?					
*40:395									9	
*44:344								8		
*45:24		2				w				
*54:01:01:01-54:02, 54:05N, 54:08N, 54:10, 54:13, 54:16-54:23, 54:25-54:29, 54:32, 54:34-54:37, 54:39, 54:41, 54:43-54:46, 55:01:01:01-55:01:03, 55:01:05-55:01:06, 55:01:08-55:02:10, 55:02:12-55:03, 55:05, 55:07, 55:09-55:12, 55:15-55:16, 55:19, 55:21-55:22, 55:24-55:26, 55:29-55:31, 55:33, 55:35-55:38, 55:40-55:41, 55:43, 55:45-55:46, 55:48, 55:50, 55:52, 55:54-55:57, 55:59-55:60, 55:62-55:63, 55:65-55:67, 55:69-55:77, 55:79-55:80, 55:82-55:85, 55:87-55:93, 55:95-55:102, 55:104-55:109, 55:111-55:117N, 55:119-55:120, 55:122-55:129, 56:01:05, 56:01:13, 56:19N		2			5					
*54:03, 54:33, 54:40, 56:01:01:01-56:01:04, 56:01:06-56:01:12, 56:01:14-56:01:18, 56:08, 56:13, 56:16, 56:20:01-56:20:02, 56:24-56:30, 56:34-56:41, 56:43, 56:45-56:49, 56:52-56:53, 56:55:01:02, 56:58-56:59, 56:61-56:62, 56:64, 56:69-56:87		2			5		7			
*54:04, 54:11, 54:15, 54:42, 55:17, 55:20, 55:27-55:28:02, 55:81, 56:05:01-56:06, 56:15			3		5					
*54:12	1	2		4	5					
*55:13, 55:58, 56:22					5	6				
*55:103	1	2			5					
*56:02:01:01-56:02:03, 56:04:01:01-56:04:04, 56:51, 56:56-56:57, 56:63, 82:01:01:01-82:04					5		7			
*56:07		2			5		7	8		
*56:21	1		3		5					
*56:65	1	2			5		7			
Well No.	1	2	3	4	5	6	7	8	9	10

Negative Control