

Well No.	1	2	3	4	5	6	7	8
HLA-B allele^{1,2,3}								
*67:01:01-67:01:03, 67:06-67:07	1		3		5			
*67:02:01:01-67:02:01:02		2	3		5			
*67:03	1				5	6		
*67:04-67:05 ⁴	1		3		5		7	
*07:02:01:01-07:02:02, 07:02:04-07:02:27, 07:02:29-07:02:66, 07:02:68-07:02:77, 07:02:79-07:02:80, 07:04:01-07:05:03, 07:05:05-07:07:03, 07:09:01-07:09:02, 07:12, 07:14-07:15, 07:17-07:26, 07:28, 07:30-07:31, 07:33:01-07:35, 07:39-07:46, 07:48-07:49N, 07:51-07:55, 07:56:02, 07:58-07:64, 07:66-07:68:03, 07:74, 07:76-07:80, 07:82, 07:84, 07:87-07:93, 07:95-07:108, 07:110-07:124, 07:126-07:148, 07:150-07:178, 07:181N-07:182N, 07:185, 07:187-07:191, 07:193-07:194, 07:196-07:203, 07:205-07:206, 07:208-07:209, 07:211-07:218, 07:220-07:227, 07:229:01-07:231N, 07:233-07:235, 07:237-07:239, 07:241-07:252, 07:254-07:256, 07:258, 07:260-07:272N, 07:274-07:279, 07:281-07:291, 07:293-07:305, 07:307-07:312, 07:314-07:327, 07:329-07:333, 07:335, 07:337-07:339, 07:341-07:351N, 07:353-07:355, 07:357-07:367, 07:369-07:382, 07:384-07:399, 42:01:01:01-42:02:02, 42:04-42:06, 42:08-42:10, 42:12-42:14, 42:16-42:21, 42:23, 42:25, 42:27-42:28, 81:01:01:01-81:02, 81:04N, 81:06-81:09	1			4	5			
*07:02:03, 07:05:04, 07:47, 07:56:01, 07:73, 07:109, 07:232, 07:280, 07:328, 07:334, 07:336, 07:356, 42:24, 54:02, 55:10, 55:82, 56:35	1				5			
*07:02:28, 07:10, 07:13, 07:83, 07:125, 07:183, 07:259, 07:292, 07:368, 81:05				4	5			
*07:02:67, 07:02:78, 07:11, 07:36, 07:57, 07:75:01:01-07:75:01:02, 07:81, 07:149, 07:179, 07:192, 07:195, 07:236, 07:240, 07:253, 07:257, 07:273, 07:340, 42:15, 42:22	1			4				
*07:03, 07:08:01-07:08:02, 07:16, 07:32, 07:37:01-07:37:02, 07:50, 07:69, 07:85:01-07:85:02, 07:207, 07:228:01-07:228:02, 07:306, 08:94, 08:226, 14:01:01:01-14:01:04, 14:01:07-14:01:09, 14:07N-14:08:02, 14:10, 14:12, 14:14, 14:19, 14:26, 14:32, 14:40, 14:46-14:47, 14:49, 14:53-14:55, 14:65, 14:71, 14:78N, 14:80-14:83, 14:86, 15:68, 15:71, 15:175, 15:381, 15:556, 18:01:01:01-18:01:12, 18:01:14-18:01:20, 18:01:22-18:03:02, 18:05:01:01-18:08, 18:10-18:15, 18:17N-18:24, 18:26-18:48, 18:50-18:53, 18:55-18:66, 18:69, 18:71-18:78, 18:80-18:106Q, 18:109-18:124, 18:126-18:131:01:02, 18:133-18:135:01:02, 18:137-18:141, 18:143-18:146, 18:148-18:167, 18:169-18:189, 27:08, 27:12:01:01-27:12:01:03, 27:18, 27:26, 27:33, 27:40, 27:44, 27:153, 27:204, 37:11, 37:37, 37:67, 38:159, 39:30, 39:32-39:34, 39:36, 39:43, 39:47, 39:50, 39:68, 39:74, 39:82, 39:102, 39:107, 39:112, 39:128, 39:136, 39:155-39:156, 40:02:01:01-40:06:14, 40:06:16-40:06:22, 40:08-40:09, 40:11:01:01-40:11:02, 40:18, 40:20:01:01-40:20:01:02, 40:24, 40:26-40:29, 40:35:01-40:35:02, 40:39-40:40, 40:44, 40:50:01:01-40:50:01:02, 40:56, 40:64:01:01-40:64:02, 40:68, 40:70:01-40:71, 40:74-40:75, 40:78, 40:82-40:83, 40:85-40:86, 40:89-40:91, 40:93-40:95, 40:97-40:99, 40:103-40:105, 40:107, 40:111, 40:119-40:120, 40:122, 40:127, 40:131, 40:133Q, 40:142N-40:145, 40:148-40:149, 40:159, 40:161-40:162, 40:165, 40:167, 40:169, 40:173-40:174, 40:176-40:177, 40:180-40:181, 40:189-40:190, 40:200, 40:202-40:203, 40:205-40:206, 40:209-40:211, 40:214, 40:220, 40:224-40:226, 40:229-40:230, 40:232, 40:243-40:244, 40:246, 40:248, 40:254-40:255, 40:266-40:271, 40:274-40:276, 40:283-40:284, 40:287, 40:289-40:291N, 40:293-40:297, 40:302-40:305, 40:307, 40:309, 40:311, 40:313-40:314, 40:317, 40:320, 40:322, 40:330-40:331, 40:334, 40:337N, 40:341, 40:343, 40:345N, 40:347-40:351, 40:354, 40:356, 40:359, 40:361N-40:364, 40:366, 40:369, 40:371, 40:373-40:376, 40:378, 40:384, 40:387-40:390, 40:399N, 40:405, 40:408-40:415, 40:425, 40:427, 40:429, 40:432, 40:434-40:435, 40:437-40:438N, 40:440-40:442, 40:444-40:445, 40:451, 42:07, 42:11, 42:26, 44:131, 48:01:01:01-48:01:10, 48:03:01:01-48:17, 48:19-48:21, 48:23-48:50, 50:43, C*03:137, C*03:182, C*03:459, C*03:504, C*07:351, C*12:82, C*16:116					5			
*07:29, 07:65, 07:70-07:72, 07:86, 07:184, 07:204, 07:210, 07:352, 07:383, 15:76, 15:101, 27:75, 37:60, 46:01:01:01-46:40:02, 46:42-46:48, 46:50-46:76, 46:78-46:81, 54:01:02, 55:35, 56:03, 56:06, 83:01, C*01:73, C*03:02:23, C*05:01:28, C*07:02:30, C*08:16:02, C*14:02:27				4				
*07:94, 81:03	1			4	5		7	
*07:186, 07:313	w			4	w			

Negative Control

