

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16
HLA-B allele ^{1,2,3}																
*78:01:01:02-78:01:02	1	2					7			10		12				Negative Control
*78:02:01	1		3							10		12				
*78:02:02	1		3				7			10		12				
*78:03	1	2		4			7			10		12				
*78:04			3		5					10		12				
*78:05	1		3			6				10		12				
*78:06	1		3			6				10		12	13			
*78:07	1	2					7					12			15	
*78:08	1	2			5		7					12				
*78:09	1		3				7			10		12	13			
*78:10	1		3		5		7			10		12	13			
*07:02:01:01-07:02:05, 07:02:07-07:02:97, 07:04:01-07:07:03, 07:09:01-07:15, 07:17-07:26, 07:28-07:31:02, 07:33:01-07:36, 07:39-07:46, 07:48-07:49N, 07:51-07:64, 07:66-07:68:03, 07:70-07:84, 07:86-07:133, 07:135N-07:179, 07:181N-07:205, 07:208-07:218, 07:220-07:227, 07:229:01-07:231N, 07:233-07:305, 07:307-07:399, 07:401-07:452, 07:454-07:478, 08:123, 08:132, 14:21, 14:57, 15:76, 15:101, 15:255, 15:594, 27:01-27:11, 27:13:01-27:15, 27:17, 27:19:01:01-27:21:02, 27:24-27:28, 27:30-27:38, 27:40-27:51, 27:53-27:58, 27:60-27:76, 27:78-27:91, 27:93-27:101, 27:103-27:118, 27:120-27:128, 27:130-27:136, 27:138-27:152, 27:154-27:203, 27:205-27:214, 27:216-27:235, 27:237-27:241, 27:243N-27:250, 27:252-27:265, 35:570, 38:26, 40:166, 40:513, 42:01:01:01-42:02:02, 42:05:01-42:06, 42:08-42:10, 42:12-42:23, 42:25, 42:27-42:34, 44:90, 44:97, 44:356, 45:06, 46:01:01:01-46:10, 46:12-46:17, 46:19-46:31, 46:33-46:39, 46:42-46:48, 46:51Q-46:98N, 54:06, 54:09, 54:26, 54:38, 55:01:07, 55:04, 55:08-55:09, 55:14, 55:22, 55:24, 55:49, 55:51, 55:86, 55:110, 56:01:13, 56:02:01:01-56:04:04, 56:09-56:12, 56:18-56:19N, 56:31-56:32, 56:51, 56:56-56:57, 56:60, 56:63, 56:66, 56:88, 67:01:01-67:07, 81:01:01:01-81:10, 82:01:01:01-82:04, 83:01									9							
*07:65		2					7		9			12				
*07:134, 35:76, 35:186, 35:229, 46:40:02									9			12				
*07:206		2					7		9							
*07:400, 18:12:01-18:12:02, 35:69, 37:11, 37:37, 40:209, 40:350						6						12				
*08:03, 08:52, 08:78, 15:361, 38:73, 40:13, 40:19, 40:109, 40:117, 40:292, 40:340, 41:46, 48:18, 49:18:01-49:18:02, 57:02:01:01-57:03:06, 57:05, 57:07, 57:09, 57:17, 57:39, 57:42, 57:46, 57:57, 57:63, 57:66, 57:70, 57:80, 57:82, 57:84, 57:94, 57:96, 57:101, 57:124, 57:137, 57:155, A*23:129, A*24:146														14		
*08:09, 08:84, 08:254, 13:46, 13:53, 13:131, 39:06:01:01-39:06:08, 39:34, 39:57, 39:62, 39:64, 39:83:01-39:83:03, 39:90, 39:109, 39:127, 39:129, 39:132, 39:140-39:141, 39:154, 39:178, 39:187, 41:70, 49:76, 55:78, 59:01:01:01-59:01:01:02, 59:03, 59:05-59:08, 59:10N, C*05:152, C*07:463, C*07:758, C*12:183, C*14:92															15	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16

Negative Control

[illegible]

[illegible]

*44:06, 51:42, 51:45, 51:62, 51:135, 51:220, 51:227, 51:281, 51:319, 51:359, 53:01:13, 53:06								8		11							
*46:11, 46:32:01-46:32:02, 54:03, 54:33, 54:40, 56:01:01:01-56:01:12, 56:01:14-56:01:18, 56:07-56:08, 56:13-56:16, 56:20:01-56:20:02, 56:22, 56:24-56:30, 56:35-56:41, 56:43, 56:46-56:50, 56:52-56:53, 56:55:01:02, 56:58-56:59, 56:61-56:62, 56:64-56:65, 56:68-56:87, 56:90, 56:92-56:93									9	10							
*46:40:01		w					7		9		12						
*49:03						6		8		11							
*50:58	1		3														
*51:01:01:01-51:01:01:109, 51:01:03-51:01:07, 51:01:10-51:01:14, 51:01:16-51:01:31, 51:01:33-51:01:38, 51:01:41-51:01:42, 51:01:44-51:01:58, 51:01:60-51:01:102, 51:03, 51:07:01, 51:08:01:01-51:08:05, 51:11N-51:12, 51:17-51:18, 51:24:01-51:24:05, 51:26-51:30, 51:32-51:33, 51:35, 51:38-51:39, 51:41N, 51:43-51:44N, 51:48-51:49, 51:51-51:53, 51:55, 51:57-51:58, 51:60-51:61:02, 51:65-51:72, 51:74-51:77, 51:79-51:80, 51:83-51:84, 51:86-51:88, 51:91, 51:94-51:96, 51:98N-51:100, 51:102, 51:105, 51:107, 51:109-51:111, 51:113-51:115, 51:117-51:118N, 51:120-51:121, 51:123-51:131, 51:133-51:134, 51:136-51:138, 51:140-51:142, 51:145, 51:149N-51:151, 51:154-51:156, 51:158:01-51:171, 51:173Q-51:174:02, 51:177-51:178N, 51:181, 51:183-51:184N, 51:186-51:191, 51:193:01-51:193:02, 51:195-51:196, 51:198, 51:200-51:207, 51:209-51:212, 51:215, 51:219, 51:221-51:222, 51:224, 51:228-51:237:01, 51:238-51:246, 51:248-51:250, 51:252-51:258, 51:260, 51:262-51:266, 51:268-51:273N, 51:275, 51:277-51:279, 51:282-51:283, 51:285-51:292, 51:294, 51:296-51:297, 51:299, 51:301-51:309, 51:311-51:318N, 51:320-51:336, 51:338-51:339, 51:342-51:344N, 51:346Q-51:351, 51:355-51:358, 51:360-51:370, 51:373-51:375, 51:377N, 51:379-51:381, 58:08:01	1							8		10	11			14			
*51:01:02:01-51:01:02:02, 51:01:09, 51:07:02, 51:08:06, 51:89, 51:103, 51:116, 51:119, 51:132, 51:176, 51:179, 51:218, 51:226, 51:237:02, 51:337, 51:354, 52:18, 52:73, 58:08:02	1									10	11			14			
*51:01:08, 51:01:15, 51:01:43, 51:13:01-51:14, 51:37, 51:50, 51:63:01-51:63:02, 51:85, 51:90, 51:92:01-51:92:02, 51:97, 51:106:01-51:106:02, 51:172, 51:182, 51:192, 51:214, 51:216, 51:223, 51:225, 51:251, 51:261, 51:274, 51:298	1							8		10	11						
*51:01:32, 51:01:39, 51:01:59, 51:16, 51:82, 51:152, 51:372	1							8		11				14			
*51:01:40, 51:78:01-51:78:02, 51:371	1									10	11						
*51:02:01:01-51:02:01:04, 51:02:03-51:02:07, 51:23, 51:146, 51:175, 51:199, 51:213, 51:345								8		10	11			14			
*51:02:02, 51:180										10	11			14			
*51:04:01-51:04:02, 51:06:01:01-51:06:03, 51:10, 51:34, 51:46, 51:56:01-51:56:03, 51:59, 51:64, 51:81, 51:293, 51:352-51:353, 51:378								8		11				14			
*51:05:01:01-51:05:01:02					w					10	11			14			
*51:06:04, 51:144, 51:148, 53:19, 58:27-58:28:02, 58:65										11				14			
*51:09:01, 51:09:03, 51:19, 51:73, 51:122, 51:259, 51:276	1				5		8			10	11			14			
*51:09:02	1				5					10	11			14			
*51:15, 58:122										10	11						
*51:20, 51:208	1	w					8			10	11			14			
*51:21, 51:101, 51:153:01-51:153:02	1						8			11				14	15		
*51:22	1			4			8			10	11			14			
*51:31	1				5		8			11				14			
*51:36, 51:143, 51:295, 51:376							8			11				14	15		
*51:40:01-51:40:02, 51:108					5		8			10	11			14			
*51:54					w					10	11						
*51:93					5					11				14			
*51:104:01-51:104:02, 51:112, 51:147, 51:267, 51:284, 52:02:01	1						8			10				14			
*51:139, 51:194					5		8			11				14			
*51:157, 51:197, 51:300							8			11					15		
*51:185	1		3				8			10	11			14			
*51:217	1				w		8			10	11			14			
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	
*51:247	1		3					8		10	11						
*51:280, 51:340	1				5		8			11				14	15		
*51:310						6		8			11			14			
*52:01:01:01-52:01:01:25, 52:01:03, 52:01:05-52:01:08, 52:01:10-52:01:18, 52:01:20, 52:01:22-52:01:28, 52:01:30-52:01:39, 52:01:41-52:01:42, 52:01:44-52:01:46, 52:01:48, 52:01:50-52:01:51, 52:01:53, 52:04-52:05, 52:06:03-52:07:02, 52:13, 52:15, 52:17, 52:19, 52:21:02-52:24, 52:27-52:31:01, 52:33-52:36, 52:38, 52:40-52:61, 52:64-52:66, 52:69-52:70, 52:72, 52:76-52:83, 52:86-52:87, 52:89N-52:96N, 52:98-52:101, 52:103N, 52:105-52:110N, 52:113	1					6				10	11			14			

Negative Control

*52:01:02:01-52:01:02:09, 52:01:04, 52:01:09, 52:01:40, 52:01:43, 52:01:47, 52:01:49, 52:06:01-52:06:02, 52:21:01, 52:31:02, 52:85, 52:102, 52:104, 52:112	1					6		8		10	11			14			
*52:01:19, 52:111	1					6				10				14			
*52:01:21, 52:08	1					6					11			14			
*52:01:29, 52:68	1		3			6				10	11						
*52:01:52, 52:12, 52:14:01-52:14:02, 52:20, 52:25:01-52:26, 52:39, 52:62-52:63, 52:75	1					6				10	11						
*52:02:02, 52:16, 52:32	1									10				14			
*52:03						5	6		8	10	11			14			
*52:09:01	1		3			6		8		10	11			14			
*52:09:02	1		3			6				10	11			14			
*52:10:01-52:10:02, 52:74	1					5	6			10	11			14			
*52:10:03	1					5	6		8	10	11			14			
*52:11:01						6				10	11			14			
*52:11:02						6		8		10	11			14			
*52:37, 52:84	1					6					11			14	15		
*52:67						6					11			14			
*52:71	1					5	6				11			14	15		
*52:97	1					6		8		10	11						
*55:02:11						5				9						15	
*55:31			3							9						15	
*55:130										9			12			15	
*56:05:01	1	2						7		9	10						
*56:05:02, 56:34	1									9	10						
*56:06	1	2						7		9	10		12				
*56:21	1									9	10				14		
*56:45										9	10		12				
*57:01:20								8									
*73:01:01:01-73:03	1									9							
A*23:01:14		w						8									
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	

Negative Control