

<b>INTERPRETATION TABLE</b>						
<b>HLA-B*81 SSP subtyping</b>						
<b>Amplification patterns of the B*81 alleles</b>						
	<b>Well</b>					
	<b>1</b>	<b>2</b>	<b>3</b>	<b>4</b>	<b>5</b>	<b>6</b>
<b>Length of spec.</b>	<b>175</b>	<b>410</b>	<b>405</b>	<b>285</b>	<b>165</b>	<b>80</b>
<b>PCR product</b>						
<b>Length of int.</b>	<b>800</b>	<b>1070</b>	<b>1070</b>	<b>1070</b>	<b>1070</b>	<b>1070</b>
<b>pos. control<sup>1</sup></b>						
<b>5'-primer<sup>2</sup></b>	<b>363</b>	<b>41</b>	<b>44</b>	<b>363</b>	<b>652</b>	<b>206</b>
	5'-AgC 3'	5'-CTg 3'	5'-ggC 3'	5'-AgC 3'	5'-CCg 3'	5'-AgA 3'
<b>3'-primer<sup>3</sup></b>	<b>499</b>	<b>282</b>	<b>282</b>	<b>605</b>	<b>774</b>	<b>244</b>
	5'-ggA 3'	5'-gCC 3'	5'-gCC 3'	5'-gCT 3'	5'-ggT 3'	5'-CTA 3'
<b>Well No.</b>	<b>1</b>	<b>2</b>	<b>3</b>	<b>4</b>	<b>5</b>	<b>6</b>
<b>HLA-B allele</b>						
<b>*8101</b>	<b>1</b>	<b>2</b>		<b>4</b>		
<b>*8102</b>	<b>1</b>		<b>3</b>	<b>4</b>		
<b>*8103</b>	<b>1</b>	<b>?</b>	<b>?</b>	<b>4</b>	<b>5</b>	
<b>*8104N</b>	<b>1</b>	<b>?</b>	<b>?</b>	<b>4</b>		<b>6</b>
<b>*070201-070218, 0704-0706, 0709-0711, 0713, 0715, 0717, 0720-0726, 0728-0731, 0734-0736, 0739, 0741-0746, 0748, 0749N, 0751, 0752, 0754-0759, 0761-0764, 0766-0768, 0770-0776, 0779-0783, 0786-0799, 420501, 420502</b>			<b>3</b>	<b>4</b>		
<b>*0703, 0708, 0716, 0727, 0732, 0737, 0738, 0747, 0750, 0769, 0785, 3707, 4015, 4016, 4032, 4098, 4805, 4808, 4810, 4812, 4814</b>				<b>4</b>		
<b>*0707, 0712, 0714, 071801-0719, 0733, 0740, 0753, 0760, 0777, 0778, 0784, 420101-4202, 4204, 4206, 4208-4210, 4212, 4213, 4506, 670101-6702, 7301</b>			<b>3</b>			
<b>*0765</b>		<b>2</b>		<b>4</b>		
<b>Well No.</b>	<b>1</b>	<b>2</b>	<b>3</b>	<b>4</b>	<b>5</b>	<b>6</b>

Length of spec.	175	410	405	285	165	80
PCR product						
Well No.	1	2	3	4	5	6
*1520, 350101-3530, 3532-3542, 3544-3556, 3558, 3559, 3561-3565Q, 3567-3572, 3574, 3575, 3577, 3578, 3580-3586, 3588-3599, 4802, 510101-512402, 5126-5141N, 5143-5146, 5148-5184, 520101-520602, 5208-5216, 530101-5317, 5319, 580101-5802, 5804-5810N, 5812-5819, 5821-5824, 7801-7806					5	
*2701-2711, 2713-2715, 2717, 2719-2721, 2725-2728, 2730-2738, 2740-2758, 5401-5419, 550101-5505, 5507-5517, 5519-5538, 560101-5604, 5606-5616, 5618-5620, 5622, 5624-5626, 8201, 8202		2				
*2724	1	2				
*3566				4	5	
*3576, 560501, 560502, 5621		2			5	
*4031, 4045, 4080, 480101, 480102, 4806, 4807, 4809, 4811, 4815, 4816, 4818-4820	1			4		
*4804	1					
*8301		2			?	
HLA-B allele						
Well No.	1	2	3	4	5	6