

Well No.	1	2	3	4	5	6	7	8	9	10	11
HLA-B allele^{1,2,3}											
*81:01:01:01-81:01:05, 81:09-81:10	1	2		4						10	
*81:02:01-81:02:02, 07:202, 07:279 ⁴	1		3	4						10	
*81:03	1	?	?	4	5					10	
*81:04N	1	?	?	4		6				10	
*81:05	1	2		4	5	6					
*81:06	1	?	?	4			7			10	
*81:07	1	?	?	4				8		10	
*81:08	1	?	?	4					9	10	
*07:02:01:01-07:02:05, 07:02:07-07:02:37, 07:02:39-07:02:73, 07:02:75:01-07:02:86, 07:02:88-07:02:90, 07:04:01-07:07:03, 07:12, 07:14:01-07:15, 07:18:01-07:26, 07:29-07:31:02, 07:33:01-07:36, 07:39-07:46, 07:48-07:49N, 07:51-07:64, 07:66-07:68:03, 07:70-07:74, 07:76-07:77, 07:79-07:82, 07:84, 07:86-07:90, 07:92-07:109, 07:111N-07:114, 07:116-07:117, 07:119-07:131, 07:133-07:146, 07:148-07:150, 07:152-07:161N, 07:163-07:173, 07:175-07:179, 07:181N, 07:184-07:196, 07:198-07:201N, 07:203-07:206, 07:208, 07:210-07:213, 07:215-07:218, 07:220-07:221, 07:223-07:224, 07:226-07:227, 07:229:01-07:231N, 07:233-07:234, 07:236, 07:238-07:245, 07:247-07:254, 07:256-07:259, 07:261-07:270, 07:272N-07:278, 07:280-07:286, 07:288-07:301, 07:303:01-07:305, 07:307-07:326, 07:328-07:350, 07:352-07:358, 07:360Q-07:367, 07:369-07:399, 07:401-07:410, 07:412-07:423, 07:425N-07:427, 07:429-07:449, 07:451, 08:123, 08:132			3							10	
*07:02:38, 07:02:74, 07:02:87, 07:09:01-07:11, 07:17, 07:28, 07:78, 07:115, 07:118, 07:132, 07:162, 07:182N-07:183, 07:197, 07:214, 07:225, 07:235, 07:237, 07:246, 07:255, 07:260, 07:287, 07:302, 07:327, 07:351N, 07:359, 07:368, 07:424, 07:428, 07:452, 14:21, 14:57, 38:26, 42:06, 42:09, 42:20, 45:06, 46:89, 56:68, 67:01:01-67:01:03, 67:03-67:04, 67:06-67:07, 73:01:01:01-73:03, C*01:214, C*16:112			3								
*07:03, 07:47, 07:232, 08:146, 39:126, 40:61, 40:166, C*03:542										10	
*07:13, 67:02:01:01-67:02:01:02			3				7				
*07:16, 07:37:01-07:37:02									9	10	
*07:50, 07:207, 07:228:01-07:228:02, 07:306, 08:23, 08:54:01-08:54:02, 08:110, 14:14, 14:18, 14:73, 15:03:01:01-15:03:09, 15:03:11-15:03:13, 15:09:01-15:10:08, 15:18:01:01-15:18:09, 15:29:01:01-15:29:01:02, 15:37, 15:47:01-15:47:02, 15:49, 15:51-15:52, 15:54, 15:62, 15:64:01-15:64:02, 15:68-15:69, 15:72, 15:74, 15:80, 15:90-15:91, 15:93, 15:98-15:99, 15:103, 15:108, 15:114, 15:123-15:124, 15:127, 15:131-15:134, 15:151, 15:153, 15:156, 15:158, 15:161, 15:173, 15:176, 15:186, 15:197:01-15:198, 15:200, 15:210, 15:220:01:01-15:221, 15:226N, 15:229, 15:235, 15:238, 15:242:01-15:243, 15:252-15:253, 15:259, 15:263, 15:266, 15:274-15:275:02, 15:281-15:282, 15:290, 15:292-15:293, 15:306-15:307, 15:311-15:314, 15:323, 15:329, 15:335, 15:337-15:338, 15:351, 15:354, 15:369, 15:379-15:380N, 15:382, 15:388-15:389, 15:414, 15:426-15:427, 15:433, 15:439, 15:444-15:445, 15:447-15:448, 15:451, 15:455, 15:475, 15:487N, 15:498, 15:502-15:504, 15:506, 15:509, 15:511, 15:517, 15:526-15:527, 15:530, 15:539, 15:548, 15:560, 15:570-15:571, 15:573, 15:582, 15:588, 15:607, 15:610-15:611, 15:614:01:01-15:614:01:02, 15:616Q-15:617, 15:620, 18:37:01-18:37:02, 18:125, 27:18, 27:23, 38:67, 39:05:01:01-39:05:05, 39:07-39:08, 39:11, 39:13:01-39:13:02, 39:20, 39:37:01:01-39:37:01:02, 39:49, 39:55-39:56, 39:113, 39:118, 39:147N, 39:149, 39:161N, 39:163, 39:165-39:168, 39:170, 39:172, 39:174, 40:12:01-40:12:02, 40:46, 40:93, 40:430, 42:11, 48:03:01:01-48:03:02, 48:05, 48:08, 48:10, 48:12-48:13, 48:17, 48:21, 48:23, 48:25-48:26, 48:40, 50:75, 55:18, 56:23, 56:42, 56:44, 56:54									9		
*07:65, 07:271, 54:04, 54:11, 54:15, 54:42, 55:04, 55:08, 55:17, 55:20, 55:27-55:28:02, 55:49, 55:51, 55:81, 55:86, 55:110, 56:06, 56:12, 56:15		2									10
*07:75:01:01-07:75:01:02, 07:91			3						9	10	
*07:83			3		5	6					
*07:110			3				7			10	
*07:147, 07:151:01-07:151:02, 07:411, 07:450			3	4						10	
*07:174, 07:222	1		3							10	
*07:209		2	3							10	
*08:120, 08:205, 40:287, 41:38, C*07:294, C*07:526:01, C*07:530, C*07:681				4							
Well No.	1	2	3	4	5	6	7	8	9	10	11
*08:213:01-08:213:02, 40:45	1										

Negative Control

*15:20, 15:228, 15:305, 15:599, 35:01:01:01-35:01:27, 35:01:29-35:01:46, 35:01:48-35:24:02, 35:27-35:30:01:02, 35:32:01-35:32:02, 35:33-35:34, 35:36-35:42:02, 35:45, 35:47-35:48, 35:50-35:56, 35:58-35:59:02, 35:61:01-35:72, 35:74-35:75, 35:77-35:78, 35:80-35:86, 35:88-35:134N, 35:136-35:184, 35:187-35:208:01, 35:209-35:228, 35:230-35:278, 35:280-35:358, 35:360, 35:362-35:367, 35:369-35:462, 35:464-35:466, 35:468-35:470, 35:472-35:474, 35:477-35:482, 35:484:01-35:508N, 35:510-35:533, 35:535-35:541, 37:80, 49:66, 51:01:01:01-51:01:55, 51:01:57-51:24:05, 51:26-51:41N, 51:43-51:46, 51:48-51:156, 51:158:01-51:163, 51:165-51:219, 51:221-51:236, 51:238-51:294, 51:296-51:299, 51:301-51:306N, 51:308-51:358, 52:01:01:01-52:01:50, 52:02:01-52:06:03, 52:08-52:106, 53:01:01:01-53:58, 53:60-53:68, 58:01:01:01-58:01:21, 58:01:23-58:02:02, 58:04-58:10N, 58:12-58:19, 58:21-58:29, 58:31N-58:104, 58:107-58:134, 78:01:01:02-78:10					5									
*15:435, 35:25-35:26, 35:49:01-35:49:02, 35:361, 48:02:01-48:02:03					5								9	
*27:01-27:11, 27:13:01-27:15, 27:17, 27:19:01:01-27:21:02, 27:25-27:28, 27:30-27:38, 27:40-27:43, 27:45-27:58, 27:60-27:76, 27:78-27:91, 27:93-27:118, 27:120-27:128, 27:130, 27:132-27:136, 27:138-27:152, 27:154-27:194, 27:196, 27:198-27:203, 27:205-27:214, 27:216-27:241, 27:243N-27:251, 37:60, 44:90, 44:97, 54:01:01:01-54:01:09, 54:03, 54:05N-54:10, 54:12-54:14, 54:16-54:29, 54:31-54:41, 54:43-54:44, 55:01:01:01-55:03, 55:05, 55:07, 55:09-55:16, 55:19, 55:21-55:26, 55:29-55:30, 55:32-55:48, 55:50, 55:52-55:77, 55:79-55:80, 55:82-55:85, 55:87-55:109, 55:111-55:121, 56:01:01:01-56:04:04, 56:07-56:11, 56:13-56:14, 56:16, 56:18-56:20:02, 56:22, 56:24-56:32, 56:34-56:35, 56:37-56:41, 56:43, 56:45-56:53, 56:55:01:02-56:66, 56:69-56:86, 82:01:01:01-82:04, 83:01		2												
*27:24	1	2												
*27:44, 27:131, 55:31		2											9	
*27:197, 35:76, 35:186, 35:229, 35:368, 56:36		2			5									
*35:463					5								10	
*35:509				4	5									
*37:52, 38:11, A*26:142, C*02:191, C*03:137, C*03:182, C*03:459, C*03:504, C*07:351, C*12:82, C*16:116							7							
*40:31, 40:80, 48:41, 48:48	1			4										
*42:01:01:01-42:02:02, 42:04-42:05:02, 42:08, 42:10, 42:12-42:19, 42:21-42:23, 42:25, 42:27-42:31			3										w	
*42:07, 42:24													w	
*48:01:01:01-48:01:10, 48:04:01:01-48:04:02, 48:06-48:07, 48:09, 48:11, 48:15-48:16, 48:18-48:20, 48:22, 48:24, 48:27-48:35, 48:37-48:39, 48:42-48:43, 48:45-48:47, 48:49-48:53	1			4									9	
*48:14				4									9	
*48:36	1			4			7						9	
*48:44	1												9	
*54:02		2					7							
*56:05:01-56:05:02, 56:21		2			5								10	
*67:05			3		5									
C*07:102							w							
Well No.	1	2	3	4	5	6	7	8	9	10	11	Negative Control		