

*04:01:01:01-04:01:15, 04:01:17-04:01:22, 04:01:24-04:01:27, 04:01:29-04:01:36, 04:05, 04:07-04:09N, 04:12, 04:15:01-04:15:03, 04:17-04:20, 04:24-04:28, 04:30-04:31, 04:35, 04:38-04:41, 04:43-04:53, 04:56-04:57, 04:59Q-04:67, 04:69-04:71, 04:73-04:76, 04:78-04:79, 04:81-04:102, 04:104-04:106						6						11		13			
*04:01:16, 04:23, 04:54, 04:72						6						11		13			
*04:01:23, 04:33						6						11		13			
*04:01:28												11		13			
*04:03, 04:80			3			6						11		13		15	
*04:04:01-04:04:02, 04:13-04:14, 04:34, 04:68						6						11		13			
*04:06			3			6						11		13		15	
*04:10						6						11					
*04:11						6											16
*04:16			3									11		13		15	
*04:29						6								13			16
*04:32, 04:77			2			6						11		13			
*04:36, 04:55						6											16
*04:37						6						11		13		15	
*04:42												11		13	14		
*04:58						6						11		13			
*04:103												11		13		15	
*04:107			3			6						11		13		15	
*05:01:01:01-05:01:18, 05:03, 05:05-05:10, 05:12-05:15, 05:18:01-05:19, 05:21, 05:24, 05:28-05:31, 05:33-05:35, 05:37-05:41, 05:44-05:45, 05:47-05:50, 05:52-05:54, 05:56-05:57, 05:58:02-05:61, 05:63-05:67, 05:69, 05:71-05:72							7					11				15	
*05:04, 05:22:01-05:22:02							7					11				15	
*05:11, 05:27, 05:32, 05:51Q, 05:70							7					11				15	
*05:16							7					11				15	
*05:17, 05:68							7					11				15	
*05:20							7					11					
*05:23							7					11				15	
*05:25, 05:55							7					11				15	
*05:26						6	7					11				15	
*05:36							7					11				15	
*05:42							7					11		13		15	
*05:43							7					11					
*05:46							7					11		13		15	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	
HLA-C allele⁴																	
*05:58:01			3				7					11				15	
*05:62							7					11				15	

*06:02:01:01-06:02:01:02, 06:02:03-06:02:11, 06:02:13-06:02:15, 06:02:17-06:02:19, 06:07, 06:10, 06:12-06:13, 06:15-06:16N, 06:18-06:30, 06:33-06:34, 06:36-06:39, 06:41-06:51, 06:53-06:66, 06:68-06:69								8				11			14		
*06:02:12								8							14		
*06:02:16								8				11					
*06:03			3					8				11			14	15	
*06:04, 06:40								8				11					
*06:05								8				11					
*06:06								8				11					
*06:08			2					8				11			14		
*06:09								8				11			14		
*06:11, 06:52								8							14		
*06:14, 06:35								8				11					
*06:17												11			14		
*06:31								8				11			14		
*06:32												11			14		
*06:67								8				11			14		
*07:01:01-07:02:07, 07:02:10-07:02:28, 07:03-07:06, 07:08, 07:11-07:19, 07:21-07:25, 07:27:01-07:27:02, 07:29-07:30, 07:32N, 07:35-07:38, 07:42, 07:44-07:48, 07:50-07:52, 07:54-07:63, 07:65-07:75, 07:77-07:91, 07:93-07:95, 07:97-07:100, 07:102-07:122, 07:124, 07:126-07:138, 07:140-07:147, 07:149-07:151, 07:153-07:155, 07:157-07:160, 07:162-07:172, 07:174-07:176, 07:178-07:183, 07:185-07:195, 07:197-07:209, 07:211-07:215, 07:217-07:220												9					
*07:02:08, 07:02:29, 07:26, 07:33N, 07:39-07:40, 07:92, 07:139, 07:152N, 07:156												9					
*07:02:09						6						9			13		
*07:07, 07:09, 07:49, 07:76												9		11			
*07:10, 07:28, 07:41, 07:43, 07:184, 07:196												9					
*07:20												9					
*07:31												9					
*07:53, 07:216												9					
*07:64												9					16
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	
HLA-C allele⁴																	
*07:96			3									9					
*07:101, 07:148, 07:161		2										9	10				
*07:123, 07:173												9					
*07:125						6						9			13	14	

*07:177									9							
*07:210									9	11						
*08:01:01-08:01:05, 08:03:01-08:03:02, 08:06, 08:08, 08:14, 08:16, 08:20, 08:22, 08:24, 08:26N, 08:28, 08:36N, 08:38, 08:40-08:42, 08:44, 08:46, 08:50, 08:56										10						
*08:02:01-08:02:07, 08:04, 08:13, 08:15:01-08:15:02, 08:17-08:19, 08:23, 08:27, 08:29-08:30, 08:32-08:35, 08:37, 08:39, 08:43, 08:45, 08:49, 08:51-08:55N										10						
*08:05										10			13			
*08:07, 08:47										10						
*08:09, 08:11										10						
*08:10							7			10	11				15	
*08:12										10						
*08:21										10			13			
*08:25										10			13			
*08:31		2								10						
*08:48										10						
*12:02:01-12:02:07, 12:10:01-12:10:02, 12:17-12:18, 12:20, 12:22, 12:30, 12:36, 12:40, 12:56, 12:64, 12:67-12:68												12	13			
*12:03:01:01-12:03:01:02, 12:03:03, 12:03:05-12:03:07, 12:03:10-12:03:12, 12:03:14-12:03:15, 12:03:18-12:03:19, 12:06-12:07, 12:11-12:13, 12:19, 12:23, 12:25, 12:29, 12:32, 12:34-12:35, 12:37-12:39N, 12:42Q-12:43, 12:45-12:48, 12:50-12:53, 12:57, 12:61-12:63, 12:65-12:66												12	13	14		
*12:03:02, 12:03:08, 12:03:16, 12:03:20, 12:26, 12:59													13	14		
*12:03:04												12		14		
*12:03:09							8					12		14		
*12:03:13												12	w	14		
*12:03:17												12	13	14		
*12:04:01, 12:33											11			14	15	
*12:04:02, 12:60											11	12	13	14	15	
*12:05											11	12		14	15	
*12:08												12	13			
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16
HLA-C allele⁴																
*12:09											11				15	
*12:14:01													13			
*12:14:02												12	13			
*12:15								8				12	13	14		
*12:16												12	13			
*12:21											11	12			15	

*16:01:01, 16:01:03-16:01:07, 16:06-16:08, 16:10-16:11, 16:13-16:15:01, 16:16Q-16:17, 16:21-16:24, 16:26-16:28, 16:30N-16:32, 16:36, 16:38-16:39, 16:41, 16:44													13			
*16:01:02													13			
*16:02:01-16:02:09, 16:09, 16:12, 16:19, 16:25										11			13		15	
*16:04, 16:33, 16:42														14		
*16:15:02											12		13			
*16:18													13			
*16:20													13			
*16:29		2												14		
*16:34			3			6							13			
*16:35																
*16:37													13		15	
*16:40							7									16
*16:43													13			
*17:01:01:01-17:01:03, 17:01:05-17:04, 17:06, 17:08-17:11		2									11				15	
*17:01:04		2									11		13		15	
*17:05		2									11				15	
*17:07											11				15	
*18:01-18:02, 18:04-18:05											11					
*18:03		2									11					
A*24:106, A*24:174, B*46:25																
B*14:03																
B*35:08:02, B*35:08:05																
B*67:02													13			
HLA-C allele ⁴																
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16

17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	Well No.
17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	Well No.
																HLA-C allele ⁴
			20						26							*01:02:01-01:02:14, 01:06-01:08, 01:11-01:13, 01:15-01:20, 01:23, 01:25-01:33, 01:37N-01:42, 01:44-01:48, 01:51-01:54, 01:56N-01:58, 01:61
			20	21					26							*01:03, 01:24, 01:34
																*01:04
																*01:05, 01:22, 01:35-01:36, B*54:18 ⁵
																*01:09
			20						26							*01:10
			20						26					31		*01:14
										27						*01:21
			20						26							*01:43
				21												*01:49-01:50, 01:55
			20						26							*01:59
	19	20							26							*01:60
							24								31	*02:02:01
							24								31	*02:02:02-02:02:03, 02:02:06-02:02:12, 02:02:15-02:02:22, 02:07-02:09, 02:11, 02:15, 02:19-02:20, 02:23-02:25Q, 02:28-02:31, 02:34-02:38N, 02:40, 02:42-02:44, 02:46, 02:48, 02:50, 02:52N-02:54
															31	*02:02:05, 02:02:14, 02:10, 02:32, 02:51
															31	*02:02:13
																*02:03, 02:16:01-02:16:02
							24								31	*02:04
															31	*02:05, 02:17
17															31	*02:06
										w					31	*02:12
	18						24								31	*02:13
															31	*02:14
	18															*02:18
							24								31	*02:21, 02:26:02-02:26:03, 02:39, 02:45
				21			24								31	*02:22
							24								31	*02:26:01
							24			27						*02:27:01
										27						*02:27:02
	18														31	*02:33
							24								31	*02:47
	18						24								31	*02:49
							24								31	*02:55
17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	Well No.
17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	Well No.
																HLA-C allele ⁴

Negative Control

