

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16
HLA-C allele ⁴																
*01:02:01-01:02:17, 01:02:19, 01:06-01:08, 01:11-01:13, 01:15-01:20, 01:23, 01:25-01:33, 01:37N-01:42, 01:44-01:48, 01:51-01:54, 01:56N-01:58, 01:61-01:77	1															
*01:02:18	1											12				
*01:03, 01:24, 01:34, 01:78	1															
*01:04	1											12		14		
*01:05, 01:22, 01:36, B*39:76, B*54:18 ⁵	1															
*01:09	1													14		
*01:10	1	2														
*01:14	1										11					
*01:21	1											12	13			
*01:35	1															
*01:43	1	2								10						
*01:49-01:50, 01:55	1															
*01:59	1										11					
*01:60	1															
*02:02:01		2	3			6					11					w
*02:02:02-02:02:03, 02:02:06-02:02:12, 02:02:15-02:02:25, 02:07-02:09, 02:11, 02:15, 02:19-02:20, 02:24-02:25Q, 02:28-02:31, 02:34-02:35, 02:37-02:38N, 02:40, 02:42-02:44, 02:46, 02:48, 02:50, 02:52N-02:54, 02:56-02:57, 02:59-02:61, 02:63-02:64, 02:66-02:67Q		2	3			6					11					15
*02:02:05, 02:02:14, 02:10, 02:32, 02:51		2	3			6					11					15
*02:02:13		2	3								11					15
*02:02:26, 02:21, 02:26-02:26:03, 02:39, 02:45		2	3								11					15
*02:03, 02:16-01-02:16:02		2	3			6					11					15
*02:04		2	w			6					11					15
*02:05, 02:17		2	3			6					11			14		15
*02:06		2	3			6		8			11					15
*02:12		2	3			6							13			15
*02:13		2	3			6					11					15
*02:14		2	w			6					11					15
*02:18		2	3			6					11					15
*02:22, 02:62		2	3			6					11					15
*02:23, 02:36, 02:68		2	3			6					11					15
*02:26:01		2									11					15
*02:27:01		2	3			6										
*02:27:02		2	3			6										
*02:33		2	3			6					11					15
*02:47		2	3			6		8			11					15
*02:49		2	3			6					11		13			15
*02:55		2	3			6					11		13			15

*02:58			3			6					11				15	
*02:65		2	3			6					11					
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16
HLA-C allele ⁴																
*03:02:01-03:02:09, 03:05, 03:17, 03:25, 03:27, 03:33, 03:35-03:36, 03:60, 03:71, 03:84, 03:89, 03:95, 03:108, 03:121N, 03:135, 03:139, 03:146, 03:154, 03:169Q, 03:178, 03:190, 03:194			3	4												
*03:02:10				4												
*03:03:01-03:03:20, 03:11:01-03:11:02, 03:20N-03:22Q, 03:30, 03:43:01-03:43:02, 03:49-03:50, 03:52-03:53, 03:55-03:56, 03:59, 03:61-03:62, 03:66, 03:68-03:69, 03:75-03:76, 03:79, 03:83, 03:85, 03:88, 03:96, 03:102-03:103, 03:112, 03:116:01-03:116:02, 03:119-03:120, 03:122, 03:124, 03:126-03:127, 03:133, 03:141-03:142, 03:150, 03:152, 03:158, 03:160-03:161, 03:168, 03:171, 03:176, 03:185, 03:187-03:189N, 03:192				4	5											
*03:03:21, 03:04:26, 03:18, 03:39, 03:64:02, 03:137, 03:144, 03:156, 03:166, 03:170, 03:182, 03:195				4												
*03:04:01:01-03:04:06, 03:04:08-03:04:24, 03:04:27-03:04:36, 03:06, 03:09, 03:14, 03:19, 03:23-03:24, 03:26, 03:28, 03:32, 03:34, 03:37:01-03:38:02, 03:40:01-03:40:03, 03:42, 03:44, 03:46-03:48, 03:51, 03:54, 03:57, 03:63-03:64:01, 03:65, 03:70, 03:72-03:74, 03:77-03:78, 03:82, 03:87, 03:90-03:93, 03:98, 03:100-03:101, 03:104-03:107, 03:109, 03:111, 03:114-03:115, 03:117-03:118, 03:123, 03:125, 03:128-03:129, 03:131, 03:134, 03:136, 03:138, 03:143, 03:147-03:149, 03:153, 03:155, 03:157, 03:159, 03:162, 03:164, 03:173-03:174, 03:179-03:181, 03:183-03:184, 03:186, 03:191, 03:193			3	4												
*03:04:07, 03:41, 03:80, 03:113			3													
*03:04:25			3	4												
*03:07, 03:45, 03:163			3	4							11				15	
*03:08			3	4												
*03:10			3	4											15	
*03:13, 03:97, 03:132, 03:167, 03:177				4	5											
*03:15			3	4							11				15	

*03:16, 03:110			3													
*03:29			3	4											15	
*03:31				4	5											
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16
HLA-C allele ⁴																
*03:58					5											
*03:67				4	w											
*03:81, 03:175				4	5											
*03:86					5											
*03:94			3													
*03:99																
*03:130, 03:140			3	4							11					
*03:145, 03:172			3	4												
*03:151, 03:165					5											
*04:01:01:01-04:01:15, 04:01:17-04:01:22, 04:01:24-04:01:27, 04:01:29-04:01:52, 04:05, 04:07, 04:12, 04:15:02, 04:17-04:20, 04:24-04:28, 04:30-04:31, 04:35, 04:38-04:41, 04:43-04:53, 04:56-04:57, 04:59Q-04:67, 04:69-04:70, 04:73-04:76, 04:78-04:79, 04:81-04:102, 04:104-04:106, 04:109-04:110, 04:113, 04:115N, 04:117-04:121, 04:123N-04:128, 04:131-04:139, 04:141-04:146, 04:148-04:152						6					11		13			
*04:01:16, 04:23, 04:54, 04:72						6					11		13			
*04:01:23, 04:33, 04:116, 04:130						6					11		13			
*04:01:28, 04:140											11		13			
*04:03:01-04:03:02, 04:80			3			6					11		13		15	
*04:04:01-04:04:02, 04:13, 04:111, 04:122						6					11		13			
*04:06			3			6					11		13		15	
*04:08						6					11		13			
*04:09N						6					11		13			
*04:10						6					11					
*04:11						6										16
*04:14, 04:68						6					11		13			
*04:15:01, 04:15:03						6					11		13			
*04:16			3								11		13		15	
*04:29						6							13			16
*04:32, 04:77		2				6					11		13			
*04:34						6					11		13			
*04:36, 04:55						6										16
*04:37						6					11		13		15	
*04:42											11		13	14		
*04:58						6					11		13			
*04:71	1					6					11		13			
*04:103											11		13		15	
*04:107			3			6					11		13		15	
*04:108						6					11		13			
*04:112						6					11		13			
*04:114						6					11		13			16

*04:129						6	7				11		13			
*04:147			3			6					11		13		15	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16
HLA-C allele ⁴																
*05:01:01:01-05:01:27, 05:03, 05:05-05:10, 05:12-05:15, 05:18:01-05:19, 05:21, 05:24, 05:28-05:31, 05:33-05:35, 05:37-05:38, 05:40-05:41, 05:44:01-05:45, 05:47-05:50, 05:52-05:54, 05:56-05:57, 05:58:02-05:61, 05:63-05:67, 05:69, 05:71-05:77, 05:80-05:84, 05:86-05:94							7				11				15	
*05:04:01-05:04:02, 05:22:01-05:22:02							7				11				15	
*05:11, 05:32, 05:51Q, 05:70							7				11				15	
*05:16							7				11				15	
*05:17, 05:68, 05:79							7				11				15	
*05:20							7				11					
*05:23							7				11				15	
*05:25, 05:55							7				11				15	
*05:26						6	7				11				15	
*05:27							7				11				15	
*05:36							7				11				15	
*05:39							7				11				15	
*05:42							7				11		13		15	
*05:43							7				11					
*05:46							7				11		13		15	
*05:58:01			3				7				11				15	
*05:62							7				11				15	
*05:78						6	7				11					
*05:85							7				11				15	
*06:02:01:01-06:02:01:02, 06:02:03-06:02:11, 06:02:13-06:02:15, 06:02:17-06:02:27, 06:07, 06:10, 06:12-06:13, 06:15-06:16N, 06:18-06:30, 06:33-06:34, 06:36-06:39, 06:41-06:51, 06:53:01-06:66, 06:68-06:71, 06:73-06:75, 06:78, 06:80, 06:83-06:88, 06:90-06:93, 06:97-06:98, 06:100, 06:102								8			11			14		
*06:02:12								8						14		
*06:02:16, 06:81, 06:101								8			11					
*06:03:01			3					8			11			14	15	
*06:03:02, 06:76:01								8			11			14	15	
*06:04, 06:40, 06:79N								8			11					
*06:05							7	8			11					
*06:06								8			11					
*06:08			2					8			11			14		
*06:09								8			11			14		
*06:11, 06:52								8						14		
*06:14, 06:35, 06:72								8			11					
*06:17											11			14		
*06:31								8			11			14		

*06:32, 06:94-06:95, 06:99											11			14		
*06:67							7	8			11			14		
*06:76:02											11	12	13	14	15	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16
HLA-C allele ⁴																
*06:77								8			11			14		
*06:82								8			11			14		
*06:89								8			11			14		
*06:96								8			11			14		
*07:01:01:01-07:01:06, 07:01:08-07:02:07, 07:02:10-07:02:28, 07:02:30-07:02:40, 07:02:42-07:04:02, 07:04:04-07:06, 07:08, 07:11-07:13, 07:15-07:19, 07:21-07:25, 07:27:01, 07:29-07:30, 07:32N, 07:35-07:38:02, 07:42, 07:44-07:48, 07:50-07:52, 07:54-07:63, 07:65-07:75, 07:77-07:91, 07:93-07:95, 07:97-07:100, 07:102-07:122, 07:124, 07:126-07:138, 07:140-07:147, 07:149-07:151, 07:153-07:155, 07:157-07:160, 07:162-07:172, 07:174-07:176, 07:178-07:183, 07:185-07:195, 07:197-07:199:01, 07:200-07:209, 07:211-07:215, 07:217-07:237, 07:239-07:241, 07:243-07:245, 07:248-07:262, 07:264N-07:266, 07:268-07:293, 07:297-07:313									9							
*07:01:07, 07:04:03, 07:14, 07:27:02, 07:53, 07:199:02, 07:216									9							
*07:02:08, 07:02:29, 07:26, 07:33N, 07:39-07:40, 07:92, 07:139, 07:152N, 07:156, 07:246, 07:267, 07:296, 07:314									9							
*07:02:09						6			9				13			
*07:02:41									9							
*07:07, 07:09, 07:49, 07:76, 07:247, 07:315									9		11					
*07:10, 07:28, 07:41, 07:43, 07:184, 07:196									9							
*07:20, 07:263									9							
*07:31:01									9					14		
*07:31:02									9					14		
*07:64									9							16
*07:96:01-07:96:02			3						9							
*07:101, 07:148, 07:161		2							9	10						
*07:123, 07:173, 07:294									9							
*07:125						6			9				13	14		
*07:177									9					14		
*07:210, 07:238									9		11					
*07:242				4					9							
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16
HLA-C allele ⁴																
*08:01:01-08:01:10, 08:03:01-08:03:02, 08:06, 08:08, 08:14, 08:16:01-08:16:02, 08:20, 08:22, 08:24, 08:26N, 08:28, 08:36N, 08:38, 08:40, 08:42, 08:44, 08:46, 08:50, 08:56-08:61, 08:65, 08:72:01-08:72:02, 08:78										10						
*08:02:01, 08:02:03-08:02:10, 08:04:01-08:04:03, 08:13, 08:15:01-08:15:02, 08:17-08:19, 08:23, 08:27, 08:29-08:30, 08:32-08:35, 08:37, 08:39, 08:43, 08:45, 08:49, 08:51-08:55N, 08:62-08:63, 08:66-08:71, 08:73-08:77										10						
*08:02:02										10						
*08:05										10			13			
*08:07, 08:47										10						
*08:09, 08:11										10						
*08:10							7			10	11				15	
*08:12										10						
*08:21										10			13			
*08:25										10			13			
*08:31		2								10						
*08:41										10						
*08:48										10						
*12:02:01-12:02:02, 12:02:04-12:02:07, 12:02:09, 12:17-12:18:01, 12:22, 12:30, 12:36, 12:40, 12:56, 12:64, 12:67-12:69, 12:73-12:74, 12:80N, 12:84N-12:86												12	13			
*12:02:03, 12:02:08, 12:10:01-12:10:02, 12:20, 12:96												12	13			
*12:03:01:01-12:03:01:02, 12:03:03, 12:03:05-12:03:07, 12:03:10-12:03:12, 12:03:14-12:03:15, 12:03:18-12:03:19, 12:03:21-12:03:22, 12:03:25, 12:06-12:07, 12:11-12:13, 12:19, 12:23, 12:25, 12:29, 12:32, 12:34-12:35, 12:37-12:39N, 12:42Q-12:43, 12:45-12:48, 12:50-12:53, 12:57, 12:61-12:63, 12:65-12:66, 12:70-12:71, 12:75-12:79, 12:82, 12:87-12:95, 12:97, 12:99												12	13	14		
*12:03:02, 12:03:08, 12:03:16, 12:03:20, 12:26, 12:59													13	14		
*12:03:04												12		14		
*12:03:09								8				12		14		
*12:03:13												12	w	14		
*12:03:17												12	13	14		
*12:03:23			3									12	13	14		
*12:03:24												12	w	14		
*12:04:01											11			14	15	

*04:01:01:01-04:01:15, 04:01:17-04:01:22, 04:01:24-04:01:27, 04:01:29-04:01:48, 04:05, 04:07, 04:09N, 04:12, 04:15:02, 04:17-04:20, 04:24-04:28, 04:30-04:31, 04:35, 04:38-04:41, 04:43-04:53, 04:56-04:57, 04:59Q-04:67, 04:69-04:70, 04:73-04:76, 04:78-04:79, 04:81-04:102, 04:104-04:106, 04:109-04:110, 04:113, 04:115N, 04:117-04:121, 04:123N-04:128, 04:131-04:139, 04:141-04:144						6						11		13			
*04:01:16, 04:23, 04:54, 04:72						6						11		13			
*04:01:23, 04:33, 04:116, 04:130						6						11		13			
*04:01:28, 04:140												11		13			
*04:03, 04:80			3			6						11		13		15	
*04:04:01-04:04:02, 04:13, 04:111, 04:122						6						11		13			
*04:06			3			6						11		13		15	
*04:08						6						11		13			
*04:10						6						11					
*04:11						6											16
*04:14, 04:68						6						11		13			
*04:15:01, 04:15:03						6						11		13			
*04:16			3									11		13		15	
*04:29						6								13			16
*04:32, 04:77			2			6						11		13			
*04:34						6						11		13			
*04:36, 04:55						6											16
*04:37						6						11		13		15	
*04:42												11		13	14		
*04:58						6						11		13			
*04:71	1					6						11		13			
*04:103												11		13		15	
*04:107			3			6						11		13		15	
*04:108						6						11		13			
*04:112						6						11		13			
*04:114						6						11		13			16
*04:129						6	7					11		13			
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	
HLA-C allele⁴																	
*05:01:01:01-05:01:26, 05:03, 05:05-05:10, 05:12-05:15, 05:18:01-05:19, 05:21, 05:24, 05:28-05:31, 05:33-05:35, 05:37-05:38, 05:40-05:41, 05:44:01-05:45, 05:47-05:50, 05:52-05:54, 05:56-05:57, 05:58:02-05:61, 05:63-05:67, 05:69, 05:71-05:77, 05:80-05:84, 05:86-05:87								7				11				15	
*05:04:01-05:04:02, 05:22:01-05:22:02								7				11				15	

*07:01:01:01-07:01:06, 07:01:08-07:02:07, 07:02:10-07:02:28, 07:02:30-07:04:02, 07:04:04-07:06, 07:08, 07:11-07:13, 07:15-07:19, 07:21-07:25, 07:27:01, 07:29-07:30, 07:32N, 07:35-07:38:02, 07:42, 07:44-07:48, 07:50-07:52, 07:54-07:63, 07:65-07:75, 07:77-07:91, 07:93-07:95, 07:97-07:100, 07:102-07:122, 07:124, 07:126-07:138, 07:140-07:147, 07:149-07:151, 07:153-07:155, 07:157-07:160, 07:162-07:172, 07:174-07:176, 07:178-07:183, 07:185-07:195, 07:197-07:199:01, 07:200-07:209, 07:211-07:215, 07:217-07:237, 07:239-07:241, 07:243-07:245, 07:248-07:262, 07:264N-07:266, 07:268-07:293, 07:297-07:299										9							
*07:01:07, 07:04:03, 07:14, 07:27:02, 07:53, 07:199:02, 07:216										9							
*07:02:08, 07:02:29, 07:26, 07:33N, 07:39-07:40, 07:92, 07:139, 07:152N, 07:156, 07:246, 07:267, 07:296										9							
*07:02:09					6					9			13				
*07:07, 07:09, 07:49, 07:76, 07:247										9	11						
*07:10, 07:28, 07:41, 07:43, 07:184, 07:196										9							
*07:20, 07:263										9							
*07:31:01										9				14			
*07:31:02										9				14			
*07:64										9							16
*07:96:01-07:96:02			3							9							
*07:101, 07:148, 07:161		2								9	10						
*07:123, 07:173, 07:294										9							
*07:125						6				9			13	14			
*07:177										9				14			
*07:210, 07:238										9	11						
*07:242				4						9							
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	
HLA-C allele⁴																	
*08:01:01-08:01:09, 08:03:01-08:03:02, 08:06, 08:08, 08:14, 08:16:01-08:16:02, 08:20, 08:22, 08:24, 08:26N, 08:28, 08:36N, 08:38, 08:40, 08:42, 08:44, 08:46, 08:50, 08:56-08:61, 08:65, 08:72											10						

*08:02:01, 08:02:03-08:02:09, 08:04:01-08:04:03, 08:13, 08:15:01- 08:15:02, 08:17-08:19, 08:23, 08:27, 08:29-08:30, 08:32-08:35, 08:37, 08:39, 08:43, 08:45, 08:49, 08:51- 08:55N, 08:62-08:63, 08:66-08:71, 08:73-08:74										10						
*08:02:02										10						
*08:05										10			13			
*08:07, 08:47										10						
*08:09, 08:11										10						
*08:10							7			10	11				15	
*08:12										10						
*08:21										10			13			
*08:25										10			13			
*08:31		2								10						
*08:41										10						
*08:48										10						
*12:02:01-12:02:02, 12:02:04- 12:02:07, 12:02:09, 12:17-12:18:01, 12:22, 12:30, 12:36, 12:40, 12:56, 12:64, 12:67-12:69, 12:73-12:74, 12:80N, 12:84N-12:86												12	13			
*12:02:03, 12:02:08, 12:10:01- 12:10:02, 12:20												12	13			
*12:03:01:01-12:03:01:02, 12:03:03, 12:03:05-12:03:07, 12:03:10-12:03:12, 12:03:14-12:03:15, 12:03:18-12:03:19, 12:03:21-12:03:22, 12:03:25, 12:06- 12:07, 12:11-12:13, 12:19, 12:23, 12:25, 12:29, 12:32, 12:34-12:35, 12:37-12:39N, 12:42Q-12:43, 12:45- 12:48, 12:50-12:53, 12:57, 12:61- 12:63, 12:65-12:66, 12:70-12:71, 12:75-12:79, 12:82, 12:87-12:94												12	13	14		
*12:03:02, 12:03:08, 12:03:16, 12:03:20, 12:26, 12:59														13	14	
*12:03:04												12		14		
*12:03:09							8					12		14		
*12:03:13												12	w	14		
*12:03:17												12	13	14		
*12:03:23			3									12	13	14		
*12:03:24												12	w	14		
*12:04:01											11			14	15	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16
HLA-C allele⁴																
*12:04:02, 12:60												11	12	13	14	15
*12:05												11	12		14	15
*12:08												12	13			
*12:09												11				15

*12:14:01													13			
*12:14:02												12	13			
*12:15							8					12	13	14		
*12:16												12	13			
*12:18:02, 16:43 ⁶													13			
*12:21							7				11	12			15	
*12:24												12	13			
*12:27												12	13			
*12:28, 12:58												12	13	14		
*12:31												12	13	14		
*12:33							7				11			14	15	
*12:41											11	12	13		15	
*12:44												12	13			
*12:49												12				
*12:54											11	12	13	14		
*12:55												12	13	14		16
*12:72												12	13		15	
*12:81												12	13	14		
*12:83												12	13			
*14:02:01-14:02:02, 14:02:04-14:02:07, 14:02:09-14:02:11, 14:07N, 14:11, 14:13-14:14, 14:18-14:19, 14:21N, 14:23-14:24:02, 14:26-14:27, 14:29-14:34, 14:36-14:37, 14:39-14:40, 14:42-14:44, 14:46-14:48																16
*14:02:03, 14:03, 14:22, 14:38, 14:41																16
*14:02:08												12				16
*14:04											11					16
*14:05																16
*14:06, 14:09, 14:28, A*30:62 ⁷																16
*14:08																16
*14:10																16
*14:12, 14:49											11					16
*14:15														14		16
*14:16														14		16
*14:17																16
*14:20																16
*14:25																16
*14:35N																16
*14:45															15	16
*15:02:01-15:02:09, 15:02:11-15:02:15, 15:13, 15:15, 15:18, 15:26, 15:28, 15:31-15:35, 15:38-15:39, 15:41, 15:44-15:45, 15:47-15:53, 15:56-15:57, 15:60, 15:64				3				8			11				15	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16
HLA-C allele ⁴																
*15:02:10				3				8			11				15	
*15:03				3				8			11		13		15	

*15:04-15:06:03, 15:09, 15:19, 15:24, 15:29-15:30, 15:40, 15:46, 15:54, 15:59, 15:66			3								11				15	
*15:07			3					8								
*15:08			3					8			11				15	
*15:10:01								8			11				15	
*15:10:02, 15:17			3					8			11				15	
*15:11			3			6		8			11				15	
*15:12					5			8			11				15	
*15:16			3					8			11		13		15	
*15:21			3					8							15	
*15:22, 15:55			3								11				15	
*15:23											11					
*15:25			3										13			
*15:27			3								11				15	
*15:36						6					11					
*15:37			3					8			11				15	
*15:42		2	3					8			11				15	
*15:43			3					8								
*15:58			3					8			11				15	
*15:61											11				15	
*15:62			3					8			11				15	
*15:63								8			11					
*15:65			3								11				15	
*16:01:01, 16:01:03-16:01:13, 16:06-16:08, 16:10-16:11, 16:13-16:15:01, 16:16Q-16:17, 16:21-16:24, 16:26-16:28, 16:30N-16:32, 16:36, 16:38-16:39, 16:41, 16:44, 16:49, 16:51-16:52, 16:54													13			
*16:01:02													13			
*16:02:01-16:02:09, 16:09, 16:12, 16:19, 16:25, 16:46-16:47											11		13		15	
*16:04:01, 16:33, 16:42, 16:55														14		
*16:15:02												12	13			
*16:18													13			
*16:20													13			
*16:29		2												14		
*16:34			3			6							13			
*16:35														14		
*16:37													13		15	
*16:40								7								16
*16:45													13			
*16:48											11			14	15	
*16:50		2											13			
*16:53								7								16
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16
HLA-C allele⁴																
*17:01:01:01-17:01:03, 17:01:05-17:04, 17:06, 17:08-17:16, 17:18-17:19		2									11				15	

*17:01:04		2									11		13		15	
*17:05		2					7				11				15	
*17:07											11				15	
*17:17											11				15	
*18:01-18:02, 18:04-18:06											11					
*18:03		2									11					
A*24:96, A*24:106, A*24:174, B*46:25																
B*14:03																
B*35:08:02, B*35:08:05																
B*35:222															15	
B*58:02																
B*67:02													13			
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16

17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	Well No.
17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	Well No.
																HLA-C allele ⁴
			20						26							*01:02:01-01:02:17, 01:02:19, 01:06-01:08, 01:11-01:13, 01:15-01:20, 01:23, 01:25-01:33, 01:37N-01:42, 01:44-01:48, 01:51-01:54, 01:56N-01:58, 01:61-01:77
			20						26							*01:02:18
			20	21					26							*01:03, 01:24, 01:34, 01:78
																*01:04
																*01:05, 01:22, 01:36, B*39:76, B*54:18 ⁵
																*01:09
			20						26							*01:10
			20						26					31		*01:14
										27			30			*01:21
												29				*01:35
			20						26							*01:43
				21												*01:49-01:50, 01:55
			20						26							*01:59
		19	20						26							*01:60
							24								31	*02:02:01
							24								31	*02:02:02-02:02:03, 02:02:06-02:02:12, 02:02:15-02:02:25, 02:07-02:09, 02:11, 02:15, 02:19-02:20, 02:24-02:25Q, 02:28-02:31, 02:34-02:35, 02:37-02:38N, 02:40, 02:42-02:44, 02:46, 02:48, 02:50, 02:52N-02:54, 02:56-02:57, 02:59-02:61, 02:63-02:64, 02:66-02:67Q
															31	*02:02:05, 02:02:14, 02:10, 02:32, 02:51
															31	*02:02:13
							24								31	*02:02:26, 02:21, 02:26:02-02:26:03, 02:39, 02:45
							24								31	*02:03, 02:16:01-02:16:02
															31	*02:04
															31	*02:05, 02:17
17															31	*02:06
										w			30		31	*02:12
	18						24								31	*02:13
															31	*02:14
	18															*02:18
				21			24								31	*02:22, 02:62
17							24								31	*02:23, 02:36, 02:68
							24								31	*02:26:01
							24			27						*02:27:01
										27						*02:27:02
	18														31	*02:33
							24								31	*02:47
	18						24						30		31	*02:49
							24						30		31	*02:55

Negative Control

												29	31			*02:58
																*02:65
17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	Well No.
17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	Well No.
																HLA-C allele ⁴
					22											*03:02:01-03:02:09, 03:05, 03:17, 03:25, 03:27, 03:33, 03:35-03:36, 03:60, 03:71, 03:84, 03:89, 03:95, 03:108, 03:121N, 03:135, 03:139, 03:146, 03:154, 03:169Q, 03:178, 03:190, 03:194
																*03:02:10
				21	22											*03:03:01-03:03:20, 03:11:01-03:11:02, 03:20N-03:22Q, 03:30, 03:43:01-03:43:02, 03:49-03:50, 03:52-03:53, 03:55-03:56, 03:59, 03:61-03:62, 03:66, 03:68-03:69, 03:75-03:76, 03:79, 03:83, 03:85, 03:88, 03:96, 03:102-03:103, 03:112, 03:116:01-03:116:02, 03:119-03:120, 03:122, 03:124, 03:126-03:127, 03:133, 03:141-03:142, 03:150, 03:152, 03:158, 03:160-03:161, 03:168, 03:171, 03:176, 03:185, 03:187-03:189N, 03:192
				21												*03:03:21, 03:04:26, 03:18, 03:39, 03:64:02, 03:137, 03:144, 03:156, 03:166, 03:170, 03:182, 03:195
					21	22										*03:04:01:01-03:04:06, 03:04:08-03:04:24, 03:04:27-03:04:36, 03:06, 03:09, 03:14, 03:19, 03:23-03:24, 03:26, 03:28, 03:32, 03:34, 03:37:01-03:38:02, 03:40:01-03:40:03, 03:42, 03:44, 03:46-03:48, 03:51, 03:54, 03:57, 03:63-03:64:01, 03:65, 03:70, 03:72-03:74, 03:77-03:78, 03:82, 03:87, 03:90-03:93, 03:98, 03:100-03:101, 03:104-03:107, 03:109, 03:111, 03:114-03:115, 03:117-03:118, 03:123, 03:125, 03:128-03:129, 03:131, 03:134, 03:136, 03:138, 03:143, 03:147-03:149, 03:153, 03:155, 03:157, 03:159, 03:162, 03:164, 03:173-03:174, 03:179-03:181, 03:183-03:184, 03:186, 03:191, 03:193
				21	22											*03:04:07, 03:41, 03:80, 03:113
				21						27						*03:04:25
				21	22											*03:07, 03:45, 03:163
				21	22							29				*03:08
				21	22											*03:10
					22											*03:13, 03:97, 03:132, 03:167, 03:177
					22											*03:15

Negative Control

						23								31		*06:32, 06:94-06:95, 06:99
						23	24		26					31		*06:67
	18													31		*06:76:02
17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	Well No.
17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	Well No.
																HLA-C allele ⁴
														31		*06:77
						23										*06:82
17						23								31		*06:89
						23						29		31		*06:96
						23										*07:01:01-01-07:01:06, 07:01:08-07:02:07, 07:02:10-07:02:28, 07:02:30-07:02:40, 07:02:42-07:04:02, 07:04:04-07:06, 07:08, 07:11-07:13, 07:15-07:19, 07:21-07:25, 07:27-01, 07:29-07:30, 07:32N, 07:35-07:38:02, 07:42, 07:44-07:48, 07:50-07:52, 07:54-07:63, 07:65-07:75, 07:77-07:91, 07:93-07:95, 07:97-07:100, 07:102-07:122, 07:124, 07:126-07:138, 07:140-07:147, 07:149-07:151, 07:153-07:155, 07:157-07:160, 07:162-07:172, 07:174-07:176, 07:178-07:183, 07:185-07:195, 07:197-07:199:01, 07:200-07:209, 07:211-07:215, 07:217-07:237, 07:239-07:241, 07:243-07:245, 07:248-07:262, 07:264N-07:266, 07:268-07:293, 07:297-07:313
						23								30		*07:01:07, 07:04:03, 07:14, 07:27:02, 07:53, 07:199:02, 07:216
						23										*07:02:08, 07:02:29, 07:26, 07:33N, 07:39-07:40, 07:92, 07:139, 07:152N, 07:156, 07:246, 07:267, 07:296, 07:314
	18					23			27							*07:02:09
						w										*07:02:41
			20			23										*07:07, 07:09, 07:49, 07:76, 07:247, 07:315
				21		23										*07:10, 07:28, 07:41, 07:43, 07:184, 07:196
						23						29				*07:20, 07:263
	18					23										*07:31:01
	18					23								30		*07:31:02
																*07:64
					22							29				*07:96:01-07:96:02
						23			26							*07:101, 07:148, 07:161
17						23										*07:123, 07:173, 07:294
						23										*07:125
	18															*07:177
			20													*07:210, 07:238
						23										*07:242
17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	Well No.

Negative Control

17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	Well No.
																HLA-C allele ⁴
	18			21						27						*08:01:01-08:01:10, 08:03:01-08:03:02, 08:06, 08:08, 08:14, 08:16:01-08:16:02, 08:20, 08:22, 08:24, 08:26N, 08:28, 08:36N, 08:38, 08:40, 08:42, 08:44, 08:46, 08:50, 08:56-08:61, 08:65, 08:72:01-08:72:02, 08:78
				21						27						*08:02:01, 08:02:03-08:02:10, 08:04:01-08:04:03, 08:13, 08:15:01-08:15:02, 08:17-08:19, 08:23, 08:27, 08:29-08:30, 08:32-08:35, 08:37, 08:39, 08:43, 08:45, 08:49, 08:51-08:55N, 08:62-08:63, 08:66-08:71, 08:73-08:77
				21			24			27						*08:02:02
				21					26	27						*08:05
		19		21						27						*08:07, 08:47
	18									27						*08:09, 08:11
	18			21												*08:10
			20	21			24		26	27						*08:12
	18			21					26	27						*08:21
				21						27						*08:25
				21						27						*08:31
	18			21						27		29				*08:41
										27						*08:48
										27			30			*12:02:01-12:02:02, 12:02:04-12:02:07, 12:02:09, 12:17-12:18:01, 12:22, 12:30, 12:36, 12:40, 12:56, 12:64, 12:67-12:69, 12:73-12:74, 12:80N, 12:84N-12:86
										27						*12:02:03, 12:02:08, 12:10:01-12:10:02, 12:20, 12:96
										27						*12:03:01:01-12:03:01:02, 12:03:03, 12:03:05-12:03:07, 12:03:10-12:03:12, 12:03:14-12:03:15, 12:03:18-12:03:19, 12:03:21-12:03:22, 12:03:25, 12:06-12:07, 12:11-12:13, 12:19, 12:23, 12:25, 12:29, 12:32, 12:34-12:35, 12:37-12:39N, 12:42Q-12:43, 12:45-12:48, 12:50-12:53, 12:57, 12:61-12:63, 12:65-12:66, 12:70-12:71, 12:75-12:79, 12:82, 12:87-12:95, 12:97, 12:99
										27						*12:03:02, 12:03:08, 12:03:16, 12:03:20, 12:26, 12:59
																*12:03:04
																*12:03:09
										27						*12:03:13
							24			27						*12:03:17
										27						*12:03:23
										w						*12:03:24
														31		*12:04:01

Negative Control

17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	Well No.
17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	Well No.
																HLA-C allele ⁴
														31		*12:04:02, 12:60
														31		*12:05
17										27				30		*12:08
			20						26						31	*12:09
	18	19								27				30		*12:14:01
	18	19								27				30		*12:14:02
17										27						*12:15
						23				27				30		*12:16
										27				30		*12:18:02, 16:43 ⁶
							24								31	*12:21
			20						26	27						*12:24
														30		*12:27
	18									27						*12:28, 12:58
				21						27						*12:31
							24								31	*12:33
														30	31	*12:41
				21						27						*12:44
										27						*12:49
										27						*12:54
										27						*12:55
										w				30	31	*12:72
17										27						*12:81
										27		29	30			*12:83
										27						*12:98
			20						26		28					*14:02:01-14:02:02, 14:02:04-14:02:07, 14:02:09-14:02:12, 14:07N, 14:11, 14:13-14:14, 14:18-14:19, 14:21N, 14:23-14:24:02, 14:26-14:27, 14:29-14:34, 14:36-14:37, 14:39-14:40, 14:42-14:44, 14:46-14:48, 14:50-14:52
			20						26	27	28					*14:02:03, 14:03, 14:22, 14:38, 14:41
			20						26		28					*14:02:08
			20						26		28					*14:04
			20						26							*14:05
										27						*14:06, 14:09, 14:28, A*30:62 ⁷
																*14:08
18		20							26	27	28					*14:10
		20							26		28				31	*14:12, 14:49
18																*14:15
																*14:16
		19	20						26		28					*14:17
			20						26		28	29				*14:20
			20		22				26		28					*14:25
			20						26	27	w					*14:35N
			20						26							*14:45
17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	Well No.
17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	Well No.
																HLA-C allele ⁴

Negative Control

