

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32
HLA-C allele ^{1,2}																																
*01:02:01-01:02:17, 01:02:19-01:02:33, 01:02:35, 01:06-01:07:01, 01:08, 01:11-01:13, 01:15-01:20, 01:23, 01:25-01:33, 01:37N-01:42, 01:44-01:48, 01:51-01:54, 01:56N-01:58, 01:61-01:66, 01:68-01:77, 01:80-01:89N, 01:91-01:96, 01:98N-01:100, 01:103-01:106	1																			20						26						
*01:02:18	1										12									20						26						
*01:02:34	1																			20						26	27					
*01:03, 01:24, 01:34, 01:78	1																			20	21					26						
*01:04	1										12		14																			
*01:05, 01:07:02, 01:22, 01:36, 01:79:01-01:79:02, B*39:76, B*54:18 ³	1																															
*01:09	1												14																			
*01:10	1	2																		20						26						
*01:14	1										11									20						26					31	
*01:21	1											12	13															27		30		
*01:35	1																												29			
*01:43	1	2								10										20						26						
*01:49-01:50, 01:55	1																				21											
*01:59	1										11									20						26						
*01:60	1																			19	20					26						
*01:67	1															16				20						26						
*01:90	1							8									17			20						26						
*01:97	1																			20						26		28				
*01:101-01:102	1																17			20						26						
*01:107	1																			20						26			29			
*02:02:01		2	3			6					11				w									24							31	
*02:02:02:01-02:02:03, 02:02:06-02:02:12, 02:02:15-02:02:25, 02:02:27-02:02:28, 02:07-02:09, 02:11, 02:15, 02:19-02:20, 02:24-02:25Q, 02:28-02:31, 02:34-02:35, 02:37-02:38N, 02:40:01, 02:42-02:44, 02:46, 02:48, 02:50, 02:52N-02:54, 02:56-02:57, 02:59-02:61, 02:63-02:64, 02:66-02:67Q, 02:69-02:71, 02:74, 02:77-02:86, 02:88, 02:90-02:93			2	3		6					11				15									24								31
*02:02:05, 02:02:14, 02:10, 02:32, 02:40:02, 02:51, 02:73, 02:76, 02:89, 02:95		2	3			6					11				15																	31
*02:02:13		2	3								11				15																	31
*02:02:26, 02:21, 02:26:02-02:26:03, 02:39, 02:45		2	3								11				15									24								31
*02:02:29		2	3								11				15								22	24								31
*02:02:30		2				6					11				15									24								31
*02:03, 02:16:01-02:16:02, 02:72		2	3			6					11				15									24								31
*02:04		2	w			6					11				15									24								31
*02:05:01, 02:17		2	3			6					11			14	15																	31
*02:05:02-02:05:03		2	3			6					11			14	15									24								31
*02:06		2	3			6		8			11				15		17															31
*02:12		2	3			6							13	15												26	w		30	31		
*02:13		2	3			6					11				15		18							24								31
*02:14		2	w			6					11				15																	31
*02:18		2	3			6					11				15		18															
*02:22, 02:62		2	3			6					11				15						21			24								31
*02:23, 02:36, 02:68		2	3			6					11				15		17							24								31
*02:26:01		2									11				15									24								31
*02:27:01		2	3			6																			24			27				
*02:27:02		2	3			6																						27				
*02:33		2	3			6					11				15		18															31
*02:47		2	3			6		8			11				15									24								31
*02:49		2	3			6					11		13	15			18							24	25	26			30	31		
*02:55		2	3			6					11		13	15										24		26			30	31		
*02:58			3			6					11				15														29			31
*02:65		2	3			6					11													24								
*02:75		2	3			6					11				15		18							24	25							31
*02:87		2										10												24								
*02:94		2	3			6	7				11				15																	31
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32

Negative Control

*08:41										10							18		21								27	29				
*08:47										10							18	19	21									27				
*08:48, 08:108										10									21									27				
*08:95										10					16		18		21									w				
*08:113										10					17				21									27				
*08:115										10									21									27		29		
*12:02:01-12:02:02, 12:02:04-12:02:07, 12:02:09-12:02:10, 12:02:13, 12:17-12:18:01, 12:22, 12:30, 12:36, 12:40, 12:56, 12:64, 12:67-12:69, 12:73-12:74, 12:80N, 12:84N-12:86, 12:103-12:105N, 12:112, 12:114, 12:117, 12:124, 12:126, 12:128, 12:130, 12:134, 12:136-12:137, 12:142, 12:145, 12:148N												12	13														26	27		30		
*12:02:03, 12:02:08, 12:02:12, 12:10:01-12:10:02, 12:20, 12:96, 12:118												12	13															26	27			
*12:02:11							8					12																26		30		
*12:03:01:01-12:03:01:02, 12:03:03, 12:03:05-12:03:07, 12:03:10-12:03:12, 12:03:14-12:03:15, 12:03:18-12:03:19, 12:03:21-12:03:22, 12:03:25, 12:03:27-12:03:28, 12:03:31, 12:03:33, 12:06-12:07, 12:11-12:13, 12:19, 12:23, 12:25, 12:29, 12:32, 12:34-12:35, 12:37-12:39N, 12:42Q-12:43, 12:45-12:48, 12:50-12:53, 12:57, 12:61-12:63, 12:65-12:66, 12:70-12:71, 12:75-12:79, 12:82, 12:87-12:95, 12:97, 12:99-12:102, 12:107-12:111, 12:115-12:116, 12:120-12:121, 12:125, 12:129, 12:131, 12:133, 12:138-12:141, 12:143												12	13	14													26	27				
*12:03:02, 12:03:08, 12:03:16, 12:03:20, 12:03:30, 12:26, 12:59																												26	27			
*12:03:04												12		14														26				
*12:03:09							8					12		14														26				
*12:03:13												12	w	14													w	27				
*12:03:17												12	13	14							24						26	27				
*12:03:23			3									12	13	14													26	27				
*12:03:24												12	w	14													26	w				
*12:03:26							8					12	13	14													26	27				
*12:03:29, 12:03:32												12	13	14													26	27		30		
*12:03:34														14													26					
*12:04:01											11			14	15												26				31	
*12:04:02, 12:60											11	12	13	14	15											26				31		
*12:05											11	12		14	15												26				31	
*12:08												12	13				17										26	27		30		
*12:09											11			15					20								26				31	
*12:14:01													13				18	19									26	27		30		
*12:14:02												12	13				18	19									26	27		30		
*12:15							8					12	13	14			17										26	27				
*12:16												12	13								23						27		30			
*12:18:02													13														26	27		30		
*12:21						7				11	12			15							24						26				31	
*12:24												12	13							20							26	27				
*12:27												12	13														26			30		
*12:28, 12:58												12	13	14			18										26	27				
*12:31												12	13	14							21						26	27				
*12:33						7				11			14	15							24						26				31	
*12:41										11	12	13	15														26			30	31	
*12:44												12	13							21							26	27				
*12:49												12															27					
*12:54										11	12	13	14														26					
*12:55												12	13	14		16											27					
*12:72												12	13	15													26	w		30	31	
*12:81, 12:113												12	13	14		17											26	27				
*12:83, 12:106												12	13														26	27		29	30	
*12:98												12		14													27					
*12:119			2									12	13	14													26	27				
*12:122												12	13	14													26	27		29		
*12:123												12															26				30	
*12:127										10		12	13														26	27			30	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32
*12:132												12	13					18								25	26	27			30	
*12:135												12	13	14	15			18									26	w			31	
*12:144													13	14							21						26	27				

Negative Control

