

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32			
HLA-C allele ^{1,2}																																			
*01:02:01:01:01:02:17, 01:02:19-01:02:33, 01:02:35-01:02:48, 01:02:50-01:02:67, 01:02:69-01:02:70, 01:02:72-01:02:77, 01:02:79-01:02:87, 01:02:89-01:02:90, 01:06-01:07:01, 01:08, 01:11-01:13, 01:15-01:01:20, 01:23, 01:25-01:33, 01:37N-01:42, 01:44-01:47, 01:51-01:54, 01:56N-01:58, 01:61-01:66, 01:68-01:77, 01:80-01:89N, 01:91-01:96, 01:98N-01:100, 01:103-01:112, 01:115, 01:117N, 01:119, 01:121Q, 01:123-01:124, 01:126-01:130, 01:132-01:135, 01:137N-01:144, 01:145:02N, 01:147-01:153, 01:155-01:157, 01:159, 01:161-01:167, 01:170-01:175, 01:178-01:185Q, 01:187-01:214, 01:216-01:217:01:02, 01:219-01:225, 01:227-01:232, 01:234-01:243, 01:245, 01:247	1																			20						26									
*01:02:18	1											12									20						26								
*01:02:34	1																				20						26	27							
*01:02:49, 01:136, 01:145:01N, 01:146:01-01:146:02, 01:233	1							8													20						26								
*01:02:68, 01:02:71	1													15							20						26								
*01:02:78, 01:02:88, 01:05, 01:07:02, 01:22, 01:35-01:36:02, 01:79:01-01:79:02, 01:120, 01:158, 01:226	1																				20														
*01:03:01-01:03:03, 01:24, 01:34, 01:78	1																				20	21					26								
*01:04	1											12	14								20														
*01:09:01-01:09:02, 01:244	1												14								20														
*01:10	1	2																			20						26								
*01:14	1										11										20						26						31		
*01:21	1											12	13								20											30			
*01:43	1	2								10											20						26								
*01:48	1		3																		20						26								
*01:49:01-01:50, 01:55, 01:176	1																				20	21													
*01:59, 01:118	1										11										20						26								
*01:60	1																		19		20						26								
*01:67, 01:122	1			4											16						20						26								
*01:90, 01:101-01:102, 01:113	1							8								17					20						26								
*01:97, 01:114:01-01:114:02, 01:168-01:169:01	1																				20						26		28						
*01:116	1																17				20						26								
*01:125, 01:218	1															16					20						26								
*01:131	1			4																	20	21					26								
*01:154	1																				20						26							31	
*01:160	1												14				18				20						26								
*01:169:02, 03:94 ⁴	1		3																		20		22				26		28						
*01:177	1															16					20		22				25	26							
*01:186	1										11										20						25	26							
*01:215	1												13								20														
*01:246	1																				20		22				26								
*02:02:01-02:02:03, 02:02:06-02:02:12, 02:02:15-02:02:25, 02:02:27-02:02:28, 02:02:30-02:02:34, 02:02:36, 02:02:39-02:02:43, 02:02:45-02:02:52, 02:02:54-02:02:66, 02:04, 02:07-02:09, 02:11, 02:15, 02:19-02:20, 02:24-02:25Q, 02:28-02:31, 02:34-02:35, 02:37-02:38:01N, 02:42-02:43:01, 02:44, 02:46, 02:48, 02:50, 02:52N-02:54, 02:56-02:57, 02:59-02:61:02, 02:63-02:64, 02:66-02:67Q, 02:69-02:71, 02:74, 02:77-02:86, 02:88, 02:90-02:93, 02:96-02:100, 02:102-02:106, 02:108, 02:112-02:114, 02:116-02:117, 02:119-02:123, 02:127-02:130, 02:132, 02:136, 02:138-02:142, 02:144-02:151, 02:153-02:160, 02:162, 02:165N-02:166, 02:170-02:173, 02:175, 02:177-02:179, 02:183-02:186, 02:188, 02:190, 02:192N-02:196, 02:198-02:210, 02:212-02:217		2	3			6					11				15									24									31		
*02:02:13		2	3									11			15							22												31	
*02:02:14, 02:02:37, 02:10:01-01-02:10:08, 02:32, 02:38:02N, 02:43:02, 02:51, 02:73, 02:76, 02:89, 02:95, 02:109-02:110, 02:124-02:125, 02:133, 02:143, 02:161, 02:168-02:169N, 02:174, 02:176, 02:181-02:182, 02:211Q		2	3			6					11				15																			31	
*02:02:26, 02:21, 02:26:01-02:26:03, 02:39, 02:45, 02:189, 02:191		2	3									11			15									24										31	
*02:02:29		2	3									11			15							22		24										31	
*02:02:35, 02:03, 02:16:02:01-02:16:02:02, 02:72, 02:135N, 02:137		2	3			6						11			15																			31	
*02:02:38		2	3			6						11			15							22		24										31	
*02:02:44		2	3			6						11	12		15									24										31	
*02:02:53		2	3			6						11			15							22												31	
*02:05:01, 02:17		2	3			6						11		14	15																			31	
*02:05:02-02:05:03		2	3			6						11		14	15									24										31	
*02:06:01, 02:36:02		2	3			6		8				11		15	17									24										31	
*02:06:02, 02:23, 02:36:01		2	3			6		8				11		15	17									24										31	
*02:12		2	3			6						11	13	15													26	w			30		31		
*02:13, 02:75		2	3			6						11		15				18						24										31	
*02:14:01-02:14:02		2	3			6						11																						31	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32			
*02:18		2	3			6						11			15			18																	
*02:22, 02:62, 02:118		2	3			6						11			15							21		24										31	
*02:27:01, 02:131		2	3			6						11												24											
*02:27:02		2	3			6						11																							
*02:33		2	3			6						11			15			18																	31
*02:40:01		2	3			6						11			15				19					24											31
*02:40:02		2	3			6						11			15				19																31
*02:47, 02:111		2	3			6		8				11			15									24											31

Negative Control

*02:49	2	3		6					11	13	15		18				24	26		30	31											
*02:55:01	2	3		6					11	13	15						24	26		30	31											
*02:55:02	2	3		6					11	13	15							26		30	31											
*02:58		3		6					11		15										31											
*02:65	2	3		6					11								24															
*02:68	2	3		6					11		15		17				24				31											
*02:87	2							10									24		27													
*02:94	2	3		6	7				11		15										31											
*02:101	2	3		w					11		15						24				31											
*02:107	2								11								24				31											
*02:115	2	3		6					11	13			18				24	26	27		30											
*02:126	2	3		6					11		15						24		w		31											
*02:134, 02:152, 02:167	2								11		15						24				31											
*02:163	2	3		6					11		15						24	25			31											
*02:164, 02:187	2	3		6					11								24				31											
*02:180	2	3		6					11		15		18	19																		
*02:197		3		6					11		15					21					31											
*03:02:01-03:02:09, 03:02:11-03:02:28, 03:05:01:01-03:05:01:03, 03:17:01-03:17:02, 03:25, 03:33, 03:35:01-03:36, 03:60, 03:71, 03:84, 03:89, 03:95, 03:108, 03:110, 03:121N, 03:135, 03:139, 03:146, 03:154, 03:169Q, 03:178, 03:190, 03:194, 03:197-03:198, 03:200-03:201N, 03:216, 03:221-03:222, 03:224N-03:226, 03:238, 03:249, 03:258, 03:292, 03:296:01-03:296:02, 03:298-03:301, 03:315, 03:326, 03:329-03:330, 03:332, 03:335, 03:338, 03:349-03:350, 03:365, 03:371, 03:373, 03:386, 03:391N, 03:402-03:403, 03:406-03:407, 03:411-03:412, 03:425, 03:429, 03:449N, 03:452, 03:464, 03:468, 03:473, 03:475, 03:482, 03:489, 03:493-03:494, 03:497, 03:505, 03:532, 03:544, 03:547, 03:549, 03:561, 03:575Q, 03:584, 03:589, 03:596, 03:612-03:615, 03:618N-03:619N, 03:623, 03:625N-03:626		3	4														22															
*03:02:10, 03:264, 03:610				3	4																											
*03:03:01:01-03:03:20, 03:03:22-03:03:50, 03:03:53-03:03:63, 03:03:65-03:03:66, 03:11:01-03:11:02, 03:18:02-03:18:03, 03:20N-03:22Q, 03:30, 03:43:01-03:43:02, 03:49-03:50, 03:52-03:53, 03:55-03:56, 03:59, 03:61-03:62, 03:66, 03:68, 03:75-03:76, 03:79, 03:83, 03:85, 03:88, 03:96, 03:102-03:103, 03:112, 03:116:01-03:116:02, 03:119:01-03:120, 03:122, 03:124, 03:126-03:127, 03:133, 03:141-03:142, 03:150, 03:152, 03:158, 03:160-03:161, 03:168, 03:171, 03:176, 03:185, 03:187-03:189N, 03:192, 03:196, 03:202-03:207, 03:214, 03:217, 03:220, 03:223, 03:227-03:230, 03:237, 03:241-03:242, 03:253-03:254, 03:272-03:273, 03:275-03:276, 03:284-03:285, 03:288-03:291, 03:304, 03:307, 03:312, 03:316N, 03:319-03:321, 03:325, 03:327, 03:336, 03:339, 03:341, 03:345-03:346, 03:352, 03:356-03:357, 03:360, 03:363N-03:364, 03:367, 03:370, 03:372:01-03:372:02, 03:374-03:375, 03:377N-03:378, 03:380N, 03:383, 03:389, 03:395, 03:398, 03:413-03:414, 03:418, 03:421N-03:422, 03:427-03:428, 03:430, 03:432N-03:433, 03:436, 03:438, 03:440, 03:444N, 03:447N, 03:457, 03:460, 03:471-03:472, 03:476-03:477, 03:481, 03:490, 03:495, 03:503, 03:506-03:507, 03:509N, 03:514-03:517, 03:519-03:520, 03:524, 03:527-03:529, 03:533, 03:537, 03:541-03:542, 03:545, 03:550-03:551, 03:556, 03:559-03:560N, 03:563, 03:567-03:568, 03:573-03:574, 03:578-03:579, 03:582-03:583, 03:585-03:587, 03:592, 03:594-03:595, 03:597-03:598, 03:606, 03:609, 03:611, 03:616-03:617, 03:622, 03:624			4	5													21	22														
*03:03:21, 03:03:51, 03:18:01, 03:39, 03:64:02, 03:144, 03:182, 03:195, 03:344:01:01-03:344:01:02, 03:504				4														21														
*03:03:52, 03:151, 03:165:01-03:165:02, 03:251, 03:267, 03:451, 03:534-03:535:01:02, 03:540, 03:590, 03:607					5													21	22													
*03:03:64				4	5								18				21			27												
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32
*03:04:01:01-03:04:06, 03:04:08-03:04:14, 03:04:16-03:04:24, 03:04:27-03:04:97, 03:04:99-03:04:100, 03:04:102, 03:06:01:01-03:06:02, 03:09, 03:14, 03:19, 03:23N-03:24, 03:26:01-03:26:02, 03:28, 03:32:01:01-03:32:01:02, 03:34, 03:37:01-03:37:02, 03:40:01-03:40:04, 03:42:01-03:42:02, 03:44, 03:46-03:48, 03:51, 03:54, 03:57:01-03:57:02, 03:63-03:64:01, 03:65, 03:70, 03:72-03:74, 03:77-03:78, 03:80:02, 03:82, 03:87:01-03:87:02, 03:90-03:93, 03:98, 03:100-03:101, 03:104-03:107, 03:109, 03:111, 03:114-03:115:02, 03:117, 03:123, 03:125, 03:128-03:129, 03:131, 03:134, 03:138, 03:143, 03:147-03:149, 03:153, 03:157, 03:159, 03:162, 03:164, 03:173-03:174, 03:179-03:181, 03:183-03:184:02, 03:186:01-03:186:02, 03:191, 03:193, 03:208N, 03:210-03:213, 03:215, 03:218-03:219, 03:232-03:236, 03:239-03:240, 03:244Q, 03:247, 03:250, 03:252, 03:255-03:257, 03:259, 03:261, 03:263:01-03:263:03, 03:265N-03:266, 03:269-03:270, 03:277N, 03:279, 03:281-03:283, 03:287:02, 03:294, 03:303, 03:305-03:306, 03:309-03:311, 03:313-03:314, 03:318N, 03:322-03:323N, 03:328, 03:331, 03:333-03:334, 03:337, 03:340, 03:343, 03:347-03:348, 03:353-03:355, 03:358-03:359, 03:361-03:362, 03:366N, 03:368-03:369, 03:376, 03:379, 03:381-03:382, 03:385, 03:387, 03:390, 03:392N-03:394, 03:396-01N-03:397, 03:399-03:401, 03:404-01-01-03:405				3	4																21	22										

Negative Control

*15:75			3					8			11	13	15	17	18		21									29	30					
*15:102	1		3								11		15	17		20									26		28	29	31			
*15:104	1		3					8			11		15	17			21										29					
*15:107							7	8			11		15	17			21															
*15:118			3			6		8			11		15	17			21										29					
*15:144					5			8						17	18		21							27			29					
*15:178			3				7	8			11		15	17			21										29					
*15:190			3								11		15	17													29					
*15:201		2	3					8			11		15	17	19		21										29					
*15:218			3	4				8			11		15	17			21										29					
*16:01:01:01-16:01:01:38, 16:01:03-16:01:15, 16:01:18-16:01:25, 16:01:27-16:01:43, 16:06-16:08, 16:11, 16:13-16:14, 16:16Q-16:17, 16:21-16:24, 16:26-16:28, 16:30N-16:32, 16:36, 16:38-16:39:02, 16:41, 16:44, 16:49:01-16:49:02, 16:51-16:52, 16:54, 16:56, 16:58-16:59, 16:65, 16:72-16:73, 16:75-16:76, 16:79-16:81, 16:83, 16:86, 16:92-16:93, 16:95-16:98, 16:100, 16:105, 16:111-16:112, 16:114, 16:116-16:119, 16:122, 16:125, 16:127-16:131, 16:134-16:135, 16:137-16:138, 16:141-16:142, 16:146-16:148, 16:151-16:152, 16:154, 16:157-16:162, 16:164-16:165, 16:168, 16:171-16:175, 16:177-16:178, 16:180, 16:183, 16:185-16:187, 16:190, 16:192-16:193, 16:196-16:198, 16:200, 16:204										11	13			18											27		30					
*16:01:02, 16:01:26, 16:57, 16:85, 16:94, 16:106, 16:126, 16:191											11	13			18													30				
*16:01:16								8			11	13			18												30					
*16:01:17											11	12	13		18										27		30					
*16:02:01:01-16:02:12, 16:02:14-16:02:20, 16:09, 16:12, 16:19, 16:46-16:47, 16:60, 16:63, 16:69, 16:74, 16:77N, 16:84, 16:89N-16:90, 16:99, 16:101-16:102, 16:104, 16:107-16:108, 16:115, 16:120, 16:123N, 16:132N-16:133, 16:136, 16:143-16:145, 16:153, 16:155-16:156, 16:163, 16:166-16:167, 16:176, 16:179, 16:181, 16:184, 16:194-16:195N, 16:202											11	13	15		18												30					
*16:02:13											11	12	13	15		18												30				
*16:04:01:01-16:04:01:03, 16:04:03-16:04:05, 16:33, 16:42, 16:55, 16:61, 16:66, 16:78, 16:82, 16:124, 16:149, 16:182, 16:203													14		18	20									27							
*16:10												13			18										27		30					
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32
*16:15:02											11	12	13			18										26	27		30			
*16:18											11	13			18												27	28	30			
*16:20								8			11	13			17	18											27		30			
*16:25											11	13	15		18											26		30				
*16:29		2											14		18	20											27					
*16:34			3			6					11	13			18										25	27		30				
*16:35													14		18												27					
*16:37											11	13	15		18												w		30			
*16:40, 16:110							7						16														27					
*16:43											11	13															27		30			
*16:45												13			18				21								27		30			
*16:48											11		14	15		18																
*16:50			2								11	13			18												27		30			
*16:53							7							16						20						26	27					
*16:62											11	13			18										25	27		30				
*16:64											11	13			17	18										26	27		30			
*16:67, 16:188												13			18												27					
*16:68															18	20										26	27					
*16:70			3								11	13	15		17	18												29	30			
*16:71											11	13			18												w		30			
*16:87											11	13			17	18											27		30			
*16:88											11			15		18	20									26			31			
*16:91											11		14	15		18	20												31			
*16:103											11		15		18																	
*16:109								8					14		17	18	20										27					
*16:113							7				11	13		16													27		30			
*16:121			3			6					11	13	15		18														30			
*16:139										10	11	13			18									24		27		30				
*16:140											11		15																			
*16:150						6							14	16		18	20								25	27						
*16:169								8				13		16		17	18										27		30			
*16:170											11	13		16		18											w		30			
*16:189											11	13			18											26			30			
*16:201											11	13			18											26	27		30			
*17:01:01:02-17:01:03, 17:01:05-17:01:09, 17:01:11-17:04, 17:06, 17:08-17:16:02, 17:18-17:19, 17:21, 17:24-17:26, 17:28-17:32, 17:34-17:39, 17:41-17:61, 17:63-17:64Q, 17:66-17:69		2									11			15			19		21							26						
*17:01:04		2									11	13	15			19		21								26						
*17:01:10		2									11		15			19		21								26			30			
*17:05		2					7				11		15			19		21								26						
*17:07, 17:23, 17:33											11		15			19		21								26						
*17:17											11		15			19		21								26			31			
*17:20		2									11					19		21			23					26						
*17:22		2														19		21								26						
*17:27N		2									11		15			19		21								w						
*17:40		2									11		15			19		21														
*17:62		2											15			19		21								26						
*17:65		2									11		15			19		21								26			31			

Negati

Negative Control

