

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
HLA-C allele ¹																								
*01:02:01-01:02:14, 01:06-01:08, 01:11-01:13, 01:15-01:20, 01:23, 01:25-01:33, 01:37N-01:42, 01:44-01:48, 01:51-01:54, 01:56N-01:58, 01:61	1																			20				
*01:03, 01:24, 01:34	1																			20	21			
*01:04	1											12		14										
*01:05, 01:22, 01:35-01:36, B*54:18 ²	1													14										
*01:09	1																							
*01:10	1	2																		20				
*01:14, 01:59	1											11								20				
*01:21	1												12	13										
*01:43	1	2																		20				
*01:49-01:50, 01:55	1											10									20			
*01:60	1																			19	20			
*02:02:01		2	3			6						11			w									
*02:02:02-02:02:03, 02:02:05-02:02:12, 02:02:14-02:03, 02:07-02:11, 02:15-02:16:02, 02:19-02:20, 02:23-02:25Q, 02:28-02:32, 02:34-02:38N, 02:40, 02:42-02:44, 02:46, 02:48, 02:50-02:54		2	3			6						11			15									
*02:02:13, 02:21, 02:26:02-02:26:03, 02:39, 02:45		2	3									11			15									
*02:04		2	w			6						11			15									
*02:05, 02:17		2	3			6						11		14	15									
*02:06		2	3			6		8				11			15		17							
*02:12		2	3			6							13	15										
*02:13, 02:18, 02:33		2	3			6						11		15			18							
*02:14		2	w			6						11												
*02:22		2	3			6						11			15						21			
*02:26:01		2										11			15									
*02:27:01-02:27:02		2	3			6									15									
*02:47		2	3			6		8				11			15									
*02:49		2	3			6						11	13	15			18							
*02:55		2	3			6						11	13	15										
*03:02:01-03:02:08, 03:05, 03:17, 03:25, 03:27, 03:33, 03:35-03:36, 03:60, 03:71, 03:84, 03:89, 03:95, 03:108, 03:121N, 03:135, 03:139			3	4																		22		
*03:03:01-03:03:18, 03:11:01-03:12, 03:20N-03:22Q, 03:30-03:31, 03:43:01-03:43:02, 03:49-03:50, 03:52-03:53, 03:55-03:56, 03:59, 03:61-03:62, 03:66, 03:68-03:69, 03:75-03:76, 03:79, 03:83, 03:85, 03:88, 03:96, 03:102-03:103, 03:112, 03:116, 03:119-03:120, 03:122, 03:124, 03:126-03:127, 03:133				4	5																21	22		
*03:04:01:01-03:04:06, 03:04:08-03:04:24, 03:06, 03:08-03:09, 03:14, 03:19, 03:23-03:24, 03:26, 03:28, 03:32, 03:34, 03:37-03:38:02, 03:40:01-03:40:02, 03:42, 03:44, 03:46-03:48, 03:51, 03:54, 03:57, 03:63-03:65, 03:70, 03:72-03:74, 03:77-03:78, 03:82, 03:87, 03:90-03:93, 03:98, 03:100-03:101, 03:104-03:107, 03:109, 03:111, 03:114-03:115, 03:117-03:118, 03:123, 03:125, 03:128-03:129, 03:131, 03:134, 03:136, 03:138			3	4																	21	22		
*03:04:07, 03:41, 03:80, 03:113			3																		21	22		
*03:04:25			3	4																	21			
*03:07, 03:45			3	4								11			15						21	22		
*03:10, 03:29			3	4											15						21	22		
*03:13, 03:97, 03:132				4	5																	22		
*03:15			3	4								11			15							22		
*03:16, 03:110			3																			22		
*03:18, 03:39, 03:137				4																		21		
*03:58					5										15						20	21	22	
*03:67				4	w																21	w		
*03:81				4	5													17			21	22		
*03:86					5										15						20	22		
*03:94			3												15						20	22		
*03:99															15						20			
*03:130			3	4								11										21	22	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24

Negative Control

*07:177								9									18							
*07:210								9	11									20						
*08:01:01-08:01:05, 08:03:01-08:03:02, 08:06, 08:08, 08:14, 08:16, 08:20, 08:22, 08:24, 08:26N, 08:28, 08:36N, 08:38, 08:40-08:42, 08:44, 08:46, 08:50, 08:56									10								18			21				
*08:02:01-08:02:07, 08:04, 08:13, 08:15:01-08:15:02, 08:17-08:19, 08:23, 08:27, 08:29-08:30, 08:32-08:35, 08:37, 08:39, 08:43, 08:45, 08:49, 08:51-08:55N									10											21				
*08:05, 08:25									10		13									21				
*08:07, 08:47									10								19			21				
*08:09, 08:11									10								18							
*08:10					7				10	11			15				18			21				
*08:12									10										20	21				
*08:21									10		13						18			21				
*08:31		2							10											21				
*08:48									10															
*12:02:01-12:02:07, 12:10:01-12:10:02, 12:17-12:18, 12:20, 12:22, 12:27, 12:30, 12:36, 12:40, 12:56, 12:64, 12:67-12:68											12	13												
*12:03:01:01-12:03:01:02, 12:03:03, 12:03:05-12:03:07, 12:03:10-12:03:12, 12:03:14-12:03:15, 12:03:17-12:03:19, 12:06-12:07, 12:11-12:13, 12:19, 12:23, 12:25, 12:29, 12:32, 12:34-12:35, 12:37-12:39N, 12:42Q-12:43, 12:45-12:48, 12:50-12:53, 12:57, 12:61-12:63, 12:65-12:66											12	13	14											
*12:03:02, 12:03:08, 12:03:16, 12:03:20, 12:26, 12:59												13	14											
*12:03:04											12	14												
*12:03:09							8				12	14												
*12:03:13											12	w	14											
*12:04:01, 12:33										11			14	15										
*12:04:02, 12:60										11	12	13	14	15										
*12:05										11	12		14	15										
*12:08											12	13					17							
*12:09										11				15						20				
*12:14:01												13					18	19						
*12:14:02											12	13					18	19						
*12:15							8				12	13	14				17							
*12:16											12	13										23		
*12:21										11	12			15										
*12:24											12	13								20				
*12:28, 12:58											12	13	14				18							
*12:31											12	13	14							21				
*12:41										11	12	13		15										
*12:44											12	13								21				
*12:49											12													
*12:54										11	12	13	14											
*12:55											12	13	14	16										
*14:02:01-14:02:07, 14:02:09-14:03, 14:05, 14:07N, 14:11, 14:13-14:14, 14:18-14:24, 14:26-14:27, 14:29-14:34																	16			20				
*14:02:08											12						16			20				
*14:04										11							16			20				
*14:06, 14:08-14:09, 14:28																	16							
*14:10																	16	18		20				
*14:12										11										20				
*14:15																	16	18						
*14:16												14					16							
*14:17																	16			19	20			
*14:25		2															16			20		22		
*15:02:01-15:02:07, 15:08, 15:10:02, 15:13, 15:15, 15:17-15:18, 15:26, 15:28, 15:31-15:35, 15:38-15:39, 15:41, 15:44-15:45, 15:47-15:53, 15:56			3					8		11				15		17				21				
*15:03, 15:16			3					8		11		13		15		17				21				
*15:04-15:06:03, 15:09, 15:19-15:20, 15:24, 15:27, 15:29-15:30, 15:40, 15:46, 15:54			3							11				15		17				21				
*15:07			3					8								17				21				
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
*15:10:01								8			11				15		17				21			
*15:11			3			6		8			11				15		17				21			
*15:12					5			8			11				15		17	18			21			
*15:21			3					8							15		17				21			

Negative Control

