

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
HLA-C allele <sup>1,2</sup>																								
*01:02:01-01:02:17, 01:02:19-01:02:40, 01:05-01:08, 01:11-01:13, 01:15-01:20, 01:22-01:23, 01:25-01:33, 01:35-01:42, 01:44-01:48, 01:51-01:54, 01:56N-01:58, 01:61-01:66, 01:68-01:77, 01:79-01:01:89N, 01:91-01:100, 01:103-01:112, 01:114-01:115, 01:117N, 01:119-01:121Q, 01:123-01:124	1																			20				
*01:02:18	1											12								20				
*01:03, 01:24, 01:34, 01:49-01-01:50, 01:55, 01:78	1																			20	21			
*01:04	1											12		14						20				
*01:09	1													14						20				
*01:10	1	2																		20				
*01:14, 01:59, 01:118	1											11								20				
*01:21	1											12	13											
*01:43	1	2										10								20				
*01:60	1																			19	20			
*01:67, 01:122	1			4												16				20				
*01:90, 01:101-01:102, 01:113	1							8										17		20				
*01:116	1																17			20				
*02:02:01-02:02:03, 02:02:05-02:02:12, 02:02:14-02:02:25, 02:02:27-02:02:28, 02:03, 02:07-02:11, 02:15-02:16:02, 02:19-02:20, 02:24-02:25Q, 02:28-02:32, 02:34-02:35, 02:37-02:38N, 02:40-01-02:40:02, 02:42-02:44, 02:46, 02:48, 02:50-02:54, 02:56-02:57, 02:59-02:61, 02:63-02:64, 02:66-02:67Q, 02:69-02:74, 02:76-02:86, 02:88-02:93, 02:95-02:100, 02:102-02:106, 02:108-02:114, 02:116-02:117, 02:119-02:120		2	3			6						11			15									
*02:02:13, 02:02:26, 02:21, 02:26:02-02:26:03, 02:39, 02:45		2	3									11			15									
*02:02:29		2	3									11			15							22		
*02:02:30		2				6						11			15									
*02:04		2	w			6						11			15									
*02:05:01-02:05:03, 02:17		2	3			6						11		14	15									
*02:06:01-02:06:02, 02:23, 02:36		2	3			6		8				11			15		17							
*02:12, 02:55:01-02:55:02		2	3			6						11	13		15									
*02:13, 02:18, 02:33, 02:75		2	3			6						11			15		18							
*02:14:01		2	w			6						11												
*02:14:02		2				6						11												
*02:22, 02:62, 02:118		2	3			6						11			15						21			
*02:26:01		2										11			15									
*02:27:01-02:27:02, 02:65		2	3			6						11												
*02:47		2	3			6		8				11			15									
*02:49		2	3			6						11	13		15		18							
*02:58			3			6						11			15									
*02:68		2	3			6						11			15		17							
*02:87		2										10												
*02:94		2	3			6	7					11			15									
*02:101		2	3			w						11			15									
*02:107		2										11												
*02:115		2	3			6						11	13							18				
*03:02:01-03:02:09, 03:02:11-03:02:12, 03:02:14-03:02:15, 03:05, 03:17, 03:25, 03:27, 03:33, 03:35-03:36, 03:60, 03:71, 03:84, 03:89, 03:95, 03:108, 03:121N, 03:135, 03:139, 03:146, 03:154, 03:169Q, 03:178, 03:190, 03:194, 03:197-03:198, 03:200-03:201N, 03:216, 03:221-03:222, 03:224N-03:226, 03:238, 03:249, 03:258, 03:292, 03:296, 03:298-03:301, 03:315			3	4																		22		
*03:02:10, 03:264				4																				
*03:02:13, 03:16, 03:110			3																				22	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24

Negative Control

*03:03:01:01-03:03:20, 03:03:22-03:03:32, 03:11:01-03:11:02, 03:18:02, 03:20N-03:22Q, 03:30-03:31, 03:43:01-03:43:02, 03:49-03:50, 03:52-03:53, 03:55-03:56, 03:59, 03:61-03:62, 03:66, 03:68-03:69, 03:75-03:76, 03:79, 03:83, 03:85, 03:88, 03:96, 03:102-03:103, 03:112, 03:116:01-03:116:02, 03:119-03:120, 03:122, 03:124, 03:126-03:127, 03:133, 03:141-03:142, 03:150, 03:152, 03:158, 03:160-03:161, 03:168, 03:171, 03:176, 03:185, 03:187-03:189N, 03:192, 03:196, 03:202-03:207, 03:214, 03:217, 03:220, 03:223, 03:227-03:230, 03:237, 03:241-03:242, 03:253-03:254, 03:272-03:276, 03:284-03:285, 03:288-03:291, 03:304, 03:307, 03:312, 03:316N, 03:319-03:321			4	5																		21	22	
*03:03:21, 03:04:26, 03:18:01, 03:39, 03:64:02, 03:137, 03:144, 03:156, 03:166, 03:170, 03:182, 03:195			4																			21		
*03:04:01:01-03:04:06, 03:04:08-03:04:24, 03:04:27-03:04:51, 03:06:01-03:06:02, 03:08-03:09, 03:14, 03:19, 03:23-03:24, 03:26, 03:28, 03:32, 03:34, 03:37:01-03:38:02, 03:40:01-03:40:04, 03:42, 03:44, 03:46-03:48, 03:51, 03:54, 03:57:01-03:57:02, 03:63-03:64:01, 03:65, 03:70, 03:72-03:74, 03:77-03:78, 03:82, 03:87:01-03:87:02, 03:90-03:93, 03:98, 03:100-03:101, 03:104-03:107, 03:109, 03:111, 03:114-03:115, 03:117-03:118, 03:123, 03:125, 03:128-03:129, 03:131, 03:134, 03:136, 03:138, 03:143, 03:147-03:149, 03:153, 03:157, 03:159, 03:162, 03:164, 03:173-03:174, 03:179-03:181, 03:183-03:184:02, 03:186:01-03:186:02, 03:191, 03:193, 03:208N, 03:210-03:213, 03:215, 03:218-03:219, 03:232-03:236, 03:239-03:240, 03:244Q, 03:246-03:247, 03:250, 03:252, 03:255-03:257, 03:259, 03:261, 03:263, 03:265N-03:266, 03:269-03:270, 03:277N, 03:279, 03:281-03:283, 03:287, 03:294, 03:303, 03:305-03:306, 03:309-03:311, 03:313-03:314, 03:318N, 03:322-03:323N			3	4																		21	22	
*03:04:07, 03:41:01-03:41:02, 03:80, 03:113:01-03:113:02, 03:260, 03:286, 03:293			3																			21	22	
*03:04:25, 03:145, 03:172			3	4																		21		
*03:07, 03:45, 03:163, 03:297			3	4					11			15										21	22	
*03:10, 03:29, 03:209			3	4								15										21	22	
*03:13:01-03:13:02, 03:97, 03:132, 03:167, 03:177, 03:271				4	5																		22	
*03:15			3	4					11			15											22	
*03:58					5							15									20	21	22	
*03:67, 03:295				4	w																	21	w	
*03:81, 03:175				4	5								17									21	22	
*03:86					5							15										20	22	
*03:94			3									15										20	22	
*03:99												15										20		
*03:130, 03:140			3	4					11													21	22	
*03:151, 03:165, 03:251, 03:267					5																	21	22	
*03:155			3	4									16									21	22	
*03:199, 03:245			3	4										17									22	
*03:231				4		6			11			16										21		
*03:243, 03:308				4	5				11													21	22	
*03:248			3	4					11															
*03:262				4	5																	?	22	
*03:268				4	5				11			15										21	22	
*03:278			w	4																		21	w	
*03:280			3	4					10													21	22	
*03:302			3	4								15											22	
*03:317			3	4									17									21	22	
*03:324				4	5																		21	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24

Negative Control



*06:02:01-01-06:02:01:03, 06:02:03-06:02:11, 06:02:13-06:02:15, 06:02:17-06:02:37, 06:02:40-06:02:42, 06:02:44-06:02:45, 06:07, 06:10, 06:12-06:13, 06:15-06:16N, 06:18-06:30, 06:33-06:34:02, 06:36-06:39, 06:41-06:51, 06:53:01-06:66, 06:68-06:71, 06:73-06:75, 06:78, 06:80, 06:82-06:88, 06:90-06:93, 06:96-06:98, 06:100, 06:102-06:117, 06:119-06:121, 06:125-06:126, 06:128N-06:131, 06:133-06:135, 06:137-06:142, 06:145-06:146, 06:148-06:152N, 06:154-06:179, 06:181-06:187								8			11			14											23
*06:02:12, 06:11, 06:52, 06:122, 06:124, 06:147								8						14											23
*06:02:16, 06:02:43, 06:04:01-06:04:02, 06:40, 06:79N, 06:81, 06:101, 06:123, 06:127:01:01-06:127:02, 06:136, 06:153								8			11														23
*06:02:38								8			11	12		14											23
*06:02:39, 06:32, 06:94-06:95, 06:99, 06:180								8			11			14											23
*06:03:01, 06:132:01	3							8			11			14	15										22
*06:03:02, 06:76:01								8			11			14	15										
*06:05							7	8			11						18		20						23
*06:06								8			11								20						23
*06:08	2							8			11			14											23
*06:09								8			11			14									21		23
*06:14, 06:35, 06:72, 06:143-06:144								8			11											21			23
*06:17											11			14							20				23
*06:31								8			11			14			18		20						23
*06:67							7	8			11			14											23
*06:76:02											11	12	13	14	15		18								
*06:77								8			11			14											
*06:89								8			11			14			17								23
*06:118								8			11			14			18	19							23
*06:132:02	3										11			14	15										22
*07:01:01:01-07:01:40, 07:01:42-07:02:07, 07:02:10-07:02:28, 07:02:30-07:02:40, 07:02:42-07:06, 07:08, 07:11-07:25, 07:27:01-07:27:02, 07:29:01-07:30, 07:32N, 07:35-07:38:02, 07:42, 07:44-07:48, 07:50-07:52, 07:54-07:63, 07:65-07:75, 07:77-07:91, 07:93-07:95, 07:97-07:100, 07:102-07:122, 07:124, 07:126-07:138, 07:140-07:147, 07:149-07:151, 07:153, 07:155, 07:157-07:160, 07:162-07:172:02, 07:174-07:176, 07:178-07:183, 07:185-07:195, 07:197-07:209, 07:211-07:215, 07:217-07:222, 07:224-07:237, 07:239-07:241, 07:243-07:245, 07:248-07:266, 07:268-07:293, 07:297-07:313, 07:318-07:321, 07:323-07:327, 07:329N-07:334, 07:336-07:337, 07:339-07:350N, 07:352-07:355, 07:357-07:366, 07:368:01-07:401, 07:404, 07:407-07:414, 07:417-07:448, 07:450-07:492, 07:494Q-07:513Q, 07:515-07:516, 07:518-07:530																									23
*07:01:41, 07:02:41, 07:223, 07:415-07:416											9														w
*07:02:08, 07:02:29, 07:26:01-07:26:02, 07:33N, 07:92, 07:139, 07:152N, 07:156, 07:246:01-07:246:02, 07:267, 07:296, 07:314:01, 07:317, 07:322, 07:351, 07:405, 07:449, 07:493, 07:517											9														
*07:02:09							6				9			13			18								23
*07:07, 07:09, 07:49, 07:76:01-07:76:02, 07:247, 07:315, 07:328, 07:403, 07:406											9	11								20					23
*07:10, 07:28, 07:41, 07:43, 07:184, 07:196, 07:367											9												21		23
*07:31:01-07:31:02, 07:514											9			14			18								23
*07:39-07:40, 07:335											9									20					
*07:53, 07:216											9	11													23
*07:64, 07:402											9					16									
*07:96:01-07:96:02, 07:314:02							3				9														22
*07:101, 07:148, 07:161							2				9	10													23
*07:123, 07:173, 07:294											9						17								23
*07:125, 07:356							6				9	11		13	14										23
*07:154											9						18								23
*07:177											9			14			18			20					
*07:210, 07:238											9	11								20					
*07:242							4				9														23
*07:316, 07:338	1										9														23
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	

Negative Control

*08:01:01-08:01:17, 08:01:19, 08:03:01-08:03:04, 08:06, 08:08:01-08:08:02, 08:14, 08:16:01-08:16:02, 08:20, 08:22, 08:24, 08:26N, 08:28, 08:36N, 08:38, 08:40-08:42, 08:44, 08:46, 08:50, 08:56-08:61, 08:65, 08:72:01-08:72:02, 08:78-08:82, 08:84-08:89N, 08:91, 08:96-08:99, 08:101-08:102, 08:105-08:106, 08:109, 08:117-08:119, 08:121N-08:122, 08:124, 08:127N-08:131, 08:133, 08:135-08:136										10						18			21					
*08:01:18, 08:02:01-01-08:02:14, 08:04:01-08:04:03, 08:13, 08:15:01-08:15:02, 08:17-08:19, 08:23, 08:27, 08:29-08:30, 08:32-08:35, 08:37, 08:39, 08:43, 08:45, 08:49, 08:51-08:55N, 08:62-08:63, 08:66-08:71, 08:73-08:77, 08:90, 08:92-08:94, 08:100, 08:103, 08:107, 08:110-08:112, 08:114-08:116, 08:120, 08:123, 08:125-08:126, 08:132, 08:134										10									21					
*08:05, 08:25										10		13							21					
*08:07, 08:104										10						19			21					
*08:09, 08:11, 08:83										10						18								
*08:10						7				10	11			15		18			21					
*08:12										10									20	21				
*08:21										10		13				18			21					
*08:31		2								10									21					
*08:47										10						18	19		21					
*08:48, 08:108										10														
*08:95										10						16	18		21					
*08:113							8			10						17								
*12:02:01-12:02:10, 12:02:12-12:02:13, 12:10:01-12:10:02, 12:17-12:18:01, 12:20, 12:22, 12:27, 12:30, 12:36, 12:40, 12:56, 12:64, 12:67-12:69, 12:73-12:74, 12:80N, 12:83-12:86, 12:96, 12:103-12:106, 12:112, 12:114, 12:117-12:118, 12:124, 12:126, 12:128, 12:130, 12:134, 12:136-12:137, 12:142, 12:145, 12:148N, 12:151, 12:153, 12:155Q, 12:161-12:162, 12:164, 12:166, 12:168, 12:175, 12:177, 12:179, 12:183												12	13											
*12:02:11							8					12												
*12:02:14, 12:18:02, B*67:02 <sup>3</sup>													13											
*12:03:01:01-12:03:01:04, 12:03:03, 12:03:05-12:03:07, 12:03:10-12:03:12, 12:03:14-12:03:15, 12:03:17-12:03:19, 12:03:21-12:03:22, 12:03:25, 12:03:27-12:03:29, 12:03:31-12:03:33, 12:03:35-12:03:38, 12:06-12:07, 12:11-12:13, 12:19, 12:23, 12:25, 12:29, 12:32, 12:34-12:35, 12:37-12:39N, 12:42Q-12:43, 12:45-12:48, 12:50-12:53, 12:57, 12:61-12:63, 12:65-12:66, 12:70-12:71, 12:75-12:79, 12:82, 12:87-12:95, 12:97, 12:99-12:102, 12:107-12:111, 12:115-12:116, 12:120-12:122, 12:125, 12:129, 12:131, 12:133, 12:138-12:141, 12:143, 12:149-12:150, 12:152, 12:156-12:160, 12:163, 12:165, 12:167, 12:170-12:174, 12:180, 12:182, 12:184, 12:186												12	13	14										
*12:03:02, 12:03:08, 12:03:16, 12:03:20, 12:03:30, 12:26, 12:59, 12:185																13	14							
*12:03:04, 12:98												12	14											
*12:03:09							8					12	14											
*12:03:13, 12:03:24												12	w	14										
*12:03:23			3									12	13	14										
*12:03:26							8					12	13	14										
*12:03:34:01-12:03:34:02																								
*12:04:01											11				14	15								
*12:04:02, 12:60											11	12	13	14	15									
*12:05											11	12			14	15								
*12:08												12	13				17							
*12:09											11				15				20					
*12:14:01													13				18	19						
*12:14:02												12	13				18	19						
*12:15, 12:113							8					12	13	14			17							
*12:16												12	13										23	
*12:21							7				11	12			15									
*12:24												12	13						20					
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
*12:28, 12:58												12	13	14			18							
*12:31												12	13	14							21			

Negative Control



*15:102			3							11				15	17			20						
*15:104	1		3					w		11				15	17					21				
*15:107							7	8		11				15	17					21				
*15:111		2	3					w		11				15	17					21				
*16:01:01-16:01:15, 16:01:18-16:01:22, 16:06-16:08, 16:11, 16:13-16:15:01, 16:16Q-16:17, 16:21-16:24, 16:26-16:28, 16:30N-16:32, 16:36, 16:38-16:39:02, 16:41, 16:44, 16:49, 16:51-16:52, 16:54, 16:56-16:59, 16:62, 16:65, 16:71-16:73, 16:75-16:76, 16:79-16:81, 16:83, 16:85-16:86, 16:92-16:98										11		13				18						23		
*16:01:16								8		11		13				18						23		
*16:01:17, 16:15:02										11	12	13				18						23		
*16:02:01-16:02:12, 16:09, 16:12, 16:18-16:19, 16:25, 16:37, 16:46-16:47, 16:60, 16:63, 16:69, 16:74, 16:77N, 16:84, 16:89N-16:90										11		13		15		18						23		
*16:02:13										11	12	13		15		18						23		
*16:04:01, 16:04:03, 16:33, 16:42, 16:55, 16:61, 16:66, 16:78, 16:82													14			18		20						
*16:10, 16:67												13				18						23		
*16:20								8		11		13			17	18						23		
*16:29		2											14			18		20						
*16:34			3				6			11		13				18						23		
*16:35													14			18								
*16:40								7							16									
*16:43										11		13										23		
*16:45												13				18			21			23		
*16:48										11			14	15		18								
*16:50		2								11		13				18						23		
*16:53								7							16			20						
*16:64, 16:87										11		13			17	18						23		
*16:68																18		20						
*16:70			3							11		13		15		17	18					23		
*16:88										11				15		18		20						
*16:91										11			14	15		18		20						
*17:01:01-17:01:03, 17:01:05-17:04, 17:06, 17:08-17:16, 17:18-17:19, 17:21, 17:24-17:32		2								11				15			19		21					
*17:01:04		2								11		13		15			19		21					
*17:05		2						7		11				15			19		21					
*17:07, 17:17, 17:23, 17:33										11				15			19		21					
*17:20		2								11							19		21			23		
*17:22		2															19		21					
*18:01-18:02, 18:04-18:08, 18:10										11								20	21			23		
*18:03		2								11								20	21	22		23		
*18:09										11				15				20	21					
A*24:64, A*24:96, A*24:106, B*08:16, B*08:123, B*08:145																						23		
A*24:174										11												23		
B*07:239, B*14:03																		20						
B*13:13:02																					22			
B*15:277, B*35:222, B*42:22														15										
B*39:76, B*54:18		1																						
B*58:02:01																						8		
B*58:74			3																					
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24

Negative Control