

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
HLA-C allele ^{1,2}																								
*01:02:01:01-01:02:17, 01:02:19-01:02:55, 01:05-01:08, 01:11-01:13, 01:15:01-01:20, 01:22-01:23, 01:25-01:33, 01:35-01:42, 01:44-01:48, 01:51-01:54, 01:56N-01:58, 01:61-01:66, 01:68-01:77, 01:79:01-01:89N, 01:91-01:100, 01:103-01:112, 01:114:01-01:115, 01:117N, 01:119-01:121Q, 01:123-01:124, 01:126-01:130, 01:132-01:135, 01:137N-01:144, 01:146:01-01:155, 01:157-01:159, 01:161-01:167, 01:170-01:175	1																			20				
*01:02:18	1											12									20			
*01:03, 01:24, 01:34, 01:49:01-01:50, 01:55, 01:78, 01:176	1																			20	21			
*01:04	1											12		14						20				
*01:09	1													14						20				
*01:10	1	2																		20				
*01:14, 01:59, 01:118	1										11									20				
*01:21	1											12	13											
*01:43	1	2									10									20				
*01:60	1																		19	20				
*01:67, 01:122	1			4												16				20				
*01:90, 01:101-01:102, 01:113	1							8									17			20				
*01:116	1																17			20				
*01:125, 14:58 ³	1															16				20				
*01:131	1			4																20	21			
*01:136, 01:145N	1							8												20				
*01:156, B*07:239, B*14:03 ³	1																			20				
*01:160	1													14				18		20				
*01:168-01:169:01																15				20				
*01:169:02, 03:94 ³				3												15				20			22	
*01:177																16				20			22	
*02:02:01-02:02:03, 02:02:06-02:02:12, 02:02:14-02:02:25, 02:02:27-02:02:28, 02:02:31-02:02:37, 02:02:39, 02:02:41-02:02:43, 02:02:45-02:03, 02:07-02:11, 02:15-02:16:02, 02:19-02:20, 02:24-02:25Q, 02:28-02:32, 02:34-02:35, 02:37-02:38:02N, 02:42-02:44, 02:46, 02:48, 02:50-02:54, 02:56-02:57, 02:59-02:61, 02:63-02:64, 02:66-02:67Q, 02:69-02:74, 02:76-02:86, 02:88-02:93, 02:95-02:100, 02:102-02:106, 02:108-02:114, 02:116-02:117, 02:119-02:130, 02:132-02:133, 02:135N-02:151, 02:153-02:163, 02:165N-02:166, 02:168-02:169N		2	3			6					11				15									
*02:02:13, 02:02:29		2	3								11				15							22		
*02:02:26, 02:21, 02:26:02-02:26:03, 02:39, 02:45		2	3								11				15									
*02:02:30, 02:02:40		2				6					11				15									
*02:02:38		2	3			6					11				15							22		
*02:02:44		2	3			6					11	12			15									
*02:04		2	w			6					11				15									
*02:05:01-02:05:03, 02:17		2	3			6					11			14	15									
*02:06:01-02:06:02, 02:23, 02:36:01-02:36:02		2	3			6		8			11				15		17							
*02:12, 02:55:01-02:55:02		2	3			6					11		13		15									
*02:13, 02:18, 02:33, 02:75		2	3			6					11				15			18						
*02:14:01		2	w			6					11													
*02:14:02, 02:164		2				6					11													
*02:22, 02:62, 02:118		2	3			6					11				15						21			
*02:26:01, 02:134, 02:152, 02:167		2									11				15									
*02:27:01-02:27:02, 02:65, 02:131		2	3			6					11													
*02:40:01-02:40:02		2	3			6					11				15				19					
*02:47		2	3			6		8			11				15									
*02:49		2	3			6					11		13		15			18						
*02:58			3			6					11				15									
*02:68		2	3			6					11				15		17							
*02:87		2									10													
*02:94		2	3			6	7				11				15									
*02:101		2	3			w					11				15									
*02:107		2									11													
*02:115		2	3			6					11		13					18						
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24

Negative Control

*03:07:01:01-03:07:02, 03:45, 03:163, 03:297			3	4							11				15					21	22			
*03:10, 03:29, 03:209			3	4											15					21	22			
*03:13:01:01-03:13:02, 03:97, 03:132, 03:167, 03:177, 03:271				4	5																22			
*03:15			3	4							11				15						22			
*03:58					5										15				20	21	22			
*03:67, 03:295				4	w															21	w			
*03:81, 03:175				4	5											17				21	22			
*03:86					5										15				20		22			
*03:130, 03:140			3	4							11									21	22			
*03:155			3	4												16				21	22			
*03:199, 03:245			3	4												17					22			
*03:231				4		6					11					16					21			
*03:243, 03:308				4	5						11										21	22		
*03:248			3	4							11													
*03:262				4	5																?	22		
*03:268, 03:461				4	5						11				15						21	22		
*03:278			w	4																	21	w		
*03:280			3	4						10											21	22		
*03:302			3	4												15						22		
*03:317, 03:388			3	4													17				21	22		
*03:324				4	5																21			
*03:351				4	5											16					21	22		
*03:416	1			4	5																21	22		
*03:450				4	5											15					21	22		
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
*04:01:01:01-04:01:01:29, 04:01:01:31-04:01:15, 04:01:17-04:01:22, 04:01:24-04:01:27, 04:01:29-04:01:60, 04:01:62, 04:01:64, 04:01:66-04:01:73, 04:01:75-04:01:113, 04:04:01:01-04:05, 04:07:01, 04:08-04:09N, 04:12-04:13:01:02, 04:15:01-04:15:03, 04:17-04:20, 04:24-04:28, 04:30-04:31, 04:34-04:35, 04:38-04:41, 04:43-04:53, 04:56-04:57, 04:59Q-04:67, 04:69-04:70, 04:73-04:76, 04:78-04:79, 04:81-04:102, 04:104-04:106, 04:109-04:111, 04:113, 04:115N, 04:117-04:128, 04:131-04:139, 04:141-04:146, 04:148-04:152, 04:154-04:159, 04:161-04:166:01, 04:167-04:168, 04:170N, 04:173N-04:177, 04:179-04:189, 04:191N, 04:193, 04:195, 04:197-04:201:01, 04:202-04:209, 04:211, 04:216-04:219, 04:221-04:226, 04:228-04:229, 04:232-04:235, 04:237-04:248, 04:250-04:255N, 04:257-04:262, 04:264, 04:266-04:277, 04:280-04:282, 04:284-04:285, 04:287-04:293, 04:295-04:298, 04:300N-04:303, 04:305N-04:310, 04:312-04:334, 04:336, 04:338-04:340, 04:342-04:346, 04:348-04:350N, 04:352-04:356, 04:358-04:359, 04:361-04:362N, 04:364N-04:366, 04:369N-04:375						6				11		13					18			21				
*04:01:16, 04:23, 04:54:01-04:54:02, 04:72, 04:341, 04:367						6					11		13					18						
*04:01:23, 04:33, 04:116, 04:130, 04:166:02, 04:231, 04:283						6					11		13								21			
*04:01:28, 04:01:61, 04:01:63, 04:01:65, 04:140, 04:196, 04:213, 04:236N, 04:304, 04:311											11		13					18			21			
*04:01:74						6					11		13				w				21			
*04:03:01:01-04:03:05, 04:06:01-04:06:02, 04:80, 04:147, 04:171, 04:256, 04:286, 04:294, 04:337, 04:363			3			6					11		13		15			18			21			
*04:07:02											11										21	22		
*04:10, 04:153, 04:192, 04:194, 04:210, 04:214-04:215N, 04:227, 04:249, 04:278, 04:347						6					11							18			21			
*04:11						6					11					16					21			
*04:14, 04:68						6					11		13					18			21		23	
*04:16, 04:351:01-04:351:02			3								11		13		15			18			21			
*04:29, 04:114						6					11		13			16		18			21			
*04:32, 04:77, 04:265			2			6					11		13					18			21	22		
*04:36, 04:55						6					11					16		18			21			
*04:37						6					11		13		15			18		20	21			
*04:42:01-04:42:02, 04:220											11		13	14				18			21		23	
*04:58, 04:368						6					11		13					18	19		21			
*04:71	1					6					11		13					18			21			
*04:103											11		13		15			18			21			
*04:107			3			6					11		13		15						21			

Negative Control

*06:09:01-06:09:02								8			11			14						21	23			
*06:14, 06:35, 06:72, 06:143-06:144								8			11									21	23			
*06:17											11			14					20		23			
*06:31								8			11			14			18		20		23			
*06:67							7	8			11			14							23			
*06:76:02											11	12	13	14	15		18							
*06:77								8			11			14										
*06:89, 06:224								8			11			14			17				23			
*06:118								8			11			14			18	19			23			
*06:132:02			3								11			14	15					22				
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
*07:01:01:01-07:01:40, 07:01:42-07:01:73, 07:01:75-07:02:07, 07:02:10-07:02:28, 07:02:30-07:02:40, 07:02:42-07:02:90, 07:02:92-07:06:02, 07:08, 07:11-07:25, 07:27:01-07:27:02, 07:29:01-07:30, 07:32N, 07:35-07:38:02, 07:42, 07:44-07:48, 07:50-07:52, 07:54-07:63, 07:65-07:75, 07:77-07:91, 07:93-07:95, 07:97-07:100, 07:102-07:122, 07:124, 07:126-07:138, 07:140-07:147, 07:149-07:151, 07:153, 07:155, 07:157-07:160, 07:162-07:172:02, 07:174-07:176, 07:178-07:183, 07:185-07:195, 07:197-07:209, 07:211-07:215, 07:217-07:222, 07:224-07:237, 07:239-07:241, 07:243-07:245, 07:248-07:266, 07:268-07:292, 07:297-07:313, 07:318-07:321, 07:323-07:327, 07:329N-07:334, 07:336-07:337, 07:339-07:350N, 07:352-07:355, 07:357-07:366, 07:368:01-07:401, 07:404, 07:407-07:414, 07:417-07:448, 07:450-07:492, 07:494Q-07:513Q, 07:515-07:516, 07:518, 07:520-07:530, 07:532-07:535, 07:537-07:562, 07:564-07:567, 07:569-07:577, 07:580-07:582Q, 07:584-07:597, 07:599-07:625, 07:627-07:713, 07:715-07:722, 07:726N-07:755, 07:757-07:774, 07:776N-07:779									9													23		
*07:01:41, 07:02:41, 07:223, 07:415-07:416									9															w
*07:01:74						6			9				13				17	18					23	
*07:02:08, 07:02:29, 07:02:91, 07:26:01-07:26:03, 07:33N, 07:92, 07:139, 07:152N, 07:156, 07:246:01-07:246:02, 07:267, 07:296, 07:314:01, 07:314:03, 07:317, 07:322, 07:351, 07:405, 07:449, 07:493, 07:517, 07:714, 07:724									9															
*07:02:09						6			9				13				18						23	
*07:07, 07:09, 07:49, 07:76:01-07:76:02, 07:247, 07:315, 07:328, 07:403, 07:406, 07:598									9	11									20				23	
*07:10, 07:28, 07:41, 07:43:01-07:43:02, 07:184, 07:196, 07:367, 07:536, 07:568									9											21			23	
*07:31:01-07:31:02, 07:514									9				14				18						23	
*07:39-07:40, 07:335									9										20					
*07:53, 07:216									9	11													23	
*07:64, 07:402									9						16							22		
*07:96:01-07:96:02, 07:314:02, 07:578, 07:756			3						9													22		
*07:101, 07:148, 07:161		2							9	10													23	
*07:123, 07:173, 07:294, 07:626, 07:725, 07:775									9								17						23	
*07:125, 07:356, 07:531						6			9	11		13	14										23	
*07:154									9									18					23	
*07:177									9				14				18		20					
*07:210, 07:238									9	11									20					
*07:242				4					9														23	
*07:293, 07:519									9						16								23	
*07:316, 07:338, 07:579	1								9														23	
*07:563									9										20				23	
*07:583		2				6			9	10			13				18							
*07:723						6			9				13											
*08:01:01:01-08:01:17, 08:01:19-08:01:28, 08:03:01-08:03:04, 08:06, 08:08:01-08:08:02, 08:14, 08:16:01-08:16:02, 08:20, 08:22, 08:24, 08:26N, 08:28, 08:36N, 08:38, 08:40-08:42, 08:44, 08:46, 08:50, 08:56-08:61, 08:65, 08:72:01-08:72:02, 08:78-08:82, 08:84-08:89N, 08:91, 08:96-08:99, 08:101-08:102, 08:105-08:106, 08:109, 08:117-08:119, 08:121N-08:122, 08:124, 08:127N-08:131, 08:133, 08:135-08:136, 08:141Q, 08:143-08:145, 08:147-08:149, 08:153-08:155, 08:157, 08:162, 08:164, 08:168, 08:173N-08:176, 08:178, 08:186-08:187									10									18			21			

Negative Control

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	
*08:01:18, 08:02:01:01-08:02:26, 08:04:01-08:04:03, 08:13, 08:15:01-08:15:02, 08:17-08:19:02, 08:23, 08:27, 08:29-08:30, 08:32-08:35, 08:37, 08:39, 08:43, 08:45, 08:49, 08:51-08:55N, 08:62:01-08:63, 08:66-08:71, 08:73, 08:75-08:77, 08:90, 08:92-08:94, 08:100, 08:103, 08:107, 08:110-08:112, 08:114-08:116, 08:120, 08:123, 08:125-08:126, 08:132, 08:134, 08:138-08:140, 08:146, 08:150-08:151, 08:156, 08:158-08:161N, 08:165-08:167, 08:169-08:172, 08:177, 08:179-08:182, 08:184										10											21				
*08:05, 08:25										10			13									21			
*08:07, 08:104, 08:142										10									19		21				
*08:09, 08:11, 08:83										10								18							
*08:10							7			10	11				15			18			21				
*08:12										10										20	21				
*08:21, 08:137										10			13					18				21			
*08:31		2								10												21			
*08:47, 08:188										10								18	19		21				
*08:48, 08:108, 08:152										10															
*08:74										10						16						21			
*08:95										10						16		18				21			
*08:113								8		10							17								
*08:163						6				10						16		18				21			
*08:183						6				10						16						21			
*08:185		2								10									19		21				
*12:02:01-12:02:10, 12:02:12-12:02:13, 12:02:16, 12:02:18, 12:02:20-12:02:27, 12:03:47, 12:10:01-12:10:02, 12:17-12:18:01, 12:20, 12:22, 12:27, 12:30, 12:36, 12:40, 12:56, 12:64, 12:67-12:69, 12:73-12:74, 12:80N, 12:83-12:86, 12:96, 12:103-12:106, 12:112, 12:114, 12:117-12:118, 12:124, 12:126, 12:128, 12:130, 12:134, 12:136-12:137, 12:142, 12:145, 12:148N, 12:151, 12:153, 12:155Q, 12:161-12:162, 12:164, 12:166, 12:168, 12:175, 12:177, 12:179, 12:183, 12:193, 12:196, 12:200, 12:204, 12:207, 12:212, 12:214, 12:219N, 12:221, 12:226, 12:228, 12:231, 12:234, 12:236N, 12:239-12:241, 12:243, 12:247, 12:250-12:251, 12:255, 12:258, 12:261, 12:263, 12:268, 12:270N, 12:275, 12:279-12:281													12	13											
*12:02:11								8				12													
*12:02:14-12:02:15, 12:02:19, 12:18:02, 12:222, 12:224, B*67:02 ³														13											
*12:02:17			3									12	13												
*12:02:28												12	w												
*12:02:29, 12:49, 12:123, B*57:01:24 ³												12													
*12:03:01:01-12:03:01:13, 12:03:03, 12:03:05-12:03:07, 12:03:10-12:03:12, 12:03:14-12:03:15, 12:03:17-12:03:19, 12:03:21-12:03:22, 12:03:25, 12:03:27-12:03:29, 12:03:31-12:03:33, 12:03:35-12:03:46, 12:03:48-12:03:56, 12:06-12:07, 12:11-12:13:01:02, 12:19, 12:23, 12:25, 12:29, 12:32, 12:34-12:35, 12:37-12:39N, 12:42Q-12:43, 12:45-12:48, 12:50-12:53, 12:57:01-12:57:02, 12:61-12:63, 12:65-12:66, 12:70-12:71, 12:75-12:79, 12:82, 12:87-12:91, 12:93-12:95, 12:97, 12:99:01-12:102, 12:107-12:111, 12:115-12:116, 12:120-12:122, 12:125, 12:129, 12:131, 12:133, 12:138-12:140:01, 12:141, 12:143, 12:149-12:150, 12:152, 12:156-12:160, 12:163, 12:165, 12:167, 12:170-12:174, 12:180, 12:182, 12:184, 12:186-12:187, 12:189-12:192, 12:197, 12:199, 12:201-12:202, 12:205-12:206, 12:209-12:211, 12:213, 12:216, 12:218, 12:220, 12:223, 12:225, 12:227, 12:229-12:230, 12:232N, 12:235, 12:237-12:238, 12:242, 12:244-12:246, 12:248-12:249, 12:253-12:254, 12:256-12:257, 12:259-12:260, 12:262, 12:264-12:267, 12:271-12:274N, 12:276-12:278												12	13	14											
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	
*12:03:02, 12:03:08, 12:03:16, 12:03:20, 12:03:30, 12:26, 12:59, 12:185													13	14											
*12:03:04, 12:98												12	14												
*12:03:09							8					12	14												

Negative Control

HLA-C low screening 2019 1K9

*16:29	2												14				18	20						
*16:34		3				6				11		13					18				23			
*16:35													14				18							
*16:40, 16:110							7									16								
*16:43										11		13										23		
*16:45												13					18			21		23		
*16:48										11			14	15			18							
*16:50		2								11		13					18					23		
*16:53							7									16								
*16:64, 16:87										11		13					17	18				23		
*16:68																	18	20						
*16:70			3							11		13		15			17	18				23		
*16:88										11				15			18	20						
*16:91										11			14	15			18	20						
*16:103										11				15			18							
*16:109								8					14				17	18	20					
*16:113							7			11		13				16						23		
*16:121			3				6			11		13		15			18					23		
*16:139									10	11		13					18					23		
*16:140										11				15										
*16:150							6						14		16		18	20						
*17:01:01-02-17:01:03, 17:01:05-17:04, 17:06, 17:08-17:16:02, 17:18-17:19, 17:21, 17:24-17:32, 17:34-17:46		2								11				15			19	21						
*17:01:04		2								11		13		15			19	21						
*17:07, 17:17, 17:23, 17:33										11				15			19	21						
*17:20		2								11							19	21	23					
*17:22		2															19	21						
*18:01:01-18:02:03, 18:04-18:08, 18:10-18:12										11								20	21	23				
*18:03		2								11							20	21	22	23				
*18:09										11				15			20	21						
A*24:64, A*24:96, A*24:106, B*08:16, B*08:123, B*08:145																						23		
A*24:174										11												23		
B*13:13:02, B*40:01:49-40:01:50																					22			
B*15:277, B*15:430, B*35:222, B*42:22, B*54:38														15										
B*39:76, B*54:18		1																						
B*44:02:44, B*56:01:09, B*58:74			3																					
B*58:02:01								8																
B*73:01																					w			
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24

Negative Control