

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
HLA-C allele <sup>1</sup>																								
*01:02:01-01:02:17, 01:02:19-01:02:23, 01:06-01:08, 01:11-01:13, 01:15-01:20, 01:23, 01:25-01:33, 01:37N-01:42, 01:44-01:48, 01:51-01:54, 01:56N-01:58, 01:61-01:66, 01:68-01:77, 01:80-01:84	1																			20				
*01:02:18	1											12								20				
*01:03, 01:24, 01:34, 01:78	1																			20	21			
*01:04	1											12	14											
*01:05, 01:22, 01:35-01:36, 01:79:01-01:79:02, B*39:76, B*54:18 <sup>2</sup>	1																							
*01:09	1												14											
*01:10	1	2																		20				
*01:14, 01:59	1											11								20				
*01:21	1											12	13											
*01:43	1	2										10								20				
*01:49-01:50, 01:55	1																					21		
*01:60	1																			19	20			
*01:67, 14:58 <sup>2</sup>	1															16				20				
*02:02:01		2	3			6					11				w									
*02:02:02-02:02:03, 02:02:05-02:02:12, 02:02:14-02:02:25, 02:03, 02:07-02:11, 02:15-02:16:02, 02:19-02:20, 02:24-02:25Q, 02:28-02:32, 02:34-02:35, 02:37-02:38N, 02:40, 02:42-02:44, 02:46, 02:48, 02:50-02:54, 02:56-02:57, 02:59-02:61, 02:63-02:64, 02:66-02:67Q, 02:70-02:72		2	3			6					11				15									
*02:02:13, 02:02:26, 02:21, 02:26:02-02:26:03, 02:39, 02:45		2	3								11				15									
*02:04		2	w			6					11				15									
*02:05:01-02:05:02, 02:17		2	3			6					11		14	15										
*02:06		2	3			6	8				11			15	17									
*02:12		2	3			6						13	15											
*02:13, 02:18, 02:33		2	3			6					11			15			18							
*02:14		2	w			6					11													
*02:22, 02:62		2	3			6					11			15							21			
*02:23, 02:36, 02:68		2	3			6					11			15	17									
*02:26:01		2									11			15										
*02:27:01-02:27:02		2	3			6																		
*02:47		2	3			6	8				11			15										
*02:49		2	3			6					11	13	15		18									
*02:55		2	3			6					11	13	15											
*02:58			3			6					11		15											
*02:65		2	3			6					11													
*03:02:01-03:02:09, 03:02:11, 03:05, 03:17, 03:25, 03:27, 03:33, 03:35-03:36, 03:60, 03:71, 03:84, 03:89, 03:95, 03:108, 03:121N, 03:135, 03:139, 03:146, 03:154, 03:169Q, 03:178, 03:190, 03:194, 03:197-03:198, 03:200-03:201N, 03:216			3	4																		22		
*03:02:10				4																				
*03:03:01-03:03:20, 03:03:22, 03:11:01-03:11:02, 03:20N-03:22Q, 03:30-03:31, 03:43:01-03:43:02, 03:49-03:50, 03:52-03:53, 03:55-03:56, 03:59, 03:61-03:62, 03:66, 03:68-03:69, 03:75-03:76, 03:79, 03:83, 03:85, 03:88, 03:96, 03:102-03:103, 03:112, 03:116:01-03:116:02, 03:119-03:120, 03:122, 03:124, 03:126-03:127, 03:133, 03:141-03:142, 03:150, 03:152, 03:158, 03:160-03:161, 03:168, 03:171, 03:176, 03:185, 03:187-03:189N, 03:192, 03:196, 03:202-03:207, 03:214, 03:217, 03:220				4	5																21	22		
*03:03:21, 03:04:26, 03:18, 03:39, 03:64:02, 03:137, 03:144, 03:156, 03:166, 03:170, 03:182, 03:195				4																		21		
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24

Negative Control



*05:42							7			11	13	15		18			21							
*05:46							7			11	13	15					21							
*05:58:01		3					7			11		15					21							
*05:78						6	7			11							21							
*06:02:01-01-06:02:01:03, 06:02:03-06:02:11, 06:02:13-06:02:15, 06:02:17-06:02:34, 06:07, 06:10, 06:12-06:13, 06:15-06:16N, 06:18-06:30, 06:33-06:34, 06:36-06:39, 06:41-06:51, 06:53:01-06:66, 06:68-06:71, 06:73-06:75, 06:78, 06:80, 06:82-06:88, 06:90-06:93, 06:96-06:98, 06:100, 06:102-06:117								8		11		14							23					
*06:02:12, 06:11, 06:52								8				14							23					
*06:02:16, 06:04, 06:40, 06:79N, 06:81, 06:101								8		11									23					
*06:03:01		3						8		11		14	15					22						
*06:03:02, 06:76:01								8		11		14	15											
*06:05							7	8		11				18		20			23					
*06:06								8		11						20			23					
*06:08		2						8		11		14							23					
*06:09								8		11		14						21	23					
*06:14, 06:35, 06:72								8		11								21	23					
*06:17										11		14					20		23					
*06:31								8		11		14		18		20			23					
*06:32, 06:94-06:95, 06:99										11		14							23					
*06:67							7	8		11		14							23					
*06:76:02										11	12	13	14	15		18								
*06:77								8		11		14												
*06:89								8		11		14			17				23					
*07:01:01-01-07:02:07, 07:02:10-07:02:28, 07:02:30-07:02:40, 07:02:42-07:06, 07:08, 07:11-07:25, 07:27:01-07:27:02, 07:29-07:30, 07:32N, 07:35-07:38:02, 07:42, 07:44-07:48, 07:50-07:63, 07:65-07:75, 07:77-07:91, 07:93-07:95, 07:97-07:100, 07:102-07:122, 07:124, 07:126-07:138, 07:140-07:147, 07:149-07:151, 07:153-07:155, 07:157-07:160, 07:162-07:172:01, 07:174-07:176, 07:178-07:183, 07:185-07:195, 07:197-07:209, 07:211-07:222, 07:224-07:237, 07:239-07:241, 07:243-07:245, 07:248-07:266, 07:268-07:293, 07:297-07:313, 07:318-07:321, 07:323-07:327, 07:329N-07:334, 07:336-07:337									9										23					
*07:02:08, 07:02:29, 07:26, 07:33N, 07:39-07:40, 07:92, 07:139, 07:152N, 07:156, 07:246, 07:267, 07:296, 07:314, 07:317, 07:322, 07:335										9														
*07:02:09							6			9		13			18				23					
*07:02:41, 07:223										9									w					
*07:07, 07:09, 07:49, 07:76, 07:247, 07:315, 07:328										9	11					20			23					
*07:10, 07:28, 07:41, 07:43, 07:184, 07:196										9								21	23					
*07:31:01-07:31:02										9		14			18				23					
*07:64										9				16										
*07:96:01-07:96:02			3							9									22					
*07:101, 07:148, 07:161		2								9	10								23					
*07:123, 07:173, 07:294										9					17				23					
*07:125							6			9		13	14						23					
*07:177										9		14			18									
*07:210, 07:238										9	11					20								
*07:242							4			9									23					
*07:316		1								9									23					
*08:01:01-08:01:12, 08:03:01-08:03:02, 08:06, 08:08:01, 08:14, 08:16:01-08:16:02, 08:20, 08:22, 08:24, 08:26N, 08:28, 08:36N, 08:38, 08:40-08:42, 08:44, 08:46, 08:50, 08:56-08:61, 08:65, 08:72:01-08:72:02, 08:78-08:82, 08:84-08:88N, 08:91										10					18			21						
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
*08:02:01-08:02:10, 08:04:01-08:04:03, 08:13, 08:15:01-08:15:02, 08:17-08:19, 08:23, 08:27, 08:29-08:30, 08:32-08:35, 08:37, 08:39, 08:43, 08:45, 08:49, 08:51-08:55N, 08:62-08:63, 08:66-08:71, 08:73-08:77, 08:89N-08:90, 08:92-08:94										10											21			
*08:05, 08:25										10		13									21			
*08:07, 08:47										10								19			21			
*08:09, 08:11, 08:83										10								18						
*08:10							7			10	11			15			18				21			

Negative Control

*08:12										10								20	21					
*08:21										10		13					18		21					
*08:31		2								10									21					
*08:48										10														
*12:02:01-12:02:10, 12:10:01-12:10:02, 12:17-12:18:01, 12:20, 12:22, 12:27, 12:30, 12:36, 12:40, 12:56, 12:64, 12:67-12:69, 12:73-12:74, 12:80N, 12:83-12:86, 12:96, 12:103-12:106												12	13											
*12:03:01:01-12:03:01:02, 12:03:03, 12:03:05-12:03:07, 12:03:10-12:03:12, 12:03:14-12:03:15, 12:03:17-12:03:19, 12:03:21-12:03:22, 12:03:25, 12:03:27-12:03:28, 12:06-12:07, 12:11-12:13, 12:19, 12:23, 12:25, 12:29, 12:32, 12:34-12:35, 12:37-12:39N, 12:42Q-12:43, 12:45-12:48, 12:50-12:53, 12:57, 12:61-12:63, 12:65-12:66, 12:70-12:71, 12:75-12:79, 12:82, 12:87-12:95, 12:97, 12:99-12:102, 12:107-12:111												12	13	14										
*12:03:02, 12:03:08, 12:03:16, 12:03:20, 12:26, 12:59														13	14									
*12:03:04, 12:98												12	14											
*12:03:09							8					12	14											
*12:03:13, 12:03:24												12	w	14										
*12:03:23		3										12	13	14										
*12:03:26							8					12	13	14										
*12:04:01										11		14	15											
*12:04:02, 12:60										11	12	13	14	15										
*12:05										11	12	14	15											
*12:08											12	13				17								
*12:09										11			15					20						
*12:14:01												13				18	19							
*12:14:02											12	13				18	19							
*12:15							8				12	13	14		17									
*12:16											12	13									23			
*12:18:02, B*67:02 <sup>2</sup>												13												
*12:21						7				11	12		15											
*12:24											12	13						20						
*12:28, 12:58											12	13	14			18								
*12:31											12	13	14							21				
*12:33						7				11		14	15											
*12:41										11	12	13	15											
*12:44											12	13								21				
*12:49											12													
*12:54										11	12	13	14											
*12:55											12	13	14	16										
*12:72											12	13	15											
*12:81											12	13	14			17								
*14:02:01-14:02:07, 14:02:09-14:03, 14:05, 14:07N, 14:11, 14:13-14:14, 14:18-14:24:02, 14:26-14:27, 14:29-14:44, 14:46-14:48, 14:50-14:52, 14:55-14:57															16			20						
*14:02:08											12				16			20						
*14:04										11					16			20						
*14:06, 14:08-14:09, 14:28, 14:53, A*30:62 <sup>2</sup>															16									
*14:10															16		18	20						
*14:12, 14:49										11								20						
*14:15												14	16			18								
*14:16												14	16											
*14:17													16				19	20						
*14:25		2											16				20		22					
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
*14:45															15	16				20				
*14:54															16						21			
*15:02:01-15:02:09, 15:02:11-15:02:16, 15:02:18-15:02:19, 15:08, 15:10:02-15:10:03, 15:13, 15:15, 15:17-15:18, 15:26, 15:28, 15:31-15:35, 15:38-15:39, 15:41, 15:44-15:45, 15:47-15:53, 15:56-15:57, 15:60, 15:64, 15:67-15:68, 15:73-15:74, 15:78-15:82			3					8			11			15		17				21				
*15:02:10, 15:02:17			3					8			11			15		17				21	22			
*15:03, 15:16			3					8			11		13	15		17				21				
*15:04-15:05:08, 15:06:01-15:06:03, 15:09, 15:19, 15:24, 15:27, 15:29-15:30, 15:40, 15:46, 15:54, 15:59, 15:66, 15:69-15:70, 15:76-15:77			3								11			15		17				21				
*15:05:09, 15:61											11			15		17				21				

Negative Control











