

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
HLA-C allele <sup>1,2</sup>																								
*01:02:01:01-01:02:17, 01:02:19-01:02:45, 01:05-01:08, 01:11-01:13, 01:15-01-01:20, 01:22-01:23, 01:25-01:33, 01:35-01:42, 01:44-01:48, 01:51-01:54, 01:56N-01:58, 01:61-01:66, 01:68-01:77, 01:79-01-01:89N, 01:91-01:100, 01:103-01:112, 01:114-01:115, 01:117N, 01:119-01:121Q, 01:123-01:124, 01:126-01:130, 01:132-01:135, 01:137N-01:144, 01:146:01-01:153	1																			20				
*01:02:18	1											12									20			
*01:03, 01:24, 01:34, 01:49:01-01:50, 01:55, 01:78	1																			20	21			
*01:04	1											12		14						20				
*01:09	1													14						20				
*01:10	1	2																		20				
*01:14, 01:59, 01:118	1										11									20				
*01:21	1											12	13											
*01:43	1	2								10										20				
*01:60	1																		19	20				
*01:67, 01:122	1			4												16				20				
*01:90, 01:101-01:102, 01:113	1							8									17			20				
*01:116	1																17			20				
*01:125, 14:58 <sup>3</sup>	1															16				20				
*01:131	1			4																20	21			
*01:136, 01:145N	1							8												20				
*02:02:01-02:02:03, 02:02:05-02:02:12, 02:02:14-02:02:25, 02:02:27-02:02:28, 02:02:31-02:03, 02:07-02:11, 02:15-02:16:02, 02:19-02:20, 02:24-02:25Q, 02:28-02:32, 02:34-02:35, 02:37-02:38N, 02:42-02:44, 02:46, 02:48, 02:50-02:54, 02:56-02:57, 02:59-02:61, 02:63-02:64, 02:66-02:67Q, 02:69-02:74, 02:76-02:86, 02:88-02:93, 02:95-02:100, 02:102-02:106, 02:108-02:114, 02:116-02:117, 02:119-02:130, 02:132-02:133, 02:135N-02:139		2	3			6					11				15									
*02:02:13, 02:02:29		2	3								11				15						22			
*02:02:26, 02:21, 02:26:02-02:26:03, 02:39, 02:45		2	3								11				15									
*02:02:30		2				6					11				15									
*02:04		2	w			6					11				15									
*02:05:01-02:05:03, 02:17		2	3			6					11			14	15									
*02:06:01-02:06:02, 02:23, 02:36:01-02:36:02		2	3			6		8			11				15		17							
*02:12, 02:55:01-02:55:02		2	3			6					11		13		15									
*02:13, 02:18, 02:33, 02:75		2	3			6					11				15			18						
*02:14:01		2	w			6					11													
*02:14:02		2				6					11													
*02:22, 02:62, 02:118		2	3			6					11				15						21			
*02:26:01, 02:134		2									11				15									
*02:27:01-02:27:02, 02:65, 02:131		2	3			6					11													
*02:40:01-02:40:02		2	3			6					11				15					19				
*02:47		2	3			6		8			11				15									
*02:49		2	3			6					11		13		15			18						
*02:58			3			6					11				15									
*02:68		2	3			6					11				15		17							
*02:87		2									10													
*02:94		2	3			6	7				11				15									
*02:101		2	3			w					11				15									
*02:107		2									11													
*02:115		2	3			6					11		13					18						
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
*03:02:01-03:02:09, 03:02:11-03:02:12, 03:02:14-03:02:17, 03:05, 03:17:01-03:17:02, 03:25, 03:27, 03:33, 03:35-01-03:36, 03:60, 03:71, 03:84, 03:89, 03:95, 03:108, 03:121N, 03:135, 03:139, 03:146, 03:154, 03:169Q, 03:178, 03:190, 03:194, 03:197-03:198, 03:200-03:201N, 03:216, 03:221-03:222, 03:224N-03:226, 03:238, 03:249, 03:258, 03:292, 03:296:01-03:296:02, 03:298-03:301, 03:315, 03:326, 03:329-03:330, 03:335, 03:338, 03:349-03:350, 03:365, 03:371, 03:373, 03:386			3	4																		22		
*03:02:10, 03:264, B*44:274 <sup>3</sup>				4																				
*03:02:13, 03:16, 03:110, 03:332			3																				22	

Negative Control

*03:03:01:01-03:03:20, 03:03:22-03:03:40, 03:11:01-03:11:02, 03:18:02, 03:20N-03:22Q, 03:30-03:31, 03:43:01-03:43:02, 03:49-03:50, 03:52-03:53, 03:55-03:56, 03:59, 03:61-03:62, 03:66, 03:68-03:69, 03:75-03:76, 03:79, 03:83, 03:85, 03:88, 03:96, 03:102-03:103, 03:112, 03:116:01-03:116:02, 03:119:01-03:120, 03:122, 03:124, 03:126-03:127, 03:133, 03:141-03:142, 03:150, 03:152, 03:158, 03:160-03:161, 03:168, 03:171, 03:176, 03:185, 03:187-03:189N, 03:192, 03:196, 03:202-03:207, 03:214, 03:217, 03:220, 03:223, 03:227-03:230, 03:237, 03:241-03:242, 03:253-03:254, 03:272-03:276, 03:284-03:285, 03:288-03:291, 03:304, 03:307, 03:312, 03:316N, 03:319-03:321, 03:325, 03:327, 03:336, 03:339, 03:341, 03:345-03:346, 03:352, 03:356-03:357, 03:360, 03:363N-03:364, 03:367, 03:370, 03:372, 03:374-03:375, 03:377N-03:378, 03:380N, 03:383				4	5																21	22		
*03:03:21, 03:04:26, 03:18:01, 03:39, 03:64:02, 03:137, 03:144, 03:156, 03:166, 03:170, 03:182, 03:195, 03:344				4																	21			
*03:04:01:01-03:04:06, 03:04:08-03:04:24, 03:04:27-03:04:58, 03:06:01-03:06:02, 03:08-03:09, 03:14, 03:19, 03:23-03:24, 03:26, 03:28, 03:32, 03:34, 03:37:01-03:38:02, 03:40:01-03:40:04, 03:42, 03:44, 03:46-03:48, 03:51, 03:54, 03:57:01-03:57:02, 03:63-03:64:01, 03:65, 03:70, 03:72-03:74, 03:77-03:78, 03:80:02, 03:82, 03:87:01-03:87:02, 03:90-03:93, 03:98, 03:100-03:101, 03:104-03:107, 03:109, 03:111, 03:114-03:115, 03:117-03:118, 03:123, 03:125, 03:128-03:129, 03:131, 03:134, 03:136, 03:138, 03:143, 03:147-03:149, 03:153, 03:157, 03:159, 03:162, 03:164, 03:173-03:174, 03:179-03:181, 03:183-03:184:02, 03:186:01-03:186:02, 03:191, 03:193, 03:208N, 03:210-03:213, 03:215, 03:218-03:219, 03:232-03:236, 03:239-03:240, 03:244Q, 03:246-03:247, 03:250, 03:252, 03:255-03:257, 03:259, 03:261, 03:263:01-03:263:02, 03:265N-03:266, 03:269-03:270, 03:277N, 03:279, 03:281-03:283, 03:287, 03:294, 03:303, 03:305-03:306, 03:309-03:311, 03:313-03:314, 03:318N, 03:322-03:323N, 03:328, 03:331, 03:333-03:334, 03:337, 03:340, 03:343, 03:347-03:348, 03:353-03:355, 03:358-03:359, 03:361-03:362, 03:366N, 03:368-03:369, 03:376, 03:379, 03:381-03:382, 03:385, 03:387				3	4															21	22			
*03:04:07, 03:41:01-03:41:02, 03:80:01, 03:113:01-03:113:02, 03:260, 03:286, 03:293, 03:384				3																	21	22		
*03:04:25, 03:145, 03:172, 03:342				3	4																21			
*03:07:01-03:07:02, 03:45, 03:163, 03:297				3	4					11				15							21	22		
*03:10, 03:29, 03:209				3	4									15							21	22		
*03:13:01-03:13:02, 03:97, 03:132, 03:167, 03:177, 03:271				4	5																	22		
*03:15				3	4					11				15								22		
*03:58					5									15							20	21	22	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
*03:67, 03:295				4	w																21	w		
*03:81, 03:175				4	5												17				21	22		
*03:86					5										15					20	22			
*03:94, 03:99:02				3											15					20	22			
*03:99:01															15					20				
*03:130, 03:140				3	4						11										21	22		
*03:151, 03:165:01-03:165:02, 03:251, 03:267					5																21	22		
*03:155				3	4											16					21	22		
*03:199, 03:245				3	4												17				22			
*03:231				4		6					11					16					21			
*03:243, 03:308				4	5						11										21	22		
*03:248				3	4						11													
*03:262				4	5																?	22		
*03:268				4	5						11				15						21	22		
*03:278				w	4																21	w		
*03:280				3	4					10											21	22		
*03:302				3	4										15							22		
*03:317, 03:388				3	4												17				21	22		
*03:324				4	5																21			
*03:351				4	5											16					21	22		

Negative Control

*04:01:01:01-04:01:15, 04:01:17-04:01:22, 04:01:24-04:01:27, 04:01:29-04:01:60, 04:01:62, 04:01:64, 04:01:66-04:01:73, 04:01:75-04:01:91, 04:04:01:01-04:05, 04:07-04:09N, 04:12-04:13, 04:15-01-04:15:03, 04:17-04:20, 04:24-04:28, 04:30-04:31, 04:34-04:35, 04:38-04:41, 04:43-04:53, 04:56-04:57, 04:59Q-04:67, 04:69-04:70, 04:73-04:76, 04:78-04:79, 04:81-04:102, 04:104-04:106, 04:109-04:111, 04:113, 04:115N, 04:117-04:128, 04:131-04:139, 04:141-04:146, 04:148-04:152, 04:154-04:159, 04:161-04:168, 04:170N, 04:173N-04:177, 04:179-04:189, 04:191N, 04:193, 04:195, 04:197-04:201:01, 04:202-04:209, 04:211, 04:216-04:219, 04:221-04:226, 04:228-04:229, 04:232-04:235, 04:237-04:248, 04:250-04:255N, 04:257-04:262, 04:264, 04:266-04:277, 04:280-04:282, 04:284-04:285, 04:287-04:293						6					11	13					18		21					
*04:01:16, 04:23, 04:54, 04:72						6					11	13					18							
*04:01:23, 04:33, 04:116, 04:130, 04:231, 04:283						6					11	13							21					
*04:01:28, 04:01:61, 04:01:63, 04:01:65, 04:140, 04:196, 04:213, 04:236N											11	13					18		21					
*04:01:74						6					11	13					w		21					
*04:03:01-04:03:04, 04:06:01-04:06:02, 04:80, 04:147, 04:171, 04:256, 04:286		3				6					11	13	15				18		21					
*04:10, 04:153, 04:192, 04:194, 04:210, 04:214-04:215N, 04:227, 04:249, 04:278						6					11						18		21					
*04:11						6					11			16					21					
*04:14, 04:68						6					11	13					18		21	23				
*04:16		3									11	13	15				18		21					
*04:29, 04:114						6					11	13		16			18		21					
*04:32, 04:77, 04:265		2				6					11	13					18		21	22				
*04:36, 04:55						6					11			16			18		21					
*04:37						6					11	13	15				18	20	21					
*04:42:01-04:42:02, 04:220											11	13	14				18		21	23				
*04:58						6					11	13					18	19	21					
*04:71		1				6					11	13					18		21					
*04:103											11	13	15				18		21					
*04:107			3			6					11	13	15						21					
*04:108						6		w			11	13					17	18						
*04:112						6					11	13					17	18		21				
*04:129						6	7				11	13					18		21					
*04:160			3			6					11	13	15				18	19	21					
*04:169						6					11						17		21					
*04:172						6					11	13		16					21					
*04:178						6		8			11	13					17	18	21					
*04:190						6					11	13	15				18		21					
*04:201:02, 04:212				4		6					11	13					18		21					
*04:230, 04:263						6					11	13					18	20	21					
*04:279N						6					11	w					18		21					
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
*05:01:01:01-05:01:38, 05:03, 05:05:01-05:15, 05:18:01-05:19, 05:21, 05:24, 05:27-05:35, 05:37-05:41, 05:44:01-05:45, 05:47-05:54, 05:56-05:57, 05:58:02, 05:59-05:61, 05:63-05:67, 05:69-05:75, 05:77, 05:80-05:84, 05:86-05:102, 05:104, 05:106:01-05:106:02, 05:108-05:111, 05:114, 05:116-05:122, 05:124-05:128N, 05:130-05:131, 05:133, 05:136-05:142, 05:144-05:145, 05:148-05:150, 05:152-05:155, 05:157-05:162, 05:164							7				11			15							21			
*05:04:01-05:04:02, 05:22:01-05:22:02, 05:103:01-05:103:02, 05:123, 05:135, 05:147							7				11			15										
*05:16, 05:85							7				11			15					20	21				
*05:17, 05:25, 05:55, 05:68, 05:76, 05:79, 05:115							7				11			15			18		21					
*05:20, 05:43, 05:113N, 05:156							7				11								21					
*05:23, 05:62, 05:146, 05:151, 05:163							7				11			15				19	21					
*05:26						6	7				11			15					21					
*05:36							7				11			15		17			21					
*05:42							7				11	13	15				18		21					
*05:46							7				11	13	15						21					
*05:58:01, 05:58:03			3				7				11			15					21					
*05:78:01-05:78:02						6	7				11								21					
*05:105			2				7				11			15					21					
*05:107							7				11			15					20					

*05:112								7													21	23		
*05:129								7														21		
*05:132								7	8															
*05:134, 05:143								7														21		
*06:02:01:01-06:02:01:12, 06:02:03-06:02:11, 06:02:13-06:02:15, 06:02:17-06:02:37, 06:02:40-06:02:42, 06:02:44-06:02:52, 06:07, 06:10, 06:12-06:13, 06:15-06:16N, 06:18-06:30, 06:33-06:34:02, 06:36-06:39, 06:41-06:51, 06:53:01-06:66, 06:68-06:71, 06:73-06:75, 06:78, 06:80, 06:82-06:88, 06:90-06:93, 06:96-06:98, 06:100, 06:102:01-06:117, 06:119-06:121, 06:125-06:126, 06:128N-06:131, 06:133-06:135, 06:137-06:142, 06:145-06:146, 06:148-06:152N, 06:154-06:179, 06:181-06:196, 06:198-06:202, 06:205-06:216, 06:218								8														23		
*06:02:12, 06:11, 06:52, 06:122, 06:124, 06:147, 06:217								8															23	
*06:02:16, 06:02:43, 06:04:01-06:04:02, 06:40, 06:79N, 06:81, 06:101, 06:123, 06:127:01:01-06:127:02, 06:136, 06:153, 06:197, 06:203								8															23	
*06:02:38								8															23	
*06:02:39, 06:32, 06:94-06:95, 06:99, 06:180								8															23	
*06:03:01, 06:132:01			3					8														22		
*06:03:02, 06:76:01								8																
*06:05								7	8													18	20	
*06:06, 06:204								8														20	23	
*06:08			2					8															23	
*06:09								8														21	23	
*06:14, 06:35, 06:72, 06:143-06:144								8														21	23	
*06:17																						20	23	
*06:31								8														18	20	
*06:67								7	8														23	
*06:76:02																						11	12	
*06:77								8																
*06:89								8														17	23	
*06:118								8														18	19	
*06:132:02			3																			11	14	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
*07:01:01:01-07:01:40, 07:01:42-07:02:07, 07:02:10-07:02:28, 07:02:30-07:02:40, 07:02:42-07:06:01:02, 07:08, 07:11-07:25, 07:27:01-07:27:02, 07:29:01-07:30, 07:32N, 07:35-07:38:02, 07:42, 07:44-07:48, 07:50-07:52, 07:54-07:63, 07:65-07:75, 07:77-07:91, 07:93-07:95, 07:97-07:100, 07:102-07:122, 07:124, 07:126-07:138, 07:140-07:147, 07:149-07:151, 07:153, 07:155, 07:157-07:160, 07:162-07:172:02, 07:174-07:176, 07:178-07:183, 07:185-07:195, 07:197-07:209, 07:211-07:215, 07:217-07:222, 07:224-07:237, 07:239-07:241, 07:243-07:245, 07:248-07:266, 07:268-07:292, 07:297-07:313, 07:318-07:321, 07:323-07:327, 07:329N-07:334, 07:336-07:337, 07:339-07:350N, 07:352-07:355, 07:357-07:366, 07:368:01-07:401, 07:404, 07:407-07:414, 07:417-07:448, 07:450-07:492, 07:494Q-07:513Q, 07:515-07:516, 07:518, 07:520-07:530, 07:532-07:535, 07:537-07:562, 07:564-07:567, 07:569-07:577, 07:580-07:582Q, 07:584-07:597, 07:599-07:625, 07:627																							23	
*07:01:41, 07:02:41, 07:223, 07:415-07:416									9														w	
*07:02:08, 07:02:29, 07:26:01-07:26:03, 07:33N, 07:92, 07:139, 07:152N, 07:156, 07:246:01-07:246:02, 07:267, 07:296, 07:314:01, 07:314:03, 07:317, 07:322, 07:351, 07:405, 07:449, 07:493, 07:517									9															
*07:02:09								6	9				13						18				23	
*07:07, 07:09, 07:49, 07:76:01-07:76:02, 07:247, 07:315, 07:328, 07:403, 07:406, 07:598									9		11										20		23	
*07:10, 07:28, 07:41, 07:43:01-07:43:02, 07:184, 07:196, 07:367, 07:536, 07:568									9													21	23	
*07:31:01-07:31:02, 07:514									9				14					18					23	
*07:39-07:40, 07:335									9												20			
*07:53, 07:216									9		11												23	
*07:64, 07:402									9							16								
*07:96:01-07:96:02, 07:314:02, 07:578								3	9													22		
*07:101, 07:148, 07:161			2						9	10													23	

Negative Control

Negative Control

*07:123, 07:173, 07:294, 07:626									9							17					23			
*07:125, 07:356, 07:531					6				9	11		13	14								23			
*07:154									9							18					23			
*07:177									9			14				18			20					
*07:210, 07:238									9	11									20					
*07:242				4					9												23			
*07:293, 07:519									9						16						23			
*07:316, 07:338, 07:579	1								9												23			
*07:563									9										20		23			
*07:583		2				6			9	10		13				18								
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
*08:01:01-08:01:17, 08:01:19-08:01:21, 08:03:01-08:03:04, 08:06, 08:08:01-08:08:02, 08:14, 08:16:01-08:16:02, 08:20, 08:22, 08:24, 08:26N, 08:28, 08:36N, 08:38, 08:40-08:42, 08:44, 08:46, 08:50, 08:56-08:61, 08:65, 08:72:01-08:72:02, 08:78-08:82, 08:84-08:89N, 08:91, 08:96-08:99, 08:101-08:102, 08:105-08:106, 08:109, 08:117-08:119, 08:121N-08:122, 08:124, 08:127N-08:131, 08:133, 08:135-08:136, 08:141Q, 08:143-08:145, 08:147-08:149, 08:153-08:155, 08:157, 08:162, 08:164										10							18			21				
*08:01:18, 08:02:01-08:02:17, 08:04:01-08:04:03, 08:13, 08:15:01-08:15:02, 08:17-08:19:02, 08:23, 08:27, 08:29-08:30, 08:32-08:35, 08:37, 08:39, 08:43, 08:45, 08:49, 08:51-08:55N, 08:62:01-08:63, 08:66-08:71, 08:73, 08:75-08:77, 08:90, 08:92-08:94, 08:100, 08:103, 08:107, 08:110-08:112, 08:114-08:116, 08:120, 08:123, 08:125-08:126, 08:132, 08:134, 08:138-08:140, 08:146, 08:150-08:151, 08:156, 08:158-08:161N, 08:165-08:166										10										21				
*08:05, 08:25										10		13									21			
*08:07, 08:104, 08:142										10							19				21			
*08:09, 08:11, 08:83										10							18							
*08:10							7			10	11			15			18				21			
*08:12										10									20		21			
*08:21, 08:137										10		13					18				21			
*08:31		2								10											21			
*08:47										10							18	19			21			
*08:48, 08:108, 08:152										10														
*08:74										10					16						21			
*08:95										10					16		18				21			
*08:113								8		10						17								
*08:163						6				10					16		18				21			
*12:02:01-12:02:10, 12:02:12-12:02:13, 12:02:16, 12:03:47, 12:10:01-12:10:02, 12:17-12:18:01, 12:20, 12:22, 12:27, 12:30, 12:36, 12:40, 12:56, 12:64, 12:67-12:69, 12:73-12:74, 12:80N, 12:83-12:86, 12:96, 12:103-12:106, 12:112, 12:114, 12:117-12:118, 12:124, 12:126, 12:128, 12:130, 12:134, 12:136-12:137, 12:142, 12:145, 12:148N, 12:151, 12:153, 12:155Q, 12:161-12:162, 12:164, 12:166, 12:168, 12:175, 12:177, 12:179, 12:183, 12:193, 12:196, 12:200, 12:204, 12:207, 12:212, 12:214, 12:219N, 12:221, 12:226, 12:228, 12:231												12	13											
*12:02:11								8				12												
*12:02:14-12:02:15, 12:18:02, 12:222, 12:224, B*67:02 <sup>3</sup>													13											
*12:02:17			3									12	13											
*12:03:01-01-12:03:01:10, 12:03:03, 12:03:05-12:03:07, 12:03:10-12:03:12, 12:03:14-12:03:15, 12:03:17-12:03:19, 12:03:21-12:03:22, 12:03:25, 12:03:27-12:03:29, 12:03:31-12:03:33, 12:03:35-12:03:46, 12:06-12:07, 12:11-12:13, 12:19, 12:23, 12:25, 12:29, 12:32, 12:34-12:35, 12:37-12:39N, 12:42Q-12:43, 12:45-12:48, 12:50-12:53, 12:57:01-12:57:02, 12:61-12:63, 12:65-12:66, 12:70-12:71, 12:75-12:79, 12:82, 12:87-12:91, 12:93-12:95, 12:97, 12:99:01-12:102, 12:107-12:111, 12:115-12:116, 12:120-12:122, 12:125, 12:129, 12:131, 12:133, 12:138-12:141, 12:143, 12:149-12:150, 12:152, 12:156-12:160, 12:163, 12:165, 12:167, 12:170-12:174, 12:180, 12:182, 12:184, 12:186-12:187, 12:189-12:192, 12:197, 12:199, 12:201-12:202, 12:205-12:206, 12:209-12:211, 12:213, 12:216, 12:218, 12:220, 12:223, 12:225, 12:227, 12:229-12:230, 12:232N												12	13	14										

Negative Control

*12:03:02, 12:03:08, 12:03:16, 12:03:20, 12:03:30, 12:26, 12:59, 12:185														13	14											
*12:03:04, 12:98												12		14												
*12:03:09								8				12		14												
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24		
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24		
*12:03:13, 12:03:24												12	w	14												
*12:03:23			3									12	13	14												
*12:03:26								8				12	13	14												
*12:03:34:01-12:03:34:02																										
*12:04:01											11			14	15											
*12:04:02, 12:60											11	12	13	14	15											
*12:05											11	12		14	15											
*12:08												12	13				17									
*12:09											11				15						20					
*12:14:01													13					18	19							
*12:14:02												12	13					18	19							
*12:15, 12:113								8				12	13	14			17									
*12:16, 12:217												12	13											23		
*12:21							7				11	12			15											
*12:24												12	13							20						
*12:28, 12:58												12	13	14				18								
*12:31												12	13	14									21			
*12:33							7				11			14	15											
*12:41											11	12	13		15											
*12:44												12	13								21					
*12:49, 12:123, B*57:01:24 <sup>3</sup>												12														
*12:54, 12:194											11	12	13	14												
*12:55												12	13	14		16										
*12:72												12	13		15											
*12:81												12	13	14			17									
*12:92												12	13	14					19							
*12:119		2										12	13	14												
*12:127											10	12	13													
*12:132, 12:169												12	13					18								
*12:135												12	13	14	15			18								
*12:144													13	14									21			
*12:146											11	12	13		15			18								
*12:147, 12:195:02												12	13	14										23		
*12:154, 12:178												12	13	14	15											
*12:176												12	13	14				18	19	20						
*12:181										9			13					18								
*12:188											11	12					17									
*12:195:01												12			14									23		
*12:195:03															14									23		
*12:198		2										12	13													
*12:203											10	12	13	14												
*12:208								8					13					17								
*12:215	1											12	13	14												
*12:233													13										21			
*14:02:01:01-14:02:07, 14:02:09-14:02:20, 14:03, 14:05, 14:07N, 14:11, 14:13-14:14, 14:18-14:24:02, 14:26-14:27, 14:29-14:44, 14:46:01-14:48, 14:50-14:52, 14:55-14:57, 14:59-14:62, 14:65-14:76, 14:78-14:86, 14:88-14:89, 14:91, 14:93N-14:96																16				20						
*14:02:08, 14:02:21												12				16					20					
*14:04, 14:64											11					16					20					
*14:06, 14:08-14:09, 14:28:01-14:28:02, 14:53, 14:63, A*29:86, A*30:62, A*33:63, B*18:64, B*35:183, B*35:252, B*39:114, B*40:138 <sup>3</sup>																16										
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24		
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24		
*14:10												11				16		18		20						
*14:12, 14:49																				20						
*14:15, 14:87														14		16		18								
*14:16														14		16										
*14:17																16		18	19	20						
*14:25		2														16				20			22			
*14:45															15	16				20						
*14:54																16							21			
*14:77											11					16										

Negative Control

*14:90														11	13			16													23	
*14:92																			16	17												
*15:02:01:01-15:02:01:07, 15:02:02:01-15:02:09, 15:02:11-15:02:16, 15:02:18-15:02:20, 15:02:22-15:02:24, 15:02:26-15:02:32, 15:06:01-15:06:03, 15:08-15:09, 15:10:02-15:10:03, 15:13:01:01-15:13:01:02, 15:15, 15:17-15:19, 15:26-15:28, 15:32Q-15:35, 15:38-15:39, 15:41, 15:44:01-15:45, 15:47-15:53, 15:56-15:57, 15:60, 15:64, 15:67-15:68, 15:73-15:74, 15:78:01-15:84Q, 15:86-15:89, 15:91-15:101, 15:103, 15:106, 15:109, 15:112-15:114, 15:119-15:124, 15:126-15:137, 15:139, 15:141-15:143, 15:145N-15:146, 15:150-15:151, 15:154-15:155			3											11				15	17												21	
*15:02:01:08N, 15:37, 15:55, 15:58, 15:72, 15:149			3											11				15	17													
*15:02:10, 15:02:17			3											11				15	17										21	22		
*15:02:21, 15:02:25, 15:10:01, 15:61														11				15	17											21		
*15:03, 15:16			3											11	13			15	17											21		
*15:04:01-15:04:03, 15:24, 15:30, 15:40, 15:66, 15:76-15:77, 15:147			3											11				15	17											21		
*15:05:01-15:05:08, 15:05:10, 15:29, 15:46, 15:54, 15:59, 15:69-15:70, 15:90, 15:105Q, 15:108, 15:110, 15:115N, 15:117, 15:140, 15:148, 15:152:02-15:153			3							w				11				15	17											21		
*15:05:09										w				11				15	17										21			
*15:05:11-15:05:12, 15:116			3							w								15	17										21			
*15:07			3							8									17										21			
*15:11, 15:118			3			6				8				11				15	17										21			
*15:12					5					8				11				15	17	18								21				
*15:21, 15:71			3							8								15	17									21				
*15:22, 15:125			3							w				11				15	17													
*15:23:01-15:23:02										w				11					17										21			
*15:25			3												13				18	19								21				
*15:31, 15:62			3							8				11				15	17	18								21				
*15:36						6				w				11				17										21				
*15:42			2	3						8				11				15	17									21				
*15:43			3							8									17									21	22			
*15:63, 15:138										8				11				17										21				
*15:65			3											11				15	17	18	19											
*15:75			3							8				11	13			15	17	18								21	23			
*15:85			3							8				11				17										21				
*15:102			3											11				15	17						20							
*15:104	1		3							w				11				15	17								21					
*15:107									7	8				11				15	17								21					
*15:111			2	3						w				11				15	17								21					
*15:144						5				8									17	18							21					
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24								
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24								
*16:01:01:01-16:01:15, 16:01:18-16:01:23, 16:06-16:08, 16:11, 16:13-16:15:01, 16:16Q-16:17, 16:21-16:24, 16:26-16:28, 16:30N-16:32, 16:36, 16:38-16:39:02, 16:41, 16:44, 16:49, 16:51-16:52, 16:54, 16:56-16:59, 16:62, 16:65, 16:71-16:73, 16:75-16:76, 16:79-16:81, 16:83, 16:85-16:86, 16:92-16:98, 16:100, 16:105-16:106, 16:111-16:112, 16:114, 16:116-16:119, 16:122													11	13					18												23	
*16:01:16								8					11	13					18												23	
*16:01:17, 16:15:02												11	12	13					18												23	
*16:02:01-16:02:12, 16:02:14-16:02:15, 16:09, 16:12, 16:18-16:19, 16:25, 16:37, 16:46-16:47, 16:60, 16:63, 16:69, 16:74, 16:77N, 16:84, 16:89N-16:90, 16:99, 16:101-16:102, 16:104, 16:107-16:108, 16:115, 16:120													11	13		15			18												23	
*16:02:13												11	12	13		15			18												23	
*16:04:01:01-16:04:01:02, 16:04:03-16:04:05, 16:33, 16:42, 16:55, 16:61, 16:66, 16:78, 16:82															14				18		20											
*16:10, 16:67														13					18												23	
*16:20								8					11	13				17	18												23	
*16:29			2																18		20											
*16:34			3				6						11	13					18												23	
*16:35															14				18													
*16:40, 16:110								7								16																
*16:43													11	13																	23	
*16:45															13				18					21							23	
*16:48													11		14	15			18													
*16:50			2											11	13				18												23	

Negative Control

*16:53							7								16				20					Negative Control
*16:64, 16:87										11		13					17	18					23	
*16:68																		18		20				
*16:70			3							11		13		15			17	18					23	
*16:88										11				15				18		20				
*16:91										11			14	15				18		20				
*16:103										11				15				18						
*16:109								8					14				17	18		20				
*16:113							7			11		13			16								23	
*16:121			3			6				11		13		15				18					23	
*17:01:01:02-17:01:03, 17:01:05-17:04, 17:06, 17:08-17:16, 17:18-17:19, 17:21, 17:24-17:32, 17:34-17:38, 17:40			2							11				15				19		21				
*17:01:04			2							11		13		15				19		21				
*17:05			2				7			11				15				19		21				
*17:07, 17:17, 17:23, 17:33										11				15				19		21				
*17:20			2							11								19		21		23		
*17:22			2															19		21				
*18:01-18:02, 18:04-18:08, 18:10										11									20	21		23		
*18:03			2							11									20	21	22	23		
*18:09										11				15					20	21				
A*24:64, A*24:96, A*24:106, B*08:16, B*08:123, B*08:145																							23	
A*24:174										11													23	
B*07:239, B*14:03																			20					
B*13:13:02, B*40:01:49-40:01:50																						22		
B*15:277, B*15:430, B*35:222, B*42:22, B*54:38														15										
B*39:76, B*54:18			1																					
B*44:02:44, B*56:01:09, B*58:74, B*82:01:01:02, B*82:02:01:02				3																				
B*58:02:01								8																
B*73:01																						w		
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24