

HLA-C02 2018 6H0

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44								
HLA-C allele^{1,2,3}																																																				
*02:02:01-02:02:03, 02:02:06-02:02:12, 02:02:15-02:02:25, 02:02:27-02:02:34, 02:02:36, 02:02:38-02:02:46, 02:48, 02:50, 02:54, 02:57, 02:59, 02:61-02:63, 02:66, 02:74, 02:77-02:80, 02:82, 02:84-02:86, 02:88, 02:90, 02:96, 02:99-02:100, 02:102-02:103, 02:108, 02:111-02:114, 02:116-02:119, 02:122-02:123, 02:127-02:130, 02:132, 02:136, 02:138-02:142, 02:144-02:149, 02:151, 02:153-02:156		1	2						9	10																																										
*02:02:05, 02:02:13, 02:02:37, 02:43:02, 02:73, 02:76, 02:95, 02:109-02:110, 02:125	1								9	10																																				42						
*02:02:14, 02:02:35, 02:51, 02:124	1								9	10																																										
*02:02:26	1	2							9																																											
*02:03			3						9	10					16			19																																		
*02:04	1	2		4					9	10																																										
*02:05:01	1				5				9	10																																					42					
*02:05:02-02:05:03, 02:22 ⁵	1	2			5				9	10																																										
*02:06:01	1					6			9	10														24																							42					
*02:06:02	1	2				6			9	10													24																													
*02:07	1	2					7		9	10																																						39				
*02:08	1	2						8	9	10																																										
*02:09	1	2							9	10	11																																									
*02:10:01:01-02:10:05, 02:89, 02:133, 02:143, 06:18 ⁴	1									10																																							42			
*02:11	1	2							9	10		12																																								
*02:12	1									10				13																																			42			
*02:13	1	2							9	10					14																																					
*02:14:01-02:14:02	1									10		12																																						42		
*02:15, 02:21 ⁵	1	2							9	10						15																																				
*02:16:02			3						9	10						16																																				
*02:17	1				5					10		12																																						42		
*02:18			3						9	10	11					16																																				
*02:19	1	2							9	10																																										
*02:20	1	2	3						9	10																																										
*02:23	1	2							9	10								17								24	25																									
*02:24	1	2							9	10											20																															
*02:25Q, 02:30 ⁵	1	2							9	10																																										
*02:26:01-02:26:03, 02:134	1	2							9										18																																	
*02:27:01, 02:131	1	2							9	10				13																																				36		
*02:27:02	1								9	10				13																																				42		

Negative Control

HLA-C02 2018 6H0

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44						
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44						
*02:28	1	2							9	10									19																															
*02:29	1	2							9	10													23																											
*02:31	1	2	3						9	10							17																																	
*02:32									9	10	11																																							
*02:33	1							8	9	10																																						42		
*02:34	1	2							9	10													22																											
*02:35, 02:69-02:70, 02:120 ⁵	1	2							9	10													23																											
*02:36:01, 02:68	1	2							9	10														24																										
*02:36:02	1								9	10														24																								42		
*02:37, 02:52N ⁵	1	2							9	10																			28																					
*02:38N	1	2							9	10																				30																				
*02:39	1	2							9																																									
*02:40:01	1	2							9	10																	26																							
*02:40:02	1								9	10																	26																						42	
*02:42	1	2							9	10																																								
*02:43:01	1	2							9	10				14			17																																	
*02:44	1	2							9	10																		27																						
*02:45	1	2							9																			27																						
*02:46	1	2				6			9	10																			28																					
*02:47	1	2				6			9	10																																								
*02:49	1	2							9	10			13																	29																				
*02:53:01-02:53:02	1	2							9	10																	26	27																						
*02:55:01	1	2							9	10																				29																				
*02:55:02	1								9	10																				29																			42	
*02:56	1	2		4					9	10																																								
*02:58	1									10																																								42
*02:60	1	2							9	10																25		28			30																			
*02:64	1	2				6			9	10																																								
*02:65	1	2							9	10																																								
*02:67Q	1	2							9	10																																								
*02:71	1	2							9	10				15																																				
*02:72									9	10																																								
*02:75, 02:126	1	2							9	10			13																																					
*02:81	1	2			5				9	10																																								
*02:83	1	2							9	10																																								
*02:87	1	2							9																						29		31																	
*02:91	1	2							9	10																																								
*02:92N	1	2							9	10																																								
*02:93	1	2							9	10																																								
*02:94	1								9	10																																								
*02:97	1	2					7		9	10																																								
*02:98	1	2							9	10																																								
*02:101	1	2							9	w																																								

Negative Control

HLA-C02 2018 6H0

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44								
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44								
*02:104	1	2							9	10																																		41								
*02:105N	1	2							9	10																											34															
*02:106	1	2							9	10																										35																
*02:107	1	2							9			12					18													31																						
*02:115	1	2							9	10			13																29								36															
*02:121N	1	2							9	10																																	41									
*02:135N									9	10																																										
*02:137										10						16																															42					
*02:150N	1	2							9	10																																							43			
*02:152	1	2							9								18														31																					
*01:02:06, 01:97, 04:01:01:01-04:01:06, 04:01:09-04:01:103, 04:05, 04:07:01- 04:07:02, 04:09N-04:12, 04:14-04:15:03, 04:17-04:20, 04:23-04:33, 04:35-04:41, 04:43- 04:57, 04:59Q-04:79, 04:81-04:93N, 04:95N- 04:102, 04:104-04:106, 04:109, 04:111- 04:121, 04:123N-04:139, 04:141-04:146, 04:148-04:159, 04:161-04:162, 04:164:01- 04:165, 04:167-04:170N, 04:172-04:177, 04:179-04:189, 04:191N-04:197, 04:199- 04:211, 04:213-04:219, 04:221-04:222, 04:224-04:240, 04:242-04:255N, 04:257- 04:264, 04:266-04:285, 04:287-04:290, 04:292-04:293, 04:295-04:298, 04:300N- 04:334, 08:33:02-08:33:03, 08:35, 08:118, 12:19, 12:24, 12:85, 12:137, 12:153, 12:233, 14:02:01:01-14:02:04, 14:02:06-14:05, 14:07N-14:14, 14:18-14:24:02, 14:26-14:40, 14:42-14:52, 14:54-14:76, 14:78, 14:80- 14:86, 14:88-14:91, 14:93N-14:100, 15:25, 16:01:01:01-16:01:20, 16:01:22-16:01:31, 16:02:08, 16:06-16:08, 16:11, 16:13-16:18, 16:22-16:24, 16:26, 16:28, 16:30N-16:32, 16:36, 16:38-16:39:02, 16:41, 16:43-16:45, 16:49, 16:51-16:54, 16:56-16:59, 16:62, 16:64-16:65, 16:68, 16:71-16:73, 16:75- 16:76, 16:79, 16:81, 16:83, 16:85-16:87, 16:92-16:98, 16:100, 16:106, 16:111-16:114, 16:116-16:119, 16:122, 16:125-16:131, 16:134-16:135, 16:137-16:139, 16:141																																																		42		
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44								

Negative Control

HLA-C02 2018 6H0

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44					
*01:02:34, 08:103, B*15:436, B*67:02																																31																	
*01:04, 04:94:01-04:94:02, 06:02:09, 06:02:21, 06:11, 06:33, 06:82, 06:104, 06:119-06:120, 06:125, 06:147, 06:188, 06:210, 06:217, 08:27, 08:160, 12:02:01-12:02:16, 12:02:18-12:03:15, 12:03:18-12:03:22, 12:03:24-12:03:26, 12:03:28-12:03:29, 12:03:31-12:03:52, 12:06-12:08, 12:10:01-12:13:01:02, 12:17, 12:22-12:23, 12:25-12:31, 12:34-12:40, 12:42Q-12:59, 12:61-12:71, 12:73-12:82, 12:84N, 12:86-12:112, 12:114, 12:116-12:118, 12:120-12:123, 12:125-12:134, 12:136, 12:138-12:143, 12:145, 12:148N-12:152, 12:155Q-12:161, 12:163-12:168, 12:170-12:174, 12:176-12:180, 12:182-12:184, 12:186-12:195:01, 12:195:03-12:207, 12:209-12:216, 12:218-12:221, 12:223-12:226, 12:228-12:232N, 12:234, 12:236N-12:257, 14:16, 16:04:01:01-16:04:01:02, 16:04:03-16:04:05, 16:33, 16:42, 16:55, 16:61, 16:66, 16:78, 16:82, 16:124	1																																															42	
*01:09	1																								25																								
*01:10, B*07:02:07, B*07:239, B*14:46, B*27:05:15, B*40:243, B*48:04:02					5																																												
*01:21, 12:227	1																															31														42			
*01:22, 01:35, 01:160, 08:04:01-08:04:03, 08:13, 08:57, 08:93, 08:104, 08:139, 08:168, 12:14:01, B*58:02:01																16																																	
*01:90, 01:101-01:102, 01:113, 01:116, 03:81, 03:175, 03:199, 03:245, 03:317, 03:388, 07:123, 07:173, 07:294, 07:626, 14:92																								24																									
*01:115							7																																										
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44					
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44					

Negative Control

