

HLA-C02 2021 5N0

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45									
HLA-C allele^{1,2,3}																																																						
*02:02:01-02:02:03, 02:02:06-02:02:12, 02:02:15-02:02:25, 02:02:27-02:02:34, 02:02:36, 02:02:38-02:02:52, 02:02:54-02:02:61, 02:50, 02:54, 02:57, 02:59, 02:61:01-02:63, 02:66, 02:74, 02:77-02:80, 02:82, 02:84-02:86, 02:88, 02:90, 02:96, 02:99-02:100, 02:102-02:103, 02:108, 02:111-02:114, 02:116-02:119, 02:122-02:123, 02:127-02:130, 02:132, 02:136, 02:138-02:142, 02:144-02:149, 02:151, 02:153-02:157, 02:159-02:160, 02:162-02:163, 02:165N-02:166, 02:170-02:173, 02:175, 02:177-02:179, 02:183-02:188, 02:190, 02:192N-02:196, 02:198-02:206	1	2							9	10																																												
*02:02:13, 02:02:37, 02:02:53, 02:73, 02:76, 02:95, 02:109-02:110, 02:125	1								9	10																																								42				
*02:02:14, 02:02:35, 02:51, 02:124	1								9	10																																												
*02:02:26, 02:191	1	2							9																																													
*02:03																16				19																																		
*02:04	1	2							9	10																																												
*02:05:01	1							5																																										42				
*02:05:02-02:05:03, 02:22 ⁴	1	2						5																																														
*02:06:01	1								6																24																								42					
*02:06:02	1	2							6															24																														
*02:07, 02:158	1	2																																																	39			
*02:08	1	2																																																				
*02:09	1	2																																																				
*02:10:01-01-02:10:08, 02:89, 02:133, 02:143, 02:161, 02:168-02:169N, 02:174, 02:176, 02:181-02:182	1																																																			42		
*02:11, 02:164	1	2																																																				
*02:12	1																																																			42		
*02:13	1	2																																																				
*02:14:01-02:14:02	1																																																				42	
*02:15, 02:21 ⁴	1	2																																																				
*02:16:02:01-02:16:02:02																																																						
*02:17	1																																																					42
*02:18																																																						
*02:19	1	2																																																				
*02:20	1	2																																																				
*02:23	1	2																																																				
*02:24	1	2																																																				
*02:25Q, 02:30 ⁴	1	2																																																				
*02:26:01-02:26:03, 02:134, 02:167	1	2							9																																													

Negative Control

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45						
*02:94	1								9	10																							33											42							
*02:97	1	2						7	9	10																																									
*02:98	1	2							9	10																																									
*02:101	1	2							9	w																																									
*02:104	1	2							9	10																																									
*02:105N	1	2							9	10																																									
*02:106	1	2							9	10																											34														
*02:107	1	2							9			12						18													31																				
*02:115	1	2							9	10			13																																						
*02:121N	1	2							9	10																																									
*02:135N									9	10																																									
*02:137										10																																									
*02:150N	1	2							9	10																																									
*02:152	1	2							9									18																																	
*02:180									9	10																																									
*02:189	1	2						8										18																																	
*02:197, 16:121 ⁵										10																																									
*01:02:06, 01:02:68, 01:97, 04:01:06, 04:01:77, 04:11, 04:29, 04:32, 04:36, 04:55, 04:59Q, 04:77, 04:85, 04:114, 04:172, 04:177, 04:249, 04:338Q, 04:382Q, 04:413, 04:428Q, 04:447, 08:33:02-08:33:04, 08:35, 08:118, 12:19, 12:24, 12:85, 12:137, 12:153, 12:233, 12:268, 14:02:01:01-14:02:04, 14:02:06-14:05, 14:07N-14:11, 14:13-14:14, 14:18-14:24:02, 14:26-14:40, 14:42-14:48, 14:50-14:52, 14:54-14:76, 14:78, 14:80-14:86, 14:88-14:91, 14:93N-14:107, 14:109-14:135, 15:25, 16:01:01:01-16:01:20, 16:01:22-16:01:34, 16:01:36-16:01:41, 16:06-16:08, 16:11, 16:13-16:18, 16:22-16:24, 16:26, 16:28, 16:30N-16:32, 16:36, 16:38-16:39:02, 16:41, 16:43-16:45, 16:49, 16:51-16:54, 16:56-16:59, 16:62, 16:64-16:65, 16:68, 16:71-16:73, 16:75-16:76, 16:79, 16:81, 16:83, 16:85-16:87, 16:92-16:98, 16:100, 16:106, 16:111-16:114, 16:116-16:119, 16:122, 16:125-16:131, 16:134-16:135, 16:137-16:139, 16:141-16:142, 16:146-16:148, 16:151-16:152, 16:154, 16:157-16:162, 16:164-16:165, 16:168, 16:170-16:175, 16:177-16:178, 16:183, 16:185-16:187																																																			
*01:02:34, 08:103, B*15:436, B*67:02:01:01-67:02:01:02																																																			
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45						

Negative Control

HLA-C02 2021 5N0

*06:223	1								18		23							42	44
*06:224, 15:74	1		6						18		24							42	44
*06:327	1								18					29	31			42	44
*07:01:74, 07:02:09, 07:125:02							12								31				
*07:02:75			3				11	14	17										
*07:101, 07:148, 07:161, 07:583, 08:195, A*01:203, A*11:166, A*30:56, A*31:85, A*80:01:01-80:09N, B*18:96, B*44:157, B*55:74																33			
*07:272, 07:326, 07:898							10												
*07:756							10	13						29			36		
*08:14, 08:80									16						31			42	
*08:24									16					29	31			42	
*08:29, 12:02:27, 12:03:16, 12:03:27, 12:03:30, 12:32, 12:144, 12:185, 12:276	1																		
*08:31	1		5																
*08:37, B*13:130, B*27:84								12											
*08:113									16		24								
*08:119			4						16										42
*12:02:17, 12:03:23	1				8														42
*12:02:32, 12:02:34, 12:03:17	1	2						14	17										
*12:15, 12:208	1			6							24								42
*12:16:01, 12:147	1							12							31				42
*12:51	1									20									42
*12:113, 12:282, 12:309, 16:109	1										24								42
*12:115	1		4																42
*12:119, 16:29	1		5																42
*12:124	1										26								42
*12:162	1		6							21									42
*12:195:02, 12:217	1							12											42
*12:323	1													29	31				42
*14:25			5					12											42
*14:41			4																42
*14:79, 16:10, 16:67				7															42

Negative Control

