

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48	Well No.			
HLA-C allele ^{1,2,3}																																																				HLA-C allele ^{1,2,3}
*03:02:01-03:02:09, 03:02:11-03:02:14, 03:108, 03:110, 03:197, 03:216, 03:222, 03:225, 03:238	1	2											13		15																																				*03:02:01-03:02:09, 03:02:11-03:02:14, 03:108, 03:110, 03:197, 03:216, 03:222, 03:225, 03:238	
*03:02:10	2												13	14																																				*03:02:10		
*03:03:01-03:03:11, 03:03:13-03:03:14, 03:03:16-03:03:20, 03:03:22-03:03:28, 03:112, 03:124, 03:133, 03:141, 03:150, 03:152, 03:158, 03:160, 03:168, 03:185, 03:188, 03:196, 03:202-03:204, 03:217, 03:220, 03:228, 03:230, 03:241-03:242, 03:253-03:254, 03:273, 03:275-03:276, 03:284			3	4			7								15																																				*03:03:01-03:03:11, 03:03:13-03:03:14, 03:03:16-03:03:20, 03:03:22-03:03:28, 03:112, 03:124, 03:133, 03:141, 03:150, 03:152, 03:158, 03:160, 03:168, 03:185, 03:188, 03:196, 03:202-03:204, 03:217, 03:220, 03:228, 03:230, 03:241-03:242, 03:253-03:254, 03:273, 03:275-03:276, 03:284	
*03:03:12, 03:03:15, 03:03:29			3	4			7				12				15																																				*03:03:12, 03:03:15, 03:03:29	
*03:03:21				4			7							14																																					*03:03:21	
*03:04:01:01-03:04:18, 03:04:20-03:04:24, 03:04:27-03:04:47, 03:91:02, 03:104-03:107, 03:111, 03:114, 03:128-03:129, 03:131, 03:134, 03:138, 03:143, 03:148-03:149, 03:157, 03:159, 03:162, 03:164, 03:173, 03:180-03:181, 03:183, 03:186:01-03:186:02, 03:191, 03:193, 03:210, 03:215, 03:232-03:235, 03:239, 03:247, 03:250, 03:255-03:257, 03:259, 03:266, 03:269-03:270, 03:280, 03:282-03:283	1			4											15																																				*03:04:01:01-03:04:18, 03:04:20-03:04:24, 03:04:27-03:04:47, 03:91:02, 03:104-03:107, 03:111, 03:114, 03:128-03:129, 03:131, 03:134, 03:138, 03:143, 03:148-03:149, 03:157, 03:159, 03:162, 03:164, 03:173, 03:180-03:181, 03:183, 03:186:01-03:186:02, 03:191, 03:193, 03:210, 03:215, 03:232-03:235, 03:239, 03:247, 03:250, 03:255-03:257, 03:259, 03:266, 03:269-03:270, 03:280, 03:282-03:283	
*03:04:19, 03:265N ⁵	1			4							12				15																																				*03:04:19, 03:265N ⁵	
*03:04:25, 03:118, 03:172	1			4																																															*03:04:25, 03:118, 03:172	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48	Well No.			
*03:04:26				4										14																																						*03:04:26
*03:05	1				5										15	17																																			*03:05	
*03:06:01-03:06:02	1			4		6									15																																					*03:06:01-03:06:02
*03:07	1		4						9	10					15	16																																				*03:07

Negative Control

*03:08	1		4			7				11					15																																	
*03:09	1		4			8									15																																	
*03:10	1		4							10					15	16																																
*03:11:01			3	4		7					12				15																																	
*03:11:02			3	4		7									15																																	
*03:13:01-03:13:02			3		5	7									15																																	
*03:14	1	2	4												15																										47							
*03:15	1	2							9	10			13	15	16							22																										
*03:16	1												13	15																																		
*03:17	1				5								13	15																																		
*03:18:01				4		7						12																																				
*03:18:02			3	4		7									15																																	
*03:19	1		4										14	15																																		
*03:20N			3	4											15	17																																
*03:21			3	4		6	7								15																										47							
*03:22Q			3	4		7	8								15									23																								
*03:23, 03:263	1		4												15	18																																
*03:24, 03:236 ⁵	1		4												15				19																													
*03:25	1			5											15	17				20																												
*03:26	1		4												15					21																												
*03:27	1			5											15	17					22																											
*03:28, 03:90 ⁵	1		4												15						23																											
*03:29	1		4			7			10	11				15	16																																	
*03:30			3	4		7									15																																	
*03:31			3	4		7									15																																	
*03:32	1		4												15																																	
*03:33	1	2											13	15					19																													
*03:34, 03:261	1		4												15																																	
*03:35, 03:178	1			5											15																																	
*03:36	1	2											13	15																											39							
*03:37:01-03:37:02, 03:208N ⁵	1		4												15																																	
*03:38:01, 03:213	1		4												15																																	
*03:38:02	1		4												15																																	
*03:39			4			7						12																													29							
*03:40:01, 03:40:03-03:40:04, 03:279	1	2	4										13	15																																		
*03:40:02	1	2	4												15																																	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48
*03:41:01-03:41:02	1														15																																	
*03:42, 03:240	1	2	4					8							13	15																																
*03:43:01		2	3	4		7							13	15																																		
*03:43:02		2	3	4		7									15																																	
*03:44, 03:123, 03:209 ⁵	1		4												15																																	
*03:45	1		4					9	10						15	16											25	27																				
*03:46	1		4												15																																	

Negative Control

*03:08
*03:09
*03:10
*03:11:01
*03:11:02
*03:13:01-03:13:02
*03:14
*03:15
*03:16
*03:17
*03:18:01
*03:18:02
*03:19
*03:20N
*03:21
*03:22Q
*03:23, 03:263
*03:24, 03:236 ⁵
*03:25
*03:26
*03:27
*03:28, 03:90 ⁵
*03:29
*03:30
*03:31
*03:32
*03:33
*03:34, 03:261
*03:35, 03:178
*03:36
*03:37:01-03:37:02, 03:208N ⁵
*03:38:01, 03:213
*03:38:02
*03:39
*03:40:01, 03:40:03-03:40:04, 03:279
*03:40:02
Well No.
Well No.
*03:41:01-03:41:02
*03:42, 03:240
*03:43:01
*03:43:02
*03:44, 03:123, 03:209 ⁵
*03:45
*03:46

*02:02:02:01-02:02:03, 02:02:05-02:02:11, 02:02:14- 02:02:28, 02:03, 02:05:01- 02:07, 02:09-02:11, 02:13, 02:15-02:25Q, 02:26:02- 02:26:03, 02:28-02:32, 02:34- 02:40:02, 02:43-02:48, 02:50- 02:54, 02:56-02:64, 02:66- 02:74, 02:76-02:82, 02:84- 02:86, 02:88-02:100, 05:58:01, 15:02:01:01- 15:02:09, 15:02:11-15:02:16, 15:02:18-15:02:20, 15:02:22- 15:02:24, 15:04:01-15:05:08, 15:05:10, 15:06:01-15:06:03, 15:08-15:09, 15:11, 15:13, 15:15, 15:17-15:19, 15:22, 15:24, 15:26-15:35, 15:38- 15:42, 15:44-15:60, 15:64- 15:76, 15:78-15:84Q, 15:86- 15:87, 15:96Q, 15:104- 15:105Q, 15:108-15:112	1								9	10																								*02:02:02:01-02:02:03, 02:02:05-02:02:11, 02:02:14- 02:02:28, 02:03, 02:05:01- 02:07, 02:09-02:11, 02:13, 02:15-02:25Q, 02:26:02- 02:26:03, 02:28-02:32, 02:34- 02:40:02, 02:43-02:48, 02:50- 02:54, 02:56-02:64, 02:66- 02:74, 02:76-02:82, 02:84- 02:86, 02:88-02:100, 05:58:01, 15:02:01:01- 15:02:09, 15:02:11-15:02:16, 15:02:18-15:02:20, 15:02:22- 15:02:24, 15:04:01-15:05:08, 15:05:10, 15:06:01-15:06:03, 15:08-15:09, 15:11, 15:13, 15:15, 15:17-15:19, 15:22, 15:24, 15:26-15:35, 15:38- 15:42, 15:44-15:60, 15:64- 15:76, 15:78-15:84Q, 15:86- 15:87, 15:96Q, 15:104- 15:105Q, 15:108-15:112															
*02:02:12, 15:05:11, 15:21	1									10																						*02:02:12, 15:05:11, 15:21																	
*02:02:13, 02:02:29	1									9	10	15	16																			*02:02:13, 02:02:29																	
*02:04, 02:14:01	w								w	w																					*02:04, 02:14:01																		
*02:08, 15:10:02-15:10:03	1								9	10								22					27								*02:08, 15:10:02-15:10:03																		
*02:12	1									10																					*02:12																		
*02:26:01, 05:58:02, 08:105, 15:23, B*44:85											14																				*02:26:01, 05:58:02, 08:105, 15:23, B*44:85																		
*02:27:01-02:27:02, 15:07, 16:34, B*40:164	1																														*02:27:01-02:27:02, 15:07, 16:34, B*40:164																		
*02:33										9	10																				*02:33																		
*02:42, 02:75, 15:113	1									9	10												27								*02:42, 02:75, 15:113																		
*02:49, 04:160	1									9	10										22		27								*02:49, 04:160																		
*02:55	1									9	10										22										*02:55																		
*02:65, 15:85	1									9																					*02:65, 15:85																		
*02:83	1									9	10																41				*02:83																		
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48	Well No.
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48	Well No.

Negative Control

*04:147	1									9	10									22						27	28																			*04:147																
*04:161																											28																			37		w						*04:161								
*04:171	1									9	10																27	28																			37	38						*04:171								
*04:178, 05:03, 05:93, 08:20, 08:40, 12:44, 12:109, B*35:178, B*35:282																											25																										*04:178, 05:03, 05:93, 08:20, 08:40, 12:44, 12:109, B*35:178, B*35:282									
*04:190																																																					*04:190									
*05:01:12, 05:29:02, 06:02:01:01-06:02:01:03, 06:02:04-06:02:07, 06:02:09-06:02:43, 06:03:02-06:04, 06:06-06:10, 06:12-06:43:02, 06:45, 06:47-06:52, 06:53:02-06:69, 06:71-06:72, 06:74Q-06:76:01, 06:77-06:81, 06:83-06:131, 06:133-06:142, 06:144-06:146, 06:148-06:152N, 07:76:01-07:76:02, 07:315, 07:328, 07:406, 12:04:01, 17:01:02-17:01:03, 17:01:05-17:01:11, 17:04, 17:06-17:21, 17:23-17:25, 17:27N-17:29, B*15:277, B*35:222																			16																																	*05:01:12, 05:29:02, 06:02:01:01-06:02:01:03, 06:02:04-06:02:07, 06:02:09-06:02:43, 06:03:02-06:04, 06:06-06:10, 06:12-06:43:02, 06:45, 06:47-06:52, 06:53:02-06:69, 06:71-06:72, 06:74Q-06:76:01, 06:77-06:81, 06:83-06:131, 06:133-06:142, 06:144-06:146, 06:148-06:152N, 07:76:01-07:76:02, 07:315, 07:328, 07:406, 12:04:01, 17:01:02-17:01:03, 17:01:05-17:01:11, 17:04, 17:06-17:21, 17:23-17:25, 17:27N-17:29, B*15:277, B*35:222										
*05:36, 12:08, 12:81, 16:64																																																					*05:36, 12:08, 12:81, 16:64									
*05:55, 07:184, 12:46N, B*35:01:38, B*35:03:15, B*35:08:04, B*35:15:02, B*37:04:02, B*44:198N, B*57:01:18, B*58:01:17																																																					*05:55, 07:184, 12:46N, B*35:01:38, B*35:03:15, B*35:08:04, B*35:15:02, B*37:04:02, B*44:198N, B*57:01:18, B*58:01:17									
*05:85																																	31	32													46						*05:85									
*05:86				5			8																																												47						*05:86					
*06:02:03, 06:46N, 17:01:01:01-17:01:01:03, 17:02-17:03, 17:05																																																			41						*06:02:03, 06:46N, 17:01:01:01-17:01:01:03, 17:02-17:03, 17:05					
*06:02:08, 14:12, 14:49													13																			28															37						*06:02:08, 14:12, 14:49									
*06:03:01	1									9	10																					27																					16						*06:03:01			
*06:44												7																																												16						*06:44
*06:53:01												7																																												16						*06:53:01
*06:70:01													12																																											16						*06:70:01
*06:73, 06:143																																																								16						*06:73, 06:143
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48	Well No.													
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48	Well No.													
*06:132:01-06:132:02	1								9	10					15	16														22																							*06:132:01-06:132:02									

Negative Control

