

Length of spec.	250	220	145	210	250	95	145	270	220	180	110	110	195	180	125	225	155	180	85	320	125	120	165	85	130	85	170	
PCR product(s)									180	220	180	110	195	180	125	270	155	180	85	320	125	120	165	85	130	85	170	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24				
*01:04, 01:54, 14:02:01-14:03, 14:05, 14:07N-14:08, 14:10-14:14, 14:16-14:19, 14:21N-14:24:01, 14:27-14:43	1																											
*01:21	1																											
*01:22, 02:03, 02:16:01-02:16:02, 02:18, 05:11, 05:17, 05:68, 05:79, 06:04, 08:01:01-08:01:06, 08:01:08, 08:03:01-08:04:02, 08:06, 08:08-08:11, 08:13-08:14, 08:16:01-08:16:02, 08:20-08:22, 08:24, 08:26N, 08:36N, 08:38-08:40, 08:42, 08:44, 08:46, 08:50, 08:54, 08:56-08:61, 12:14:01-12:14:02, 12:18:01-12:18:02, 12:20, 15:02:01-15:07, 15:09-15:13, 15:16-15:19, 15:21-15:24, 15:26-15:35, 15:37-15:50, 15:52-15:61, 16:35, 16:40, 16:48, 17:01:01:01-17:06, 17:08-17:13					5																							
*01:35, 05:27, 08:41, 15:15, 17:07, B*58:02					5			8																				
*01:50	1																	17										
*01:59, 05:20, 06:82, 12:54	1																											
*02:58, 05:39	1							8																				
*03:05, 03:13:01, 03:25, 03:27, 03:35																16												
*03:34, 03:142							7																					
*03:81																												
*03:130, 03:140, 07:49, 07:210, 07:238, 07:247, A*68:46																												
*03:135																16				19								
*05:25, 05:42, 06:05, 06:76:02, 08:28, 12:28	1										11																	
*05:49	1																											
*05:64, 08:19	1																											
*05:78	1						7			10												20						
*07:02:09, 15:25, 16:26, 16:46											11																	
*07:08, 07:108																												
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24				
Length of spec.	250	220	145	210	250	95	145	270	180	110	110	195	180	125	225	155	180	85	320	125	120	165	85	130	85	170		
PCR product(s)									180	220	180	110	195	180	125	270	155	180	85	320	125	120	165	85	130	85	170	

									220										545	240		215		
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24
*07:64		2					7																	
*07:125												12												24
*07:167																								
*08:01:07					5											16								
*08:02:07, 08:33:02	1															16								
*12:10:01-12:10:02	1																18							
*14:04	1																							
*14:06, 14:15					5																			
*14:09	1															16								
*14:20	1							8																
*14:25	1						7																	
*15:36					5		7			10										20				
*16:12																								
*16:45, A*24:96, A*24:146															15									
*18:03	1						7										18							
A*33:52																				20				
B*07:90																								
B*15:27:01-15:27:03, B*15:109																	18							
B*15:78:03																16								
B*40:100																								
HLA-C allele ^{4,5}																								
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24

200	215	280 125	215 120	185 105	75	180 130	275 235 105	180 80	275 200 75	175 145 85	115	135 110	135 95	330 135	270 140	280 135	100	260 170 145	135 90	160 135	300 125	50	240 155	Length of spec.	PCR product(s)
25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48	Well No.	HLA-C allele ^{4,5}
																									*04:01:01-01-04:01:09, 04:01:11-04:01:22, 04:01:24- 04:01:27, 04:01:29-04:01:41, 04:85-04:86, 04:90-04:92, 04:97-04:99, 04:101-04:102, 04:104, 04:109, 04:112- 04:113, 04:117-04:119, 04:121, 04:124-04:127
																									*04:01:10, 04:01:23, 04:116
																									*04:01:28
																									*04:03, 02:49 ⁶
																									*04:04:01-04:04:02, 04:122
																									*04:05
																									*04:06
																									*04:07, 04:89
																									*04:08
																									*04:09N
																									*04:10
																									*04:11
																									*04:12
									34																*04:13
																									*04:14
																									*04:15:01, 04:15:03
																									*04:15:02
																					45				*04:16
																									*04:17
																									*04:18
																									*04:19
																									*04:20
																									*04:23, 04:38, 04:39 ⁹
																									*04:24
																									*04:25, 04:40, 04:41 ¹⁰
																									*04:26
																									*04:27
																									*04:28
																									*04:29
																									*04:30
																									*04:31
																									*04:32
																									*04:33
25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48	Well No.	HLA-C allele ^{4,5}
200	215	280 125	215 120	185 105	75	180 130	275 235 105	180 80	275 200 75	175 145 85	115	135 110	135 95	330 135	270 140	280 135	100	260 170 145	135 90	160 135	300 125	50	240 155	Length of spec.	PCR product(s)

200	215	280	215	185	75	180	275	180	275	175	115	135	135	330	270	280	100	170	135	160	300	50	240	155	Length of spec.	PCR product(s)
25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48	Well No.	Length of spec.	PCR product(s)
																41										*01:04, 01:54, 14:02:01-14:03, 14:05, 14:07N-14:08, 14:10-14:14, 14:16-14:19, 14:21N-14:24:01, 14:27-14:43
														39												*01:21
																										*01:22, 02:03, 02:16:01-02:16:02, 02:18, 05:11, 05:17, 05:68, 05:79, 06:04, 08:01:01-08:01:06, 08:01:08, 08:03:01-08:04:02, 08:06, 08:08-08:11, 08:13-08:14, 08:16:01-08:16:02, 08:20-08:22, 08:24, 08:26N, 08:36N, 08:38-08:40, 08:42, 08:44, 08:46, 08:50, 08:54, 08:56-08:61, 12:14:01-12:14:02, 12:18:01-12:18:02, 12:20, 15:02:01-15:07, 15:09-15:13, 15:16-15:19, 15:21-15:24, 15:26-15:35, 15:37-15:50, 15:52-15:61, 16:35, 16:40, 16:48, 17:01:01:01-17:06, 17:08-17:13
																										*01:35, 05:27, 08:41, 15:15, 17:07, B*58:02
																						47				*01:50
																										*01:59, 05:20, 06:82, 12:54
																										*02:58, 05:39
																										*03:05, 03:13:01, 03:25, 03:27, 03:35
																										*03:34, 03:142
																			44							*03:81
																							47			*03:130, 03:140, 07:49, 07:210, 07:238, 07:247, A*68:46
																										*03:135
					30																					*05:25, 05:42, 06:05, 06:76:02, 08:28, 12:28
		27																								*05:49
										34																*05:64, 08:19
																										*05:78
																										*07:02:09, 15:25, 16:26, 16:46
								33	34																	*07:08, 07:108
25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48	Well No.	Length of spec.	PCR product(s)
200	215	280	215	185	75	180	235	180	200	145	115	135	135	330	270	280	100	170	135	160	300	50	240	155	Length of spec.	PCR product(s)

25	26	27	28	29	30	31	275	33	275	175	36	37	38	39	40	41	42	260	44	45	46	47	48	Well No.
																								*07:64
																								*07:125
																						48		*07:167
																								*08:01:07
25																								*08:02:07, 08:33:02
																41							47	*12:10:01-12:10:02
																41								*14:04
																41								*14:06, 14:15
																41								*14:09
																41								*14:20
																41								*14:25
																								*15:36
																	42							*16:12
25																								*16:45, A*24:96, A*24:146
																								*18:03
																								A*33:52
					30																			B*07:90
25																								B*15:27:01-15:27:03,
																								B*15:109
																								B*15:78:03
																								B*40:100
																								HLA-C allele ^{4,5}
25	26	27	28	29	30	31	32	33	34	35	36	37	38	39	40	41	42	43	44	45	46	47	48	Well No.