



*12:58	1		3						10	12							20																							
*12:60	1		3	4			7		9	10			14																											
*12:63	1		3						10	12											23																			
*12:72	1	2							9	w																														
*12:80N	1	2						8	9		12																					31								
*12:84N	1	2							9		12																				31									
*12:86, 12:104N <sup>5</sup>	1	2							9		12																				29									
*12:98			3			5				10	12																													
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38		
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38		
*12:101	1		3						9	10	12																													
*12:102	1		3				6		9	10	12																													
*12:105N	1	2							9		12																													
*12:108	1		3							10	12																													
*12:109	1		3						9	10	12																													
*12:110	1		3						9	10	12																													
*12:111	1		3						9	10	12																													
*01:02:01-01:02:17, 01:02:19-01:03, 01:06-01:08, 01:10-01:13, 01:15-01:16, 01:18-01:20, 01:23-01:31, 01:33-01:34, 01:37N-01:48, 01:51-01:54, 01:56N-01:58, 01:61-01:78, 01:80-01:90, 03:58, 03:86, 03:94, 03:99, 14:02:01-14:02:02, 14:02:04-14:02:07, 14:02:09-14:02:15, 14:05, 14:07N, 14:11, 14:13-14:14, 14:18-14:21N, 14:23-14:27, 14:29-14:34, 14:36-14:37, 14:39-14:40, 14:42-14:48, 14:50-14:52, 14:55-14:60, B*14:03											11																													
*01:02:18										10	11						17																							
*01:04			2	3																								27												
*01:09, 06:02:12, 06:52, 14:16				3																																				
*01:14, 06:06, 14:12, 14:49					4						11																													
*01:17									9		11																													
*01:22, 01:35, 14:06, 14:15, 15:43																																								
*01:32											11																													
*01:59				4							11																													
*01:60, 14:17											11					16																								
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38		
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38		
*02:02:01-02:02:03, 02:02:05-02:02:07, 02:02:09-02:02:11, 02:02:14-02:02:20, 02:02:22-02:04, 02:06-02:11, 02:13-02:16:02, 02:18-02:26:03, 02:28-02:40, 02:43-02:48, 02:50-02:54, 02:57-02:62, 02:64, 02:66-02:74, 04:10, 04:153, 05:01:01:01-05:01:11, 05:01:13-05:01:23, 05:01:25-05:01:27, 05:03-05:10, 05:12-05:15, 05:18:01-05:19, 05:21-05:22:02, 05:24-05:26, 05:28-05:29:01, 05:30-05:35, 05:37-05:41, 05:43-05:45, 05:47-05:54, 05:56-05:61, 05:63-05:67, 05:69-05:78, 05:80, 05:82-05:84, 05:86-05:92N, 05:94-05:97, 05:99N, 15:08, 15:51, 15:74-15:75, 15:84Q, 16:02:01-16:02:11, 16:09, 16:12, 16:19, 16:46-16:47, 16:60, 16:63					4	5		7																																
*02:02:08, 02:02:13, 02:63, 03:07, 03:45, 04:01:10, 04:01:23, 04:130, 05:01:12, 05:29:02, 06:02:16, 06:35, 06:40, 06:72, 06:79N, 06:81, 06:101, 06:123, 07:09, 07:76, 07:315, 07:328, 18:01-18:07N																																								
*02:02:12											5																													
*02:02:21				4	w			7																																
*02:12	1								9		w																													

Negative Control

Negative Control

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38		
*02:27:01-02:27:02, 03:04:25, 04:11, 04:36, 08:02:01-01-08:02:01:02, 08:02:04-08:02:10, 08:15:01-08:15:02, 08:17-08:19, 08:23, 08:27-08:35, 08:37, 08:43, 08:45, 08:48-08:49, 08:51-08:53, 08:55N, 08:62-08:63, 08:67-08:71, 08:74-08:77, 08:90, 08:92, 08:94, 08:100, 14:08, 14:54, 16:01:01, 16:01:03-16:01:15, 16:06-16:08, 16:10-16:11, 16:16Q-16:18, 16:20-16:24, 16:26-16:28, 16:30N-16:32, 16:34, 16:38-16:39:02, 16:41, 16:43-16:45, 16:49-16:52, 16:54, 16:56, 16:58-16:59, 16:62					5							12																												
*02:42, 05:98				4	5		7						14																											
*02:49, 04:03:01, 04:06, 04:16, 04:103, 04:147, 05:42	1			4			7																																	
*02:55, 04:107, 05:46, 16:25	1			4			7		9																															
*02:56, 05:81				4	5		7																				28													
*02:65, 05:20, 16:57				4	5		7																															32		
*03:08, 03:29, 03:31						6																																		
*03:15	1			4																																				
*03:27, B*07:13	1								9																															
*03:38:01-03:38:02	1								9																		26													
*03:53	1																18																							
*03:69, 03:136	1																										26													
*03:102, 07:24, 07:81, 07:168, 07:218																												28												
*03:104, 07:01:20, 07:01:27, 07:02:38, B*18:77, B*40:29																	19																							
*03:130	1			4					9																		26											32		
*03:140, 07:49, 07:210, 07:238, 07:247				4																																			32	
*03:163	1			4					9																		26													
*03:171, 03:211:01																																						34		
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38		
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38		
*04:01:01:01-04:01:04, 04:01:06-04:01:09, 04:01:11-04:01:22, 04:01:24-04:01:28, 04:01:30-04:01:57, 04:03:02-04:05, 04:07-04:09N, 04:13-04:15:03, 04:17-04:20, 04:23-04:28, 04:30-04:32, 04:34-04:35, 04:38-04:51, 04:53-04:54, 04:56-04:57, 04:59Q-04:79, 04:81-04:99, 04:101-04:102, 04:104-04:106, 04:108-04:111, 04:113, 04:115N-04:129, 04:131-04:139, 04:141-04:143, 04:145-04:146, 04:148-04:152, 04:154-04:159, 04:161-04:165				4			7																																	
*04:01:05				4			7						13											24	25															
*04:12				4			7										18																							
*04:29, 07:01:13, 07:04:01-07:04:09, 07:11-07:12, 07:45, 07:63, 07:68, 07:101, 07:139, 07:142, 07:199:01-07:199:02, 07:272, 07:302, 07:323-07:324, 07:329N, 07:338, 08:02:03												12																												
*04:33				4			7		9																															
*04:37				4			7				11																													
*04:52				4			7																																20	
*04:55					5							12																											20	
*04:58				4			7									16																								
*04:80	1			4			7																				26													
*04:100				4			7																				26													
*04:112				4			6	7																																
*04:114				4			7																																32	
*04:140				4			7						14																											

Negative Control

Negative Control





Well No.
<b>HLA-C allele<sup>1,2,3</sup></b>
*12:02:01-12:02:05, 12:02:07, 12:02:09-12:02:11, 12:56, 12:64, 12:67-12:69, 12:73-12:74, 12:85, 12:96, 12:103, 12:106, 12:112, 12:114
*12:02:06, 12:02:08
* <b>12:03:01:01</b> -12:03:01:02, 12:03:03, <b>12:03:05</b> -12:03:07, <b>12:03:10-12:03:12, 12:03:14-</b> <b>12:03:15, 12:03:17</b> -12:03:18, 12:03:21-12:03:23, 12:03:25- 12:03:29, 12:51-12:53, 12:57, 12:61-12:62, 12:65-12:66, 12:70- 12:71, 12:75-12:79, 12:82, 12:87- 12:95, 12:97, 12:99-12:100, 12:107, 12:113, 12:115
*12:03:02, <b>12:03:08</b> , 12:03:16, 12:03:20, 12:59
* <b>12:03:04</b>
* <b>12:03:09</b>
* <b>12:03:13</b>
* <b>12:03:19</b>
*12:03:24
*12:04:01
* <b>12:04:02</b>
* <b>12:05</b>
*12:06, 12:48, 12:81 <sup>5</sup>
* <b>12:07</b>
*12:08
* <b>12:09</b> , 05:16, 05:85 <sup>4</sup>
*12:10:01-12:10:02
*12:11
*12:12
* <b>12:13</b>
*12:14:01
*12:14:02
*12:15
* <b>12:16</b> , 01:21 <sup>4</sup>
<b>Well No.</b>
<b>Well No.</b>
*12:17
* <b>12:18:01</b> , 12:83
*12:18:02, 08:21 <sup>4</sup>
* <b>12:19</b>
*12:20
* <b>12:21</b>
*12:22
* <b>12:23</b>
* <b>12:24</b>
* <b>12:25</b>
*12:26
* <b>12:27</b>
* <b>12:28</b>
*12:29, <b>12:38</b> <sup>5</sup>
* <b>12:30</b> , 12:36 <sup>5</sup>
*12:31
*12:32, <b>12:34</b> <sup>5</sup>
*12:33, 02:05:01-02:05:03, 02:17 <sup>4</sup>
* <b>12:35</b>
* <b>12:37</b>
* <b>12:39N</b>
*12:40
* <b>12:41</b>
* <b>12:42Q</b>
*12:43
* <b>12:44</b>
*12:45, <b>12:50</b>
*12:46N
* <b>12:47</b>
*12:49
* <b>12:54</b>
*12:55

*12:58
*12:60
*12:63
*12:72
*12:80N
*12:84N
*12:86, 12:104N <sup>5</sup>
*12:98
<b>Well No.</b>
<b>Well No.</b>
*12:101
*12:102
*12:105N
*12:108
*12:109
*12:110
*12:111
*01:02:01-01:02:17, 01:02:19-01:03, 01:06-01:08, 01:10-01:13, 01:15-01:16, 01:18-01:20, 01:23-01:31, 01:33-01:34, 01:37N-01:48, 01:51-01:54, 01:56N-01:58, 01:61-01:78, 01:80-01:90, 03:58, 03:86, 03:94, 03:99, 14:02:01-14:02:02, 14:02:04-14:02:07, 14:02:09-14:02:15, 14:05, 14:07N, 14:11, 14:13-14:14, 14:18-14:21N, 14:23-14:27, 14:29-14:34, 14:36-14:37, 14:39-14:40, 14:42-14:48, 14:50-14:52, 14:55-14:60, <i>B*14:03</i>
*01:02:18
*01:04
*01:09, 06:02:12, 06:52, 14:16
*01:14, 06:06, 14:12, 14:49
*01:17
*01:22, 01:35, 14:06, 14:15, 15:43
*01:32
*01:59
*01:60, 14:17
<b>Well No.</b>
<b>Well No.</b>
*02:02:01-02:02:03, 02:02:05-02:02:07, 02:02:09-02:02:11, 02:02:14-02:02:20, 02:02:22-02:04, 02:06-02:11, 02:13-02:16:02, 02:18-02:26:03, 02:28-02:40, 02:43-02:48, 02:50-02:54, 02:57-02:62, 02:64, 02:66-02:74, 04:10, 04:153, 05:01:01:01-05:01:11, 05:01:13-05:01:23, 05:01:25-05:01:27, 05:03-05:10, 05:12-05:15, 05:18:01-05:19, 05:21-05:22:02, 05:24-05:26, 05:28-05:29:01, 05:30-05:35, 05:37-05:41, 05:43-05:45, 05:47-05:54, 05:56-05:61, 05:63-05:67, 05:69-05:78, 05:80, 05:82-05:84, 05:86-05:92N, 05:94-05:97, 05:99N, 15:08, 15:51, 15:74-15:75, 15:84Q, 16:02:01-16:02:11, 16:09, 16:12, 16:19, 16:46-16:47, 16:60, 16:63
*02:02:08, 02:02:13, 02:63, 03:07, 03:45, 04:01:10, 04:01:23, 04:130, 05:01:12, 05:29:02, 06:02:16, 06:35, 06:40, 06:72, 06:79N, 06:81, 06:101, 06:123, 07:09, 07:76, 07:315, 07:328, 18:01-18:07N
*02:02:12
*02:02:21
*02:12

Well No.
Well No.
*02:27:01-02:27:02, 03:04:25, 04:11, 04:36, 08:02:01:01- 08:02:01:02, 08:02:04-08:02:10, 08:15:01-08:15:02, 08:17-08:19, 08:23, 08:27-08:35, 08:37, 08:43, 08:45, 08:48-08:49, 08:51-08:53, 08:55N, 08:62-08:63, 08:67-08:71, 08:74-08:77, 08:90, 08:92, 08:94, 08:100, 14:08, 14:54, 16:01:01, 16:01:03-16:01:15, 16:06-16:08, 16:10-16:11, 16:16Q-16:18, 16:20- 16:24, 16:26-16:28, 16:30N-16:32, 16:34, 16:38-16:39:02, 16:41, 16:43-16:45, 16:49-16:52, 16:54, 16:56, 16:58-16:59, 16:62
*02:42, 05:98
*02:49, 04:03:01, 04:06, 04:16, 04:103, 04:147, 05:42
*02:55, 04:107, 05:46, 16:25
*02:56, 05:81
*02:65, 05:20, 16:57
*03:08, 03:29, 03:31
*03:15
*03:27, B*07:13
*03:38:01-03:38:02
*03:53
*03:69, 03:136
*03:102, 07:24, 07:81, 07:168, 07:218
*03:104, 07:01:20, 07:01:27, 07:02:38, B*18:77, B*40:29
*03:130
*03:140, 07:49, 07:210, 07:238, 07:247
*03:163
*03:171, 03:211:01
Well No.
Well No.
*04:01:01:01-04:01:04, 04:01:06- 04:01:09, 04:01:11-04:01:22, 04:01:24-04:01:28, 04:01:30- 04:01:57, 04:03:02-04:05, 04:07- 04:09N, 04:13-04:15:03, 04:17- 04:20, 04:23-04:28, 04:30-04:32, 04:34-04:35, 04:38-04:51, 04:53- 04:54, 04:56-04:57, 04:59Q-04:79, 04:81-04:99, 04:101-04:102, 04:104-04:106, 04:108-04:111, 04:113, 04:115N-04:129, 04:131- 04:139, 04:141-04:143, 04:145- 04:146, 04:148-04:152, 04:154- 04:159, 04:161-04:165
*04:01:05
*04:12
*04:29, 07:01:13, 07:04:01- 07:04:09, 07:11-07:12, 07:45, 07:63, 07:68, 07:101, 07:139, 07:142, 07:199:01-07:199:02, 07:272, 07:302, 07:323-07:324, 07:329N, 07:338, 08:02:03
*04:33
*04:37
*04:52
*04:55
*04:58
*04:80
*04:100
*04:112
*04:114
*04:140



*04:144
*04:160
*05:01:24
<b>Well No.</b>
<b>Well No.</b>
*05:11, 05:17, 05:27, 05:68, 05:79, 08:10, 15:02:01-15:02:09, 15:02:11-15:02:13, 15:02:15- 15:02:16, 15:02:18-15:02:19, 15:04-15:06:03, 15:09-15:11, 15:13, 15:15, 15:17-15:19, 15:22- 15:24, 15:26-15:42, 15:44, 15:46- 15:50, 15:52-15:62, 15:64, 15:66, 15:68-15:70, 15:72-15:73, 15:76- 15:82, 16:48
*05:23, 05:62
*05:36
*05:55
*05:93
*06:02:01:01-06:02:01:03, 06:02:03-06:02:11, 06:02:13- 06:02:15, 06:02:17-06:02:36, 06:07-06:10, 06:12-06:13, 06:15- 06:19, 06:21-06:29, 06:31-06:34, 06:36-06:39, 06:42-06:43:02, 06:45-06:51, 06:53:01-06:71, 06:74Q-06:75, 06:77-06:78, 06:80, 06:83-06:86, 06:88-06:100, 06:102 06:117, 06:119-06:121
*06:03:01-06:03:02, 06:76:01
*06:04, 15:02:10, 15:02:17
*06:05
*06:11
*06:14
*06:20, 06:87
*06:30
*06:41
*06:44
*06:73
*06:76:02
*06:82
*06:118
*06:122
<b>Well No.</b>
<b>Well No.</b>
*07:02:09
*07:07
*07:10, 07:43, 07:196
*07:16, 07:51
*07:26, 07:92, 07:96:01-07:96:02, 07:314, 07:317, B*07:15, B*07:160
*07:181
*07:204, A*02:211:01, A*24:261, A*68:46, A*68:76
*07:214
*08:01:01-08:01:13, 08:03:01- 08:04:03, 08:06, 08:08:01-08:09, 08:11, 08:13, 08:16:01-08:16:02, 08:22, 08:24, 08:26N, 08:36N, 08:38-08:39, 08:41-08:42, 08:44, 08:46, 08:50, 08:54, 08:56-08:61, 08:65-08:66, 08:72:01-08:72:02, 08:78-08:79, 08:81-08:89N, 08:91, 08:93, 08:96-08:99, 08:101- 08:102, 14:53, 15:07, 16:35, 16:40
*08:02:02
*08:05, 08:25, 16:15:01
*08:07, 08:47
*08:12, 14:02:03, 14:03, 14:22, 14:35N, 14:38, 14:41, 14:61, 16:53

*08:14, 08:80
*08:20, 08:40
*08:73, B*35:08:02, B*35:08:05
*08:95
*14:02:08
*14:04
*14:10
*15:02:14
*15:03, 15:16
*15:12
*15:21
<b>Well No.</b>
<b>Well No.</b>
*15:25
*15:45, 15:67
*15:63
*15:65
*15:71
*15:83
*16:04:01, 16:29, 16:33, 16:42, 16:55
*16:13
*16:14
*16:15:02
*16:36
*16:37
*16:61
*17:01:01:01-17:01:03, 17:01:05- 17:10, 17:12-17:14, 17:16, 17:18- 17:19, 17:21, 17:23
*17:01:04
*17:11, 17:15
*17:17
*17:20
*17:22
A*26:67, A*68:95
B*15:181N, B*57:50
B*46:51Q
B*67:02
<b>Well No.</b>