

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31
HLA-C allele^{1,2,3}																															
*14:02:01, 14:02:04-14:02:19, 14:30, 14:36-14:37, 14:39-14:40, 14:42, 14:46, 14:50-14:52, 14:56, 14:59, 14:62, 14:65, 14:67-14:69, 14:71	1				5				9																						
*14:02:02	1				5																										
*14:02:03		2		4	5				9																						
*14:03, 14:38, 14:61		2		4	5			8	9																						
*14:04	1		3		5		7		9																						
*14:05	1					6			9																						
*14:06	1								9	10																					
*14:07N	1				5				9		11																				
*14:08		2		4					9			12																			
*14:09	1					6																									
*14:10		2		4	5			8	9				13																		
*14:11	1				5				9					14																	
*14:12	1		3		5				9						15																
*14:13	1				5				9							16															
*14:14	1				5				9				13																		
*14:15	1								9	10				14																	
*14:16	1								9								17														
*14:17	1				5													18													
*14:18, 14:29 ⁵	1				5				9										19												
*14:19	1				5				9												20										
*14:20	1				5				9													21									
*14:21N	1				5				9	10																					
*14:22	1			4	5			8	9									17													
*14:23	1				5				9			12																			
*14:24:01-14:24:02, 14:31 ⁵	1				5				9													22									
*14:25	1				5				9														23								
*14:26	1				5				9										18												
*14:27	1				5				9									17													
*14:28:01	1																				20										
*14:28:02	1																												29		
*14:32	1				5				9													21		23							
*14:33	1				5				9															24							
*14:34					5				9															24							
*14:35N		2		4				8	9												20										

Negative Control

*14:41		2		4	5			8	9													24									
*14:43	1				5				9															27							
*14:44, 14:55 ⁵	1				5				9														25								
*14:45, 14:63	1								9																						
*14:47N	1				5				9	10							17														
*14:48	1				5				9								18	19													
*14:49	1		3		5		7		9					15	16																
*14:53		2		4				8	9	10																					
*14:54		2		4		6		8																							
*14:57	1				5				9															26							
*14:58, 01:97 ⁴					5				9																						
*14:60	1				5				9																28						
*14:64	1		3		5		7		9						16																
*14:66	1				5				9																			30			
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31

<p>*01:02:34, 01:21, 02:87, 07:02:09, 08:01:01-08:01:06, 08:01:08-08:02:06, 08:02:08-08:02:10, 08:02:12-08:09, 08:11-08:33:01, 08:33:03-08:63, 08:65- 08:94, 08:96-08:104, 08:106-08:117, 08:120-08:123, 12:02:01-12:02:10, 12:02:12-12:03:03, 12:03:05-12:03:08, 12:03:10-12:03:19, 12:03:21-12:03:23, 12:03:25-12:03:33, 12:03:35-12:03:37, 12:06-12:08, 12:11-12:20, 12:22-12:26, 12:28-12:32, 12:34-12:40, 12:42Q- 12:53, 12:56-12:59, 12:61-12:69, 12:71, 12:73-12:114, 12:116-12:122, 12:124-12:134, 12:136-12:139, 12:141- 12:145, 12:147-12:153, 12:155Q, 12:157-12:162, 16:01:01, 16:01:03- 16:01:05, 16:01:07-16:01:15, 16:01:17- 16:01:19, 16:04:01, 16:04:03, 16:06- 16:08, 16:10-16:11, 16:13-16:18, 16:20- 16:24, 16:26-16:33, 16:35-16:36, 16:38- 16:45, 16:49-16:56, 16:58-16:59, 16:61- 16:62, 16:64-16:68, 16:72-16:73, 16:75- 16:76, 16:78-16:83, 16:86, <i>B*35:08:02</i>, <i>B*35:08:05</i>, <i>B*67:02</i></p>	4							
<p>*01:14, 02:02:13, 02:02:29, 05:01:12, 05:29:02, 06:44, 06:61, 06:124, 06:138, 07:328, 15:02:10, 15:02:17, 15:99, <i>B*15:277</i>, <i>B*35:222</i></p>				15				Negative Control

*02:27:01-02:27:02, 08:105, 12:140, 15:07, 15:25, 16:34			4		8																	
*02:106								12														
*03:02:01-03:02:07, 03:02:09- 03:03:14, 03:03:16-03:04:16, 03:04:18- 03:04:24, 03:04:26, 03:04:28-03:04:29, 03:04:31-03:04:48, 03:06:01-03:06:02, 03:08-03:09, 03:13:01-03:13:02, 03:16, 03:19-03:22Q, 03:24, 03:26, 03:28, 03:30-03:31, 03:33-03:36, 03:40:01, 03:40:03-03:43:01, 03:44, 03:46-03:49, 03:51-03:57, 03:59, 03:61-03:63, 03:65- 03:68, 03:70, 03:72-03:85, 03:87:02, 03:91:01-03:93, 03:95-03:98, 03:100- 03:101, 03:104-03:106, 03:108-03:114, 03:116:01-03:118, 03:120-03:121N, 03:123-03:127, 03:129, 03:131-03:133, 03:137-03:139, 03:141-03:142, 03:144- 03:152, 03:154-03:162, 03:164-03:166, 03:168-03:177, 03:179-03:189N, 03:191-03:201N, 03:203-03:208N, 03:210-03:216, 03:218-03:219, 03:221- 03:230, 03:232-03:239, 03:241-03:242, 03:244Q-03:245, 03:247-03:248, 03:250-03:261, 03:264-03:266, 03:269- 03:270, 03:272-03:273, 03:277N- 03:280, 03:282-03:295				6	8																	
*03:03:15				6	w																	
*03:04:17, 03:11:01-03:11:02, 03:18:01- 03:18:02, 03:37:01-03:37:02, 03:39, 03:50, 03:64:01-03:64:02, 03:107, 03:115, 03:134, 03:140, 03:217, 03:243, 03:275, 15:77, B*15:78:03				6																		
*03:04:25			4	6	8																	
*03:07, 03:268				6				15														
*03:10, 03:29				6	w			15														

Negative Control

*03:15, 03:297					6	7									15	16																
*03:17, 03:71, 03:249					6		8	9																								
*03:23					6		8															22										
*03:27, 07:20, 07:96:01-07:96:02, 07:127, 07:263, 07:390							7	8																								
*03:32, 03:60					6		8										16															
*03:38:01-03:38:02, 03:69, 03:246, 03:274					6	7	8																									
*03:45					6												15	16														
*03:88					6		8															20										
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	
*03:130					6	7																										
*03:136					6	7	8										16															
*03:163					6	7											15															
*03:231					6								13																			
*03:271								8	9	10												21										
*04:01:01:01-04:01:04, 04:01:06, 04:01:08, 04:01:10-04:01:22, 04:01:24- 04:01:28, 04:01:30-04:01:32, 04:01:34- 04:01:61, 04:01:63-04:01:68, 04:04:01, 04:05, 04:07-04:10, 04:12-04:14, 04:18- 04:20, 04:23-04:25, 04:27-04:28, 04:30- 04:32, 04:34-04:35, 04:38-04:41, 04:43- 04:51, 04:53, 04:57-04:70, 04:72- 04:79, 04:81-04:89, 04:91-04:94:01, 04:95N-04:97, 04:99, 04:101-04:102, 04:104-04:106, 04:108-04:139, 04:141- 04:144, 04:146, 04:148-04:159, 04:161- 04:165, 04:167-04:168, 04:170N, 04:173N-04:177, 04:179, 04:181- 04:189, 04:191N-04:197, 04:199- 04:201, 04:203N-04:205N		2	3														16															
*04:01:05, 04:01:07, 04:01:09, 04:01:33, 04:01:62, 04:04:02, 04:33, 04:94:02, 04:98:01-04:98:02, 04:169, 04:178, 04:180, 05:78, 15:36		2	3																													

*04:01:23			3			7								15	16																																								
*04:01:29			2												16																																								
*04:03:01-04:03:02, 04:06, 04:16, 04:42:01-04:42:02, 04:71, 04:103, 04:147, 04:171, 04:190, 04:206															16																																								
*04:11, 04:172, 12:55			2																																																				
*04:15:01-04:15:03, 04:17, 04:100			2	3											16																																								
*04:26, 04:90, 04:145			w	w											16																																								
*04:29, 04:36			2				4								16																																								
*04:37			2	3								9			16																																								
*04:52			2	3										13	16																																								
*04:54			2	3											16																																		29						
*04:55			2				4							13	16																																								
*04:56				3											16																																								
*04:80								6							16																																								
*04:140, 04:166			2	3											16				19																																				
*04:198			2												16																																								
*04:202			2	3											16																																								
*06:02:01:01-06:02:01:03, 06:02:03- 06:02:07, 06:02:09-06:04, 06:06-06:10, 06:12-06:43:02, 06:45-06:60, 06:62- 06:76:01, 06:77-06:81, 06:83-06:100, 06:102-06:123, 06:125-06:137, 06:139- 06:141, 06:143-06:146, 06:148-06:164, 07:07, 07:09, 07:76:01-07:76:02, 07:315, 07:406, 12:04:01, 17:01:01:01- 17:01:03, 17:01:05-17:21, 17:23-17:26, 17:28-17:30															15																																								
*06:02:08						6	7		9						15	16																																							
*06:101							7								15																																					29			
*06:142							7								15	17																																							
*07:01:41, 07:02:41, 07:223, 07:357, 07:361, 07:415-07:416								w																																															
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31																								
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31																								
*07:02:32, 07:242							7								16																																								
*07:08, 07:108:01-07:108:02							7																																														29		
*07:64	1						7																																																
*07:267, 07:322							7																																																24

Negative Contro

*07:402	1					7									19																					
*08:01:07, 08:02:07, 08:33:02				4		6																														
*08:95, 12:03:24, 12:72, 12:135, 12:154, 16:37, 16:71				w																																
*08:119, 12:115				4																																
*12:03:20, 16:01:06				4		6		9																												
*12:10:01-12:10:02				4																															29	
*12:70				4																														30		
*12:156				4																																
*17:27N							w																													
*18:01-18:09							7																													
A*24:225:02, A*24:248													13																							
B*15:06, B*15:27:01-15:27:03, B*15:84, B*15:109, B*15:195, B*15:327, B*15:344																																				29
B*15:258N											11																									
Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31					

Negative Control