

Well No.	1	2	3	4	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22	23	24	25	26	27	28	29	30	31	32	
HLA-C allele^{1,2,3}																																	
*14:02:01:01-14:02:01:06, 14:02:04-14:02:09, 14:02:11-14:02:20, 14:30, 14:36-14:37, 14:39-14:40, 14:42, 14:50-14:52, 14:56, 14:59, 14:62, 14:65, 14:67-14:69, 14:71-14:72, 14:75-14:76, 14:80, 14:83-14:84, 14:89, 14:91, 14:94-14:96	1				5				9																							31	
*14:02:02, 14:02:21, 14:82	1				5																											31	
*14:02:03		2		4	5				9																							31	
*14:02:10, 14:85, 14:88	1				5				9																								
*14:03, 14:38, 14:61		2		4	5			8	9																								
*14:04	1		3		5		7		9																							31	
*14:05	1					6			9																							31	
*14:06	1								9	10																						31	
*14:07N	1				5				9		11																					31	
*14:08		2		4					9			12																				31	
*14:09	1					6																										31	
*14:10		2		4	5			8	9				13																				
*14:11	1				5				9				14																			31	
*14:12	1		3		5				9					15																		31	
*14:13	1				5				9						16																	31	
*14:14, 14:46:01-14:46:02 ⁴	1				5				9				13																			31	
*14:15, 14:87	1								9	10			14																			31	
*14:16	1								9								17															31	
*14:17	1				5												18															31	
*14:18	1				5				9									19															
*14:19	1				5				9											20												31	
*14:20	1				5				9												21											31	
*14:21N	1				5				9	10																						31	
*14:22	1			4	5			8	9							17																	
*14:23	1				5				9			12																				31	
*14:24:01-14:24:02, 14:31 ⁴	1				5				9													22										31	
*14:25	1				5				9														23									31	
*14:26	1				5				9								18															31	
*14:27	1				5				9								17															31	
*14:28:01	1																			20												31	

ive Control

<p>*01:14, 02:02:13, 02:02:29, 06:44, 06:61, 06:124, 06:138, 06:183-06:184, 06:198, 07:328, 15:02:10, 15:02:17, 15:99, <i>B*15:277, B*15:430, B*35:222, B*42:22</i></p>				15				
<p>*01:17, 01:23, 01:128, 01:152, 06:11, 06:82, 06:210, 07:01:01:01-07:01:40, 07:01:42-07:02:07, 07:02:10-07:02:28, 07:02:30-07:02:31, 07:02:33-07:02:40, 07:02:42-07:06:01:02, 07:10-07:17:02, 07:17:04-07:19, 07:22-07:33N, 07:35- 07:38:02, 07:41-07:63, 07:65-07:68, 07:70- 07:71, 07:74-07:75, 07:77-07:91, 07:93- 07:95, 07:97-07:107, 07:109:01-07:126, 07:128-07:151, 07:153-07:155, 07:157- 07:171, 07:173-07:176, 07:178-07:209, 07:211-07:222, 07:224-07:237, 07:239- 07:241, 07:243-07:262, 07:264N-07:266, 07:268-07:294, 07:296-07:314:03, 07:316- 07:321, 07:324-07:327, 07:329N-07:334, 07:336-07:356, 07:358-07:360, 07:362- 07:389, 07:391-07:401, 07:403-07:404, 07:407-07:414, 07:417, 07:419-07:492, 07:494Q-07:558, 07:560-07:561, 07:564- 07:577, 07:579-07:582Q, 07:584-07:597, 07:599-07:627, <i>B*07:13, B*07:15, B*07:160, B*08:123, B*08:145, B*42:18</i></p>		7						
<p>*01:48, 03:02:08, 03:03:32, 03:04:27, 03:04:30, 03:05, 03:14, 03:25, 03:40:02, 03:43:02, 03:58, 03:86-03:87:01, 03:89- 03:90, 03:94, 03:102-03:103, 03:119:01, 03:122, 03:128, 03:135, 03:143, 03:153, 03:167, 03:178, 03:190, 03:202, 03:209, 03:220, 03:240, 03:262-03:263:02, 03:267, 03:276, 03:296:01-03:296:02, 03:300, 03:330-03:331, 03:335, 03:338, 03:346, 03:353, 03:355, 03:361, 03:386, 15:43, <i>B*07:234, B*15:315, B*18:126, B*55:30</i></p>		8						Negative Control

<p>*02:87, 07:583, 08:01:01:01-08:01:06, 08:01:08-08:02:06, 08:02:08-08:02:10, 08:02:12-08:09, 08:11-08:13, 08:15:01- 08:33:01, 08:33:03-08:57, 08:59-08:63, 08:65-08:79, 08:81-08:94, 08:96-08:102, 08:104, 08:106-08:118, 08:120-08:133, 08:135-08:154, 08:156-08:162, 08:164- 08:166, 12:02:01-12:02:10, 12:02:12- 12:03:03, 12:03:05-12:03:08, 12:03:10- 12:03:19, 12:03:21-12:03:23, 12:03:25- 12:03:33, 12:03:35-12:03:47, 12:06-12:08, 12:11-12:15, 12:17-12:20, 12:22-12:26, 12:28-12:32, 12:34-12:40, 12:42Q-12:53, 12:56-12:59, 12:61-12:69, 12:73-12:103, 12:105N-12:114, 12:116-12:122, 12:124- 12:134, 12:136-12:139, 12:141-12:145, 12:148N-12:153, 12:155Q, 12:157-12:187, 12:189-12:194, 12:195:02, 12:196-12:208, 12:210-12:226, 12:228-12:233, 16:01:01:01-16:01:01:04, 16:01:03- 16:01:05, 16:01:07-16:01:15, 16:01:17- 16:01:23, 16:04:01:01-16:04:01:02, 16:04:03-16:04:05, 16:06-16:08, 16:10- 16:11, 16:13-16:18, 16:20-16:24, 16:26- 16:33, 16:35-16:36, 16:38-16:45, 16:49- 16:56, 16:58-16:59, 16:61-16:62, 16:64- 16:68, 16:72-16:73, 16:75-16:76, 16:78- 16:79, 16:81-16:83, 16:87, 16:92-16:93, 16:95-16:98, 16:100, 16:105, 16:109- 16:114, 16:116, 16:118-16:119, 16:122</p>	4							31
*02:106			12					Negative Control

*03:27, 07:20, 07:96:01-07:96:02, 07:127, 07:263, 07:390, 07:562, 07:578			7 8										
*03:32, 03:60			6 8			16							
*03:38:01-03:38:02, 03:69, 03:246, 03:274			6 7 8										
*03:45			6			15 16							
*03:88			6 8					20					
*03:130			6 7										
*03:136			6 7 8			16							
*03:163			6 7			15							
*03:231			6			13							31
*03:247			6 8							26			
*03:271			8 9 10					21					
*03:302			8 9										
*03:378			6 8			12							
*04:01:01:01-04:01:02, 04:01:04, 04:01:06, 04:01:08, 04:01:10-04:01:22, 04:01:24-04:01:28, 04:01:30-04:01:32, 04:01:34-04:01:61, 04:01:63-04:01:74, 04:01:76, 04:01:78-04:01:91, 04:04:01:01-04:04:01:02, 04:05, 04:07-04:10, 04:12-04:14, 04:18-04:20, 04:23-04:25, 04:28, 04:30-04:32, 04:34-04:35, 04:38-04:41, 04:43-04:51, 04:53, 04:57-04:70, 04:72-04:79, 04:81-04:89, 04:91-04:94:01, 04:95N-04:97, 04:99, 04:101-04:102, 04:104-04:106, 04:108-04:132, 04:134-04:139, 04:141-04:144, 04:146, 04:148-04:159, 04:161-04:165, 04:167-04:168, 04:170N, 04:173N-04:177, 04:179, 04:181-04:189, 04:191N-04:197, 04:199-04:201:02, 04:203N-04:205N, 04:207-04:213, 04:215N-04:216, 04:218-04:219, 04:221, 04:223:02, 04:225N-04:229, 04:232-04:235, 04:237-04:238, 04:240-04:241, 04:244-04:247, 04:250-04:255N, 04:257-04:260, 04:262, 04:264-04:285, 04:287-04:293	2 3					16						31	Negative Control

